




UVC FITTINGS

# VIRUSTRON 3

ULTRAVIOLET GERMICIDAL LAMP

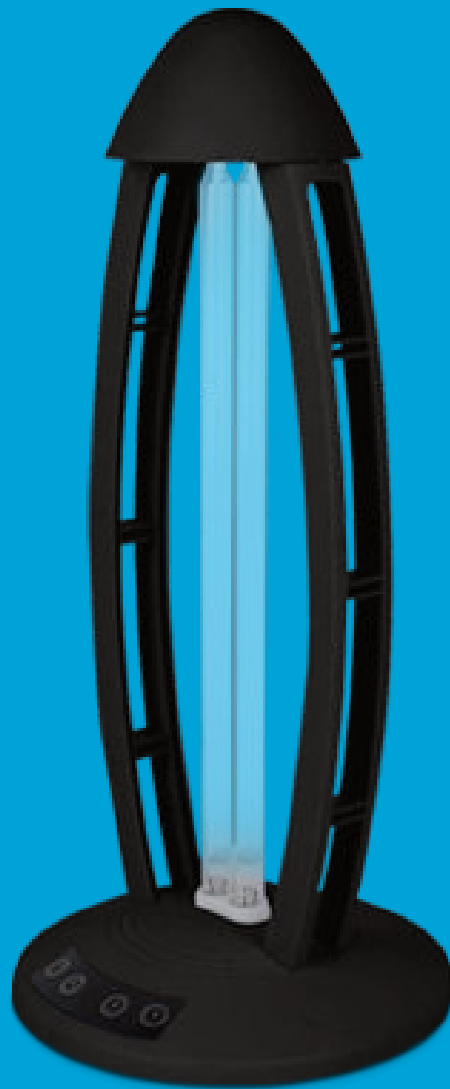

[WWW.SPAZIOMEDICAL.COM](http://WWW.SPAZIOMEDICAL.COM)

## Description

Ultraviolet germicidal lamp with ABS body and quartz tube.

The fitting has a 99.99% disinfection rate utilising UVC and also releases Ozone (O<sub>3</sub>) which assists in the disinfection of hard to reach areas. It can be used in different rooms.

The Virustron 3 is operated with a remote control, as humans and animals cannot be in the room while the fitting is operational. It includes 3 time settings 15, 30 and 60 mins.

The Virustron is CE certified.




UVC and Ozone (O<sub>3</sub>)

## Specifications

Type + Code	Virustron 3	5002.01
Material	ABS	
Illumination	Quartz Tube	
Wattage and Voltage	38W / 230V	
Frequency	50Hz	
Coverage Area	40m <sup>2</sup>	
Dimensions	Diam. 210 x H.460	
Control Method	Remote Control	

## Instructions

1. Plug in the unit.
2. Ensure the inline switch is switched on as per the image below.

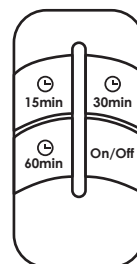

It is recommended that you ventilate the room for at least 40min after disinfection.

3. Ensure the remote battery is installed correctly.
4. Press the remote button on/off **once only** to switch on the fitting.
5. Press the treatment duration button on the remote **only once**, 15 min. **or** 30min. **or** 60min.
6. The UV lamp will turn on after 15 - 20 seconds allowing you time to evacuate the room. Once the sterilization duration is complete, the unit will switch off automatically.

**Each remote will only work with the fitting it is supplied with.**

**Please note:** As an additional safety feature the Virustron 3 has a microwave sensor installed which will automatically switch off the lamp should movement be detected.

During testing or demonstration it is normal for the lamp to turn off and on until no movement is detected.