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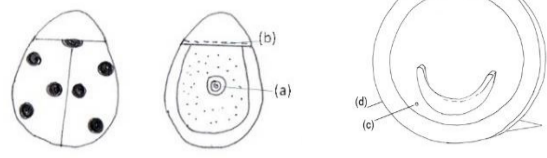
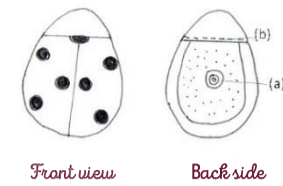
INSTRUCTION MANUAL

Ladybird - applique
with integrated sensor unit

Front view

Back side

Receiver/transmitter station with
Qi wireless charging function
(hereinafter called "station")

- Med. Cert. Infrared (hereinafter called IR) temperature sensor
- Opening for removal of the sensor unit
- Charging and operating LED (blue, orange-red)
- Connection for Micro-USB power cable

Please read the instruction manual carefully before using the device and keep it for further use.

INTENDED PURPOSE

Mary is a Healthcare Lifestyle device for the contactless observation of defined vital functions of a person over a longer period of time at home use. Mary consists of the removable smart applique with the sensor unit, the corresponding underwear, the receiver/transmitter station and a smartphone APP.

Correctly attached to the corresponding underwear, the removable smart applique transmits the data via Bluetooth® LE to the receiver/transmitter station. From this station, the data is transferred via a secure GSM Internet connection to a secure cloud and displayed on the user's smartphone with the appropriate APP.

INTENDED USE AND DEFINED VITAL FUNCTIONS

The device is a support in the care and is not a medical product according to 93/42/EWG and offers no diagnostic and therapeutic methods. It supplies only the measured data as information. Mary monitors body temperature, chest movements, position (prone position, supine position, upright), activity level (calm or active). It reacts to rapid temperature rise, when the set temperature threshold value is exceeded, to undetectable breathing movements or when the prone position is detected.

Whether the child's state of health is critical or changes accordingly, requires the use of a medical device as defined by the EU (e.g. a clinical thermometer). Diagnoses or corresponding therapy procedures are always the responsibility of the treating physician.

(2) USAGE

ATTACHING THE APPLIQUE TO THE CLOTHING

Put on your kid's corresponding clothing from the Mary by sticklett collection. This must fit tightly to the body, otherwise it can come to measuring errors. Attach the ladybird appliqué with integrated sensor unit to the velvet sewn-on area on the Mary by sticklett clothes. (see illustration below). This area may look slightly different depending on the design.

(1) Note that the IR temperature sensor (at the bottom of the ladybird) is inside the red circle.

(2) Position the ladybird-applique crosswise.

(3) With light pressure and up and down movements, ensure that the ladybird-applique has a good hold on the underwear.

MONITORING OF VITAL VALUES AND ALARMING

To ensure a good connection between the sensor unit and the station, please observe the recommended maximum distance and the upright positioning of the station with the appropriate stand.

On the main page of the Mary by sticklett Smartphone APP you can see the temperature display, the state of the breathing movements as well as the current movement behavior.

Keep the APP active during the monitoring period to receive alarms.

You can activate and deactivate alarms according to your needs.

In case of an alarm, the APP reacts with a signal tone on your smartphone (the signal sounds via the media volume of your smartphone).

Vital values can be monitored from an unlimited range using the smartphone.

For further information on the app, please refer to the help in the app.

NOTE

The device is in standby mode (sleeping) if a temperature below 30°C has been detected by the IR temperature sensor. As soon as the IR temperature sensor detects a temperature above 30°C, the sensor unit will automatically wake up and send data.

SAFETY NOTES

INSTRUCTIONS FOR CORRECT USE

Use the device only for its intended purpose.

The device works with appropriate clothing from the Mary by sticklett collection and the Mary by sticklett smartphone APP.

The device is not a toy for children. Please do not let your child play with it.

It is intended for indoor use or for travelling in prams.

The electronic devices (sensor unit and station) are not waterproof (protection class IP40). Avoid splashing water and running water. If liquid is poured over the ladybird application, remove the sensor unit immediately and dry if necessary. In case of water contact with the receiver/transmitter station, make it dry immediately.

The unit can be used for a longer period of time during the day and at night.

We recommend a maximum distance of 1 meter between the ladybird on the clothing and the station. Walls, pieces of furniture or adult persons can affect the quality of the connection.

If the ladybird is correctly attached, the IR temperature sensor adjusts to the child's body temperature within one minute.

The clothing under the ladybird must fit snugly and not hang down limply.

Internet (WIFI / mobile data) and media volume are required to use the Mary by sticklett smartphone APP.

Charge the device completely before using it for the first time.

When the smartphone APP is installed for the first time, the device is automatically activated. Attention: This process can take up to 12 hours.

LITHIUM POLYMER BATTERY - SAFETY INSTRUCTIONS

Do not disassemble, open or crush the lithium polymer battery.

Do not expose the lithium polymer battery to great heat or fire. Avoid storing in direct sunlight.

Do not expose the lithium polymer battery to strong mechanical loads.

Keep lithium polymer battery out of the reach of children.

Use the receiver/transmitter station only with the USB cable supplied. Overheating, explosion or fire may occur if the device is used differently.

NOTES ON BATTERY REMOVAL

Mary by sticklett is powered by two lithium polymer batteries, which cannot be replaced. To remove it, the sensor unit must be carefully cut open with a sharp object and is therefore destroyed. The station must be opened to remove the battery.

NOTES ON DISPOSAL

This device must not be disposed of together with household waste. Every consumer is obliged to hand in all electrical or electronic devices, regardless of whether they contain harmful substances or not, at a local council waste disposal facility so that they can be disposed of in an environmentally friendly manner.

(3) CARE INSTRUCTIONS & TECHNICAL DATA

CLEANING AND DISINFECTION

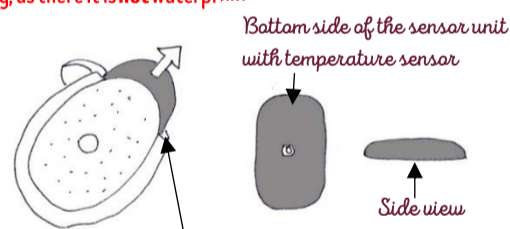
Washing instructions for the clothes & ladybird-applique without sensor unit.

Important notes:

Always remove the applique from the clothing before each wash!

The applique can be washed in the washing machine in the supplied washing bag if required. **Please remove the sensor unit from the applique before washing, as there it is not waterproof**


Remove the sensor unit when opening the head on the bottom side of the ladybird.

The sensor unit is coated with biocompatible silicone.

To clean the sensor unit, use a soft cloth slightly moistened with mild soapy water. Do not allow dripping liquid to enter the deeper areas on the top and the temperature sensor on the bottom.

Do not push the sensor unit back into the smart applique until both parts are completely dry.

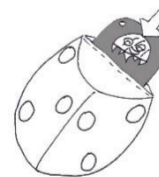For correct function insert the sensor unit into the applique as shown in the illustration on the right.


TECHNICAL DATA

Name und Model:	Mary by sticklett® - Vitalmonitoring SS A10 S2
Article Nr.:	19001-1
Power supply:	5 V = 1A
Measuring range:	27° C - 50°C
Measurement accuracy:	±0,2°C - 0,5°C
Operating conditions:	Room temperature 15°C - 28°C
Storage conditions:	Store in a dry place and do not store under direct sun
Protection class of sensor unit & receiver/ transmitter station:	IP40
Weight of sensor unit:	23 g
Weight of receiver/ transmitter station:	118 g
Size of applique:	65mm x 80 mm x 10 mm
Size of sensor unit:	45 mm x 60 mm x 10 mm
Size of station:	Diameter 150 mm x 30 mm
Voltage/ Capacity sensor unit:	3,7V / 75mAh
Voltage/ Capacity receiver/ transmitter station:	3,7 V / 520mAh

(1) CHARGING & SETTING UP

CHARGING THE RECEIVER/TRANSMITTER STATION

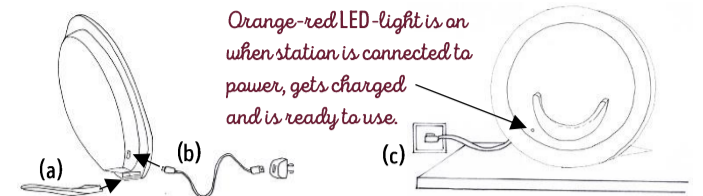- Insert the stand into the slot on the receiver/transmitter station.
- Plug the micro-USB connector into the receiver/transmitter station and the USB connector into a commercially available power supply. (not included in delivery)
- When the receiver/transmitter station is connected to the power, the battery inside is charged and the orange-red LED light is on.

The station works on battery (without orange-red LED light) and lasts approx. 8 hours.

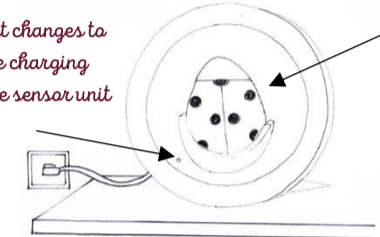
CHARGE THE SENSOR UNIT OF THE LADYBIRD-APPLIQUE

Place the ladybird with integrated sensor unit in the half moon of the upright station according to the following illustration.

When the orange-red LED changes to blue and remains constantly blue, the ladybird is charging. Starting the charging process can often take a few seconds.

The charging process takes approx. 90 minutes. The LED may flash frequently during this time.

The LED-light changes to blue when the charging process of the sensor unit is started.


To start the charging process, place the ladybird upright in the half moon and press it a little on the canvas.

NOTE: The charging process of the smart application only works when station is connected to the power supply. The battery in the ladybird lasts approx. 3-4 days

DOWNLOAD THE MARY BY STICKLETT-APP

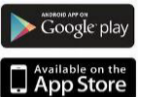The corresponding APP with the designation "MARY by sticklett" can be downloaded for iOS & Android from V 6.0 free of charge in the App and Google Play Store. Install the APP on your smartphone and open it.

SETTING UP - COUPLING THE DEVICE WITH THE APP

Start the APP on your smartphone, scan the QR code in the enclosed envelope when requested, or type in the number below the QR code and tap on "Connect".

The APP will then automatically connect to your device, initiate the activation process and show you the main page of the APP.

To pair the device with the smartphone, the device does not have to be active. The device can also be paired with multiple smartphones at the same time.

PRIVACY POLICY

All data recorded by the device are anonymous and subject to data protection. At no time it is possible to deduce your identity. The data will not be passed on to third parties and you have the right to delete your collected data at any time.

WE RESERVE THE RIGHT TO MAKE TECHNICAL AND DESIGN CHANGES IN THE PROCESS OF PRODUCT IMPROVEMENTS.

These instructions for use were last revised in February 2019.

The current version of the instructions for use and safety information can be found at www.sticklett.com/doc

DECLARATION OF CONFORMITY

The CE marking of the device refers to the EU directive 2014/53/EU. Here we declare that

Mary by sticklett® - Vitalmonitoring SS A10 S2, Art. 19001-1, complies with the basic requirements of the European Directive 2014/53/EU.

You can request the complete EC Declaration of Conformity via sticklett technologies GmbH or download it from the sticklett website www.sticklett.com/doc.

ELECTROMAGNETIC COMPATIBILITY

The device meets the requirements of the EN 301489-1 & EN 301489-17 standard for electromagnetic compatibility. You can request details of these measurement data from sticklett technologies GmbH or download them from the sticklett website www.sticklett.com/doc.

SCOPE OF DELIVERY

- 1 smart ladybird applique with integrated sensor unit
- 1 receiver/transmitter station with Qi wireless charging function
- 1 Micro USB power cable (power supply not included)
- 1 Stand to plug into the receiver/transmitter station
- 1 laundry bag
- 1 envelope with instructions for use, safety instructions and activation code
- 1 mini children's booklet with the story about Mary to read aloud.
- optional 1 piece of clothing (included in Newborn-Edition, otherwise available separately)

MANUFACTURER AND DISTRIBUTOR

STICKLETT

sticklett technologies GmbH
Primoschgasse 3
9020 Klagenfurt am Wörthersee
AUSTRIA
E-Mail: hello@sticklett.com
E-Mail Support: mary@sticklett.com
Internet: www.sticklett.com