

## LIMITED LIFETIME WARRANTY

Your Dynarex Product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser. This item was built to exacting standards and carefully inspected prior to shipment. This Limited Lifetime Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

*The Warranty shall not apply under the following conditions:*

problems arising from normal wear; problems arising from failure to adhere to the product instructions; problems arising from misuse, negligence, accident or improper operation, maintenance or storage; problems arising from modifications or unauthorized repairs, parts or attachments; products where the serial number has been removed or defaced; Problems with non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement. Dynarex shall not be liable for any consequential or incidental damages whatsoever. Dynarex shall repair or replace defective products at its option. The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein.

Reorder No. 10462



# Half Length Bed Rail



**SYMBOL GLOSSARY**  
For an explanation of symbols  
used in Dynarex packaging, visit  
[dynarex.com/symbols.php](http://dynarex.com/symbols.php)

R180131


Manufactured for: Dynarex Corporation  
10 Glenshaw Street • Orangeburg, NY 10962  
USA • [www.dynarex.com](http://www.dynarex.com)

Made in China


## MOUNTING INSTRUCTIONS

1. Attach the bed rail to the frame of the moving head deck. **DO NOT** attach to the fixed bed frame.
2. While holding the rail on an angle, slide the rail so that the bottom edge of the bed frame sits inside the fixed clamp jaw.
3. Slide the movable clamp jaw over the top of the frame. Ensure that the movable clamp jaw at the top of the mechanism attaches to the top edge of the bed frame, and the bed frame sits between the clamp jaws. (Figure 1)

**FIGURE 1**

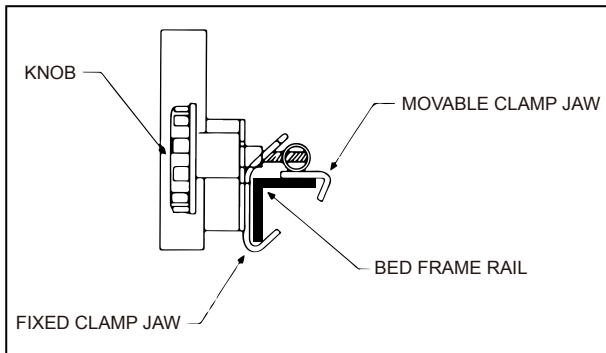

4. Tighten knob to secure it into place.

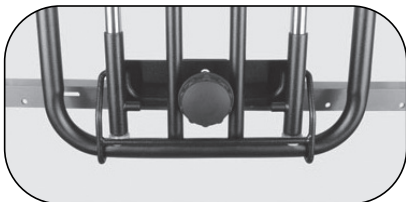

## RAIL OPERATION

To Raise Bed Rail: Pull out retainer bar, raise rail to highest position, and release retainer bar, ensuring that rail is securely trapped by retainer hooks.

To Lower Bed Rail: Raise rail slightly to release retainer hooks, pull out retainer bar, lower rail and release retainer bar.

## WARNING

Read instructions carefully before installing.

Before use, ensure that the Bed Rail is assembled and installed as described, that all attaching hardware is securely tightened, and that the Bed Rail is securely in place; inspect frequently to ensure that all attaching hardware is securely tightened, and that the Bed Rail is securely in place.

**DO NOT** place two bed rails on the same side of the bed.

When installing bed rail, only place one rail on bed per side.

Incorrect mounting may cause injury or death.

**DO NOT** use the Bed Rail as a handle for getting into or out of bed; this could result in personal injury and damage to the Bed Rail.

**DO NOT** exert side pressure on the Bed Rail; this could deform or break it.

**DO NOT** use the Bed Rail as a push handle for moving the bed; this could deform or break it.