

HOW TO ASSEMBLE YOUR




FOLDING CUP HOLDER

Item # CH-1101



WHAT'S INCLUDED

- a. (1) Cup Holder
- b. (1) Hardware
- c. (1) Long Rubber Mats for Express Rollator
- d. (2) Short Rubber Mats for Star Rollators
- e. (2) Mounting Brackets

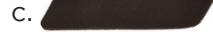

Quick & Easy
Assembly





Requires Phillips
Screwdriver

Follow these
SIMPLE STEPS

- 1** Determine on which side, left or right, the cup holder will be installed on the **Express rollator**. Peel off the yellow strip from the long rubber mat (c.) (**Figure 1**), and wrap around the location where cup holder is to be installed (**Figure 2**).

EXPRESS Model
4328

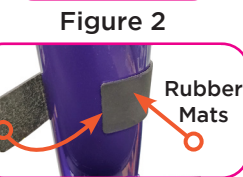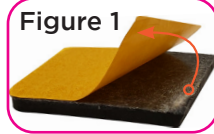

- Note:** The 2 shorter rubber mats (d) are for the **STAR rollators only**. Place a rubber mat on the center of each mounting bracket (**Figure 3**).


STAR Model Series
**4262/4263/4276/4278/4279/
4286/4288/4289**



Figure 3


- 2** Place the mounting brackets around the rubber mat to secure the position. Ensure that the inner plastic gear of the mount faces the side the cup holder will attach to (**Figure 4**). Insert two screws through the mounting bracket and tighten with a phillips screw driver.

NO POWER TOOLS.

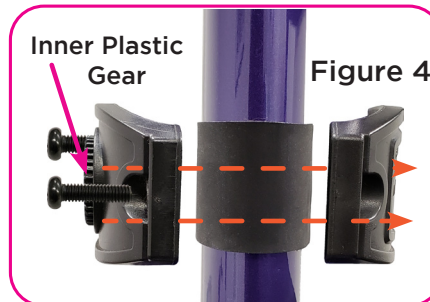


- 3** Attach folding cup holder on the inner plastic gear and ensure the base of the cup holder is parallel to the ground (rotate cup holder to adjust along gear if needed). Secure with washer and small screw (**Figure 5**).

Figure 5

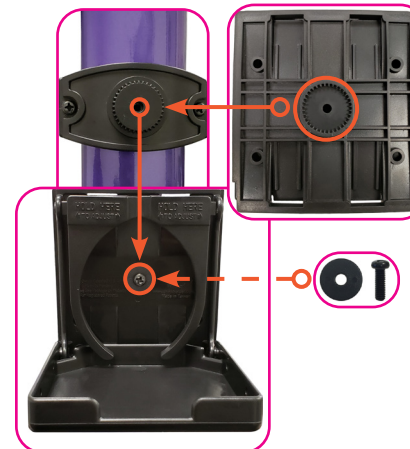

- 4** Drink holder is designed to collapse down to convenient size (**Figure 6**). Swing the clamps down, and flip up the base to fold the drink holder. Drink holder can be adjusted by sliding the two clamps to the appropriate size.

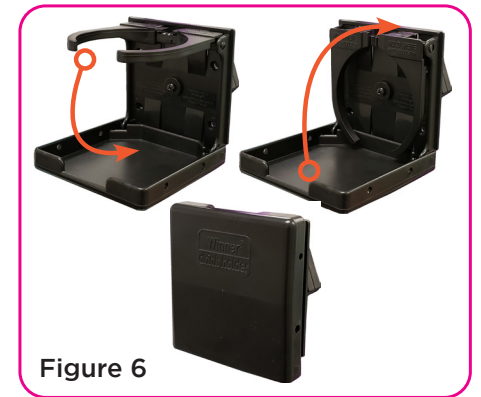

WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame

Non-Metal/Metal parts

Limited Lifetime Warranty

One Year Limited Warranty

(rubber tips, plastic or metal hardware)

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.


THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

IMPORTANT WARNING

FOLLOW ALL DIRECTIONS CAREFULLY BEFORE USE. FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.

CONTACT US!
We're here to help





 info@novajoy.com

 1 800 557 6682

novajoy.com

Connect with us!

novajoy.com

 [novamedicalprod](https://www.facebook.com/novamedicalprod)
 [novamedicalprod](https://twitter.com/novamedicalprod)
 [novawalkerstalk](https://twitter.com/novawalkerstalk)
 [NOVA Medical Products](https://www.youtube.com/channel/UC...)

CARE • QUALITY • INNOVATION



THE **NOVA** PROMISE