**MEDLINE**

# Momentum

## Fold & Go

Rollator

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pliable

## Andador

plegable

MDS86870G, MDS86870R, MDS86870W

# General Warnings and Cautions

**DO NOT use this product without first reading and understanding the instructions contained in this manual.** If you are unable to understand the warnings, cautions, or instructions, contact a healthcare professional, dealer, or technician before use. Failure to do this may result in bodily injury or damage to the device.

**DO NOT** self-propel the rollator while seated.

Rollators are intended for individual use only and are **NOT TO BE USED AS A WHEELCHAIR**. Serious injury to the user and/or damage to the rollator's frame or wheels may result from improper use.

**DO NOT** use the rollator to walk backwards, down gradients or to climb stairs, curbs, or to go over obstacles. Serious risk of fall or injury may occur.

√ The backrest is intended to provide back support while seated. The backrest is not intended to support full body weight. Avoid leaning while sitting as this may result in a fall.

√ All wheels must be in contact with the floor and/or ground at **ALL TIMES**. This will ensure the rollator is properly balanced.

**DO NOT** exceed the maximum weight capacity (including bag contents) or serious injury could result:

## **Maximum weight capacity: 250 lbs (113 kg)**

- Items should be placed completely in the tote bag and should not protrude. Excess weight may cause the rollator to tip, resulting in bodily injury or rollator damage.
- The tote bag located under the seat has a weight limitation of 10 lbs (4.5 kg).
- √ Before each use make sure the rollator frame is fully and securely opened.
- √ Periodically wipe down the rollator with a soft cloth and non-abrasive cleaner.

# Seat Warnings

The brakes **MUST** be in the locked position **BEFORE** sitting on the seat.

**DO NOT** sit on the rollator while it is being loaded into a vehicle or while it is being transported in a vehicle.

**DO NOT** use the seat to transport people or objects.

**DO NOT** sit on the seat when the rollator is on an incline.

The rollator was designed to be used as a walking aid. **DO NOT** use the rollator in a manner that could cause personal injury; such as a scooter, a knee walker, or a jumping/hopping aid.

# Brake Warnings

Check the brakes for proper operation. Make sure that all parts are secure and that the moving parts are in good working order. Wheels should roll easily and not wobble.

When rising from a lower position, **DO NOT** use the rollator to provide assistance to the user. If the rollator is used to pull the user up to a standing position, the rollator may move even if the brakes are in the locked position and this could result in a fall.

**DO NOT** push the rollator while brakes are enabled. Brakes are to be used to keep the wheels from rolling. However, it may not prevent the rollator from moving if it is pushed.

# What's Included

- A** Rollator
- B** Under The Seat Bag
- C** Backrest

- D** Arms
- E** Seat Cushion

# Part Identification

**A** Handles w/ brakes

**B** Seat

**C** Backrest

**D** Under the seat bag

**E** Front wheels

**F** Back wheels

**G** Handle height-adjust push button

**H** Seat height-adjust push button

**I** Handle fold/unfold button

**J** Seat lift lever

**K** Rollator fold/unfold handle

**L** Brake cable

**M** Brake shoe

**N** Seat cushion

# Assembly Instructions

- 1** Unfold the rollator by pressing down on the seat.

- 2** Slide handle/brake assemblies onto each side of the rollator until they lock onto frame. **Push excess loose cable into slot in lower assembly to remove slack, otherwise the brakes will remain locked.**

- 3** Attach the backrest by sliding the two plastic ends into the brackets on the rollator. Make sure to slide them all the way to the bottom. Tighten the screw with a Philips screwdriver until the screw is flush.

**4** With plastic seat flat, add seat cushion to the seat.

- 5** Adjust the handle height. Stand next to the rollator with your arms on the side. Make sure that top of the handles are at your wrists. Adjust to the correct height using push buttons on the handles. Make sure both arms are set to the same height. Min. handle height: 35" (88.9 cm) – Max. handle height: 38.75" (98.4 cm).

- 6** Adjust the seat height by standing next to the rollator and making sure the top of the seat is even with top of your knee caps. Adjust to the correct height using push buttons on the legs. Make sure that all 4 legs are set to the same height.

## Use and Care

To use hand brakes while walking, squeeze brakes to slow down.

To lock the brakes when using the rollator as a seat, push down to lock the brakes. **Release brakes** by pulling up when you are ready to use as a walker again.

# Accessing Storage under the Seat

Lift the seat by pulling up on the lever. To use the rollator step-in feature (to stand closer to tables or countertops) push the bag in by sliding it forward.

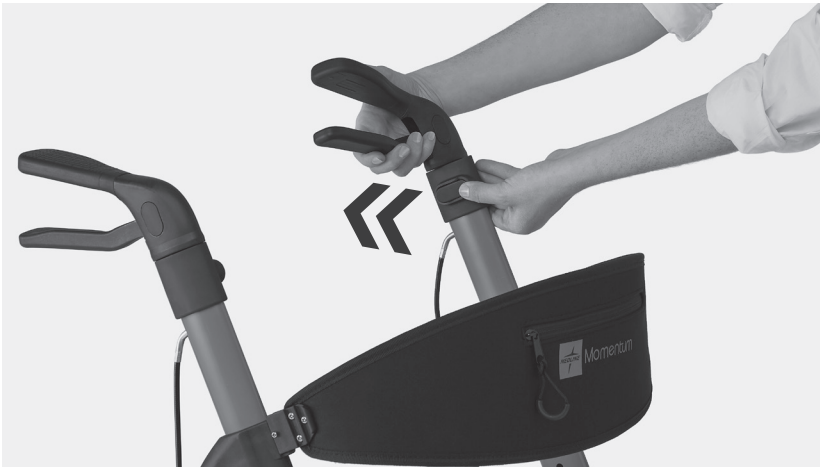
To remove storage bag, unfasten the hook & loop straps.

## Folding and Unfolding the Rollator

For use, make sure the seat is put down and securely locked by pressing it down firmly with two hands. It should make an audible click sound. To fold, grab the seat by the handle and pull up until rollator is folded. Push down on the seat to unfold.

To fold in the handles, press the handle push button while folding the handles in.

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Some products may not be available for sale in Mexico or Canada. We reserve the right to correct any errors that may occur within this brochure. © 2020 Medline Industries, Inc. All rights reserved. Medline is a registered trademark of Medline Industries, Inc.
Medline Industries, Inc.