



INFECTION CONTROL DESIGN

Mor-Medical International
People Matter™

Tel: 956-627-4406 Fax: 956-928-9574 • 208 E Cano St • Edinburg, Texas 78539
www.mormedicalinternational.com • info@mormedicalintl.com

Mor-Medical International wants to thank you for buying the New Era PVC Shower Commode Chair.

Seamless Infection Control Patent Design-No Fittings Exclusively by Mor-Medical designed for efficient infection control. Structural frame held in place with high tensile strength stainless steel bolts.

Patent No.: 11529023



DNE – 310-3TWL-FF-DDA
310 LBS. CAP



DNE – 385-3TWL-FF-DDA
385 LBS. CAP



DNE – 435-4TWL-FF-DDA
435 LBS. CAP

Please remove all the components from the package / box and lay them on the floor / table etc. The list below will detail all components / parts.

1. 2 each side panels, Left and Right



2. 2 drop arms



3. 1 each seat frame with seat, 1 each cross brace to be assembled on the front LOWER SECTION and will have a GREEN tag.

Seat may vary (depends on item ordered)



4. 2 pail rails, already installed on seat frame (if ordered with pail).



5. Commode Pail



6. Snap On Seat

Seat may vary (depends on item ordered)



7. PVC back with Vinyl straps (remove temporary cross brace after installation of top retaining screws. Proceed to install bottom retaining screws after temporary cross brace is removed)



8. 4 each 3" Locking Casters for 18" & 22" chairs, 4" Locking Casters for 26"



All Components / Parts



Assembly Steps

Install PVC back onto left and right-side walls (Step 1)



Both left and right-side wall (Blue tag) secured with bottom cross brace. (Step 2)



Cross brace to be removed once back is installed (slides off). Serves as bottom front brace.



Both Left and Right-side walls secured with seat frame braces. (Step 3)



Install left and right side arms. Secure with bolt and nut provided. Blue strap towards front end (Step 4)



Secure back of arms with screws and cap provided. Secure front of arms with strap and snap (Step 5)



Back of Arm



Front of Arm

Slide rail on rails to install. (Step 6)



Snap on seat to frame (Step 7)



Measure from the bottom of the PVC side wall leg up exactly 1 1/8" (Step 8)



Make a line with a pencil to show the correct distance. 1 1/8" (mark all four PVC legs on the frame) Two per PVC wall – panel (Step 9)



Insert the PVC Cap and Caster together into the PVC leg. Push the two together until you reach your pencil mark of 1 1/8." (Step 10)



Once all four Cap and Casters are INSERTED TO THE **MAXIMUM INTERNAL** DEPTH of 1" (ONE INCH) proceed to secure with the ½" screws utilizing the pilot hole. Screw should face inside of leg (Step 11)



The screws will help retain the Cap and Casters in place.



New Era Chair fully assembled and ready for use!
Note: make sure all screws are tightened properly, wheels are installed correctly and are in the lock position before placing patient in chair.



Tools needed to fully assemble the New Era PVC Shower Commode Chair.



Allen Wrench



Crescent Adjustable Wrench



Phillips Screwdriver

Assembly Components



(6 each) ¼-20 x 3-1/2 pan head metal screws
For side panels



(8 each) 3/8 Flat Washer 18-8 Stainless
For PVC Back



(6 each) 1/4-20 x 3 3/4 Hex C/S Stainless

(4) For PVC Back Rest – 2 per side
(2) For PVC Arms




(4 each) 1/4-20 Acorn Cap Nut Stainless

PVC Back Rest – 2 per side



(2 each) 1/4-20 Lock Nut

PVC Arms

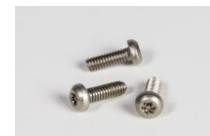

(4 each) 8 x 1/2 Phil Pan T/S SS 18-8

To secure caster wheel and PVC Cap to PVC Frame (front and back)