



RAISINGBELT

The Raising Belt uses the principles of frictionless surface and leverage to assist the patient in a sitting to standing motion. The force needed by the caregiver is greatly reduced thanks to this Bestcare Belt. The Raising Belt is the perfect accessory for the BestMove and BestStand family of products.



PART NUMBER	SIZE	INCHES	CENTIMETERS	UNIT
TS 30760	standard	5.5 x 77	14 x 195.58	ea

HANDI SLIDE OPTIONS

- BED TO BED
- BED TURNING
- ✓ **BED TO CHAIR**
- ✓ **BED SITTING UP**
- BED TO STRETCHER
- BED REPOSITIONING
- STANDING UP** ✓
- PIVOT TURNS
- CAR TRANSFERS** ✓
- LIFT FROM FLOOR
- SEATING POSITIONING
- WHEELCHAIR TO BED** ✓


RAISINGBELT

INSTRUCTION FOR USE



OVER THE BED:

Sit the patient on the bed, and cushion part of the band around the patient.

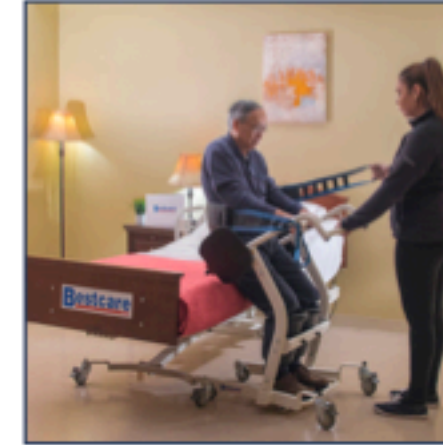

Hook one of the opening on the band to the lift. The caregiver, using the other free end of the band, will pull the patient up.



Once the patient is up, hook the remaining handle to the lift. At this moment, the caregiver can secure the patient to their lift.



WHEELCHAIR OR CAR:

While the patient is on the wheelchair, place the cushion part of the band around the patient.

For sedans or cars, patients feet are firmly on foot rest and their knees are pressed against knee brace.



Hook one of the opening on the band to the lift. The caregiver, using the other free end of the band, will pull the patient up.



Once the patient is up, hook the remaining handle to the lift. At this moment, the caregiver can secure the patient to their lift.