



# NIGHTWATCH<sup>®</sup>

## QUICK START GUIDE

For detailed information on operation and use, please refer to the **NIGHTWATCH<sup>®</sup> User Guide**.  
Available for download via the Resources tab within the App.

Model: NW-1000  
Model: NWC-2000


Offered exclusively by:




**PROTEQUUS<sup>®</sup>**  
Equine Health & Safety

# Powering Your Device

Fully charge your NIGHTWATCH® device prior to use.

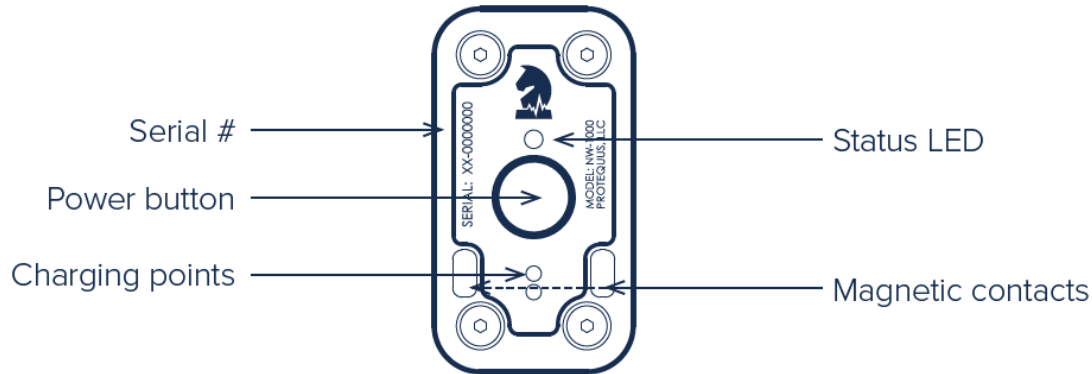

## Power Your Device **ON/OFF**

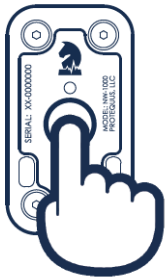

**ON:** **PRESS** and **HOLD** power button ~2 secs (*release when you see **white** LED*)

**OFF:** **PRESS** and **HOLD** power button ~5 secs (*release after **white** LED turns off*)

## **CHARGE** Your Device



**CHARGE:** **ATTACH** magnetic charger to ID plate on halter (*turn on device if you do not see **green** LED*)



**CHARGING** (*blinking*)



**FULLY CHARGED** (*solid green*)

# Setting Up Your Device

Follow these steps to get your NIGHTWATCH® device ready for use.

## Create ACCOUNT

- 1 DOWNLOAD** the NIGHTWATCH® App on the App Store or Google Play to your mobile device.\*
- 2 CREATE** an account by launching the NIGHTWATCH® App and selecting “Create Account.”



## Add Device to ACCOUNT

- 3 ADD** a new device to your NIGHTWATCH® account by tapping the (+) button and selecting “Add New Device.”
- 4 ACTIVATE** your device by scanning or manually entering the Activation Code and Serial # (within original packaging).
- 5 CONNECT** your activated device to a local WiFi network by following the User Guide Instructions (link available in App).




## Assign Device to HORSE

- 6 ADD** a new horse to your NIGHTWATCH® account by tapping the (+) button and selecting “Add New Horse” and follow on-screen wizard to complete horse profile.
- 7 ASSIGN** a device to a horse by selecting “Assign Device” and choose from drop-down menu of available devices by Serial #.  
**NOTE:** If you do not see any devices, return to step 3 to add a new device.



\* The NIGHTWATCH® mobile App requires iOS 12 or higher and Android 11.0 or higher.

# Connecting Your Device

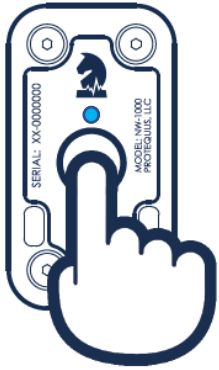

## **STEP 1: Temporarily place device in broadcast mode**

- Confirm your device is OFF.
- Press and HOLD power button ~10 secs (release when you see **blue** LED).
- Proceed to step 2 when you see **blue** LED flashing. This may take several mins.

## **STEP 2: Connect device to a nearby smartphone or computer**

- Open WiFi settings on your smartphone or computer.
- Select your device by serial # among list of available WiFi networks.
- Enter “nightwatch” as password, when prompted.
- Confirm smartphone or computer is successfully connected to device.

**NOTE:** Ensure your device is within range of your desired WiFi network and you have that network’s password before proceeding.

## **STEP 3: Connect device to a WiFi network via your smartphone or computer**

- Open a web browser and enter **connect.smarthalter.com** or **192.168.44.1** to launch WiFi menu of options.
- Select “Setup WiFi,” identify a WiFi network, and enter that network’s password, when prompted.
- Confirm device is successfully connected to a WiFi network.

## **STEP 4: Restart device**





- Power device OFF then ON to exit broadcast mode.



Scan QR Code to access step-by-step guide to connecting your device to WiFi.

# Fitting Your Device



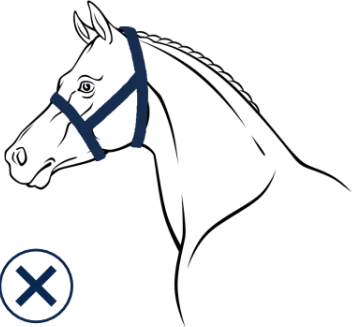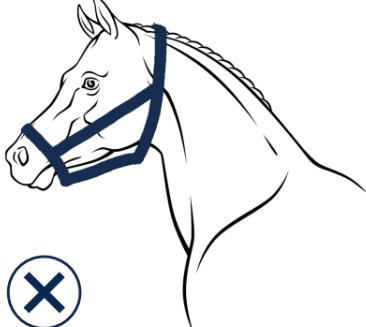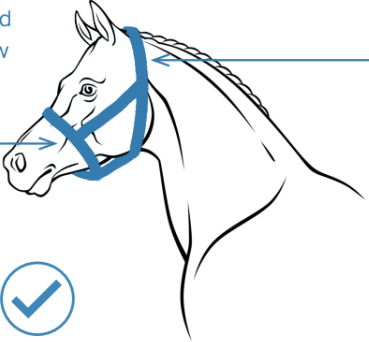




Your NIGHTWATCH® device requires a proper fit\*.

Too Tight	Too Loose
	
Proper Fit	
<p data-bbox="498 876 799 993"><b>NOTE:</b> Noseband should be ~1 inch below cheekbone and loose enough to allow for jaw movement when eating.</p>   <p data-bbox="1199 1029 1421 1176"><b>STOP:</b> The stainless-steel ID plate on your halter's crown should be positioned on the left side of your horse.</p>	

\* An ill-fitting device may detach or result in skin irritation, inaccurate or inconsistent results, and/or damage to your device.

# Status LED Definitions

Your NIGHTWATCH® device will communicate a wealth of information via the status LED.

HORSE Status	
 1 Blink (every 10 secs)	Smart alerts enabled (EDI® ≤ 4.0) or Manual alerts enabled (threshold not breached)
 1 Blink (every 10 secs)	Smart alerts enabled (4.0 < EDI® ≤ 7.0)
 1 Blink (every 10 secs) Solid	Smart alerts enabled (EDI® > 7.0) Lockout Mode (alert issued, device needs to be reset)
DEVICE Status	
 Solid Blinking 1 Blink (every 10 secs)* 3 Blinks (every 10 secs)	Powering ON / OFF, Error, or Updating of Firmware Start Up / Initialization Standby Mode Alerts disabled
 Solid Blinking 3 Blinks (every 10 secs)	Broadcast Mode - Initialization Broadcast Mode - Connect to WiFi via a wireless device Broadcast Mode - Connect to WiFi via mobile App
 Blinking Solid	On Charger - Charging† On Charger - Fully Charged
 No illumination	Powered OFF (0% Battery) or Sleep Mode

\* Device will remain in Standby Mode for 5 mins when first powered ON and/or removed from charger.

† LED will blink rapidly if device battery is critically low.



**Need support? We're here to help.**

Customer Care  
800-757-3856 (+512-515-1095)  
M-F 8:30 AM - 5:30 PM CST  
[support@protequus.com](mailto:support@protequus.com)

© 2022 Protequus®. All Rights Reserved. Patent-Protected Technology.  
All Trademarks and logos herein are owned by Protequus®.

Offered exclusively by:



**PROTEQUUS®**  
Equine Health & Safety