


# Steri-Pak LF Bowie-Dick Test Sheet BD 110

The Steri-Pak LF Bowie-Dick Test Sheet is for daily monitoring of pre-vacuum steam sterilizers operating at 134°C for 3.5 minutes. The test sheet is designed for use with Bowie-Dick towel packs assembled at the health center as described in ISO 11140-3.

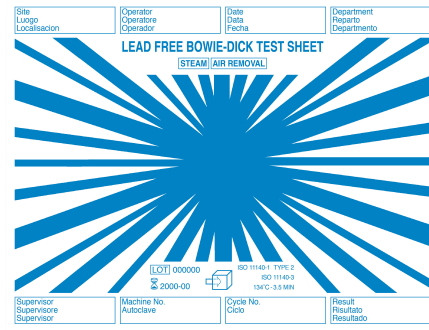

## Key Features

- Complete color change - 3.5 minutes at 134°C
- Blue color remains if an air leak is present
- Lead and toxic heavy metal free

## Performance Characteristics

If the sterilizer is functioning properly, the indicator graphic will show a color change from blue to black throughout the entire indicator image. When excessive residual air or non-condensable gases are present, the indicator will fail. Major failures will leave a blue spot in the center of the sheet. Typical failures will appear as a smaller blue spot or a lighter shade of black in the center of the sheet compared to the edge of the sheet.

## Physical Characteristics

- Dimensions: L 279mm x W 216mm

## Storage

The storage conditions are on the product packaging. Store exposed indicator sheet away from light sources.

## Shelf Life

36 months from date of manufacture. The expiry date is printed on the product packaging.

## Standards Classification

ISO 11140-1 Type 2 Indicators for use in specific tests; ISO 11140-3

## SteriTec Part No.

BD 110, Steri-Pak Bowie-Dick Test Sheet

## Packaging

6 bundles of 100 sheets