Cross-Checks F

REF CI 118

Directions For Use

INDICATIONS FOR USE

SteriTec Cross-checks F chemical process indicator strips are designed to be used in a LTSF (low temperature steam/formaldehyde) sterilizer. Their purpose is to give visual confirmation of the process of LTSF sterilization conditions at their location within the sterilizer chamber.

COLOR CHANGE

During sterilization, the indicator changes from purple to green.

INSTRUCTIONS FOR USE

1. Place a Cross-Checks F process indicator strip in each pack, peel pouch, or tray to be steam/formaldehyde sterilized.
2. Process the package as usual in LTSF sterilizer.
3. After processing, visually verify that the indicator spots have changed color from purple to green.

SAFETY PRECAUTIONS

- *If there is any doubt about sterility of item, it must be considered NOT sterile.*
- *Do not use any strips whose indicator marks are not purple in color prior to exposure to sterilization processing.*

STORAGE

SteriTec Cross-Checks F LTSF process indicator strips should be stored at room temperature 50°- 100°F (10° - 38°C) and normal humidity (10-90% R.H.)

EXPIRY DATE

The expiry date is printed on the product packaging.

LOT NUMBER

A unique identification code, **LOT**, is printed on each indicator strip and packaging labeling.

INTERFERING SUBSTANCES OR CONDITIONS

Do not store near or expose to any formaldehyde or aldehyde based chemicals.

RELEASE OF TOXIC SUBSTANCES

The indicator releases NO KNOWN TOXIC SUBSTANCES in sufficient quantities to cause either a health hazard, or a hazard to the intended properties of the product being sterilized before, during or after the sterilization process.

DECLARATION OF CONFORMITY

SteriTec Products Mfg. Co., Inc. declares that the Cross-Checks F indicator conforms to all applicable portions ISO 11140-1

*SteriTec Products Mfg. Co., Inc.
74 Inverness Drive East
Englewood, CO 80112 USA
(303) 660-4201 FAX (303) 660-4213*

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ID&CO S.r.l.