

Fill the tank to 1 or 2 users' line (for both water and ice)



**Connect air tubing from** Aquilo control unit to each air attachment on the Boots



**Connect water tubing from** Aquilo control unit to each water attachment on the **Boots** 



Put on the Boots and zip until highest point



## **Upper screen** (water pump)



Press Power button on right corner (metal button) in order to switch on the device

Turn on water pump by pressing on the farleft button for 3 seconds, then press again 2 to start

25mins)

1

Pulse button will run the water pump for 2 minutes on, 30 seconds off, repeat

Set timer by either pressing + or – to increase or decrease time (standard time is

## Bottom screen (air pump)





#### **Press Play for Standard Pressure settings**

Standard setting: 75 bottom up(Mode 1)

#### **Changing compression Modes**

Press Set button on the top left and then the + or – button on the top right . A Red Arrow will appear under the Mode (ex. M1, M2, ...)

Mode 2 Uniform compressi

Mode 3 Sequential compression – top to bottom

Mode 4 sequential compression: bottom up with 1 channel at the time; releasing previous channel

Mode 5 sequential compres

It is also possible to (1CH, 2CH..)

Standard setting: 75mmHg, 30min and sequential compression -

Uniform compression – all air channels are filled at the same time

sequential compression: bottom to top with 2 channels at a time

It is also possible to select specific air channels by clicking on them

## Bottom screen (air pump)





### **Changing compression levels**

Press OK button below Set button and then + or – to increase or decrease levels. A Red Arrow will appear under the mmHg

Minimum compression level is 20mm of mercury

Maximum compression level is 150mm of mercury

#### **Changing time**

Press OK button belo decrease time

Minimum time: 20mins Maximum time: 200mins

Press and hold Lock changes to settings

Press Start/Pause bu compression pump

**Press OK button below Set button and then + or – to increase or** 

Press and hold Lock button, if desired, to lock settings and prevent

Press Start/Pause button at bottom left in order to launch the

# www.aquilosports.com

