**EN** Thank you for purchasing the Align-Right Therapeutic Sleeping Pillow. This is the first step in enjoying a good night's sleep. Follow the directions below to ensure you receive the full potential of this innovative pillow. We want all our customers to be Aligned Right.

### Side Sleeping

Place your head on top of the pillow. Your ear should be centered on the pillow like the image below.

### Back Sleeping

Tuck the pillow to your shoulders, but not under the shoulder. The pillow will comfortably support the curve of your neck. (A gentle muscle stretching is felt as the pillow aligns your cervical curve.)

### Important Information

1. Neck muscles may require 2-12 days to adjust to the change in support.
2. When pillow is placed correctly, your head should feel naturally supported.

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## EN Cleaning:

With the pillowcase removed first, machine wash in lukewarm water with the zipper closed.

## Drying:

1. Put in dryer with zipper still closed for 10 minutes on low heat.
2. After the 10 minutes, open the zipper, and continue drying on high heat until dry.

## Refluff Instructions:

Once a month, sanitize and refluff your pillow by unzipping, and placing in dryer on high heat for 20 minutes.

NOTE: The Align-Right Sleeping Pillow should always be dried on its own in the dryer to avoid lint from other clothing collecting on the fibre.

## Warranty:

90 Day Limited Warranty for manufacturers defects. Please check website for full details.

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Align-Right® is a brand of KEGO Corporation.