


verilux[®]
The Healthy Lighting Company[®] Est. 1956

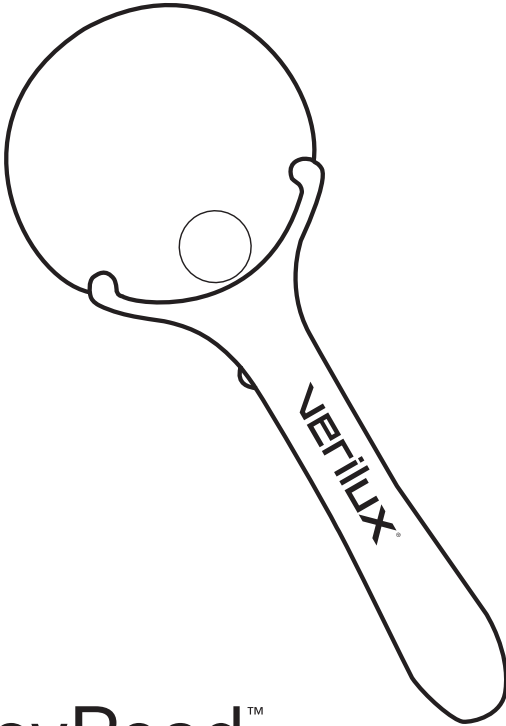

EasyRead[™]

Natural Spectrum[®]
Illuminated Magnifier

Model: VM04

Job#: 0.0	Title: VM04 Manual trim to 3.875"x5.75"
Date: 06/18/13	Version: VM04-MAN-Rev8



Dear Customer,

Thank you for purchasing the EasyRead™ Natural Spectrum® Illuminated Magnifier. You now own an innovative product, manufactured to the highest standards and backed by a one-year limited warranty.

As a Verilux customer, your satisfaction means everything to us. We look forward to serving you now and in the future.

Many other healthy-lighting products are available through our catalog and online. Visit us online at www.verilux.com to request a free catalog and learn more about all our quality Verilux products, or call us toll-free at 1-800-454-4408.

Have a bright day!

A handwritten signature in black ink that reads "Nicholas Harmon".

Nicholas Harmon
President, Verilux, Inc.

Features

The comfortable, handheld EasyRead™ Illuminated Magnifier will help you see better by magnifying subject matter by 2.5X with bright, Natural Spectrum® light. Two LEDs evenly illuminate reading surfaces without glare, and the 5X bifocal inset provides additional magnification for seeing the finest details.

- 2.5X high-quality, optical-grade lens with 5X spot lens
- 2 Natural Spectrum LEDs
- Easy-grip, comfort handle
- Zippered, padded case and lens cloth
- On/Off button
- Easy-access compartment holds 2 AAA batteries (not included)

Operation


You have the option of using the EasyRead Magnifier with or without illumination. Simply press the button on the upper left side of the handle once, and the two LEDs will illuminate. Press the button a second time to turn them off.

Replace batteries with two AAAs only.

Locate the battery compartment on the underside of the handle. Slide the battery compartment open, tilt the compartment cover upward and remove any dead batteries. Please dispose of batteries in a proper manner to help minimize environmental impact.

Insert two AAA batteries into the compartment with the “+” end pointing toward the bottom of the handle. Make sure the “+” and “-” symbols are facing in the correct direction; otherwise, the LEDs will not operate.

Once the batteries are inserted, lower the compartment cover and slide upward towards the lens until the cover clicks into place.

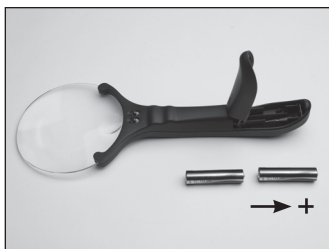

3

Job#: 0.0	Title: VM04 Manual trim to 3.875"x5.75"
Date: 06/18/13	Version: VM04-MAN-Rev8

One-Year Limited Warranty

ATTENTION! ONCE OPENED, PLEASE **DO NOT** RETURN THIS PRODUCT TO THE STORE WHERE IT WAS PURCHASED FOR REPAIR OR REPLACEMENT!

Many questions may be answered by visiting www.verilux.com, or you may call our Customer Service Department at 800-786-6850 during normal business hours.

This limited warranty is provided by: Verilux, Inc., 340 Mad River Park, Waitsfield, VT 05673

Verilux warrants this product to be free from defects in material and workmanship for a period of one year from the date of the original retail purchase from Verilux or an authorized Verilux distributor. **Proof of purchase is required for all warranty claims.** During the limited warranty period, Verilux, Inc. will, at its option, repair or replace defective parts of this product, at no charge to the customer, subject to the following limitations: This limited warranty does not include any postage, freight, handling, insurance or delivery fees. This warranty does not cover damage, defect or failure caused by or resulting from accident, external destruction, alteration, modification, abuse, misuse or misapplication of this product.

This warranty does not cover damage to the product resulting from return shipping or handling. The use of shipping insurance is recommended to help protect your product.

Return Authorization is required for all returns. To obtain a Return Authorization, please contact the Verilux Customer Service Department at 800-786-6850.

If, during the first year of ownership, this product fails to operate properly, it should be returned as specified at www.verilux.com/warrantyreplacement or as instructed by a Verilux customer service representative at 800-786-6850.

Due to ongoing improvements, actual product may have slight variations from the product described in this manual.

**Please visit our website at: www.verilux.com or call
our Customer Service Department at**

1-800-786-6850

Representatives are available Monday — Friday, 9am — 5pm EST

verilux[®]
The Healthy Lighting Company[®] Est. 1956

340 Mad River Park, Waitsfield, VT 05673

VM04-MAN-Rev8

Printed in China for: Verilux, Inc.
© Copyright 2012 Verilux, Inc. All rights reserved.