# pumpsets and accessories

Instructions for use | Gebrauchsanweisung | Mode d'emploi | Istruzioni per l'uso | Instrucciones de uso | Indicações de utilização | Gebruiksaanwijzing | Bruksanvisning | Käyttöohje | Bruksanvisning | Brugsanvisning | Инструкција по експлуатацији | Instrukcja obsługi | Οδηγίες χρήσης | Kullanım Kılavuzu | دليل الاستعمال | 사용 설명서 | 使用说明

	<b>Languages</b>		<b>Page</b>
en	Instructions for use	Ardo Pumpsets and accessories	1–5
de	Gebrauchsanweisung	Ardo Pumpsets und Zubehör	6–10
fr	Mode d'emploi	des Pumpsets Ardo et accessoires	11–15
it	Istruzioni per l'uso	Pumpsets e accessori Ardo	16–20
es	Instrucciones de uso	Pumpsets Ardo y accesorios	21–25
pt	Indicações de utilização	Pumpsets e acessórios Ardo	26–30
nl	Gebruiksaanwijzing	Ardo Pumpsets en accessoires	31–35
sv	Bruksanvisning	Ardo Pumpset och tillbehör	36–40
fi	Käyttöohje	Ardo Pumpsets ja lisävarusteet	41–45
no	Bruksanvisning	Ardo Pumpsets og tilbehør	46–50
da	Brugsanvisning	Ardo Pumpsets og tilbehør	51–55
ru	Инструкция по эксплуатации	Наборы для сцеживания и принадлежности Ardo	56–60
pl	Instrukcja obsługi	Ardo Pumpsets i akcesoriów	61–65
el	Οδηγίες χρήσης	Ardo Pumpsets και αξεσουάρ	66–70
tr	Kullanım Kılavuzu	Ardo Pumpsets ve Aksesuarlar	71–75
ar	دليل الاستعمال	والملاحقات التكميلية Ardo طقم المضخة	76–80
ko	사용 설명서	아르도 유축기 세트 및 부속품	81–85
zh	使用说明	Ardo 水泵机组及附件	86–90

## 1. Packaging contents

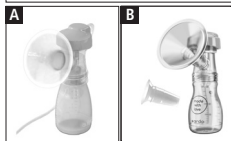
### Pumpset:

- a Adapter tube cover
- b Flexible membrane pot
- c Breast shell 26 mm
- d 1 lip valve plus 1 spare valve
- e Bottle 150 ml (illustration not binding)
- f Tube
- g Tube connector
- h Cap
- i Cap liner

### Versions

- A Single Pumpset
- Double Pumpset
- Clinic Pumpset
- Pumpset Ardo Alyssa & Bellis
- B Pumpset Ardo Alyssa


**WARNING: Danger of suffocation** – contains small parts. Keep small parts away from babies and small children.

## 2. Supplementary accessories

- j 36 mm breast shell
- k 31 mm breast shell
- l 28 mm breast shell insert
- m 22 mm breast shell insert
- n 28 mm breast shell insert, silicone
- o 22 mm breast shell insert, silicone
- p 18 mm breast shell insert, silicone
- q Optiflow breast shell insert
- r Bottle holder
- s Bottle stand

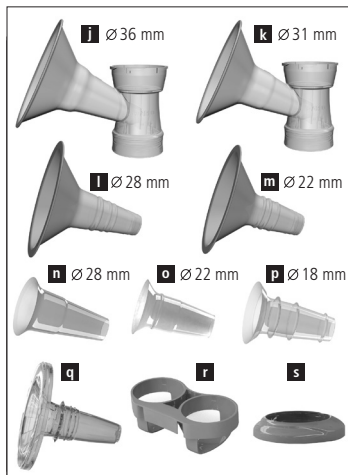The choice of two Ardo breast shells and breast shell inserts enables you to find the size that is right for your nipples. The Ardo Optiflow massage insert guarantees even gentler expressing and the massage effect encourages the flow of milk (can only be used with the 31 mm breast shell). The bottle holder is compatible with the Calypso Double Plus breastpump and the bottle stand is compatible with Calypso and Ardo Alyssa.

### 3. Intended use

The Ardo pumpset should be used in combination with the Ardo breastpumps and is intended for the expression, collection and storage of breast milk. The pumpset can be used as a single or double pumpset.

### 4. User risk

There are no known contraindications.

**Safety regulations:** Only use the pumpset if it is clean (Sections 6 and 7). Check the pumpset for damage after cleaning.

According to Regulation (EU) 2017/745 on medical devices, any serious incidents that occur in connection with the product must be reported to the manufacturer and the competent authorities of the Member State in which the user resides.

### 5. Assembling the pumpset

Assemble as illustrated. Attach the adapter tube cover so that both marks match up and then rotate to the right.

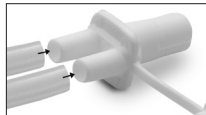
Important: Boil before first use (see the section 'Cleaning at home') and check whether the flexible membrane pot is intact with the smooth side facing outwards.

Calypso, Carum, Ardo Alyssa, Ardo Bellis


Calypso, Ardo Alyssa, Ardo Bellis

Push the insert into the breast shell and fix it by rotating it slightly.

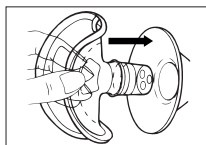Insert the active Optiflow massage insert into the 31 mm breast shell and pull the edge of the insert, but not the lug on the top edge, over the breast shell.

## 6. Cleaning at home

After each use, wash the pumpset with dishwashing liquid and rinse well. Once a day, disassemble the pumpset and boil the parts for 10 minutes or clean in the dishwasher, microwave (in the microwave bag) or steam steriliser (follow the manufacturer's instructions), then dry with a clean cloth and cover.

This product can be washed and re-used 40 times.

**ATTENTION:** After boiling, allow the parts to cool down before you touch them.

The illustrated parts can be wiped down with a cloth as needed. It is not necessary to flush the tube.

Do not boil the bottle holder.

Do not clean the lip valve with a brush.

## 7. Cleaning in hospitals (only for the Clinic Pumpset)

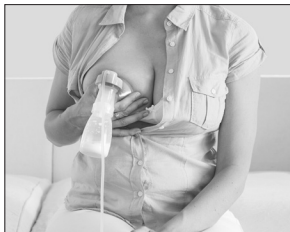
Sterilise the disassembled pumpset at 121°C (250°F) for 20 min / 134°C (273°F) for 5 min.

## 8. Use

Wash your hands thoroughly. Sit in a comfortable and relaxed position.


### 8.1 Expressing with one pumpset

- Connect the pumpset to the Ardo breastpump.
- Centre your nipple so that it moves freely in the breast shell (breast shell insert) when pumping.
- Switch the breastpump on.


### 8.2 Expressing with a double pumpset

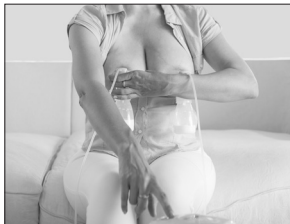- Connect the second pumpset to the Ardo breastpump.
- Procedure for the **Ardo Carum**, **Ardo Alyssa** and **Ardo Bellis** breastpumps: Place a pumpset on the breast, switch the pump on and place the second pumpset on the other breast. Alternatively, you can also attach both pumpsets at the same time.
- Procedure for the **Ardo Calypso** breastpump: Attach both pumpsets at the same time (see illustration). Switch the pump on and hold each pumpset separately in your hands.

## 9. Choosing a breast shell and Optiflow massage insert

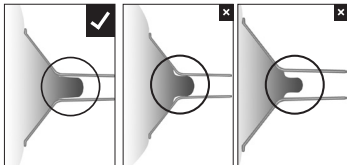If your nipple rubs uncomfortably on the inside of the funnel, use a larger breast shell (breast shell insert). If the areola is also sucked into the funnel, use a smaller breast shell. The Optiflow can only be used with the 31 mm breast shell. Store the Optiflow insert protected from light (silicone).

## 10. Storing breast milk

Recommended storage period at room temperature: 3–4 hours at 16–29°C (60–85°F); in the refrigerator: 3 days at ≤4°C (39°F); in the freezer: 6 months to 1 year at <-17°C (0°F)

**ATTENTION:** Do not put more than 150 ml of breast milk in the bottle because milk expands when it freezes. Do not heat breast milk in the microwave.

## 11. Possible problems and troubleshooting

### Weak or no vacuum:

- Ensure that the pumpset is correctly assembled and is not damaged.
- Check if the lip valve is cracked or stuck together.
- Check if the flexible membrane pot is intact with the smooth side facing outwards.
- Ensure that the adapter tube cover forms a tight seal.

### Pain when pumping:

- Ensure that the nipple is in the middle of the breast shell.
- Check that the breast shell size is right for your nipples.
- Check that the vacuum intensity is not set too high.
- Contact a professional for help.

If you have problems with breastfeeding, please contact your midwife, lactation consultant or doctor.

## 12. Warranty

There are legally prescribed minimum warranty periods from the date of purchase that are specific for each country. The warranty coverage is limited to the first purchaser.

## 13. Spare parts

63.00.267 Lip valves (2 pieces)

63.00.269 Flexible membrane pot, adapter tube cover (2 pieces)

## 14. Technical information

Service life of the pumpset: As long as the pumpset is not faulty, it can be used. Please check the pumpset for damage before every use.

 0123	Compliance with the valid European regulations for medical devices		Manufacturer
	Follow instructions for use		Follow instructions for use
	Medical device		Pumpset is a BF type applied part
	Materials: Polypropylene (PP) / silicone / ABS		Relative humidity: 20%–95%
	Temperature: Operation +5°C (41°F) to +40°C (104°F) / Transport -20°C (-4°F) to +50°C (122°F)		Air pressure: 700–1060 hPa
	Dispose of with polypropylene		Protect packaging from sunlight
	Packaging: Caution, fragile		Store in a dry place

## 15. Disposal

Waste must be disposed of in accordance with local regulations.

The German text is the reference for the contents.

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Ardo medical AG