

Thank you for purchasing this Equipro equipment.

This booklet is designed to familiarize you with this product. It explains assembly, operation, use and maintenance, as well as some precautionary measures. **Please read instructions carefully. This will save you time and unnecessary repairs.** Only qualified and licensed technicians should operate this unit, Equipro will not be held responsible for injuries or accidents resulting from improper use.

Operating instructions

- Make sure that the on/off switch is in the off position, and the intensity control at minimum setting.
- Select and install one of the diamond tip on the stainless steel handpiece connected to the tube.
- Turn on unit.
- Gradually adjust the intensity control switch, to the desired suction intensity.
- Once the desired intensity is reached, gently go over the area to be treated. ALWAYS KEEP A CONSTANT MOTION ON THE SKIN.
- Once treatment is completed, turn the intensity control back to the minimum setting before shutting off the unit.

Sterilization

Never forget to clean the diamond tips and the hand-piece after each use, with a mild soap or alcohol, then with an ultraviolet sterilizer. We recommend using the **Hot-cabi / UV sterilizer** by Equipro, which is specifically conceived for this use. Please note that the cotton filters are not reusable.



In case of malfunction

If the **Diamonderm** does not turn on :

- Verify if the unit is properly plugged in and if the wall outlet is in good working condition.
- Verify the fuse, located at the back of the unit.
- Never replace it with a higher amp fuse. Maximum 1 amp/250V.
- Verify if the tube is correctly plugged and it is not twisted.
- The power cable may be cut, also check if there is a defective suctiontube.

After having verified all of the possibilities above, if your unit is still not functioning properly, or not at all, contact your distributor or salesperson **ONLY**.

Precautionary measures

It is recommended to wash the diamond tips with a mild soap before use, in order to prevent any possible allergic reaction to the cleaning product which may be left on the abrasive tips.

Intensity level	Unit of pressure
MIN	4 inHg
1	5 inHg
2	8 inHg
3	12 inHg
4	14 inHg
5	18 inHg
6	20 inHg
MAX	22 inHg



Intensity level (suggested)

Min – 1	Eye contour, sensitive area
2	Neck, thin, sensitive skin
2.5 – 3	Face
4+	Body

Diamond tips (suggested)



Body, knee, elbow, stretchmarks



Face, larger blemishes



Face scars, imperfections, fine lines



Face, blackheads, acne



Smaller area, nose, eyes, wrinkles



Our Mission

Making you our top priority for more than 3 generations!
For more than fifty years, our family has been supporting beauty and wellbeing professionals.

As a manufacturer of high-end beauty and spa equipment, we have always understood that our products are your work tools. That is why excellence is our commitment: so you can provide clients the very best care day after day.

Quality Equipment

Quality, innovation, design & technology are the results of a dynamic production team working hand in hand with the research and development department in order to design top quality aesthetic appliances and equipment.



www.equipro-bty.com



Contact Us

+1 877 324-2226

info@equipro-bty.com

Canadian Division


ÉQUIPRO ÉQUIPEMENT D'ESTHÉTIQUE
11970 Albert Hudon blvd.
Montréal, (Qc), Canada H1G 3K3
Toll free : 1 877 324-2226
Email : info@equipro-bty.com

U.S. Division

EQUIPRO BEAUTY USA
2425 Main Street, P.O.Box 154
Isle La Motte, VT, USA 05463
Toll free : 1 877 324-2226
Email : info@equipro-bty.com



Diamonderm Instruction manual

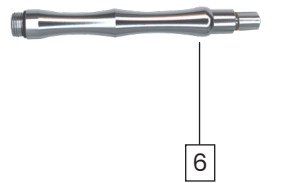

1. On/Off switch
2. Intensity control
3. Vacuum connector
4. Handpiece mount
5. Diamond abrasive tips
6. Stainless steel handpiece

Accessories

Each Diamonderm unit includes the following :

- 3 stainless steel handpieces
- 10 interchangeable cotton filters
- 9 individual diamond abrasive tips
- 1 vacuum tube

