

Distributed by:  
ZEPF SURGICAL  
800-552-3441



Stille artery forceps




## Stille artery forceps

Distributed by:

ZEPF SURGICAL

800-552-3441

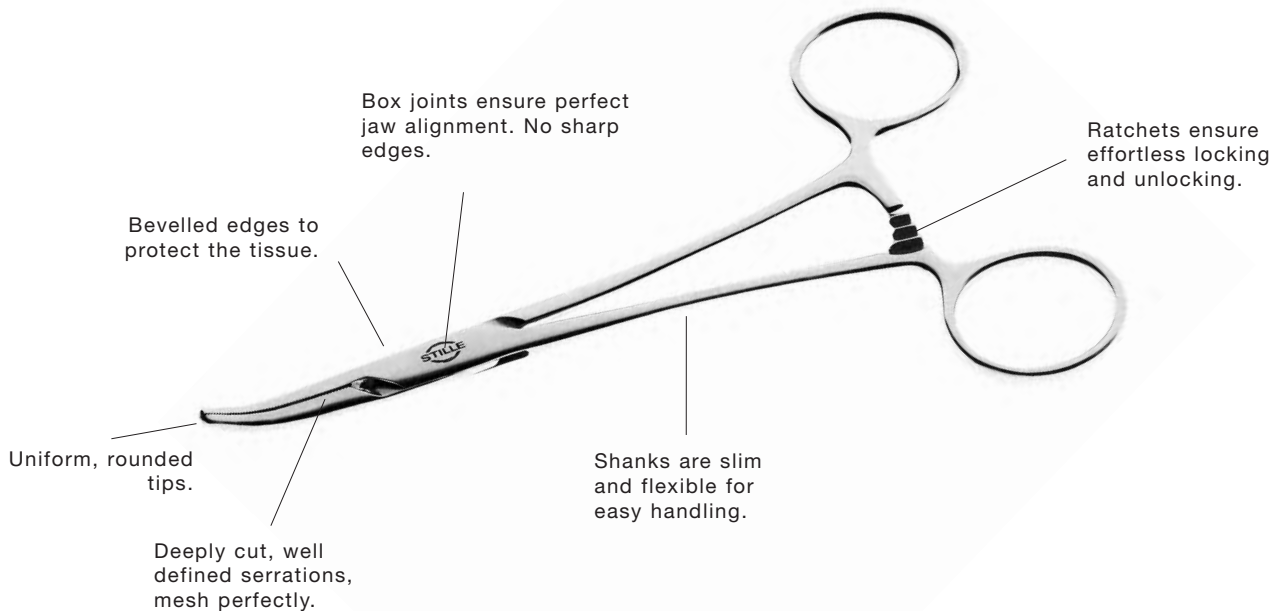

**Stille artery forceps** are manufactured in stainless steel of the highest quality available and are designed with the specific needs of the surgeon in focus. They are always easily opened and closed, enabling perfect control. The steady grip ensures that the haemostatic forceps does not slip off during surgery. Serrations mesh perfectly. When a bleeding vessel is clamped, blood flow is arrested and no valuable time is lost in making adjustments. The long life –surpassing that of other instrument brands- means reduced costs for the hospital.

### How to test Stille Artery Forceps:

Close the jaws slowly until they meet.  
If they overlap, the jaws are out of alignment.

If the instrument has teeth, inspect to ensure that the teeth are meshing properly.

While holding the rings of the instrument in each hand, open the instrument as far as possible. If the box lock has considerable play and it is very loose, this will cause misalignment. If this occurs, the instrument is faulty and should be repaired.

Cleaning and sterilizing of a haemostat should always be carried out with the ratchet open.

**ZEPF**®


Distributed by:  
**ZEPF SURGICAL**  
 800-552-3441



**HALSTED-MOSQUITO DELICATE**

Delicate hemostat without teeth for small operating field.


- \* 103-10026-121 Straight 12 cm 4.75"
- 103-10028-121 Curved 12 cm 4.75"

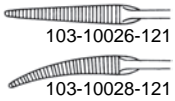

**HALSTED-MOSQUITO**

Hemostat without teeth, used in small operative fields.  
 For plastic surgery, neuro, hand or any delicate surgery.


- \* 103-10030-121 Straight 12 cm 4.75"
- 103-10032-121 Curved 12 cm 4.75"
- 103-10032-181 Curved 18 cm 7"

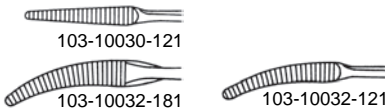

**HALSTED-MOSQUITO**

Hemostat with 1x2 teeth, used in small operative fields.  
 For plastic surgery, neuro, hand or any delicate surgery.

- \* 103-10035-121 Straight 1x2 Teeth 12 cm 4.75"
- 103-10037-121 Curved 1x2 Teeth 12 cm 4.75"

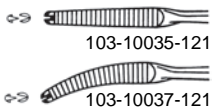

ARTERY FORCEPS

\* Photo shows this instrument

Distributed by:  
**ZEPF SURGICAL**  
800-552-3441



## **PÉAN**

Hemostat with long jaws and relatively blunt tip.  
Widely used when a short instrument is needed.

\* 103-10110-141 Straight 14 cm 5.5"



## **KOCHER**

Hemostat with 1x2 teeth. Widely used when a short instrument is needed. Can also be used as tissue forceps to retract or lift an organ.


\* 103-10115-141 Straight 1x2 Teeth 14 cm 5.5"



## **CHRISTOPHE-STILLE**

Curved hemostat without teeth.

\* 103-10172- Curved 16 cm 6.25"

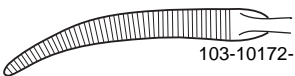


\* Photo shows this instrument

**CRILE**

General purpose hemostat without teeth. Used in relatively small operating fields, when more strength is needed than a Mosquito can offer.


- \* 103-10180-141 Straight 14 cm 5.5"
- 103-10180-161 Straight 16 cm 6.25"
- 103-10180-181 Straight 18 cm 7"
- 103-10182-141 Curved 14 cm 5.5"
- 103-10182-161 Curved 16 cm 6.25"
- 103-10182-181 Curved 18 cm 7"

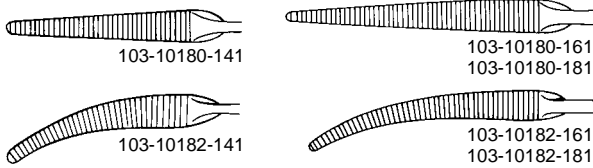

**CRILE**

General purpose hemostat with 1x2 teeth. Used in relatively small operating fields, when more strength is needed than a Mosquito can offer.



- \* 103-10185- Straight 1x2 Teeth 14 cm 5.5"

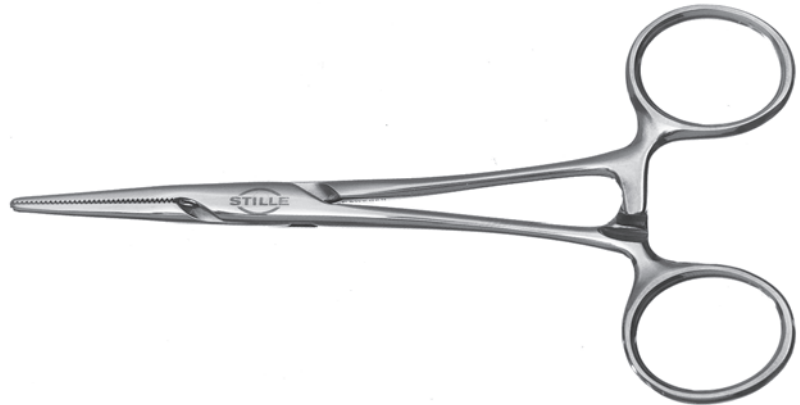

**LERICHE**

Fairly tapered hemostat. For use in small operative fields.

- \* 103-10190- Straight 15 cm 6"
- 103-10192- Curved 15 cm 6"
- 103-10195- Straight 1x2 Teeth 15 cm 6"
- 103-10197- Curved 1x2 Teeth 15 cm 6"

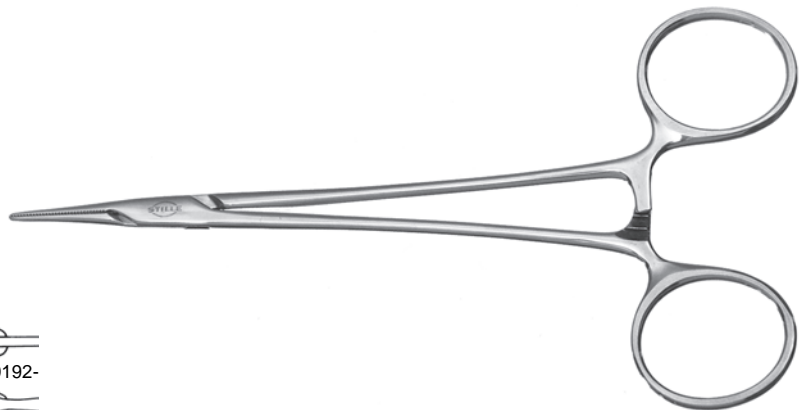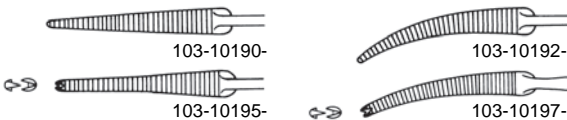

\* Photo shows this instrument

Distributed by:

ZEPF SURGICAL



800-552-3441



### ROCHESTER-PÉAN

Sturdy hemostat. Relatively blunt tips. Used in all types of surgery.



- \* 103-10200-161 Straight 16 cm 6.25"
- 103-10200-181 Straight 18 cm 7"
- 103-10202-161 Curved 16 cm 6.25"
- 103-10202-181 Curved 18 cm 7"

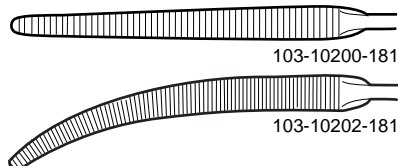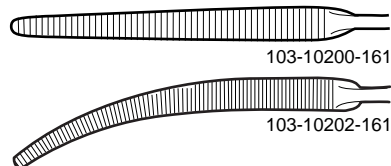

### ROCHESTER-OCHSNER

Sturdy hemostat with 1x2 teeth. Used when a strong forceps is needed, both to occlude blood vessels and to lift or retract organs.




- 103-10205-161 Straight 1x2 Teeth 16 cm 6.25"
- \* 103-10205-181 Straight 1x2 Teeth 18 cm 7"
- 103-10207-161 Curved 1x2 Teeth 16 cm 6.25"
- 103-10207-181 Curved 1x2 Teeth 18 cm 7"

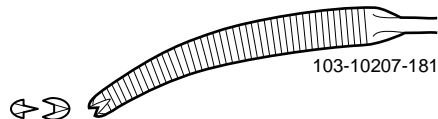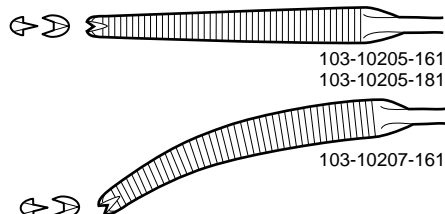

### ROCHESTER-PÉAN

Sturdy hemostats. The 20 cm has short and the 24 cm has long jaws. Also referred to as hysterectomy forceps.

- \* 103-10210-201 Straight 20 cm 8"
- 103-10210-241 Straight 24 cm 9.5"
- 103-10212-201 Curved 20 cm 8"
- 103-10212-241 Curved 24 cm 9.5"

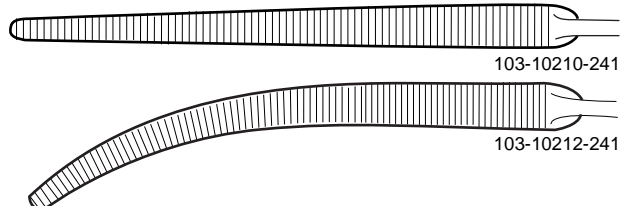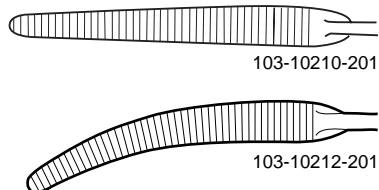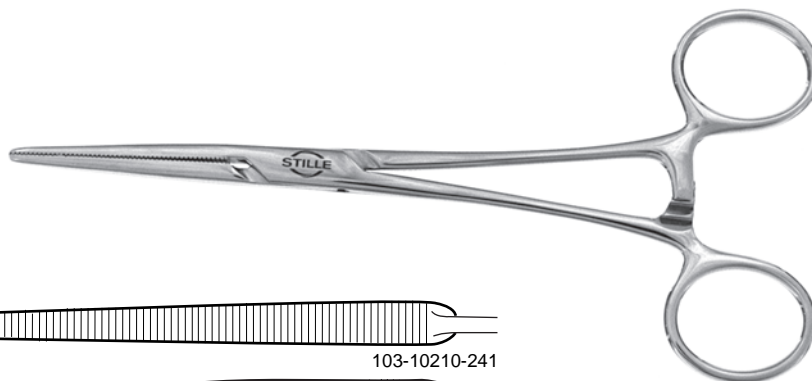





\* Photo shows this instrument

ARTERY FORCEPS

**ROCHESTER-OCHSNER**

Sturdy hemostat with 1x2 teeth. Used when a strong and long forceps is needed, both to occlude blood vessels and to lift or retract organs.


* 103-10215-201	Straight	1x2 Teeth	20 cm 8"
103-10215-241	Straight	1x2 Teeth	24 cm 9.5"
103-10217-201	Curved	1x2 Teeth	20 cm 8"
103-10217-241	Curved	1x2 Teeth	24 cm 9.5"

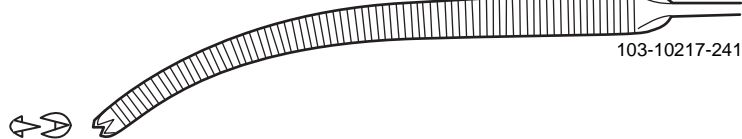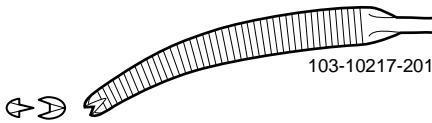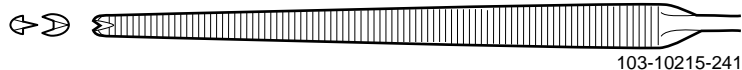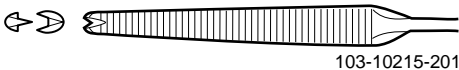

**BENGOLEA STILLE**

A long hemostat with straight or curved jaws without teeth.

103-10220-201	Straight	20 cm 8"
103-10220-241	Straight	24 cm 9.5"
* 103-10222-201	Curved	20 cm 8"
103-10222-241	Curved	24 cm 9.5"

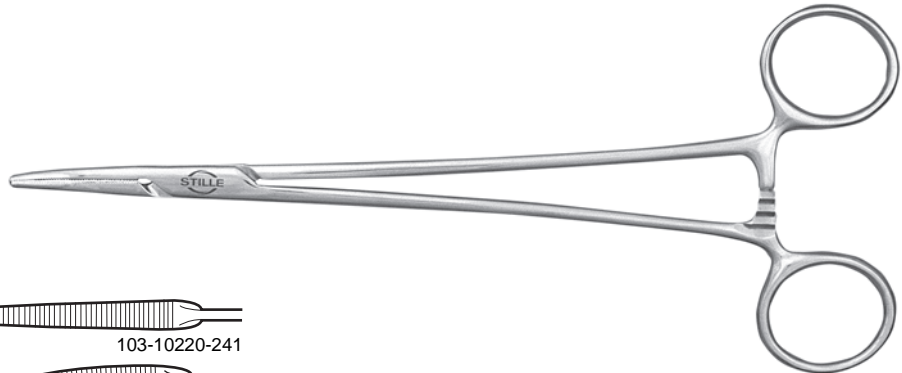

**OLIVECRONA-STILLE**

16 cm straight hemostat. Useful as a long hemostat with fine points.

* 103-10240-161	Straight	16 cm 6.25"
-----------------	----------	-------------



\* Photo shows this instrument

### DANDY-STILLE

Side curved hemostat without teeth. Often used in neuro surgery.


\* 103-10310- Side curved 14 cm 5.5"



### SCHNIDT

A hemostat popular in general surgery.




\* 103-10411-201 Curved 20 cm 8"

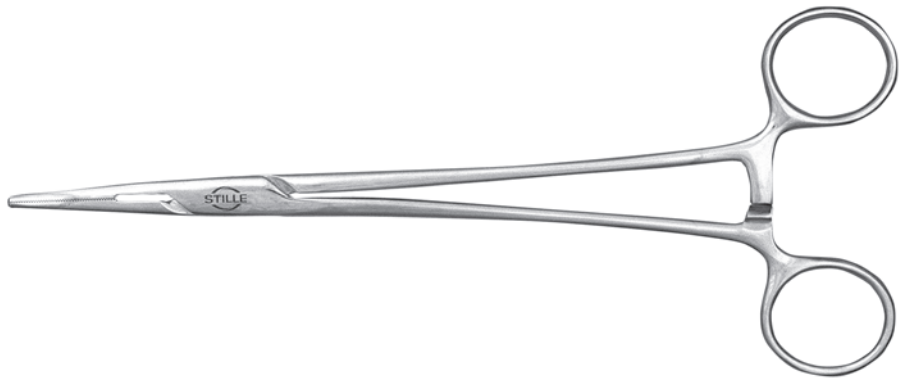

### SAWTELL

A hemostat with half-serrated and fairly strongly curved jaws.

\* 103-10413-211 Curved 21 cm 8.25"  
103-10413-241 Curved 24 cm 9.5"

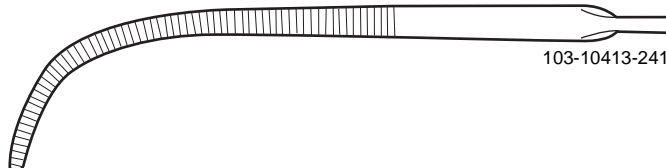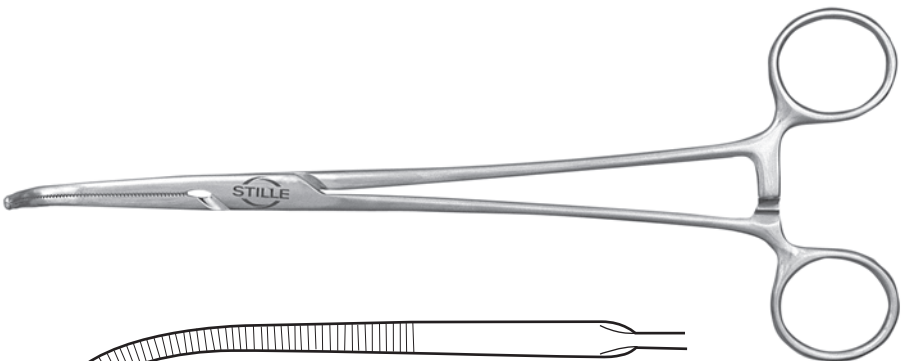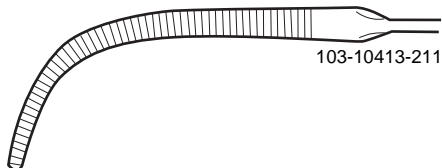

\* Photo shows this instrument



Distributed by:

ZEPF SURGICAL



800-552-3441



**CRAFOORD-STILLE**

Designed for thoracic and cardiovascular surgery. The fine tips make this a very useful instrument. Offers the surgeon Mosquito type tips, but with a length that allows working in a deep cavity.



- \* 103-10416- Curved 18 cm 7"
- 103-10430-241 Curved 24 cm 9.5"

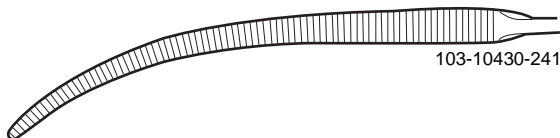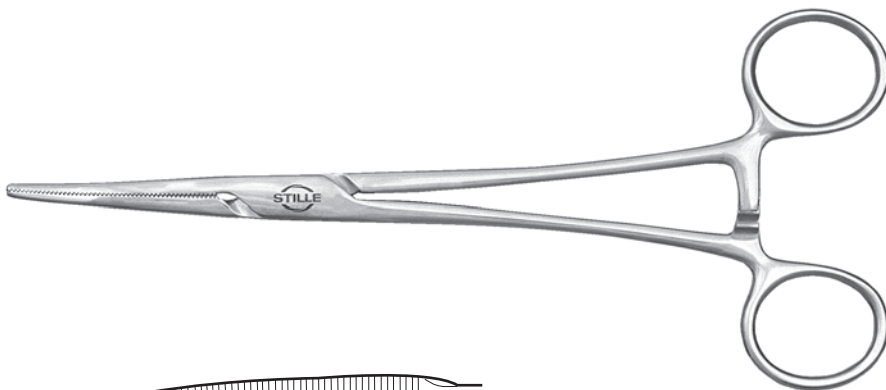

**HEISS-STILLE**

Dissecting and ligature clamps with half-serrated, fully curved jaws.






- \* 103-10420-201 Strongly curved 20 cm 8"

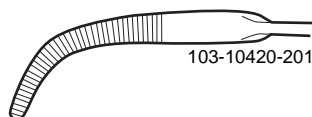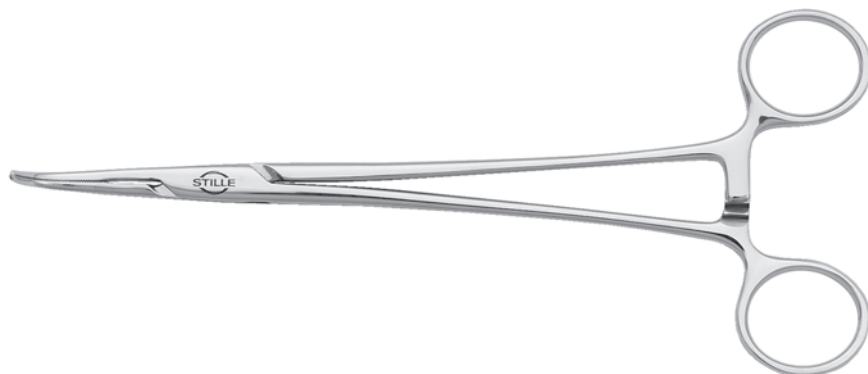

**STILLE ARTERY AND LIGATURE CLAMP**

Four models of dissecting forceps with different jaw curvature.

- \* 103-10423-1 Slightly curved 20 cm 8"
- 103-10423-2 Curved 20 cm 8"
- 103-10423-3 Strongly curved 19.5 cm 7.75"
- 103-10423-4 Angled 90 19 cm 7.5"

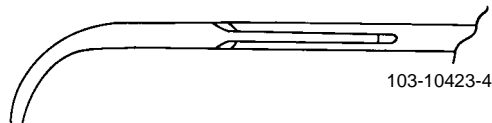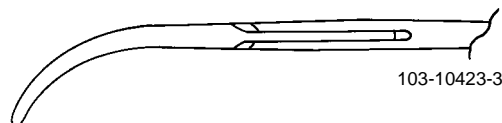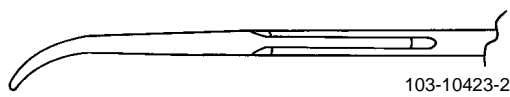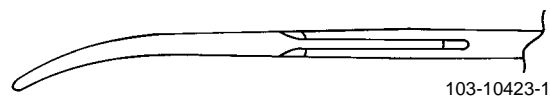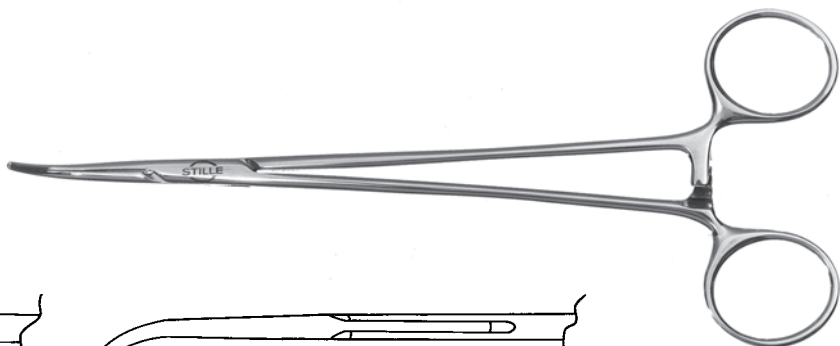

ARTERY FORCEPS

\* Photo shows this instrument



Distributed by:  
**ZEPF SURGICAL**  
800-552-3441



**OVERHOLT**

Strongly curved dissecting forceps.

\* 103-10425- Strongly curved 14 cm 5.5"

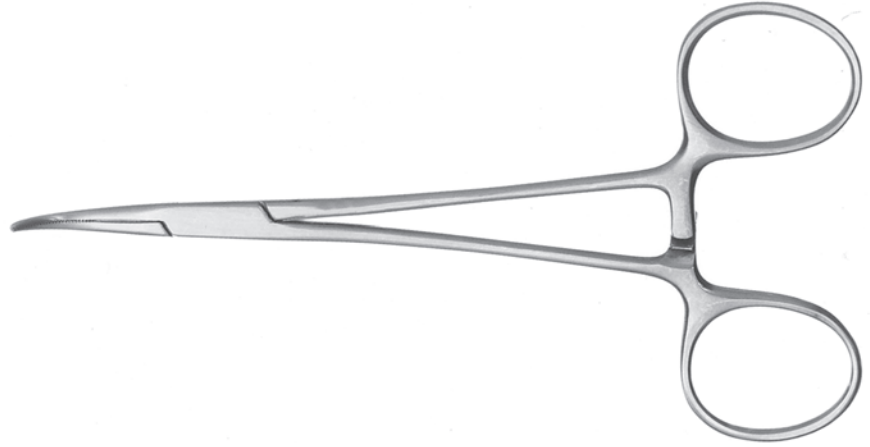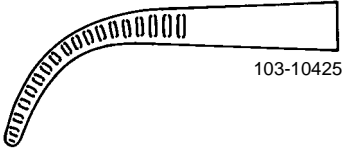

**A  
R  
T  
E  
R  
Y  
  
F  
O  
R  
C  
E  
P  
S**

\* Photo shows this instrument