

SAM ThoraSite®

**POSITION.
ALIGN.
VENT.**

DEVICE TECHNOLOGY

ThoraSite is an anatomical landmark guide, designed to provide a safer approach to the pleural space by pinpointing the procedure site. The device aids in identifying the appropriate intercostal space for lateral thoracostomies and similar procedures. ThoraSite is non-sterile and intended to be single use.



FREQUENTLY ASKED QUESTIONS

DEVICE

Q: What are the key benefits for using ThoraSite?

A: ThoraSite simplifies locating the anatomical landmarks necessary for proper thoracostomy placement.

Q: How does ThoraSite help reduce the overall procedure time?

A: ThoraSite reduces the time necessary to find proper anatomical landmark for thoracostomy placement.

Q: Is there a competitive product on the market?

A: Currently SAM ThoraSite has no direct competitor.

Q: Is ThoraSite right or left specific?

A: No, the single device design accommodates procedures on both the patient's left and right side.

Q: Is ThoraSite Radiolucent?

A: Yes, ThoraSite is composed of materials compatible with X-ray imaging, eliminating the need for guide removal.



FREQUENTLY ASKED QUESTIONS

PROCEDURE AND USE

Q: How does SAM ThoraSite aid with identifying the procedure site?

A: The patented design references the patient's own anatomical landmarks to position the Safe Zone Window directly over the ideal procedure site.

Q: How does SAM ThoraSite aid with accurate placement?

A: Despite differences in patient's general anatomy, ThoraSite accurately orientates the Safe Zone Window over their 3rd, 4th, and 5th intercostal spaces, the proper thoracostomy location.

Q: What body size range can ThoraSite be used on?

A: ThoraSite is designed to accommodate a large range of patient's sizes. The device is intended for adult and adolescent use. It is not recommended for use in pediatric or neonatal patients.

Q: Can ThoraSite be reused.

A: No. ThoraSite is a single use device. Reuse of device may cause illness or injury.

Q: Can ThoraSite be used with an anterior or midclavicular approach?

A: No, ThoraSite is only indicated for the lateral thoracostomies.


Q: What is ThoraSite's Indication for Use?

A: ThoraSite is indicated for use by trained professionals following their standard of care to treat patients with pneumothorax, hemothorax and similar conditions requiring a thoracostomy.

Q: What is ThoraSite's Intended Use?

A: ThoraSite's intended purpose and use is as an anatomical location aid to identify an appropriate location for lateral thoracostomies in adult and adolescent patients. Not intended for pediatric, or neonatal use.

PRODUCT SPECIFICATIONS

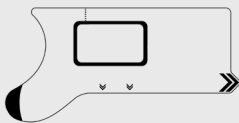

MODELS AVAILABLE


SAM ThoraSite
5-PACK (TS200-5P-EN)
25-PACK (TS200-25P-EN)

PACKAGED SIZE

Height: 2.7 in. (68.71 mm)
Width: 0.04 in (1.02 mm)
Length: 5.6 in. (141.04 mm)
Weight: 0.18 oz (5.1 g)

GLOBAL SALES

sammedical.store
sammedical.com
+1.503.639.5474
+1.800.818.4726 | U.S. Toll Free

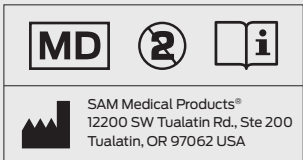

US Patent № 10,595,898

SAM® is a registered trademark in the United States and other countries.
©2020 SAM Medical Products. All rights reserved.