# EmeTerm<sup>®</sup> Anti-emetic Device

INSTRUCTIONS FOR USE

## Note

- Before first use, charge the battery to over 60% capacity.
- Please clean the skin before wearing. The device should be applied to the correct position (P6).
- The time of use varies according to individual situation and users could adjust the intensity of the device according to the feeling of the symptoms.
- Under low temperature and low moisture climate (e.g. winter), it is recommended to start the treatment at least 10 minutes before experiencing nausea.


Video operation introduction

[https://youtu.be/IGEt\\_n5uDVE](https://youtu.be/IGEt_n5uDVE)

## Quick Start Guide

### I. Device features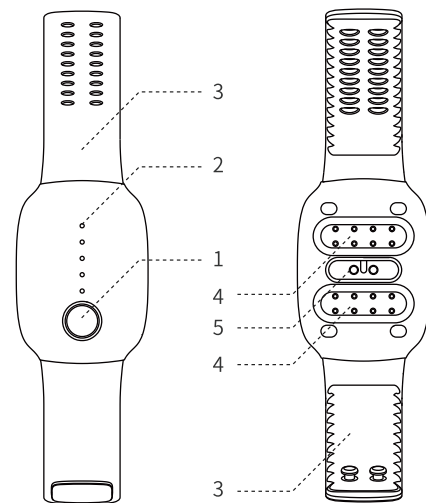

1. Control button
2. Indicator lights
3. Wristband
4. Electrodes
5. Charging port


Innovation Self-Care Medical Device Supplier


[www.watmedical.com](http://www.watmedical.com)

## II. Battery charging

- 1 Connect the magnetic connector to the charging port.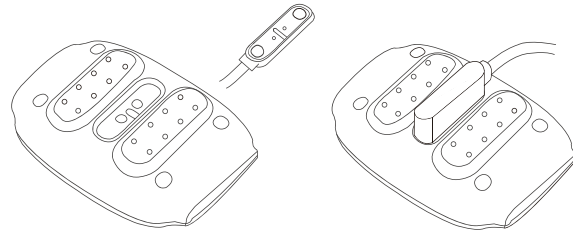

- 2 Display the capacity of battery.

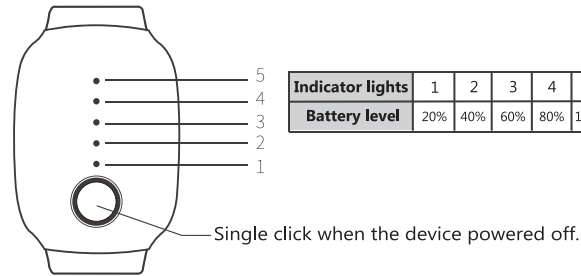
## III. Wearing instructions

1. Position (P6): 2-3cm away from the transverse wrist lines, with a width of about 2 fingers.


### Attention:

The device should be applied to clean and healthy skin.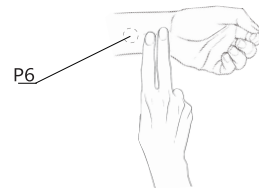

- 2 Wear the device.

Ensure skin is sufficiently in contact with electrodes.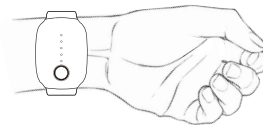


### Attention:

The device should be applied to the correct position (P6).

## IV. Operating instructions

Turn on	Press the button for 3 seconds
Turn off	Press the button for 3 seconds
Increase	Single click for 1 gear increase
Decrease	Double click for 1 gear decrease

## V. Operating principles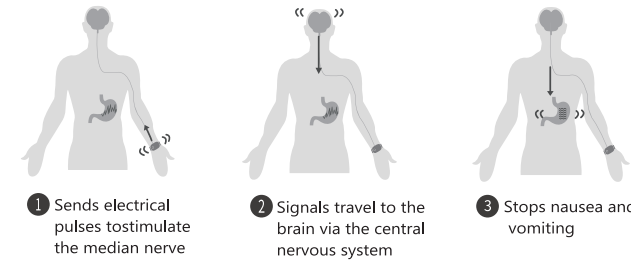

## Manual Instruction

【Name of Product】  
TENS device-EmeTerm

【Specification and Model】  
YF-ZTY-E1

### 【Packing Specification】

The packing specification for each model is one piece per case. The product configuration list is shown in the following table:

Name of Accessory	Quantity
Main unit	One set
Wristband	One set
Instruction files	One set

### 【Product Structure】

1.1. Structure of the product:  
Main unit, rubber wristband.

No	Name
1	Electrodes
2	Wristband
3	Control button
4	Front side of host
5	Buckle

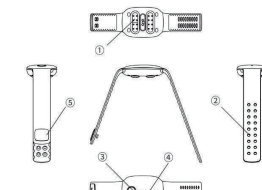
Figure 1 Structure Drawing for Device

Please use CE certificated adapter and charging line.

### 2. Technical Parameters of Product.

Output	Parameters				
Gear range	1	2	3	4	5
Voltage	22.8V	28.8V	32.4V	34.0V	36V
Range of amplitude	-39.5V ~ +39.5V (±3V)				
Oscillogram	Mode		Pulse width	Frequency	
	Orthogonal pulse sharp-wave		100μs (±10%)	33Hz (±3)	

3. Battery information: 3.7V DC. 140mAh Lithium battery

### 【Range of Application】

TENS device-EmeTerm is a wearable device for the treatment of nausea and vomiting induced by:

- Medicine including: Chemotherapy, general anesthesia and morphine.
- Motion sickness including: Sea-sickness, car-sickness, airsickness, altitude stress and VR dizziness

### 【Usage】

**Wearing:** Place the contact plate (two metal electrodes) of the device at the center of the inner wrist, about 2 to 3cm from the wrist crease. Please ensure the control button is close to your body side, and then fasten the wrist wrap at a suitable tightness.

**Starting up:** Press the control button for 3 seconds to start up the device, and then the display lights for the first gear to the fifth gear will be on in turn and at last stay at the first gear. If the button light and the gear light flash intermittently at the same time, it means the product is working normally..

**Gear range:** The product has five gears (1, 2, 3, 4, 5 indicating the function goes from weak to strong) which are used to adjust the intensity of function. Users can select their suitable gears.

1. A quick press on the control switch will increase one gear. Users can know the current gear through the display light (For instance, the first three display lights and the button light will be on if the device is at the third gear).
2. Two quick presses on the control switch will decrease one gear. Users can know the current gear through the display light.

**Power off:** Press the control switch for 3 seconds, and the display lights for the first gear to the fifth gear will be on in turn and then go out, indicating that the device is powered off. State of charge: 1. A short press (<1 second) on the control switch under the power-off state will enable the gear display light to display the current state of charge of device. The first gear to the fifth gear indicate that the state of charge goes from 20% to 100%. 2. Reminder of too-low power when the product is working: the product will be power-off automatically after the button light and the gear light flash for 15s at the same time.

**Charging:** Charge the device with the attached USB cable which can be connected to the attached charger or the computer's USB port. While in charging, the device's charging indicator light is on while the gear display light displays the progress of charging (after it is fully charged, all of the five gear lights will be on).

### 【Classification de Contact】

Electrical Safety Applied Part is Type BF.

### 【Duration of Contact】

The maximum time of contact per use would depend on the capacity of the rechargeable battery, the rechargeable battery will only last 7 hours maximum with full charged battery, and typically the patient will remove the device when their symptoms are gone, which the minimum duration of contact would be determined by different individual to maximum of 7 hours.

### 【Contraindication】

1. Patients who suffer from arrhythmia and epilepsy;
2. People who have metal piece or cardiac pacemaker within their bodies;
3. People who have local acute inflammation, local hemorrhagic tendency, local skin ulceration, local allergy, local bleeding, local infections and other types of local open wounds;
4. Please consult your physician if you have any other contraindications.

### 【Warning】

1. Please use the device by the instruction manual in order to ensure a safe use.
2. The patient is not allowed to use the product together with a high-frequency device. It is forbidden to put this device and a high-frequency device in the same treatment room.
3. It is not allowed to put the electrode on the unspecified part for use.
4. It is normal that a slight tingling will develop on the wrist while wearing the device. For users who have pruritus and tingling, they may wear the device on the other wrist. Generally, the pruritus and tingling will disappear in 24 hours.
5. It is not suitable for patients to adjust the device at a high gear in their first use of the device because they have not got prepared psychologically. After 2 to 3 times of treatment, patients can have gear up gradually.

### 【Maintenance】

1. Keep the device clean, and use it following the usage steps.
2. Put the device in a dry and well-ventilated place. Do not put it under the window or beside the heating installation to protect it from heating and dampness.
3. It is not allowed to expose the device to environment with strong electromagnetic interference. Please turn off the power after use.

### 【Disposal】

Dispose device according to your local state/country laws.

### 【Operating Conditions】

Normal operating conditions should comply with the following requirements:  
Environment temperature: 5 C ~ 40 C  
Relative humidity: 10 ~ 80%  
Atmospheric pressure: 86 ~ 106kPa

### 【Storage】

The product should be stored in accordance with the following requirements:  
Environment temperature: -20 C ~ +55 C  
Relative humidity: 10 ~ 80%  
Atmospheric pressure: 50 ~ 106kPa  
Indoor, dry and well-ventilated, free from corrosive substances Great pressure is not allowed to be put on the product.

### 【Labels and packaging marks】

No	Symbol	Meaning
1		Manufacturing date of product
2		Serial number of product
3		Note! Please check all documents attached
4		Informations du fabricant
5		Information of EU authorized representative
6		The product has passed CE certification
7		BF type
8		The product contains battery, so please don't throw it away freely.
9		Refer to the Instruction Manual
10		The waterproof and dustproof grades of this product reach the standard

### 【Date of Manufacturing】

See the packaging.

### 【Manufacturing Enterprise and After-sale Service Agency】

Manufacturer: WAT Medical Technology Inc.  
Add.: Room703-711, No.2 North Taoyuan Road, 315600, Ningbo, Zhejiang Province, P.R.C  
Tel.: 86-574-65060811  
Fax: 86-574-65060815  
Email: watmed@hotmail.com

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Fabricant : WAT Medical Technology Inc.