

**Sizing chart for T.E.D.™ Anti-Embolism Stockings,
Kendall SCD™ Compression Sleeves and A-V Impulse™ Foot Compression System**

Help reduce the risk of deep vein thrombosis

T.E.D.™ Anti-Embolism Stockings All stockings are latex-free

Measuring instructions for thigh length stockings	Thigh length					
	1. Thigh circumference	2. Calf circumference	3. Length			
			Short <29 in.	Medium 29–33 in.	Long >33 in.	
1. Measure upper thigh circumference at buttock fold.	<25 in.	Small	<12 in.	3071LF-	3130LF	3222LF
2. Measure calf circumference at greatest point.		Medium	12–15 in.	3310LF	3416LF-	3549LF
		Large	15–17½ in.	3634LF	3728LF-	3856LF
3. Measure length from heel to buttock fold.	25–32 in.	X-Large	17½–21½ in.	3180LF	3181LF	3182LF
	32–36 in.	XX-Large	21½–26 in.	3183LF	3184LF	3185LF

Measuring instructions for knee length stockings	Knee length*					
	1. Calf circumference	2. Length				
		Regular	Long			
1. Measure calf circumference at greatest point.	Small	<12 in.	7071	<16 in.	7339	>16 in.
2. Measure length from back of heel to bend in the knee.	Medium	12–15 in.	7115-	<17 in.	7480-	>17 in.
	Large	15–17½ in.	7203-	<18 in.	7594	>18 in.
	X-Large	17½–20 in.	7604	<18 in.	7802	>18 in.
	XX-Large	20–23 in.	7470LF	<18 in.	7471LF	>18 in.
	XXX-Large	23–26 in.	7472LF	<18 in.	7473LF	>18 in.

*Also available in beige, black and white with closed toe for the recuperating patient. Please contact 800.964.5227 for information regarding these additional stockings.

Kendall SCD™ Compression Sleeves

Measuring instructions for compression sleeves	Thigh length					
	1. Thigh circumference	Size	Kendall SCD™ Smart Compression™ Comfort Sleeve	Kendall SCD™ Smart Compression™ Comfort Sleeve tear-away	Kendall SCD™ Smart Compression™ Express Sleeve	Kendall SCD™ Smart Compression™ Express Sleeve tear-away
1. For thigh length: Measure upper thigh circumference at buttock fold.	<16 in.	X-Small	74010-*	N/A	N/A	N/A
	<22 in.	Small	74011*	74041*	9545-	9545T
	<28 in.	Medium	74012-*	74042*	9530	9530T
	<36 in.	Large	74013*	74043*	9780-*	9780T*
	<28 in.	Sterile	N/A	N/A	9736	N/A

2. Calf circumference	Size	Kendall SCD™ Smart Compression™ Comfort Sleeve	Kendall SCD™ Smart Compression™ Express Sleeve	Kendall SCD™ Smart Compression™ Express Foot Cuffs		
				Women's shoe size (U.S.)	Men's shoe size (U.S.)	Item code
<14 in.	Small	74021*	N/A	<9	<7½	5897 Medium
<21 in.	Medium	74022*	9529-	>9½	>8	5898 Large
<26 in.	Large	74023*	9789-*			
<32 in.	X-Large	N/A	9790*			

*Indicated for use in the bariatric and morbidly obese patient population.

A-V Impulse™ Foot Compression System

ImPad™ Rigid Sole Foot Cover					Sterile ImPad™ Rigid Sole Foot Cover	
Women's shoe size (U.S.)	Men's shoe size (U.S.)	Foot circumference	Size (in pairs, left and right)	Item code	One size fits all	Item code
5½–9	4½–8	<12 in.	Medium	5065-	<9	6066
9½–13½	8½–12½	<13 in.	Large	5075-	>9½	6067-
9½–14½	8½–13½	13–18 in.	X-Large	5107-		

For more information on products to help reduce the risk of deep vein thrombosis visit cardinalhealth.com/getsmartcompression

