

ASM

200

User Manual



Content

Components	3
-------------------	---

H/W Parts

Device Functions	4
Device Settings	5

System Settings



Date & Time	6
Wi-Fi	7

Application


Application Settings	8
Customer	9
Hair Loss Status	10
Scalp Status	11
Hair Density	12
Keratin of Scalp	13
Scalp Sensitivity	14
Hair Thickness	15
Hair Pore Status	16
Cuticle Status	17
Summary	18
Send Result -Email	19
Mirroring	20
View Mode	22

Warranty	23
-----------------	----


Components

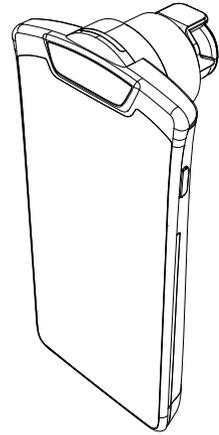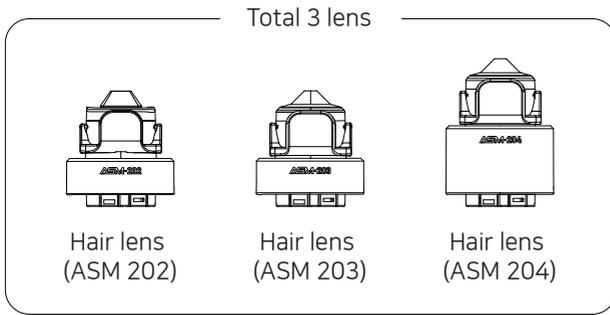


Main unit with hair lens (ASM 200)

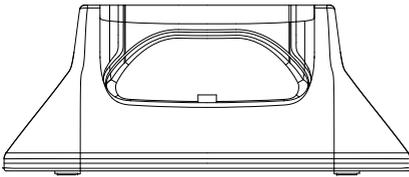

Cradle

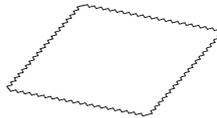


Cleaning cloth

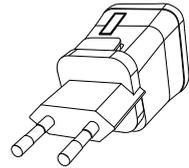


Charger



Manual


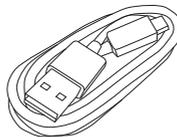


USB cable

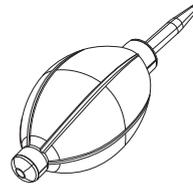

Air blower


Device Functions

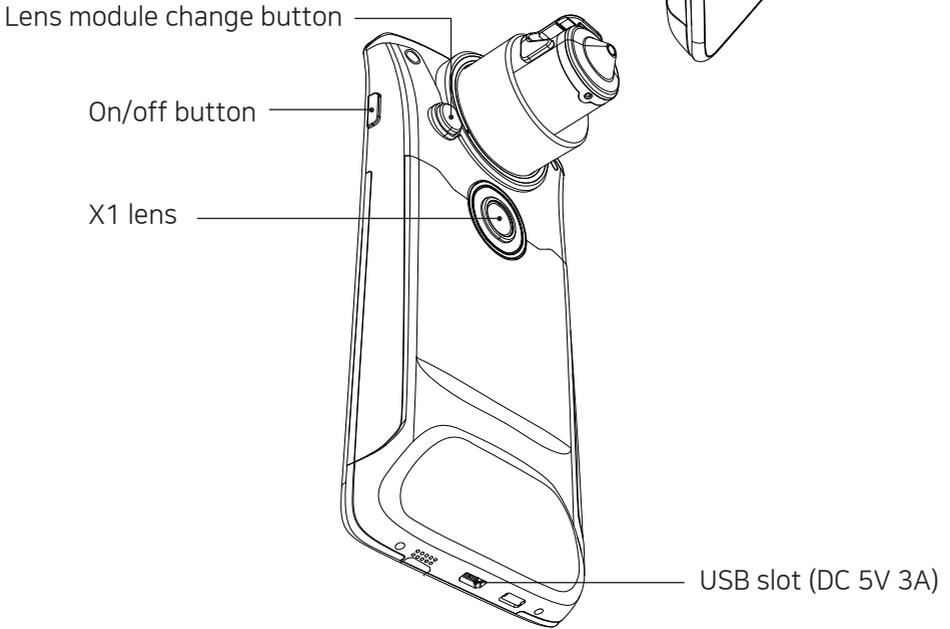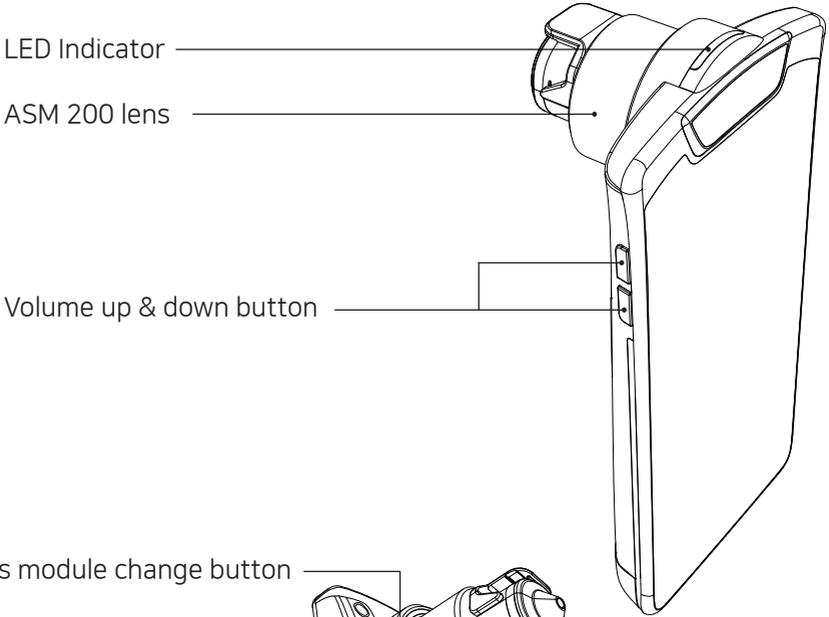

Device Settings


To turn on & off the power

- ① Press and hold "Power" button.
- ② Then "LED Indicator" will be twinkle and turn to white.

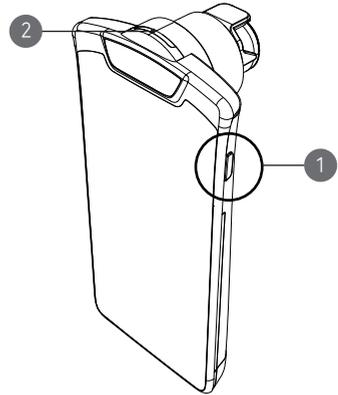


Charger can be used for American type & European type. The default is European type.


- ① Push the plug up.

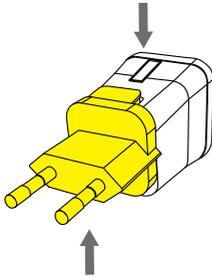

- ② Then European type is disassembled and you can see the American type inside of the charger.

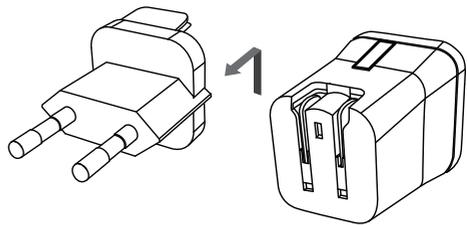

- ③ Choose the one you want to use and connect USB cable in adaptor.

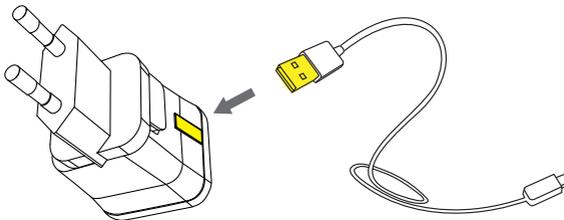


Date & Time




Touch  button.



Touch  button.

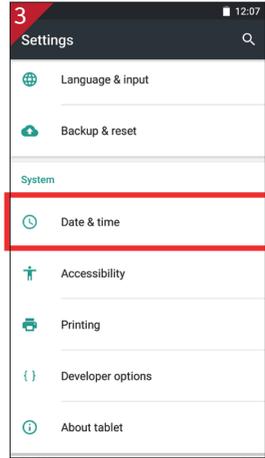


Go to "Date & time"

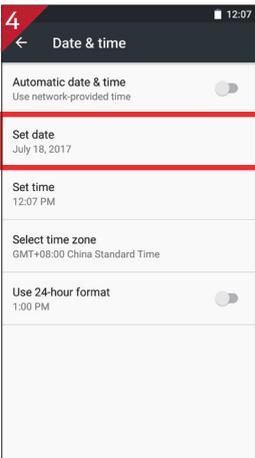

Go to "Set date".



Select the date and touch "OK" button.

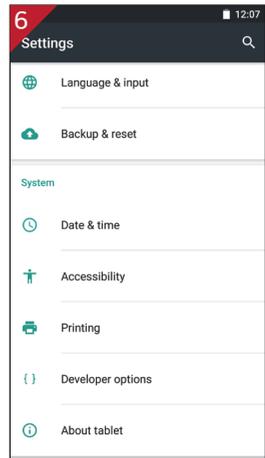


Check if the setting is correctly applied.


Wi-Fi




Touch  button.

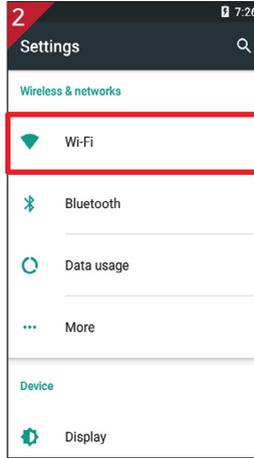


Touch "Wi-Fi".

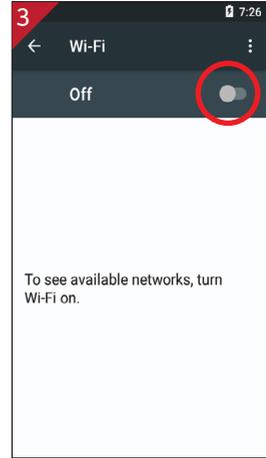


Turn on toggle switch.

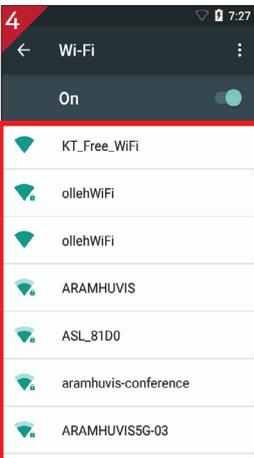

Wi-Fi list will appear.

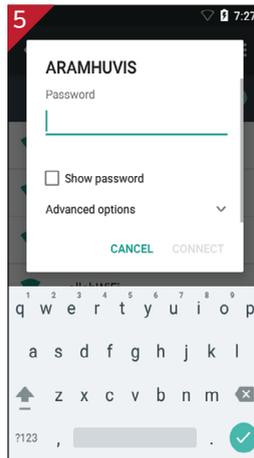

Among the lists, select the Wi-Fi connection you could use.

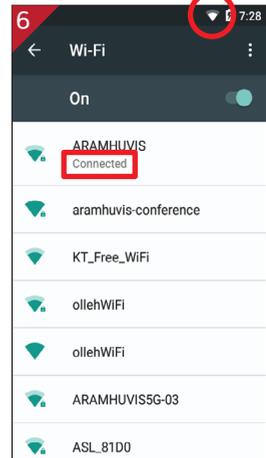


Wait until the device is connected to Wi-Fi.



Application Settings



Touch  button.

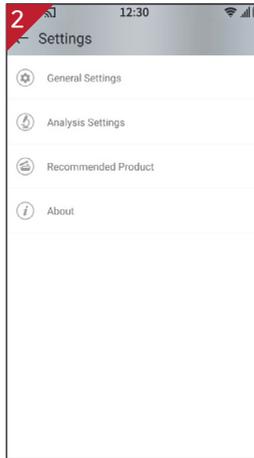


Go to "General Settings".

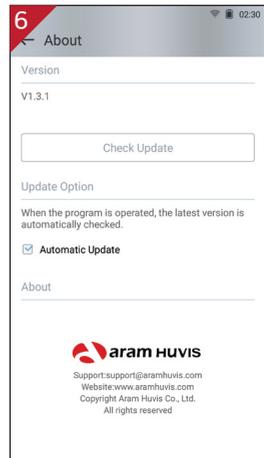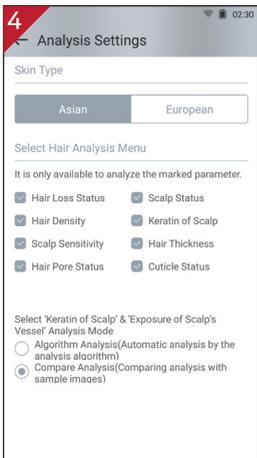


Customer




Touch  button.

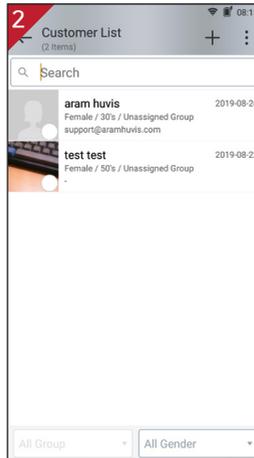


Touch  to register new customer.

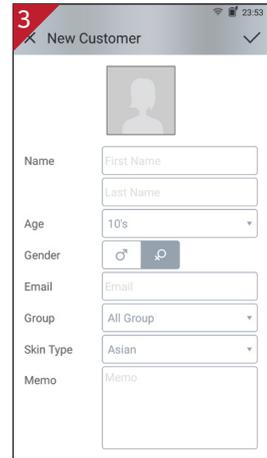

Input the information.

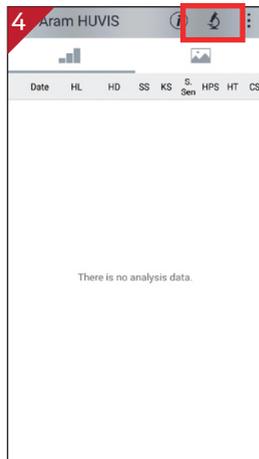

Touch  button to start analyzing.

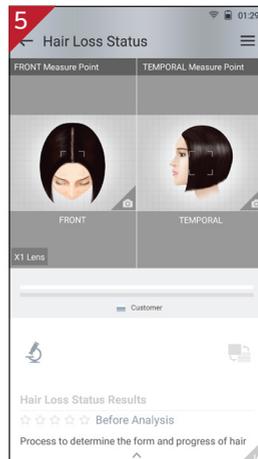


Touch  button to activate camera mode.




Hair Loss Status



Touch  button.

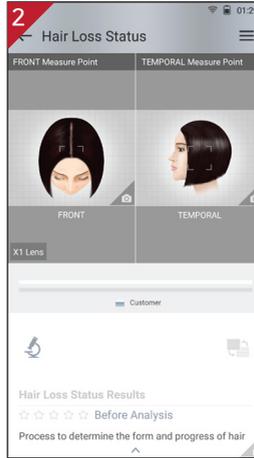


Touch  button to activate camera mode.

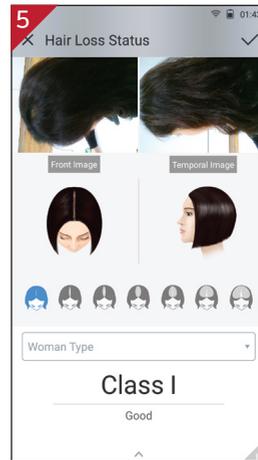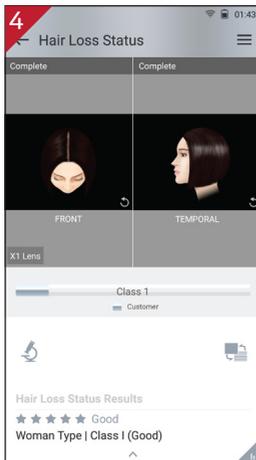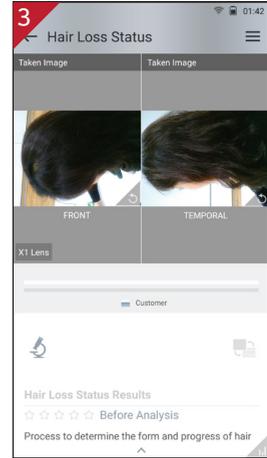


Then select the type which is similar with your hair loss shape.

Scalp Status

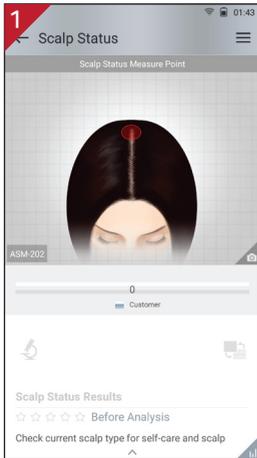

Touch  button to activate camera mode.

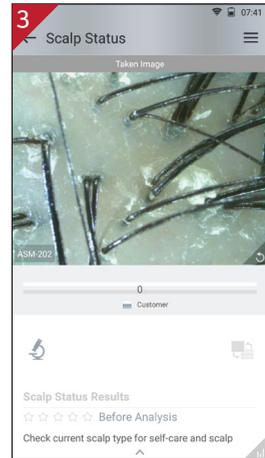

Put the camera onto the hair and touch  button to capture the image.




Touch "Analysis" to check analysis result.




Among the samples, choose the one which is similar with your image.


Hair Density

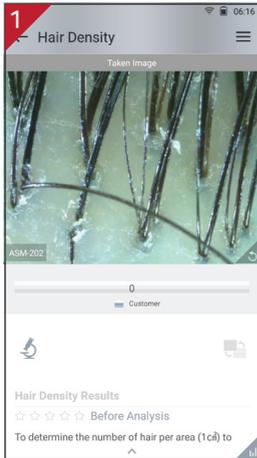

Touch "Analysis" to check analysis result.

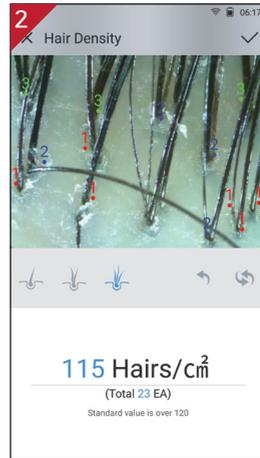


Count the hair number in the image.

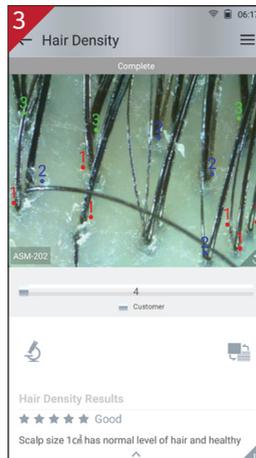


Check the result.
And swipe to next step.


Keratin of Scalp

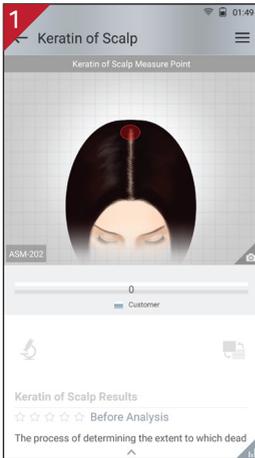


Touch  button to activate camera mode.

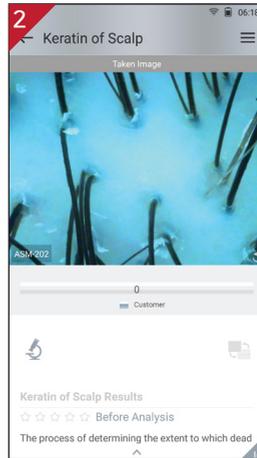

Put the camera onto the scalp and touch  button to capture the image.

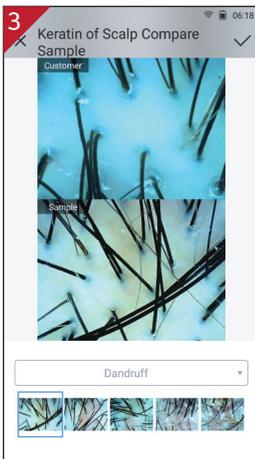


Touch "Analysis" to check analysis result.

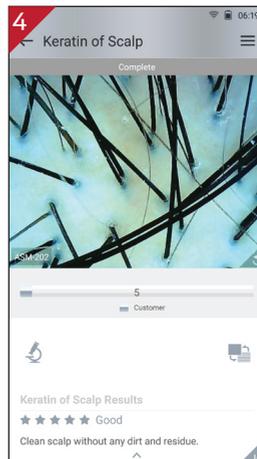


Touch "Analysis" to check analysis result.

Scalp Sensitivity

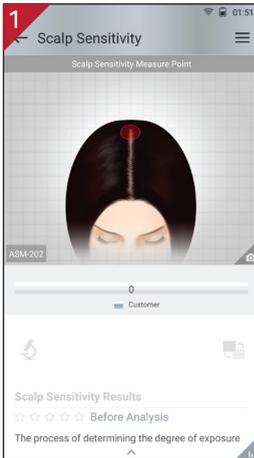


Touch  button to activate camera mode.

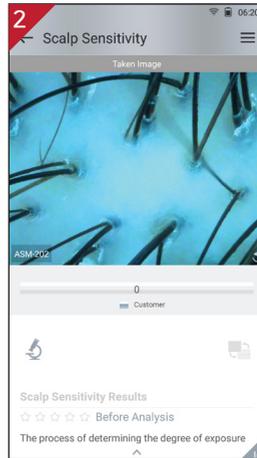

Put the camera onto the scalp and touch  button to capture the image.




Touch "Analysis" to check analysis result.

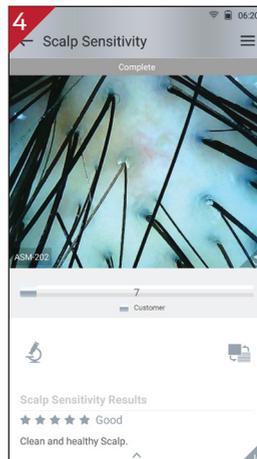

Touch "Analysis" to check analysis result.


Hair Thickness

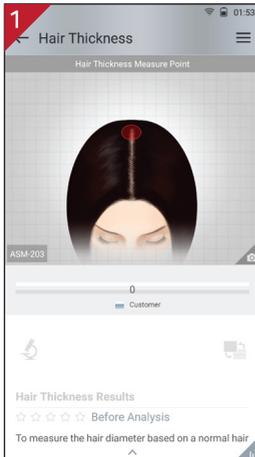


Touch  button to activate camera mode.



Put the camera onto the scalp and touch  button to capture the image.

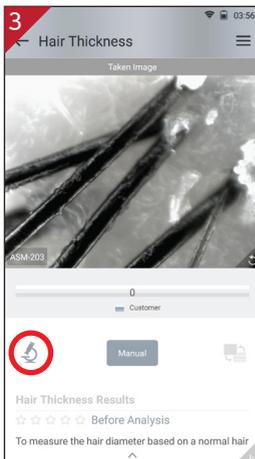


Touch "Analysis" to check analysis result.

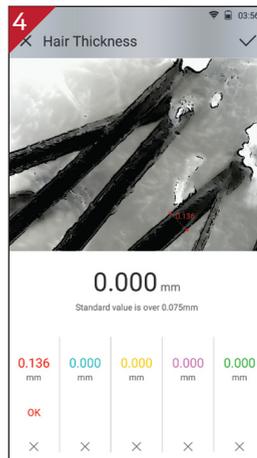


Choose one hair and touch little bit down side of the hair. Then touch little bit upside of the hair. It will make a green line to measure the thickness. (If the line is not correct, please try again)


Hair Pore Status

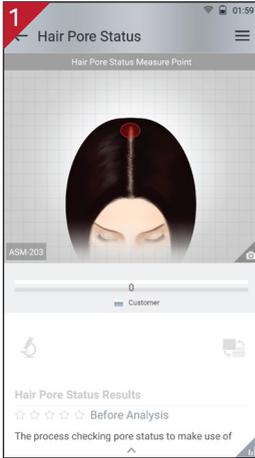


Touch  button to activate camera mode.

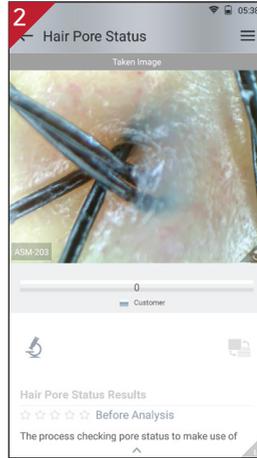

Touch "Analysis" to check analysis result.

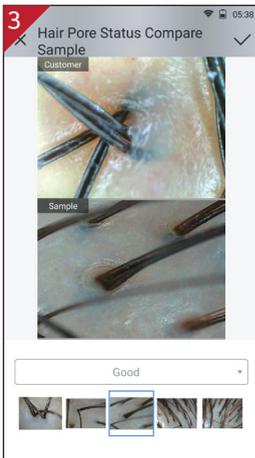


Among the samples, choose the one which is similar with your image.

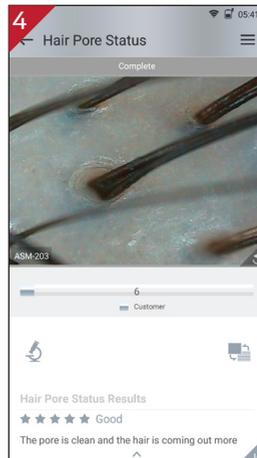


Check the result.
And swipe to next step.


Cuticle Status

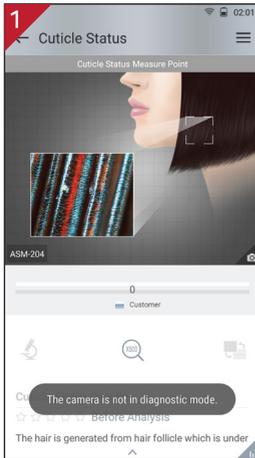


Touch  button to activate camera mode.

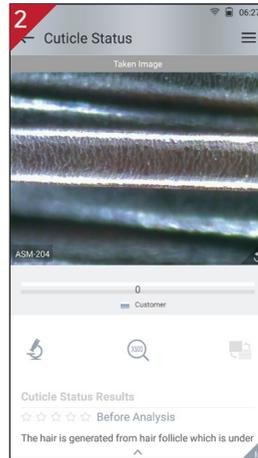

Touch "Analysis" to check analysis result.

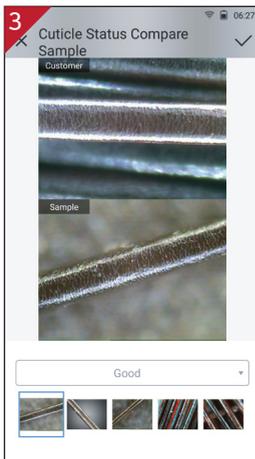


Among the samples, choose the one which is similar with your image.

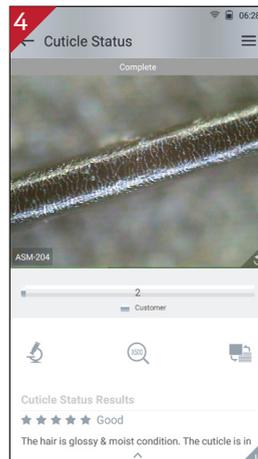


Check the result. And swipe to next step.


Summary

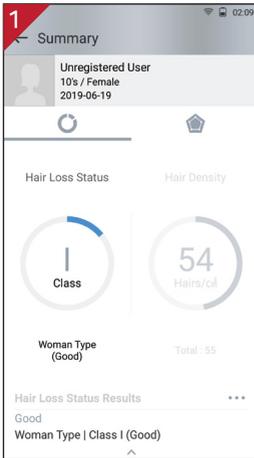


The results will be shown in two pages.

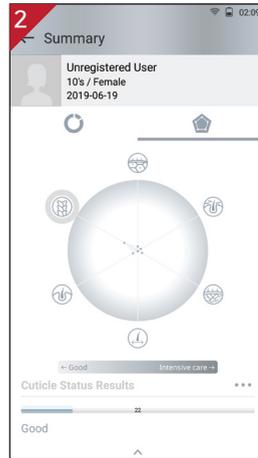

On second page, the description of each parameter would be available by clicking ^ icon.

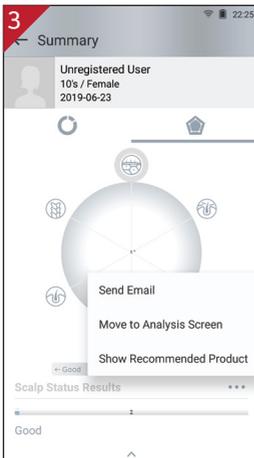


Once you select "Show Recommended Product", product image will pop up as 3.

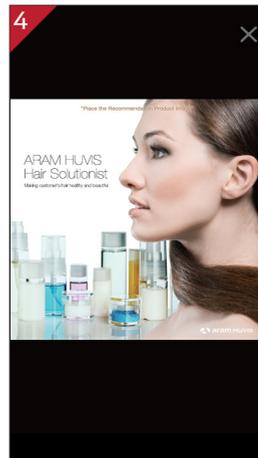


Available to check product image.


Send Result - Email

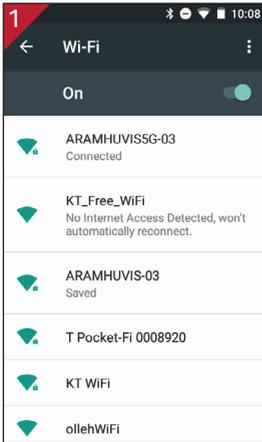


Before sending Email, the device shall be connected to Wi-Fi.

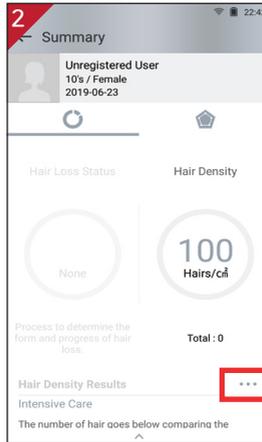


Touch **:** button.

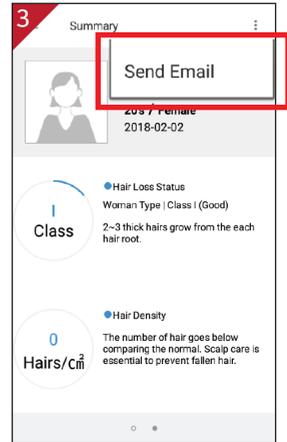


Go to "Send Email".

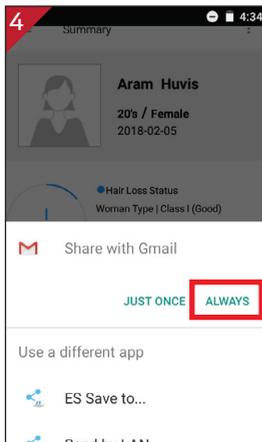

Select "Gmail" and "always".

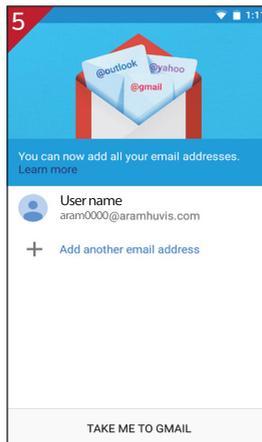

Select the mail account to register on the list.

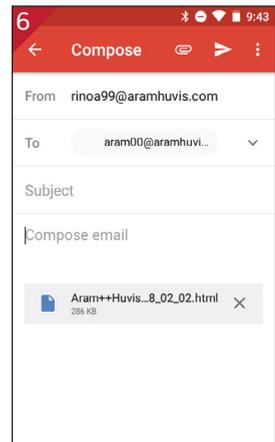


After selected sender's mail address, put receiver's mail address. Then touch "Send" button.

Mirroring



Go to "Settings".

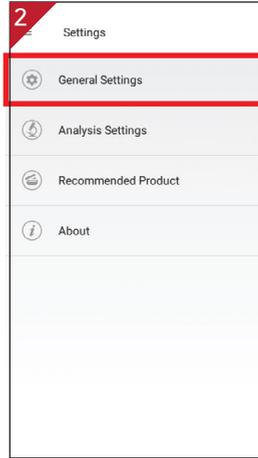

Go to "General Settings".




Set the mirroring option "On".



Capture QR code using QR code reader app. Automatically moved to App store.

Mirroring

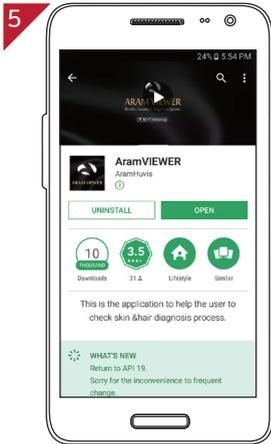

Install "Aram VIEWER".



Then, go to "Wi-Fi Setting".



In your mobile device go to Wi-Fi setting. Then find SSID and connect it.




Mirroring will be started.


View Mode




Touch  button.

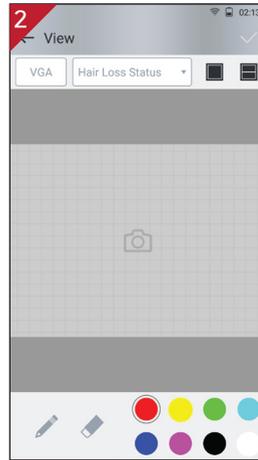

In view mode, it is able to take pictures with capture button.

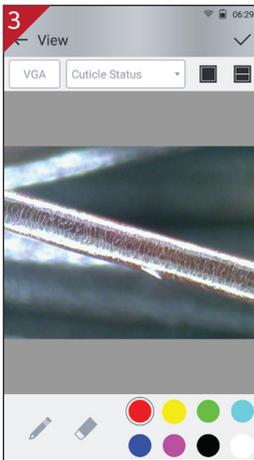

The user can mark the defect or concern according to the image with note function.

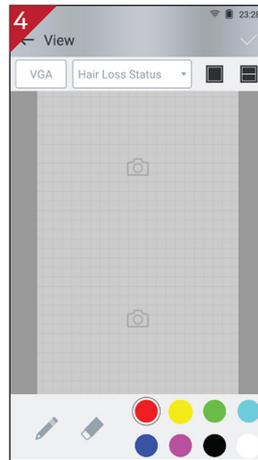

The user can select analysis parameter to take pictures in proper mode.

Warranty

- ① The product undergoes strict quality management and inspection process before production and introduction to market.
- ② In case of functional or operational defect occurs under normal condition of use within 10 days after purchase, we will repair it for free or replace in new product.
- ③ When this product goes out under the normal use, we will repair it free of charge under warranty.
- ④ Please keep the warranty card when you ask for repair service.
- ⑤ Customer needs to pay repair charge after warranty runs out.
- ⑥ For this product, warranty period is 14 months from the Invoice date.
- ⑦ Cost
Within warranty : Sending delivery(Buyer), repair cost and returning delivery(Aram)
 - If the product is broken by user's fault, repair cost could be charged.
 - After warranty : Sending delivery and repair cost(Buyer), returning delivery(Aram)
- ⑧ How to send :
 - As sending a present from person to person by EMS or FEDEX.
(Do not write the company name)
 - Total value should be under US \$ 99.(Write non-commercial value)

Warranty

Service request form			
Date of purchase	/	/	Company
Contact person			
Address :			
Tel :			
Product name, serial No :			
Problems :			

SHIPPING ADDRESS :

#801 Byucksan Technopia, 560, Dunchon-daero, Jungwon-gu, Seongnam-si,
Gyeonggi-do, 13230 Republic of Korea

Tel: 070-4687-1712 / Fax: 031-732-6330 / Email: support@aramhuvivis.com
<http://www.aramhuvivis.com>

Memo

Memo

Memo



HEAD OFFICE

#401~402 Bundang Seoul National University Hospital Health Care Innovation Park, 172, Dolma-ro, Bundang-gu, Seongnam-si, Gyeonggi-do, 13605 Republic of Korea

PRODUCTION DIVISION & FACTORY

#909 Byucksan Technopia, 560, Dunchon-daero, Jungwon-gu, Seongnam-si, Gyeonggi-do, 13230 Republic of Korea

Tel: 070-4687-1712 / Fax: 031-732-6330 / Email: info@aramhuvis.com
<http://www.aramhuvis.com>