

ASM

100

User Manual




Contents

Components	3
<hr/>	
H/W Parts	
i. Both Sides	4
ii. Device Settings	5
<hr/>	
System Settings	
i. Date & Time	7
ii. Wi-Fi	9
iii. Sleep	10
iv. Language	11
<hr/>	
Application	
i. Settings	12
ii. Full Analysis	
CRM	13
Moisture	14
Sebum	15
Pore	16
Melanin	17
Acne	18
Wrinkle	19
Sensitivity	20
Summary	21
i. Email	22
ii. Mirroring	23
iii. View Mode	25
<hr/>	
Warranty	26


※ Notes on use

- The images of this manual can be different with the actual things.
- The information contained in this manual and future improvements are subject to change without notice.
- The latest version of the manual will be updated at the below link.
<http://www.aramhuvis.com/downloads>


Components

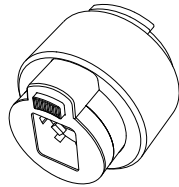

Skin lens (ASM-100)

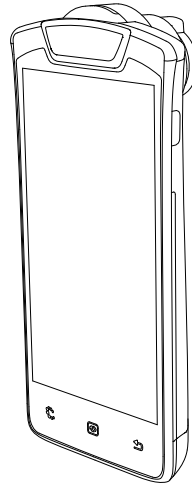


Main unit with skin lens

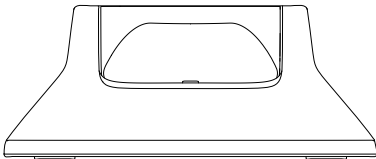


Cradle




Sebum paper
(100ea)

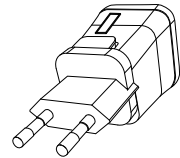


Charger

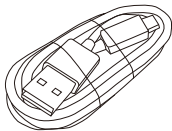


USB cable

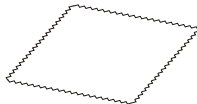

Cleaning cloth

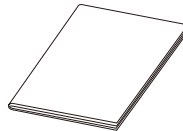


Manual

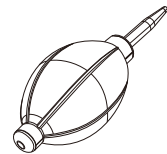


Air blower

Both Sides

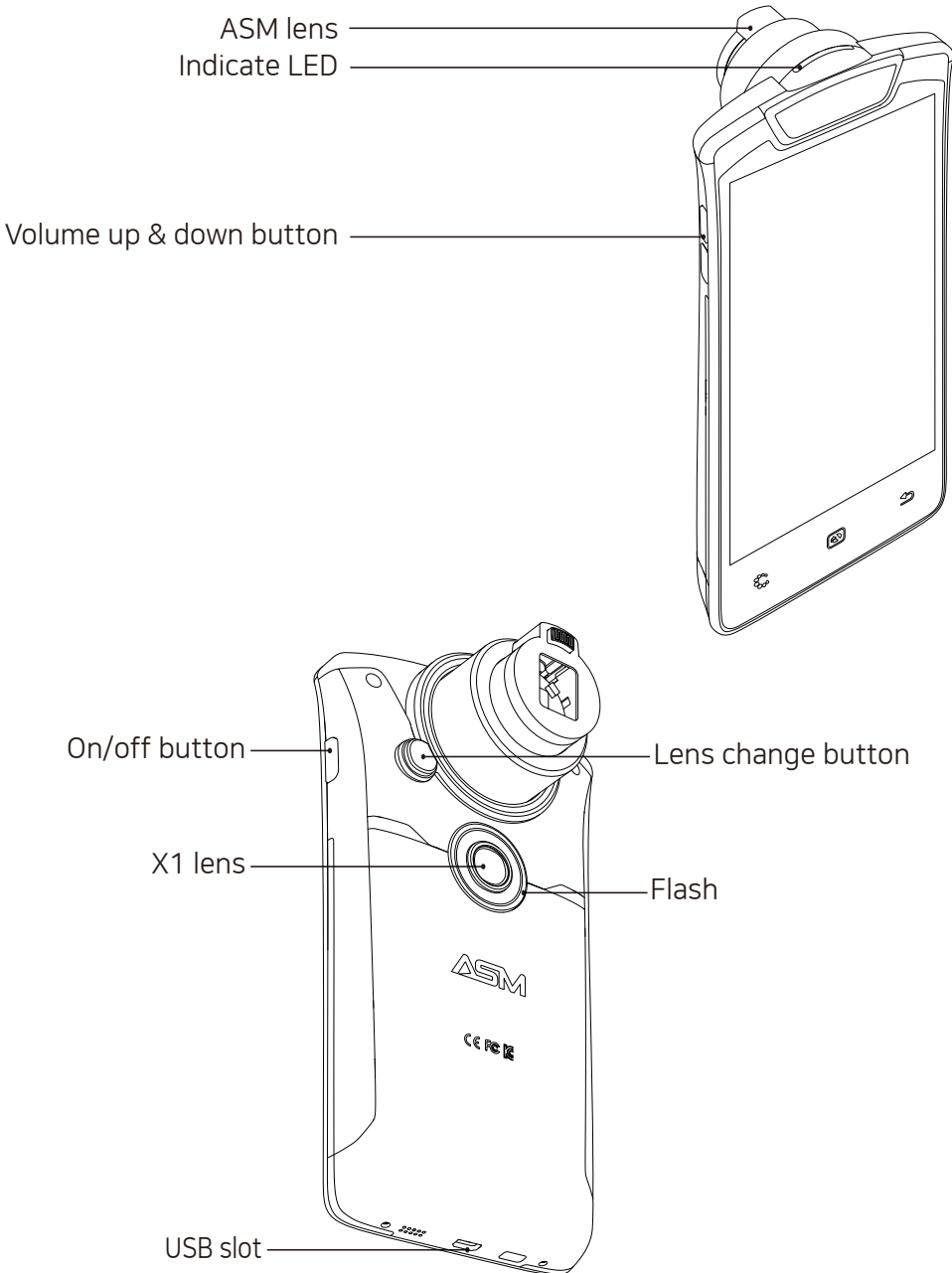


Settings

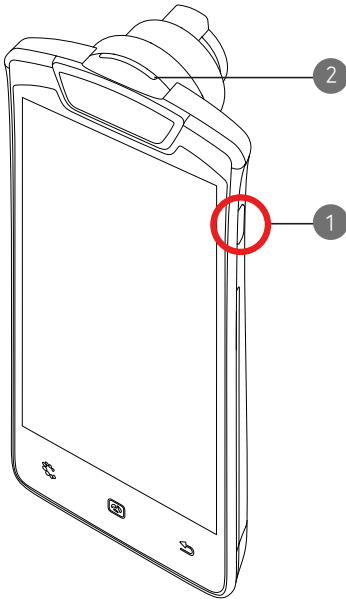

How to turn on the power

- 1 Press and hold "Power" button.
- 2 Take your finger off from the button after "Indicate LED" turned on blue.

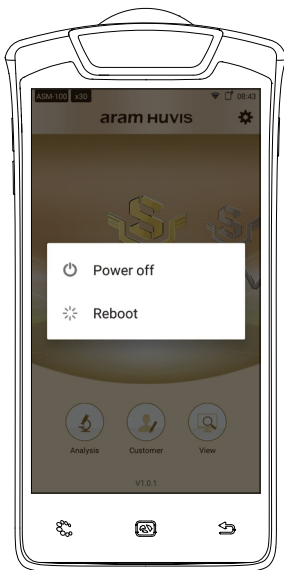


How to turn off on the power

- 1 Press and hold "Power" button.
- 2 Touch "Power off".


Device Settings

Charger can be used for American type & European type. The default is European type.


① Push the plug up.

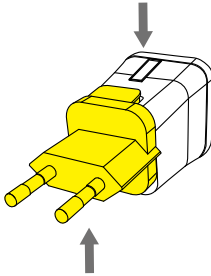

② Then 220V European type is disassembled and you can see the 110V-220V American type inside of the charger.

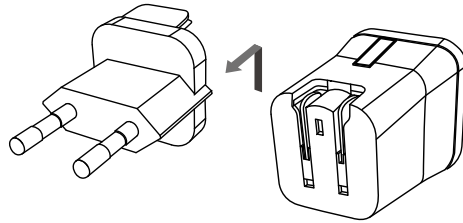

③ Choose the one you want to use and connect USB cable in adaptor.

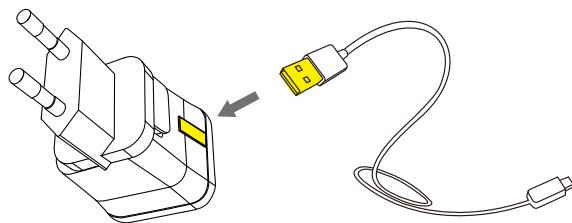


Date & Time




Touch  button.




Touch  button.

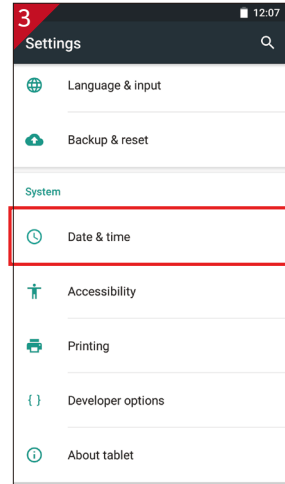


Go to "Date & time"

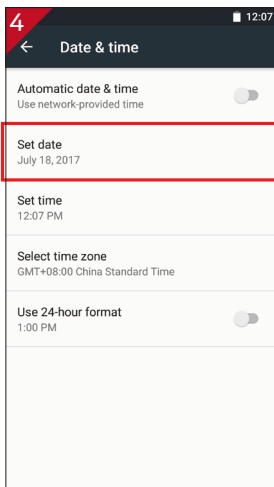

Go to "Set date".

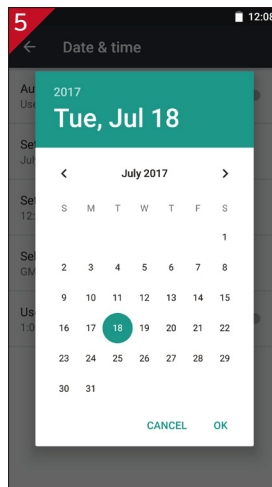


Select the date and touch "OK" button.

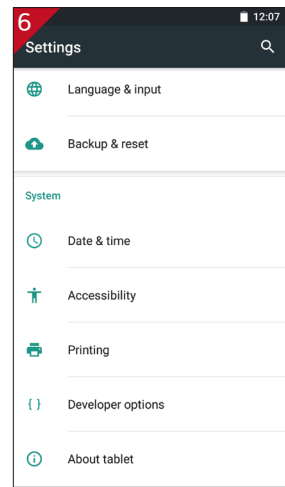


Check if the setting is correctly applied.


Date & Time

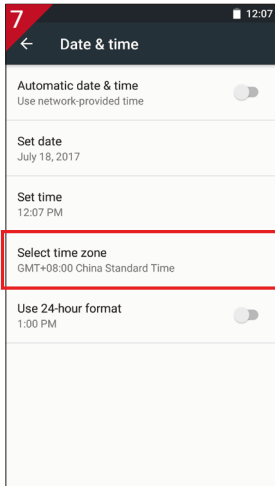


Touch "Select time zone"

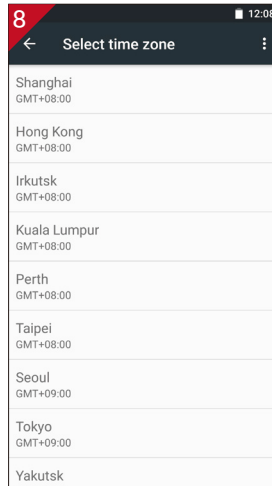


Select proper time zone.

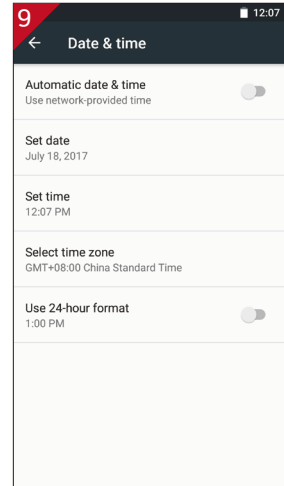


Check if the setting is correctly applied.

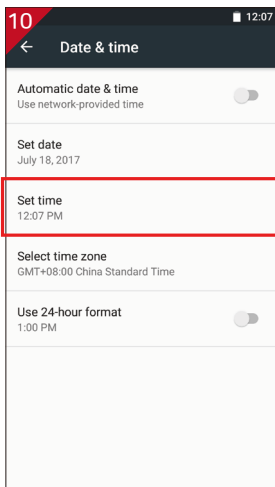

Touch "Set time".

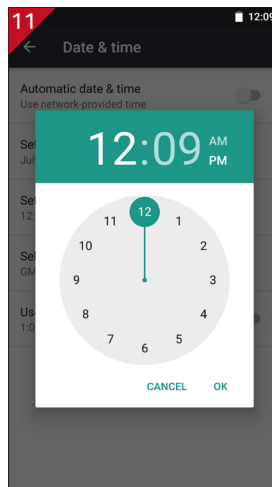


Change into the current time and touch "OK" button.

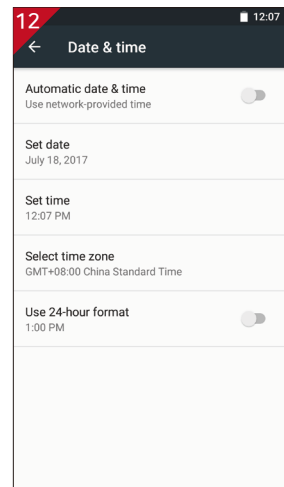


Check if the setting is correctly applied.


Wi - Fi

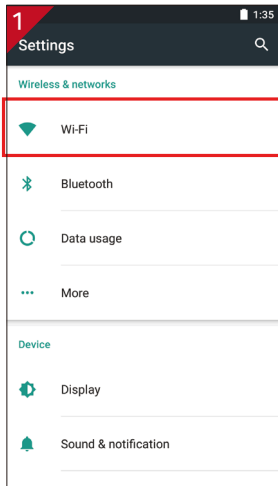


Touch "Wi-Fi".

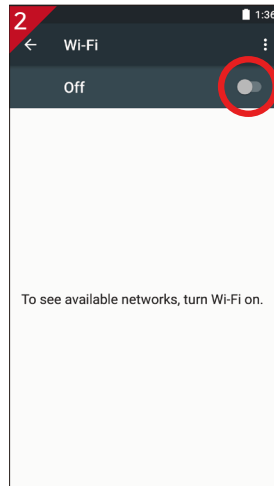


Turn on Wi-Fi using On/Off button

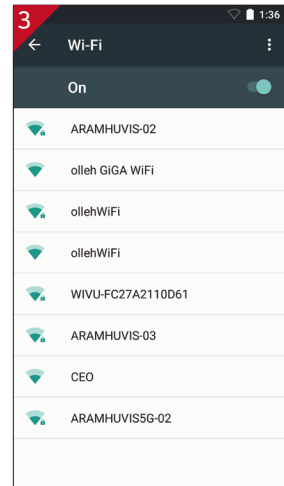

Choose proper Wi-Fi signal.

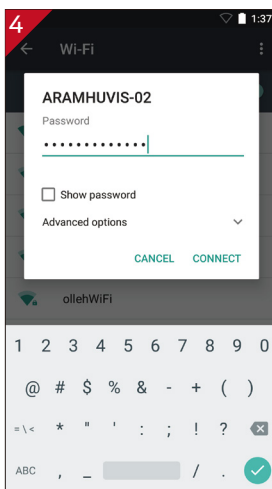


Put Wi-Fi password.

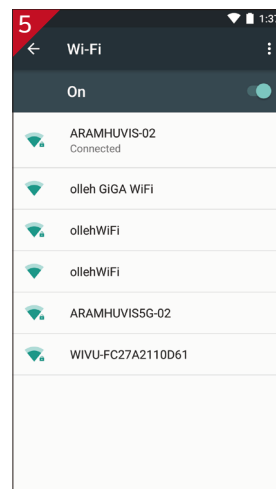

Wait until the device is connected to Wi-Fi.

Wi - Fi

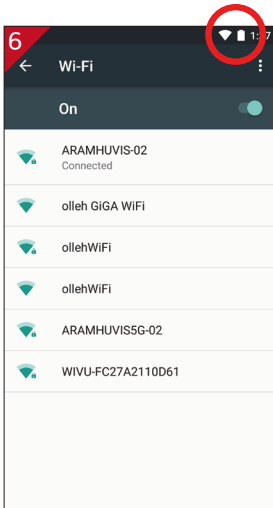


Once the device is connected to Wi-Fi, you could see the Wi-Fi mark on upper side of the screen.


Wi-Fi Usage

- Email
Wi-Fi connection is needed to send analysis data by email.
*Reference : page 7 (Wi-Fi), page 20(Email).
- Software update
In order to update the software version.


Sleep

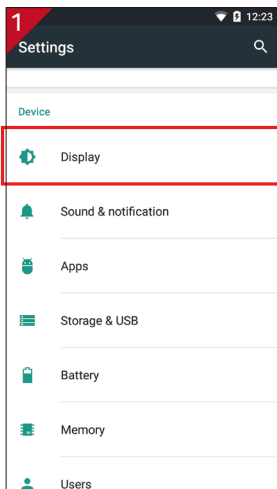

Touch "Display".

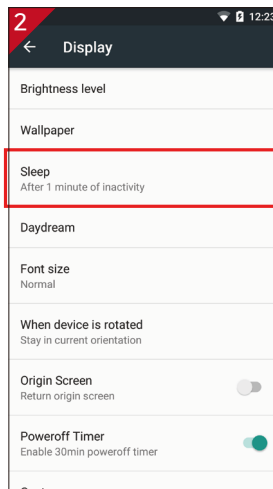


Go to "Sleep".

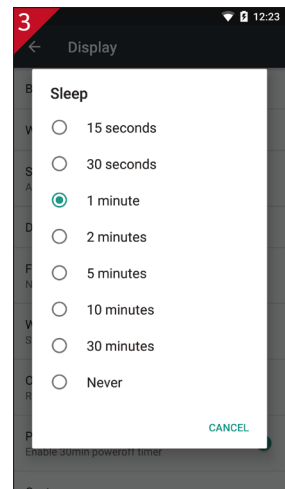


Select the option you want to apply.


Language

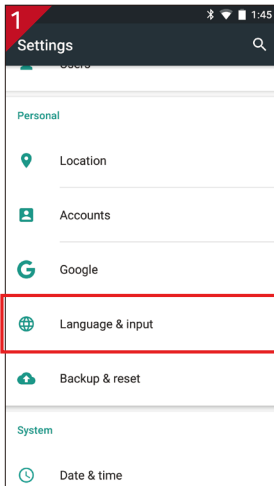

Touch "Language & input".

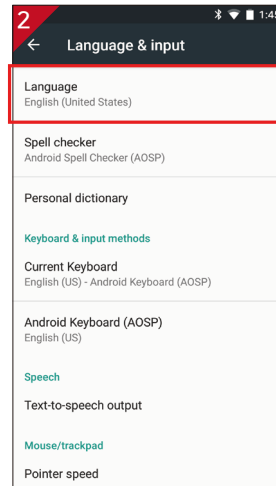

Go to "Language".

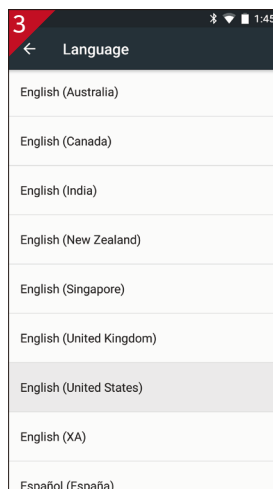


Select preferred language.


Settings



Touch  button.

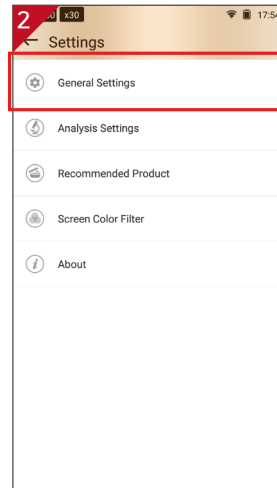


Go to "General Settings".

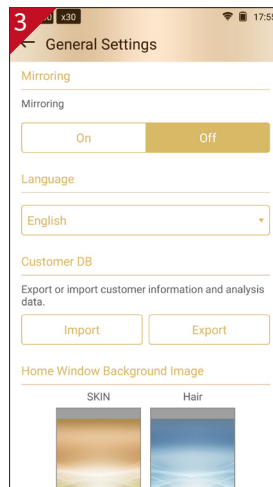


Available to change Language,
Version and Email setting.


CRM

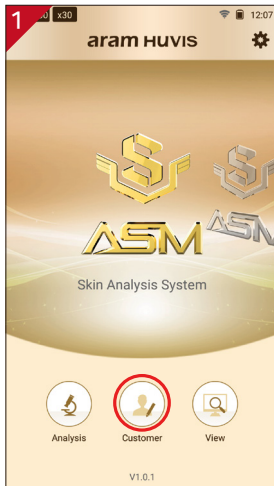


Touch "Customer".

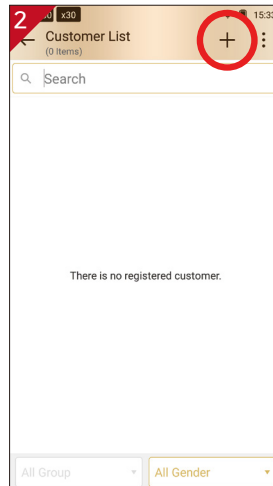


Touch + to register new customer.

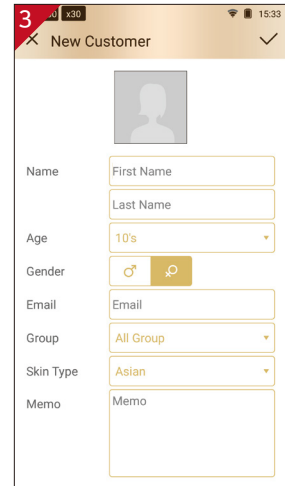


Input the information.

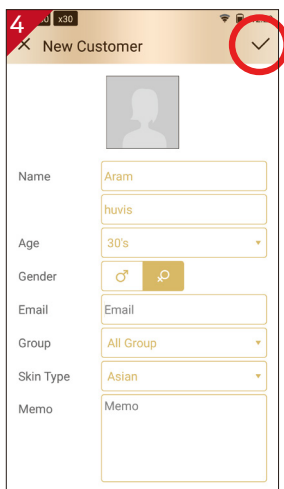


After putting all information, touch ✓.

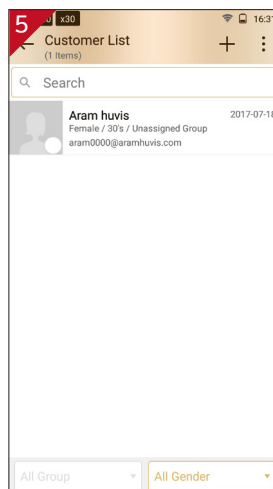


User's profile is created on the list. Then touch user's profile to start analyzing.

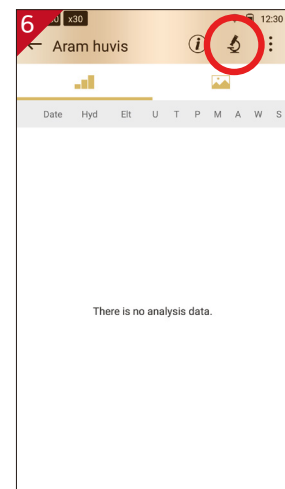


Touch  button to start analyzing.


Moisture

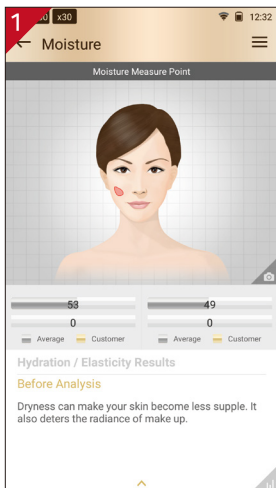


Touch the  button.
Please wait until vibration and sound.
*** Never put the sensor on the skin before the message "Touch to skin please".**

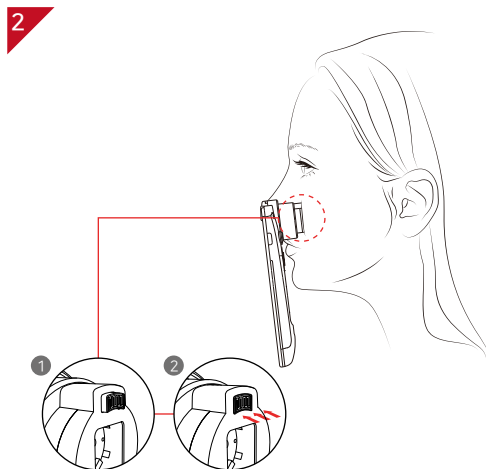

After checking the message "Touch to skin please", push the sensor completely to marked area of the face.
*** All surface of the sensor should be pushed completely.**

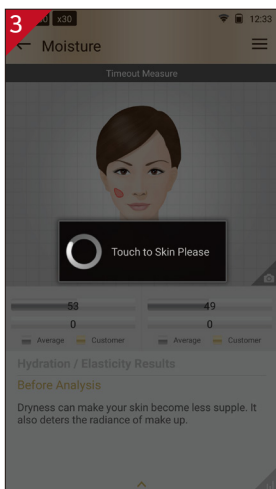


Analyzing hydration and elasticity...

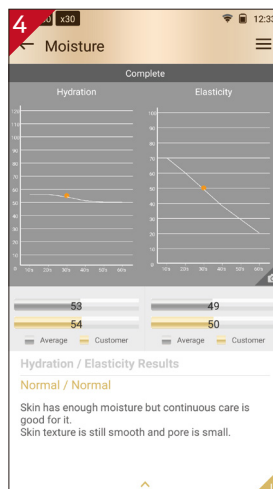


Check the result.
To move on next page, slide the screen from right to left.


Sebum

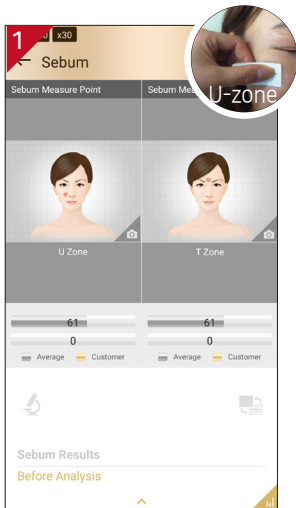


Put the sebum paper on U-zone of the cheek, then press it for 2~3 seconds.

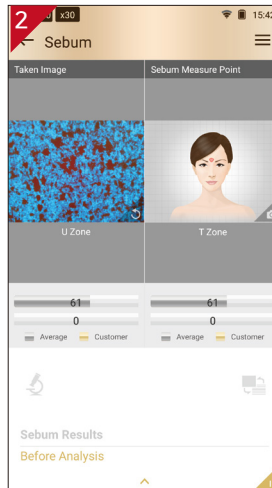


Take a picture of pink area of the sebum paper.

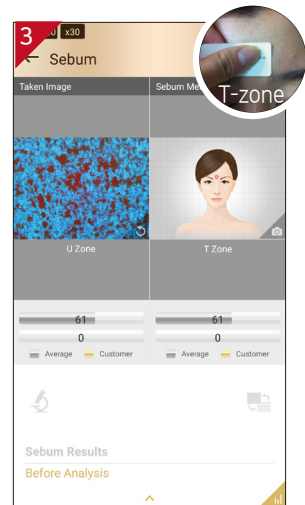

Put the sebum paper on T-zone of forehead, then press it for 2~3 seconds.

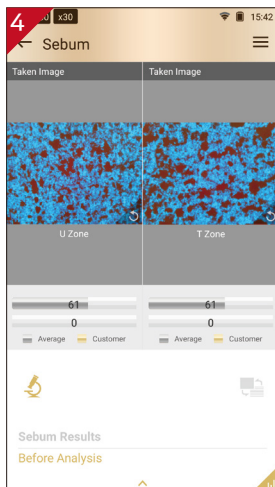



Take a picture of pink area of the sebum paper.
Then, touch  button to analyze.

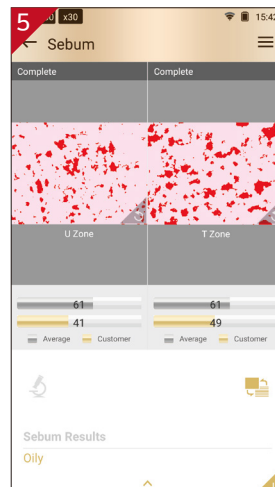



Sebum result will appear.


Pore

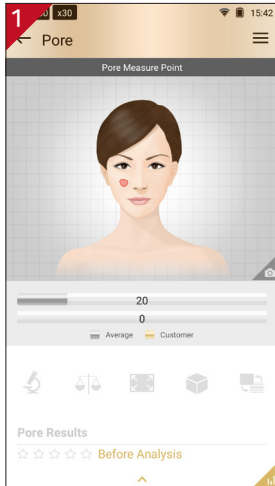


Touch  to activate camera and put the camera onto the marked area of the face.

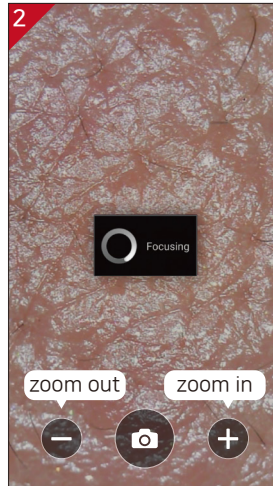


Touch screen to bring a camera into focus and then touch  to take a picture.

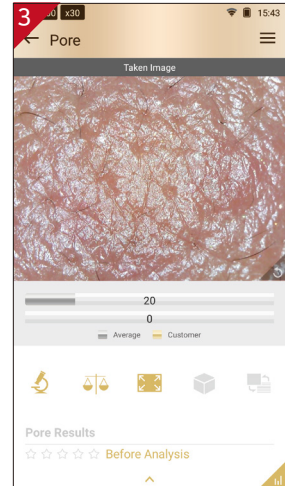


Touch  button to analyze.

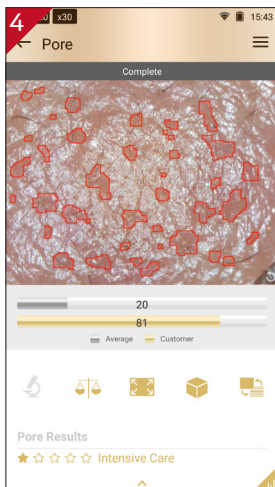


Result will appear.

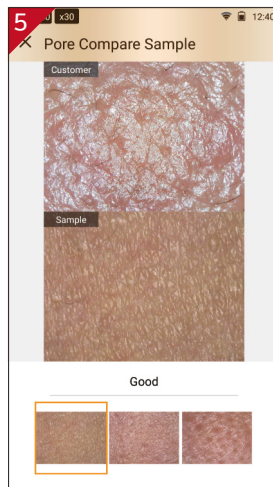


Touch  button to compare taken image with 3 sample images.
- Good/Normal/Bad





Touch  button to convert taken pore image in 3D.


Melanin




Touch  to activate camera and put the camera onto the marked area of the face.

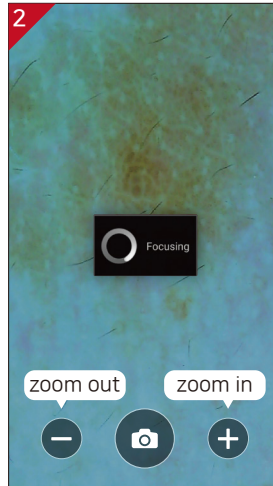


Touch screen to bring a camera into focus and then touch  to take a picture.

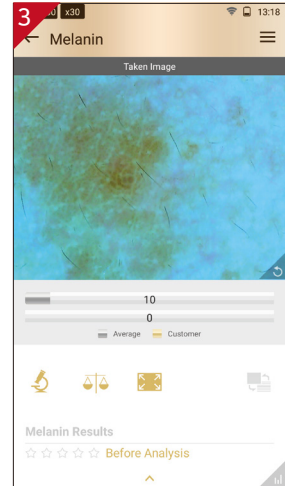


Touch  button to analyze.

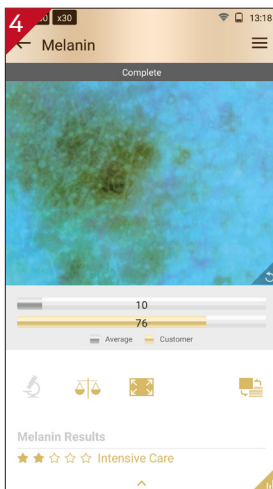


Result will appear.

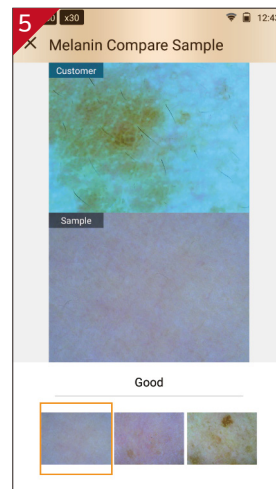



Touch  button to compare taken image with 3 sample images.
- Good/Normal/Bad


Acne




Touch  to activate camera and put the camera onto the marked area of the face.

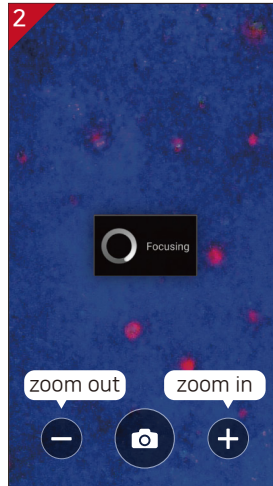


Touch screen to bring a camera into focus and then touch  to take a picture.

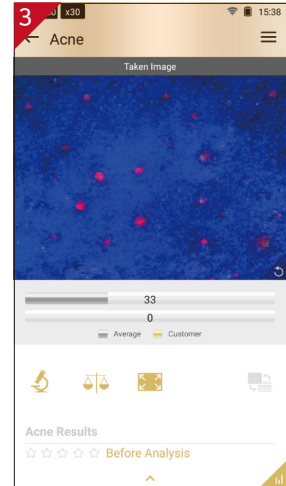


Touch  button to analyze.

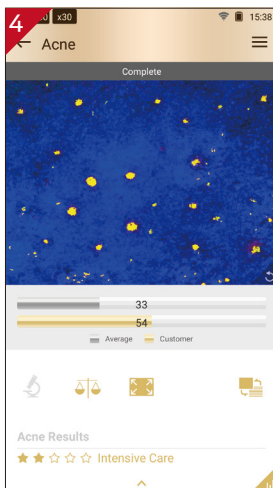



Result will appear.

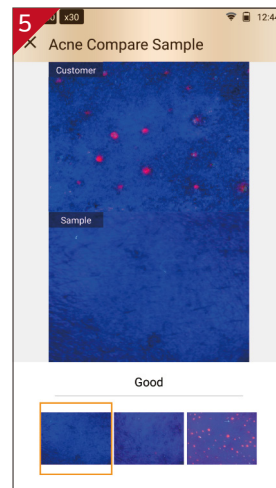


Touch  button to compare taken image with 3 sample images.
- Good/Normal/Bad


Wrinkle

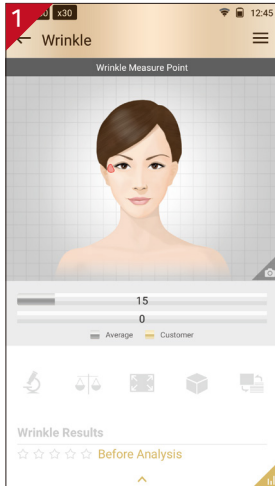


Touch  to activate camera and put the camera onto the marked area of the face.

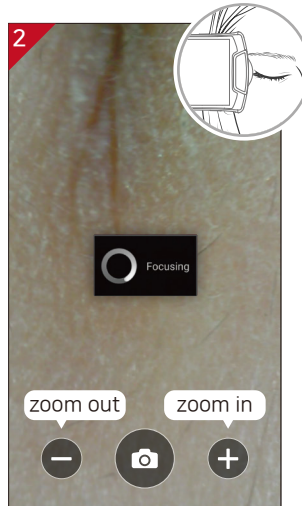


Touch screen to bring a camera into focus and then touch  to take a picture.
* For wrinkle photo, take picture of outer eye area in widthways to check crow's feet.

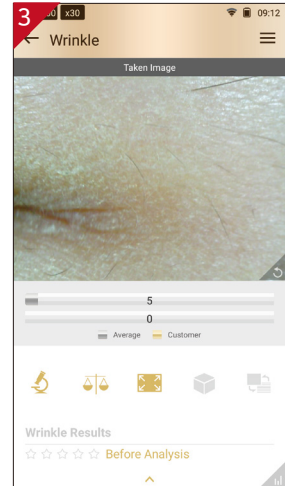


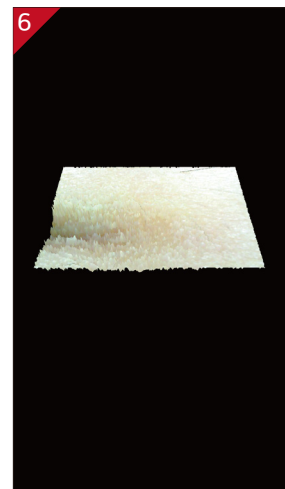
Touch  button to analyze.

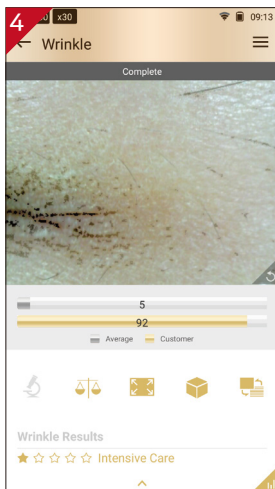


Result will appear.

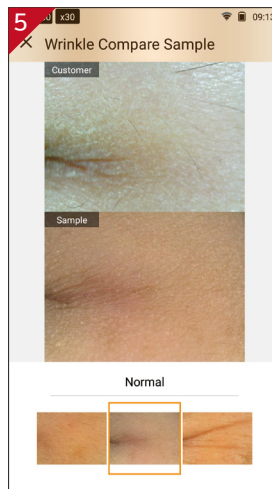



Touch  button to compare taken image with 3 sample images.
- Good/Normal/Bad





Touch  button to convert taken pore image in 3D.


Sensitivity

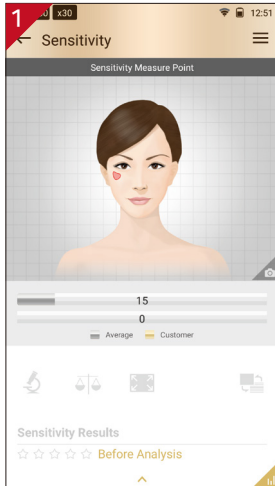


Touch  to activate camera and put the camera onto the marked area of the face.

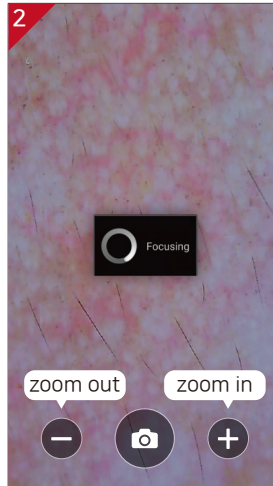

Touch screen to bring a camera into focus and then touch  to take a picture.

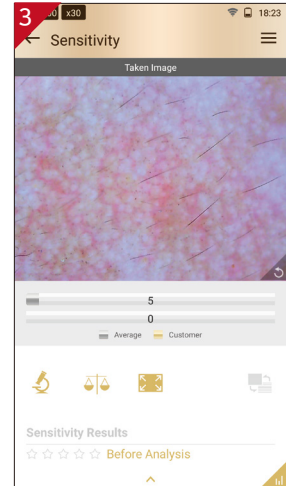


Touch  button to analyze.


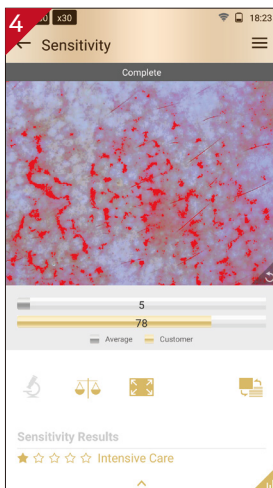


Result will appear.





Touch  button to compare taken image with 3 sample images.
- Good/Normal/Bad

Summary

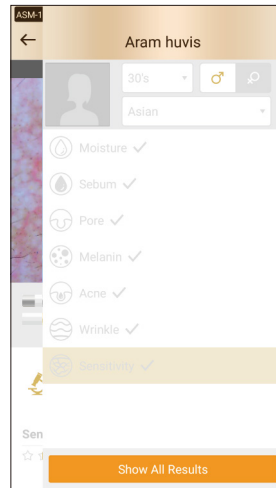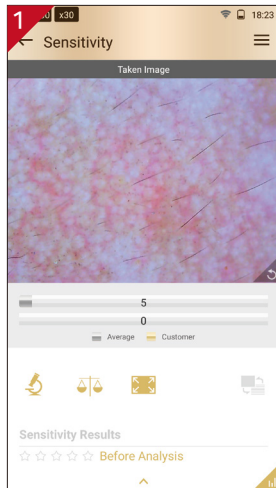


Touch  button or "Show All Result" to see overall result.

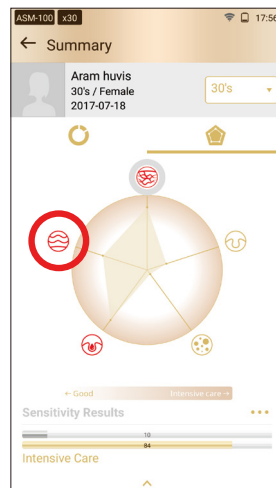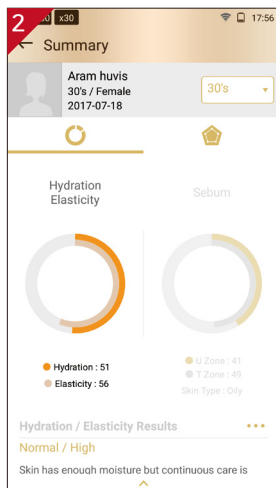


Available to check the result. The worst results will be marked in red.


Email

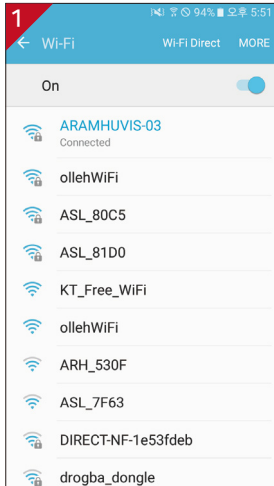


Before sending Email, the device shall be connected to Wi-Fi.* Reference : page 7.

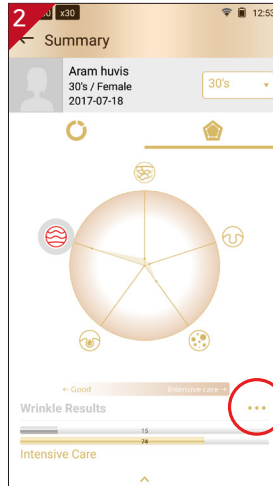


Touch ...

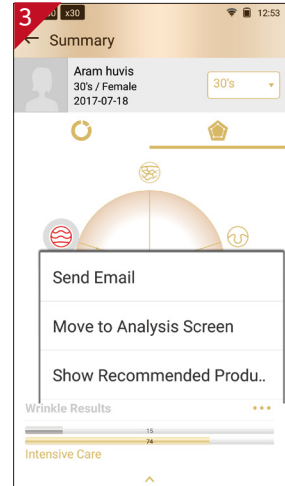


Go to "Send Email".

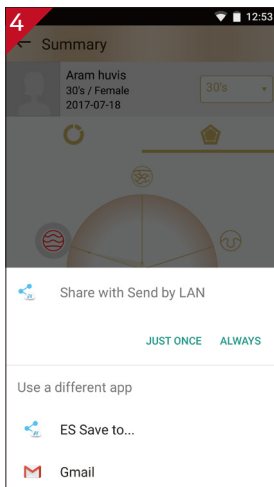

Select "Gmail"

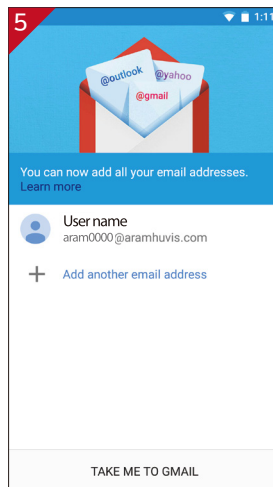

Select the mail account to register on the list.

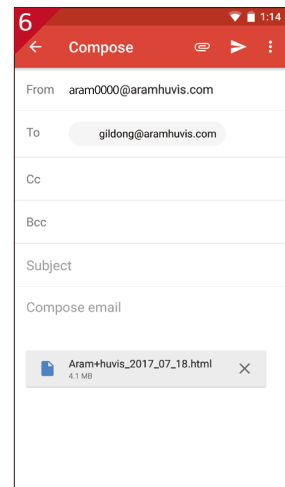


After selecting sender's mail address, put receiver's mail address. Then touch "Send" button.


Mirroring



Touch  button.

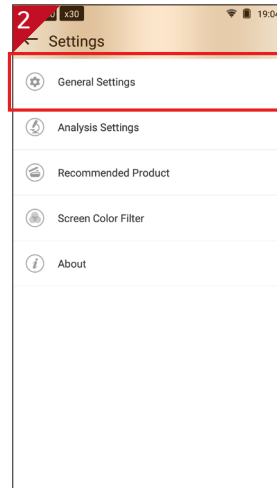

Go to "General Settings".

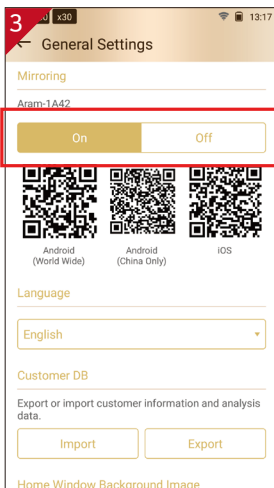


Set the mirroring option "On".



Capture QR code using QR code reader app. Automatically moved to App store.


Mirroring

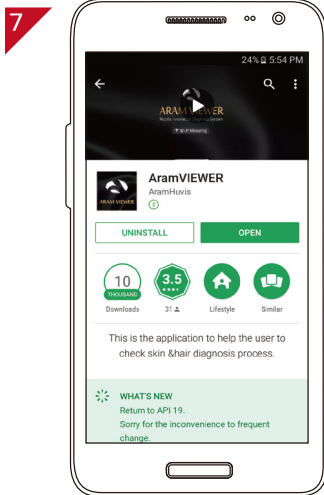


Install "Aram VIEWER".



Then, go to "Wi-Fi Setting".

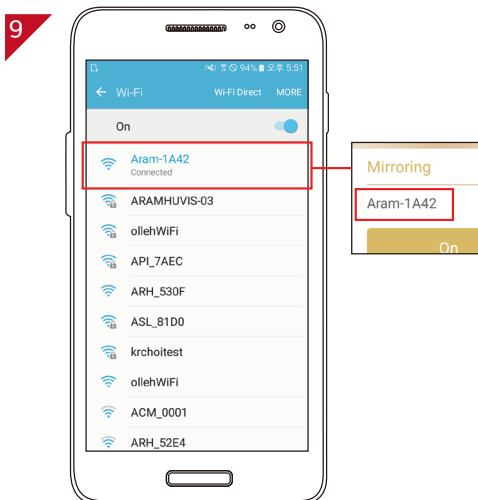

In your mobile device go to Wi-Fi setting. Then find SSID and connect it.

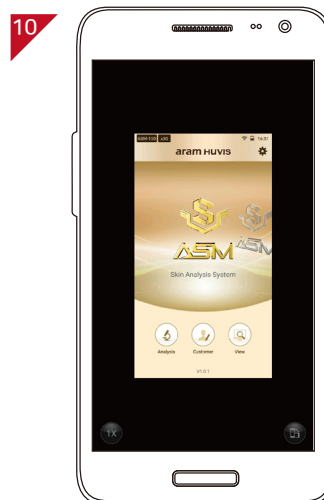


Mirroring will be started.


View Mode




Touch "View".

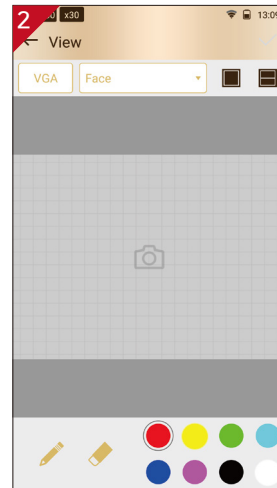

In view mode, it is able to take pictures with capture button.

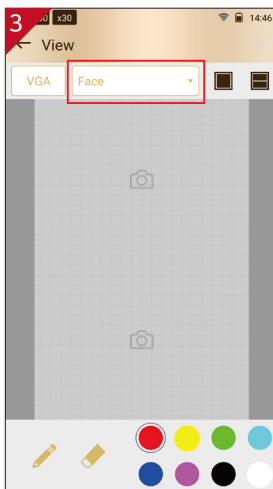

The user can select analysis parameter to take pictures in proper mode.

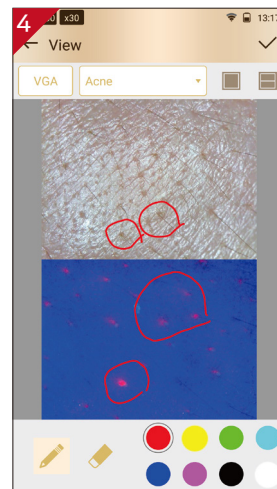

The user can mark the defect or concern according to the image with note function.

Warranty

- ① The product undergoes strict quality management and inspection process before production and introduction to market.
- ② In case of functional or operational defect occurs under normal condition of use within 10 days after purchase, we will repair it for free or replace in new product.
- ③ When this product goes out under the normal use, we will repair it free of charge under warranty.
- ④ Repair cost could be charged after warranty period.
- ⑤ For this product, warranty period is 14 months from the Invoice date.
- ⑥ Cost
 - Within warranty : Sending delivery(Buyer), repair cost and returning delivery(Aram)
 - If the product is broken by user's fault, repair cost could be charged.
 - After warranty : Sending delivery and repair cost(Buyer), returning delivery(Aram)
- ⑦ How to send :
 - Please mail to support@aramhuvis.com to receive information and proper instruction.
 - Tel : +82 70 4687 1712

Memo



HEAD OFFICE

#401~402 Bundang Seoul National University Hospital Health Care Innovation Park, 172, Dolma-ro, Bundang-gu, Seongnam-si, Gyeonggi-do, 13605 Rep. Korea

PRODUCTION DIVISION & FACTORY

#909 Byucksan Technopia, 560, Dunchon-daero, Jungwon-gu, Seongnam-si, Gyeonggi-do, 13230 Rep. Korea

Tel: 070-4687-1712 / Fax: 031-732-6330 / Email: info@aramhuvis.com
<http://www.aramhuvis.com>