Instructions

Suction Cup Grab Bars

Item # 521-1561-1912, 521-1561-1916,
521-1562-1912, 521-1562-1916, 521-1562-1901

Installation

NOTE: Grab bar must be placed on a smooth, non-porous surface. If placed over cracks or on grout between tiles, the grab bar will not be stable, and you may get a false reading on the Indicator.

1. Thoroughly clean and dry the suction cups and surface where the suction cups will be attached. Occasionally clean both suction cups and attachment surface as needed.
2. Release the suction cups by flipping the latches on the grab bar so they are in the upright position.
3. Firmly push the suction cups against the surface to remove all excess air, Fig. 1.
4. Push the latches down and make sure the Indicators are green.

NOTE: If the Indicator is red, Fig. 2, repeat steps 1-4 to reattach.




5. Always test grab bar suction before each use even though both Indicators may still be green. Do not use the grab bar to support your body weight; it is only for balance assistance.

Removal:

1. Flip both latches up.
2. Pull the vinyl tabs on the side of each suction cup to release the suction, Fig. 3. You may also slide the grip to the edge of the surface to break the suction.

CAUTION:

- Do not use the grab bar to support your body weight; it is only for balance assistance.
- Grip loses strength after time, so you should reattach the grab bar before every use.
- The Indicators are designed to ensure successful installation; they do not indicate proper adhesion.

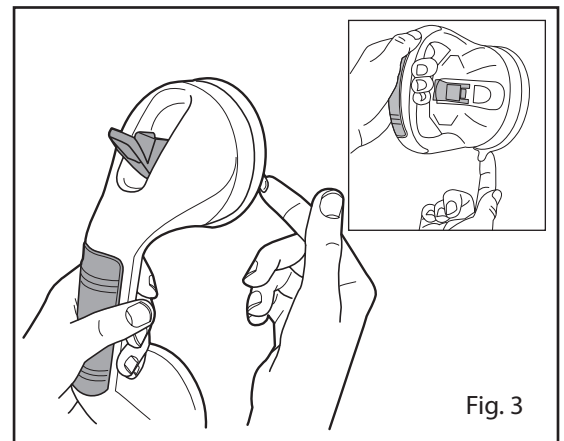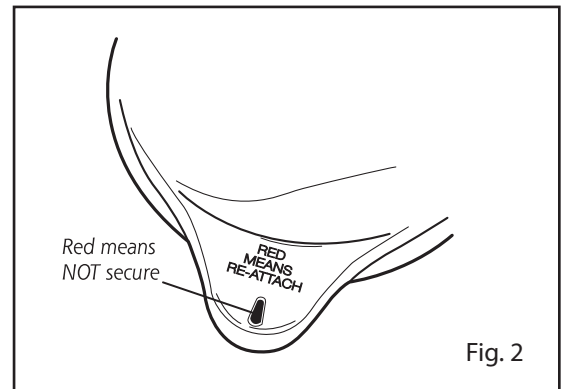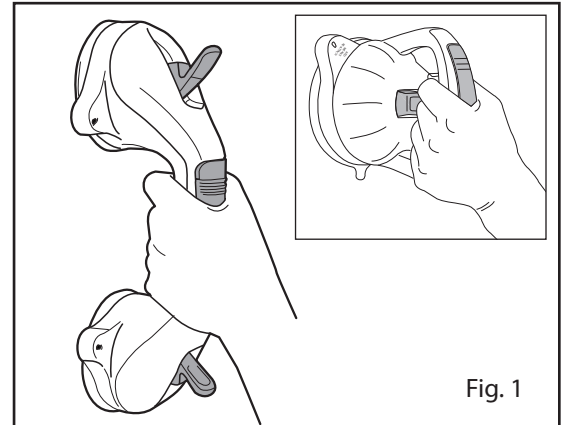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