

HALO Histoacryl[®]

Topical Skin Adhesive

Histoacryl[®]

Histoacryl[®] Flexible



HALO Histoacryl[®] TissueSeal Comparison

	Histoacryl [®]	Histoacryl [®] Flexible
Composition	n-butyl 2 - cyanoacrylate	n-butyl 2 - cyanoacrylate with plasticizers
Color	Clear Violet (D&C violet n°2)	Violet (D&C violet n°2)
Indications	Skin Closure	Skin Closure
Application	Direct from ampule	Direct from ampule
Time for Closure	30 Seconds	30 Seconds
Color Code (ampule)	Translucent & Yellow bar	Opaque ampule & Yellow bar
Color Code (packaging)	White and Orange	Dark and Light Blue
Storage	24 months @ 22°C	24 months @ 25°C

Why Halo Histoacryl[®] Flexible?

- Histoacryl[®] Flexible has been designed to combine flexibility and strength to close and protect wounds
- Peel strength is significantly higher for Histoacryl[®] Flexible than the market leader
- Closure of incisions up to 25cm
- Only one layer of adhesive is necessary to achieve a full-strength bond
- Special applicator tip provides better control and improved application
- Topical skin adhesives yield excellent cosmetic results
- Generates less heat during polymerization, making it more comfortable for the patient
- Adapts easier to the contours of the surgical incision



Microbial Barrier

In vitro studies have shown that Histoacryl® acts as a barrier to microbial penetration as long as the adhesive film remains intact. Histoacryl® inhibits the growth of Staphylococcus aureus, Staphylococcus epidermidis, Escherichia coli, Pseudomonas aeruginosa, Enterococcus faecium, Brevundimonas dimuta and Candida albicans for up to 7 days on inoculated agar plates.

Part #	Description	UOM	#/UOM
1051260P	Halo Histoacryl® Flexible 10 / 0.5 mL ampules 10 Applicator Tips	Box	10
TS1050044FP	Halo Histoacryl® - Blue 10 / 0.5 mL ampules 10 Applicator Tips	Box	10
TS1050071FP	Halo Histoacryl® - Clear 10 / 0.5 mL ampules 10 Applicator Tips	Box	10

OPEN




Open the package and take out the Histoacryl® Tip.

PREPARE APPLICATOR TIP



Open the ampule by twisting off the ribbed tip. Attach the Histoacryl® Tip to opened ampule.

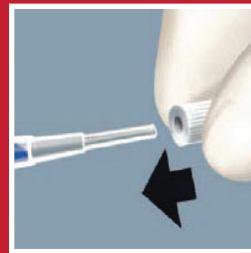

APPLY



Apply the glue in a thin layer to the approximated wound edges and hold in position for 60 seconds.

Polymerization of Halo Histoacryl® begins when moisture is introduced to the glue, via contact with skin. Histoacryl, in a capped container, will remain useable until moisture is introduced to the ampule. Single patient use.

Packaging



Box
10 Ampules
10 Applicator Tips




Ampules
10 / box



Applicator Tips
10 / box

Medical Director Authorization is required to purchase this product. Please consult your Medical Director to establish guidelines for use and protocol.

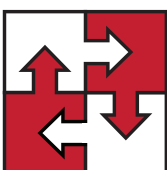

**Integrated
MedCraft**

Toll Free: 855.752.1011 | info@integratedmc.com | www.integratedmc.com
Cage Code: 7YLA3
Med/Surg DAPA: SP0200-18-H-0034
Equipment DAPA: SP0200-18-H-0045

Woman Owned
Small Business