CITY LITE *Model*

Operating Manual

1-888-56-BUGGY

contact@thetravelbuggy.com

Contents

Introduction.....	3
Specifications	4
Specification Diagram	5
Safe Operation	6
Precautions	7
Maintenance.....	8
Electromagnetic Interference (EMI)	8
Setup	9
Controller.....	11
Basic Operation.....	12
Charging Batteries	13
Free Wheel Mode.....	15
Transporting Your City Lite	16
Troubleshooting	17
Diagnostics.....	18
Limited Warranty	19

Introduction

Thank you for choosing The Travel Buggy for your portable wheelchair transportation needs. Congratulations on your choice of the CITY LITE model, which will provide you with years of dependable service with proper use and care.

It is important to read through this manual before operating your CITY LITE to become familiar with its features as well as its limitations and safety information. You will find the operation and care of your CITY LITE very simple and easy, but should you have any questions please contact us at The Travel Buggy at our customer hotline, 1.888.56.BUGGY or by email contact@thetravelbuggy.com.

Disclaimer: The Travel Buggy, its affiliates and partners cannot be held responsible for personal injury or property damage resulting from the unsafe or improper operation or maintenance of the CITY LITE portable electric wheelchair.

As part of our ongoing product improvement initiative, The Travel Buggy reserves the right to change specifications and design without notice. As a result, there may be minor differences between your CITY LITE and its accessories and the photos, illustrations and instructions in this manual.

Get to know your CITY LITE Model by The Travel Buggy:

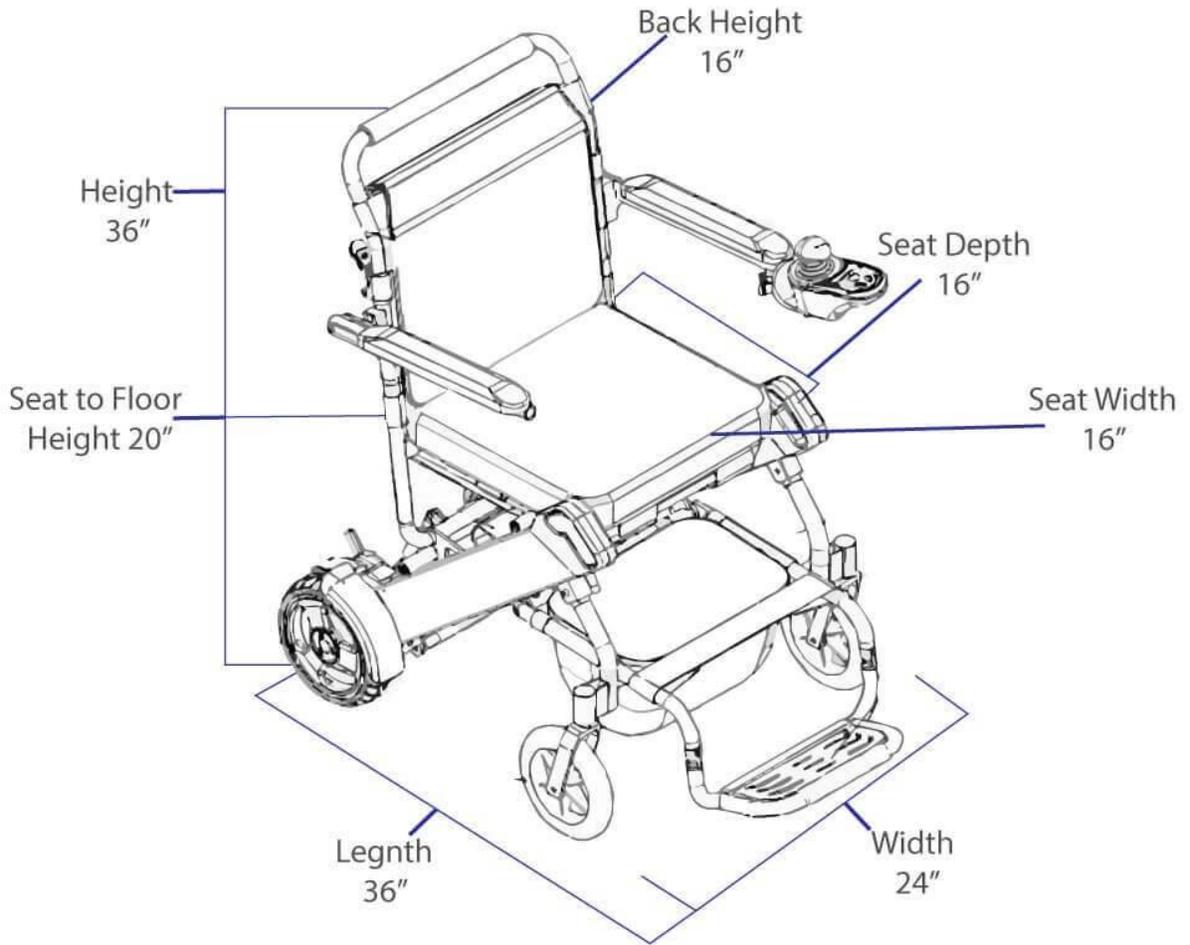
Locate Serial Number on the bottom of the bar, under seating area for registration online and note here:

Specifications

Max load:	120 kg (265 lbs.)
Tires:	6" PU solid tire (Front) 8" PU solid tire (Back)
Max slope capacity:	12°
Max speed:	6 km (3.7 miles) per hour
Max distance (full charge):	20 km (12 miles) (single battery) 40 km (24 miles) (double battery)
Steering:	360° joystick controller
Motor:	2 x 180W brushless motors
Seat depth:	40 cm (15.75 inches)
Seat width:	40 cm (15.75 inches)
Seat height:	47 cm (18.5 inches)
Braking distance:	1.5 m (when at top speed)
Turning radius:	85 cm (33 inches)
Battery:	Lithium 24V/10AH (a second 10AH is optional)
Charger:	24V/2A
Time required to fully charge battery:	6-8 hours
Net weight (including battery):	21 kg (46 lbs.) (single battery) 23 kg (51 lbs.) (double battery)
Net weight (without battery):	19 kg (42 lbs.)
Frame material:	Aluminum alloy
Unfolded (L x W x H):	89 x 61 x 94 cm (35 x 24 x 37 inches)
Folded (L x W x H):	61 x 33 x 78.75 cm (24 x 13 x 31 inches)

Specification Diagram

Safe Operation

We think this is the most important section of the manual; how to safely operate the CITY LITE portable wheelchair. Please review this section thoroughly and contact us if you have any questions.

General Safety...

- Adjust the CITY LITE to your personal comfort level as much as possible.
- Take your time to learn how the CITY LITE works in a private space before you begin operating around others.
- Practice bending, reaching and transferring in and out of the CITY LITE.
- Always be aware of your surroundings.
- If you're not sure the CITY LITE can surpass an obstacle, try to find another way around.
- The CITY LITE is designed for a single user only; more than one user may result in personal injury to you and others.
- Make sure that electronic components such as the control panel, battery charger, and other electronic controlled components avoid moisture. If these components come into contact with moisture, dry before use.
- The CITY LITE seat is designed for comfort, not specifically for pressure relief; if you suffer from pressure sores, or if you are at risk that they will occur, please consult your doctor.

Before using your CITY LITE...

- Make sure the CITY LITE is operating smoothly.
- Check for any unusual noises, vibration, or a change in the ease of use; if you detect a problem, use the tool kit to perform maintenance or get in touch with your service professional or The Travel Buggy directly.
- Failing to maintain your CITY LITE could increase the risk for injury; get in touch with The Travel Buggy so that we can help you find and correct the problem.

Getting into the CITY LITE...

- Before sitting, make sure the CITY LITE is OFF, this will ensure the CITY LITE will not move upon touching the joystick.
- Make sure the foot rest is down.
- Once onboard, make sure your feet are safely on the footrest.

Driving the CITY LITE...

- Be alert to the danger of motor vehicles on roads or in parking lots.
- At night, or when it is hard to see, use reflective tape on your CITY LITE.
- The CITY LITE is designed for use on firm, even surfaces such as concrete, asphalt or indoor flooring.
- Do not operate the CITY LITE in sand, loose soil or over rough terrain, which could damage wheels, bearings, axles or motors, or loosen fasteners.

Getting out of the CITY LITE...

- Make sure to bring the CITY LITE to a complete stop.
- Turn OFF the CITY LITE to avoid accidentally operating the joystick and moving the wheelchair.
- Raise the armrest so that you can conveniently exit the CITY LITE.
- Carefully and safely get out of the seat.

Weight Limit – Do not exceed or the warranty will be void...

- Never exceed a total weight of 120 kg (265 lbs.) total for the rider plus items carried.
- Exceeding the weight limit is likely to damage the seat, frame or fasteners and may cause severe injury to you or others.

When transporting the CITY LITE...

- Never sit in the CITY LITE while in a moving vehicle; an accident or sudden stop may cause you to be thrown from the power chair.
- Never transport the CITY LITE in the front seat of a vehicle where it may shift and interfere with the driver.
- Always secure the CITY LITE so that it cannot roll or shift when being transported in a vehicle.
- The CITY LITE is NOT approved to be used with any tie down system, which is generally used to transport a seated user in a powered wheelchair in a motor vehicle.

Precautions

CAUTION: OPERATE SAFELY

**Do not operate the CITY LITE in water of any kind.
Do not operate the CITY LITE up or down staircases or escalators.**

**Do not operate the CITY LITE while on a moving vehicle such as a bus.
Do not lean your body to reach out of the CITY LITE, causing a balance issue.
Do not exceed the weight limit total of 120 kg (265 lbs.) for rider plus any items.**

**DO turn off the CITY LITE when exiting the chair for safety.
DO practice driving the CITY LITE in a private, enclosed area.**

**Be aware of local regulations regarding operating the CITY LITE on a public road.
Modifying your CITY LITE in any way may void all warranties.
Do not expose the CITY LITE or batteries to extreme temperatures.**

Maintenance

It is important to perform regular maintenance of your CITY LITE for longevity and warranty eligibility. Even a little cleaning now and again will go a long way to keeping your device looking and functioning like new.

Certified repair shops with trained professionals who specialize in electric wheelchairs and scooters will be qualified to perform the service and maintenance on your CITY LITE electric wheelchair should it be required. Please be sure to keep a log of the service and maintenance performed. Your shop or technician may require parts, which The Travel Buggy has available.

We recommend you inspect your CITY LITE two to three times per year for in the following areas:

- Frame: Tighten loose screws as necessary using the tool kit included with your CITY LITE.
- Wheels: make sure they rotate easily. Significant resistance may require new bearings.
- Armrests: look for excessive wear or if they are loose.

Replacement parts and more major maintenance issues should be done by a professional. We do not recommend that you do intense types of maintenance work to the CITY LITE yourself.

Battery...

- Use batteries that have been supplied by The Travel Buggy only; our batteries are special lithium batteries designed specifically for use in the CITY LITE.
- Read all battery instructions before attempting to change or charge battery.
- To remove the battery, you must first unplug the wire harness from the back of the battery.
- Pull the battery out slowly and evenly. Replace the battery by reversing these steps.

Fasteners...

Many of the screws, bolts and nuts on the CITY LITE are special high-strength fasteners. Replacing these fasteners with non-approved parts may cause your CITY LITE to result in faulty operation and may cause harm to you or other people, therefore:

- Use fasteners that have been supplied by The Travel Buggy only.
- Using the tools included with your CITY LITE, tighten any fasteners that become loose as soon as possible.

Change and Adjustments...

Never use non-CITY LITE parts or make changes to your CITY LITE unless authorized in writing by a representative from our company. Doing so will void the Warranty and may create a safety hazard.

Electromagnetic Interference (EMI)

Powered vehicles such as the CITY LITE may be susceptible to electromagnetic interference (EMI), which is interfering electromagnetic energy (EM) emitted from sources such as radio stations, TV stations, amateur radio (HAM) transmitters, two-way radios, and cellular phones. The interference (from radio wave sources) can cause the CITY LITE to release its brakes, move by itself, or move in unintended directions.

Setup

Your CITY LITE will arrive carefully packaged in a cardboard box. We suggest that two people are available to remove the packaging and set up the CITY LITE for the first time.

1. The CITY LITE ships upright; carefully open the top of the box then turn the box on the side for easier removal of the contents.
2. Once all packaging has been removed, you will find the following components::

1. Raise the backrest until it snaps into place
 2. Place one hand on the backrest and use your other hand to push the footrest down.
 3. Pull the cable located under the seat to set the locking mechanism. *NOTE: Important step.
 4. Push the back rest away from the seat cushion, locking it in place.
 5. Rotate both armrests up and over towards the front of the chair.
- NOTE: Make certain that the back rest is locked in place and does not collapse forward before using.

1. Raise backrest

2. Expand chair

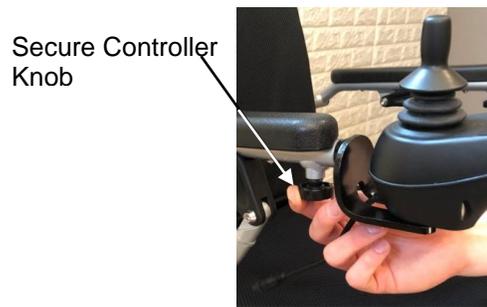
3. & 4. Pull cable to lock;
Push back rest away
From seat cushion

5. Raise armrests

- Slide the Controller into the open slot on the arm of the CITY LITE on the right side (unless pre-ordered for left-handed users). Tighten knob located under the arm rest to secure the controller in place.

- Connect the controller wire harness to the CPU wire harness that runs from the chair itself, taking care to line up the pins properly so that they do not get bent, broken or damaged. Secure the connection by twisting the coupling ring over the connected harness.

Connect CPU to Controller wire harness

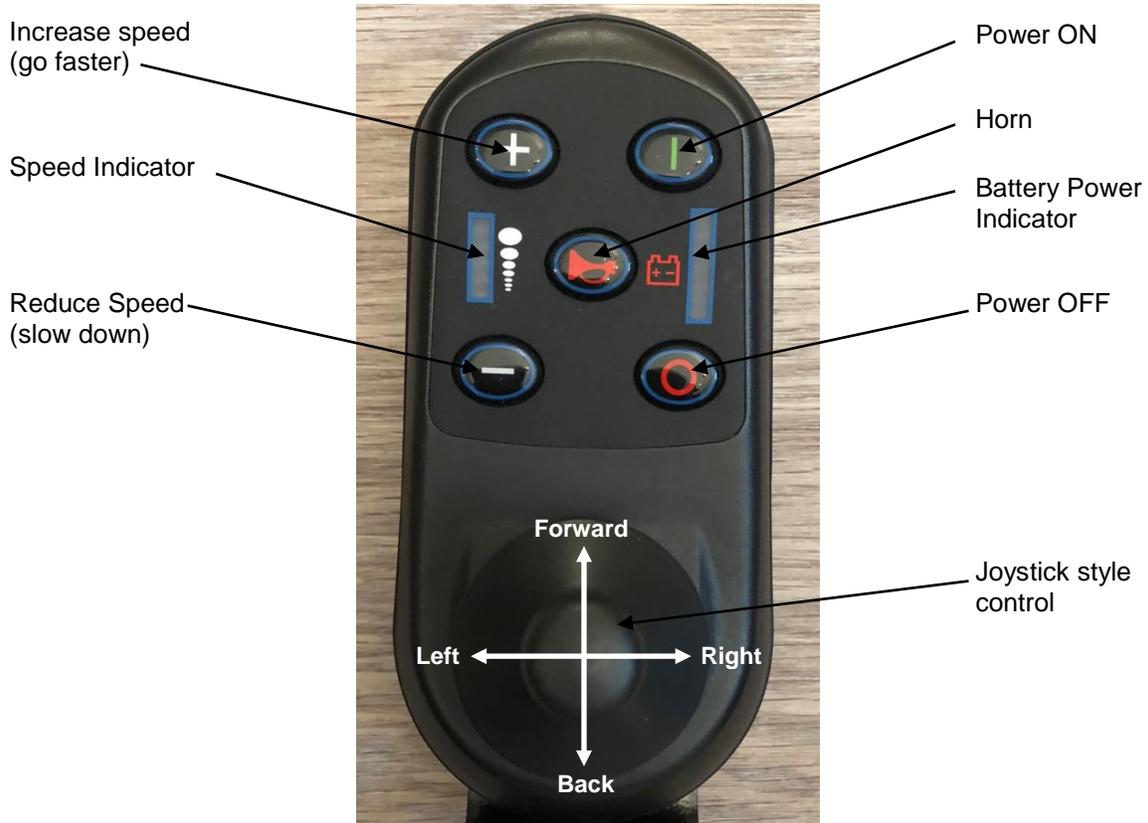
NOTE: Make sure connector pins in wire harness are lined up properly; do NOT force the wire harness connections together or you will bend or break the pins.

- Connect battery harness located near the rear wheel of the CITY LITE.

Controller

The CITY LITE is equipped with a joystick style controller which is intuitive and easy to understand and use. The Controller on your CITY LITE also features a horn, speed controls, battery indicator and power buttons as shown:

Controller and Driving Tips...

- Take some time in a safe, controlled environment to learn the operation of the controller, the speed and the maneuverability of the CITY LITE.
- When turning, reduce your speed, which will help avoid any potential tilting or tipping of the CITY LITE.
- Be sure to adjust the speed depending on your environment.
- To slow down, slowly bring the joystick back toward the initial position.
- To stop, just bring the joystick back to the initial position.
- Use extra care when you drive the CITY LITE in reverse. You may lose control or fall if one of the rear wheels hits an object. When operating in reverse:
 - Operate the CITY LITE slowly and at an even speed.
 - Stop often and check to make sure your path is clear of obstacles.

Basic Operation

Before operating your CITY LITE, be sure to read all safety information and precautions included in this manual. Operate only in the appropriate environment after you have practiced operating the CITY LITE and know how to work the controller.

GET IN	Be certain that the power is OFF before sitting in the seat of your CITY LITE, with no lights on the controller. Make sure the foot rest is down before getting into your CITY LITE.
POWER ON	Press the Power button, you will hear a beep and a light will indicate that power is on.
TURNING & STEERING	Use the joystick style control to steer and turn left, right or go forward and backward.
STOPPING	Simply remove your hand from the joystick to stop.
POWER OFF	When stopping, it is advised that you power OFF the CITY LITE to prevent accidents. Press the Power button to turn off the CITY LITE.
GET OUT	Make sure there is no light on the controller before getting off the CITY LITE.

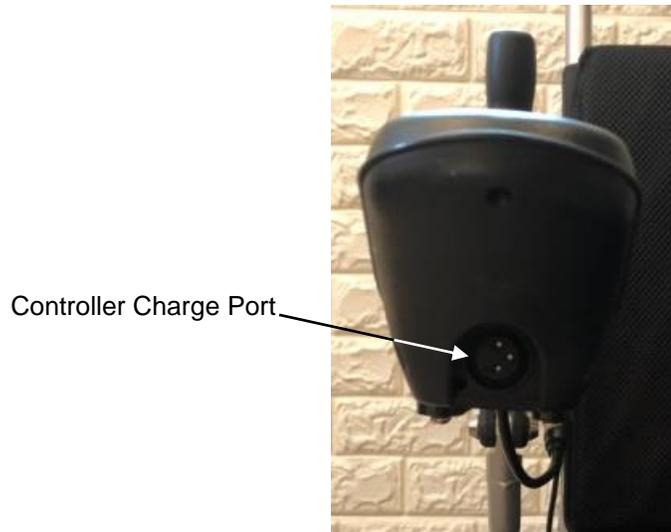
NOTE: To ensure complete safety, always power off your CITY LITE when you aren't moving, no matter how briefly you have stopped. This will ensure that you or someone else doesn't accidentally operate the joystick control causing unexpected movement of the wheelchair.

Charging Batteries

Your CITY LITE will arrive with a fully charged battery, which should last a maximum of 20 km or 12 miles under normal operating conditions. A second battery can be purchased by contacting The Travel Buggy, which will extend the run time. We recommend that you run the CITY LITE until the battery is completely dead in order to get the most out of the battery life.

When you need to charge the battery in your CITY LITE, use the following procedure:

1. Turn the power OFF using the button on the controller.
2. Plug in the charger to the controller port (see below) and leave to charge for 6 to 8 hours.

1. Plug the charger power harness into the charger and then into a grounded wall outlet.
2. Allow your CITY to charge for 6 to 8 hours or when the light on the charger turns green, which means your batteries are fully charged.
3. Should you require them, the lights and what they mean are located on the back of the charger.

Front of Charger

Charger Light:

Light should come on after plugging in the charger.
Light color indicates battery life:
RED – battery is fully drained
AMBER – battery is charging
GREEN – battery is fully charged; unplug from power

Back of Charger with definitions of what the light colors mean.

Your CITY LITE may require a new battery at some point; to change it:

1. Unplug the battery wire harness located on the back of the chair near the rear wheel.
2. Slide out the battery completely. Replace by sliding the new battery in and plugging in the wire harness.

More information on the CITY LITE battery...

- The CITY LITE comes with one battery installed, charged 24 volt 10Ah lithium ion battery.
- Charging the battery for 8 hours will maximize the overall life-cycle of the battery.
- Turn the main unit's power off while charging.
- Do not leave children unsupervised near the CITY LITE while it is charging.
- Do not attempt to open the battery housing.
- Red means the power is on and the unit is charging; green means the unit is fully charged. Note: this information is also located on the back of the charger.
- When the battery is fully charged, remove the cable. Then take out the DC charging connector and AC input power socket.
- The average time taken to re-charge workable batteries will vary from 2 to 8 hours depending on the level of usage.
- Do not charge more than 8 hours.
- If you do not operate the CITY LITE for two months, we recommend that you charge the battery for 6-8 hours before use.
- Over Current Protection: this system is included on the CITY LITE and will shut off the current from the batteries if triggered, thus shutting the motor down, which will ensure that the motor and battery do not experience any damage.
- The lifecycle of a lithium battery is measured in charge cycles with an average of 500 charge cycles.
- Check the battery indicator light daily if you use the CITY frequently.
- Do not use any battery chargers that have not been certified by The Travel Buggy.
- If damaged, do not handle the battery without protective gloves and eyewear.
- If the battery needs to be disposed of, locate the nearest hazardous materials disposal facility, and call them for instructions.
- The CITY LITE is equipped with one lithium battery and will go up to approximately 20 km or 12 miles. This maximum operating time per charge can be affected by:
 - Uneven, sloped or soft terrain
 - Exceeding total weight limit
 - Varying speed excessively
 - Stopping and starting frequently
- Before storing your CITY LITE for long periods of time, take these steps to maintain the battery:
 - Fully charge the battery before storing, then

- Remove the battery from the unit
- Store the battery in a room temperature environment that is kept dry
- Avoid temperature extremes and do not allow the battery to freeze
- Do not attempt to charge your battery if it is frozen from extreme weather conditions, instead first allow it to thaw for several hours.
- Additional or replacement batteries are available from The Travel Buggy; contact us at 1.888.56.BUGGY or by email at contact@thetravelbuggy.com.

Free Wheel Mode

In order to have a care giver push the CITY LITE, it must be put in Free Wheel mode. This mode disengages the drive mechanism to the wheels, making them free to push manually. We recommend that you turn OFF the power to the CITY LITE when in this mode, both for safety and to save battery power.

1. To put the CITY LITE in Free Wheel Mode, come to a full stop and turn OFF the power on the controller.
2. Push UP on the two red levers located on each rear wheel as shown:

Push UP two red levers on each wheel

Transporting Your City Lite

To transport your CITY LITE, it is advised that you fold it up following the procedures below.

NOTE: Never transport the CITY LITE in a tie down system, as the CITY LITE is not compatible with them. Never sit in the CITY LITE and be transported in a moving vehicle. Do not place the folded CITY LITE in the front seat with the driver where it could move or slide.

1. Turn off the controller and unplug the wire harness. Remove the controller by loosening the knob and sliding it out of the chair.
2. Rotate both armrests up and toward the back of the chair.
3. Store the cushion underneath the chair.
4. Squeeze both levers to fold the back rest down.
5. Pull the cable located under the seat to release the locking mechanism.
6. Push the back rest toward the seat and fold up the foot rest.

1. Unplug and remove

2. Rotate arm rests

3. Store cushion

4. Squeeze levers and fold back rest down

5. Pull cable

6. Fold up

For instructions on how to unfold the CITY LITE, see the Setup portion of this manual.





Troubleshooting

The following represents the most common questions asked about the CITY LITE with regard to everyday use; check this helpful guide or the operator's manual for more information if you have a question about using your CITY LITE:

Question	Possible Causes	Answer
Why is there NO POWER to the CITY LITE?	<ol style="list-style-type: none"> 1. Controller system power is not connected. 2. Controller system CPU and joystick are not connected. 3. Battery power is too low. 	<ol style="list-style-type: none"> 1. Connect the Battery. 2. Ensure all connections between Joystick, CPU and Battery are securely tightened. 3. Charge the Battery.
Why is the CITY LITE noisy or vibrating when turning?	<ol style="list-style-type: none"> 1. Speed is set too low. 2. Motor is damaged. 	<ol style="list-style-type: none"> 1. Raise the speed. At low speeds, the motor may sound or feel strained. 2. Replace the motor.
Why can't I remove the battery?	<ol style="list-style-type: none"> 1. Battery must be unplugged before removing. 	<ol style="list-style-type: none"> 1. Unplug the battery wire harness located on the back of the wheelchair.
Why is the driving distance shorter than normal?	<ol style="list-style-type: none"> 1. Chair is being operated at below 0 degrees Celsius. 2. Chair is being driven on uneven surfaces, rough terrain or slopes often. 	<ol style="list-style-type: none"> 1. This is a normal characteristic of Lithium material in a battery when operating at a cold temperature. 2. Energy consumption increases on uneven surfaces, rough terrain or slopes. This is normal.
Why can't I charge the battery?	<ol style="list-style-type: none"> 1. Charger light does not turn on. 2. Charger light is always green. 3. Charging time is stopped before full charge is complete. 	<ol style="list-style-type: none"> 1. Replace the charger. 2. Battery is not connected, or may need to be replaced. 3. Capacity of the battery has decreased over time. Replace the battery.
Why can't I connect the Controller?	<ol style="list-style-type: none"> 1. Connector pins (male) on the controller have become bent and misaligned with connector holes (female) on CPU connector. 	<ol style="list-style-type: none"> 1. Using a small tool, carefully straighten the connector pins (male) on the Controller to align with holes (female) on the CPU connector.

Diagnostics

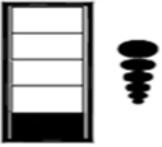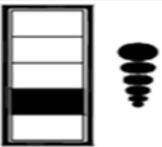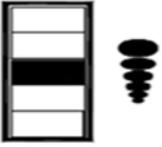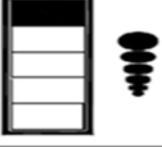Your controller has an indicator light that, during normal operation, is a steady GREEN. This light also functions as a diagnostic tool by flashing in patterns that indicate a diagnostic code. The following is a chart of the diagnostic light codes that your CITY LITE may use to indicate problems. It is important to pay attention to these diagnostics in case the CITY LITE must be put out of use for safety reasons until maintenance can be performed.

Diagnostic Code (left side – speed indicator light)	Diagnosis Description	Trouble Shooting
1 	Battery Empty	Charge the battery immediately.
2 	Joystick malfunction	Will likely require Joystick replacement. Call for service.
3 	Wheelchair is charging	Unplug charging cable to use wheelchair.
4 	Motor connection damaged	Will likely require motor or cable replacement. Call for service.
5 	Brake is not connected or damaged	Make sure the chair is not in freewheel mode (push red levers to power mode). If error continues, call for service.

Limited Warranty

The purchaser of this product is entitled to a limited warranty as offered by The Travel Buggy and its affiliates on the following components and timeframes:

- Frame - 5 Years
- Motor - 2 years
- Battery - 1 year
- Joystick and CPU System - 1 year
- Wear Parts* - 3 months

*Includes tires, seat and back rest, arm rests, and support straps.

This warranty is valid for the replacement of dysfunctional parts only. Any parts under warranty will be replaced and shipped to your door. Any service and labor fees, if applicable, to replace parts under warranty must be paid by the user.

Due to its straightforward design, most parts can be easily exchanged by the end user without a professional service tech required. However, it is always recommended you seek professional help for maintenance and service, to make sure the work is done properly.

The warranty only covers NON-DELIBERATE Damage.

The warranty does not cover:

- Products damaged by user negligence.
- Products damaged accidentally.
- Products damaged intentionally.
- Products that have been subjected to negligence.
- Products that have been subjected to abuse.
- Products that have been improperly stored.
- Products that have been improperly handled.
- Products that have been improperly operated.
- Products that have experienced general misuse.
- Products that have been modified in an unauthorized, unapproved way.

Warranty is non-transferable and only valid for the original wheelchair purchaser.

Warranty is only valid so long as product is registered online at www.thetravelbuggy.com within 30 days of receipt. The serial number for registration can be found at the top of the chair's back, simply remove the roll cushion to locate it.

The Travel Buggy will, at its sole discretion, replace any warrantable item that fails within the warranty period.

The Travel Buggy team promises to try and make right any issue that may arise with your purchase or the experience with our products.

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