# CRUZ

by Travel Buggy

Foldable Mobility Scooter

# TABLE OF CONTENTS

NOTICES	3	LOADING & TRANSPORTING	17
SAFETY INFORMATION	4	BATTERY MAINTENANCE	18
BATTERY SAFETY INFORMATION	6	CHARGING	20
SPECIFICATIONS	8	MAINTENANCE & CARE	21
GETTING STARTED	10	LIMITED WARRANTY	22
OPERATING	16	CONTACT & SUPPORT	23

# NOTICES

Thank you for choosing CRUZ foldable mobility scooter by Travel Buggy!

It is important to read through this manual before operating your CRUZ to become familiar with its features as well as its limitations and safety information

You will find the operation and care of your CRUZ very simple and easy, but should you have any questions please contact Travel Buggy

## OUR CUSTOMER HOTLINE

1.888.56.BUGGY (Canada)

1.855.56.BUGGY (USA)

## EMAIL

[contact@travelbuggy.com](mailto:contact@travelbuggy.com)

## DISCLAIMER:

The notes, warnings, and cautions contained within the manual and marked by the triangular Caution Symbol  should be given special care. Users should also pay special attention to information marked in this manual beginning with **NOTICE**.

Travel Buggy, its affiliates and partners cannot be held responsible for personal injury or property damage resulting from the unsafe or improper operation or maintenance of the CRUZ foldable mobility scooter.

As part of our ongoing product improvement initiative, Travel Buggy reserves the right to change specifications and design without notice. As a result, there may be minor differences between your CRUZ and the photos, illustrations and instructions in this manual

# SAFETY INFORMATION

---

- DO make certain that the seat is in a locked position when you are operating your CRUZ
- DO reduce your CRUZ's speed when you are driving around a corner
- DO proceed with caution as you approach the downgrade of a ramp or other incline and when driving near a raised surface or unprotected ledges and drop offs (curbs, porches, stairs, etc.)
- DO avoid exposure to rain, snow, ice, salt, or standing water whenever possible
- DO maintain and store in a clean and dry condition
- DO keep your hands away from the tires when driving. Be aware that loose fitting clothing can become caught in drive tires
- DO make certain that the tires are properly inflated (30 psi)
- DO keep tools and other metal objects away from battery terminals. Contact with tools can cause electrical shock
- DO follow the recommended weight capacity (150 kg / 330 lbs)
- DO follow the recommended incline slope rating (14 degrees/25% slope)
- DO NOT drive your CRUZ across the side of a hill or diagonally up or down a hill, and do not stop (if possible) while driving up or down an incline
- DO NOT operate your new CRUZ for the first time without completely reading and understanding the owner's manual
- DO NOT carry additional passengers on your CRUZ
- DO NOT use your CRUZ as a seat in a moving vehicle
- DO NOT allow unsupervised children to play near the scooter
- DO NOT operate your CRUZ while under the influence of alcohol, as this may impair your ability to drive safely
- ALL servicing must be performed by qualified personnel only. Unauthorized modifications to scooter, its power system or its control system may compromise the performance and safety of the system. UNDER NO CIRCUMSTANCES should you attempt any repair or service that is not covered in this manual.
- ALWAYS ensure all safety warning labels are in place and legible.

# SAFETY INFORMATION

---

## PRE-RIDE SAFETY CHECK

Get to know the feel of your CRUZ foldable mobility scooter and its capabilities in a safe driving environment before taking it in public. Travel Buggy recommends that you perform a safety check before each use to make sure your CRUZ operates smoothly and safely.

## PERFORM THE FOLLOWING INSPECTIONS PRIOR TO USING YOUR CRUZ:

- Check for proper tire inflation. Maintain but do not exceed the psi/bar/kPa air pressure rating indicated (30psi) on each tire.
- Check all electrical connections. Make sure they are tight and not corroded.
- Check the brakes.
- Check battery charge.

 **WARNING:** The operator should read all of the warnings signs and instructions here and on the lift before using scooter.

 **WARNING:** Always protect batteries from freezing temperatures and never charge a frozen battery. This will damage the battery and may cause personal injury. Attempting to charge a battery in freezing conditions does not prevent a battery from freezing.

 **WARNING:** You should not operate your scooter on public streets and roadways. Be aware that it may be difficult for traffic to see you when you are seated on your scooter. Obey all local pedestrian traffic rules. Wait until your path is clear of traffic, and then proceed with extreme caution.

 **NOTE:** If you discover a problem, contact Travel Buggy or an authorized Travel Buggy dealer for assistance.

 **WARNING:** Do not modify your scooter in any way including accessories if they have not authorized by Travel Buggy . Do not use accessories if they have not been tested or approved for CRUZ foldable scooter.

 **WARNING:** Be aware that cell phones, two-way radios, laptops, and other types of radio transmitters may cause unintended movement of your electrically-powered mobility vehicle due to EMI. Exercise caution when using any of these items while operating your mobility vehicle and avoid coming into close proximity of radio and TV stations.

# BATTERY SAFETY INFORMATION

## WHEN USING THE BATTERY

1. Lithium-ion battery packs may get hot, explode or ignite and cause serious injury if exposed to abuse conditions. Be sure to follow the safety warnings listed below:
  - DO NOT place the battery near fire or heat the battery
  - DO NOT install the battery backwards so the polarity is reversed
  - DO NOT connect the positive and negative terminal of the battery to each other with any metal object (such as a wire)
  - DO NOT carry or store the battery together with necklaces, hairpins, or other metal objects
  - DO NOT pierce the battery with nails strike the battery with a hammer, step on the battery or otherwise subject it to strong impacts or shocks
  - DO NO solder directly onto the battery
  - DO NOT expose battery to water or salt water, or allow the battery to get wet
2. DO NOT disassemble or modify the battery. The battery contains safety and

- protection devices, which if damaged, may cause the battery to generate heat, explode or ignite
3. DO NOT place the battery in or near fire, on stoves or other high temperature locations. Do not place the battery in direct sunlight, or use or store batteries inside cars in hot weather. Doing so may cause the battery to generate heat, explode or ignite. Using the battery in this manner may also result in a loss of performance and a shortened life expectancy.
  4. If the device is to be used by children, the caregiver should explain the contents of this document to the children and provide adequate supervision to ensure the device is being used appropriately
  5. When the battery is worn out, insulate the terminals with adhesive tape or similar materials before disposal
  6. Immediately discontinue use of the battery if, while using, charging or storing the battery, the battery emits an unusual smell, feels hot, changes color, or shape, or appears abnormal in any way. Contact Travel Buggy if any of these problems are observed
  7. DO NOT place battery in microwave over, high pressure containers or on induction cookware

# BATTERY SAFETY INFORMATION

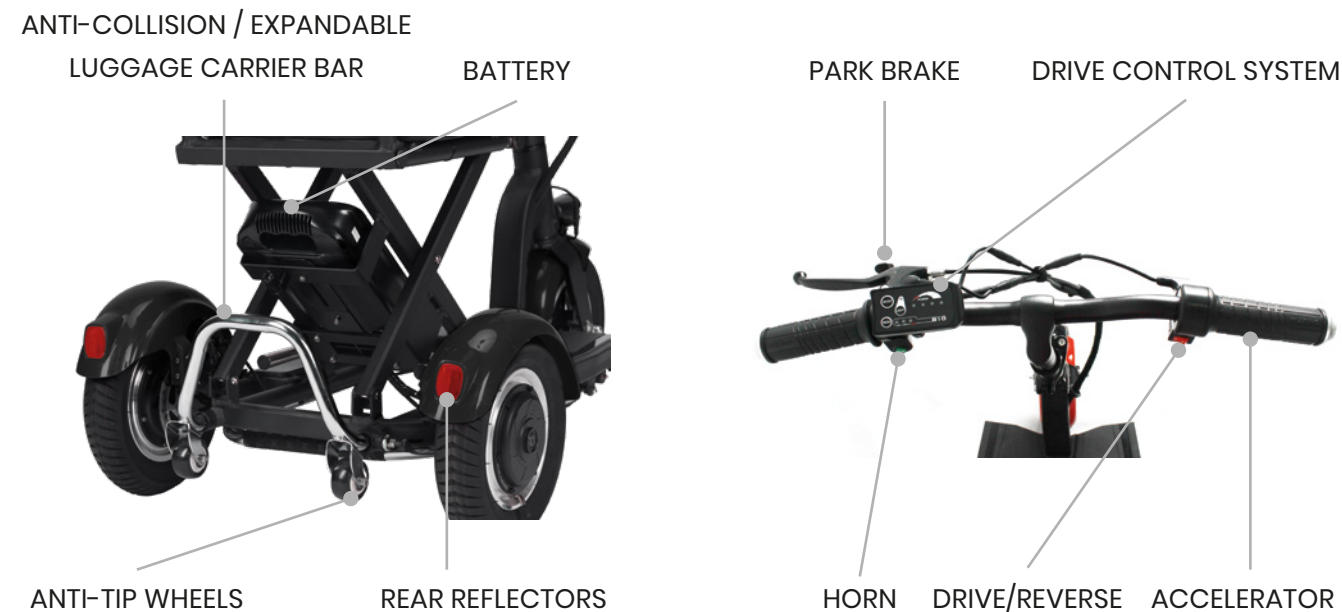
## CHARGING PRECAUTIONS

1. BE SURE to follow the warnings listed below while charging the battery. Failure to do so may cause the battery to become hot, explode or ignite and cause serious injury.
  - When charging the battery, ONLY USE THE SPECIFIED CHARGER PROVIDED WITH THE PRODUCT
  - DO NOT attach the battery to a power supply plug or directly to a car's cigarette lighter
  - DO NOT place the battery in or near fire, or in direct sunlight. Heating the battery can damage the safety circuitry, which can cause additional heating, rupture or ignition of the battery
2. DO NOT continue charging the battery if it does not recharge in the timelines specified in the operating manual
3. THE TEMPERATURE range over which a battery can be charged is 0°C to 45°C (32°F to 113°F). Charging the battery at temperatures outside of that range may cause severe damage to the battery or reduce battery life expectancy

## WHEN DISCHARGING THE BATTERY

1. DO NOT discharge the battery using any other device except for the specified device. When the battery is used in devices other than the specified device, it may damage the battery or reduce its life expectancy. If the device causes abnormal current flow, it may cause the battery to become hot, explode or ignite and cause serious injury.
2. THE TEMPERATURE range over which the battery can be discharged is -20°C to 60°C (-4°F to 140°F). Use of the battery outside this temperature range may damage the performance of the battery or may reduce its life expectancy.

# SPECIFICATIONS

# SPECIFICATIONS

## GENERAL SPECIFICATIONS

MAXIMUM USER WEIGHT CAPACITY: 330 LBS

PRODUCT WEIGHT (WITH BATTERY): 52 LBS

PRODUCT WEIGHT (WITHOUT BATTERY): 47 LBS

CLIMBING INCLINE: 14 DEGREES/25% GRADE

TURNING RADIUS: 36"

## FUNCTIONAL SPECIFICATIONS

TRAVEL DISTANCE: 30 KM / 18.5 MI

SPEED: THREE SPEED LEVELS 4/8/12 KPH, 2.5/5/7.5 MPH

MOTOR: 36V 300W REAR HUB MOTOR

BATTERIES: 36V 8AH / 288WH LITHIUM BATTERY

CHARGER: 110-240V 2A

CHARGING TIME FROM EMPTY: 4-6 HOURS

LIGHT: LED HEADLIGHT, REAR REFLECTOR

BRAKES: REAR DISC BRAKE

ADJUSTABLE HANDLEBAR, FOLDABLE, REAR LUGGAGE RACK, CARRYING HANDLE

## MATERIALS

FRAME MATERIAL: ALUMINIUM ALLOY

TIRE: AIR FILLED PNEUMATIC TIRES

## DIMENSIONS SPECIFICATIONS

GROUND CLEARANCE (AT LOWEST POINT): 3"

DIMENSIONS FOLDED (L - W - H): 39"-23"-21"

DIMENSIONS UNFOLDED (L - W - H): 39"-23"-36"

SEAT HEIGHT: 20"

SEAT WIDTH: 12.5"

SEAT LENGTH: 12"

DECK TO GROUND: 3"

TIRE SIZE: FRONT 8 INCH, REAR 10 INCH

WHEEL TYPE: 3 WHEELS PNEUMATIC

# GETTING STARTED

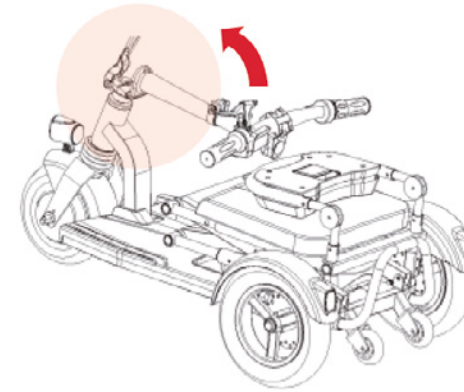
- Unbox your new CRUZ foldable mobility scooter
- Remove all packaging and protective wrap
- Locate the components provided with scooter


## INCLUDED:

- CRUZ Foldable Mobility Scooter
- Lithium Ion Battery (installed)
- Battery Charger
- Operating Manual
- Rear fenders



# GETTING STARTED

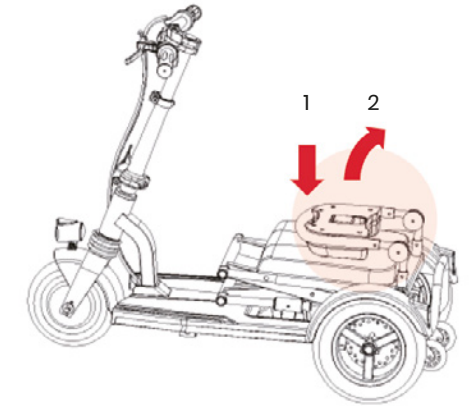- Assemble the handlebar and steering column by lifting up and locking it into position
- Ensure the steering column latch is pressed firmly down to allow for the column to lock into position. Once the steering column is flush, lock the steering column latch by pulling up.

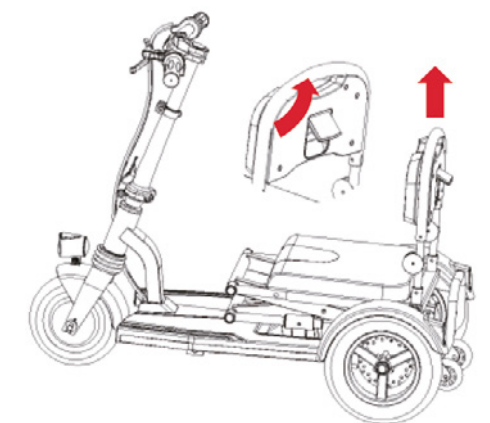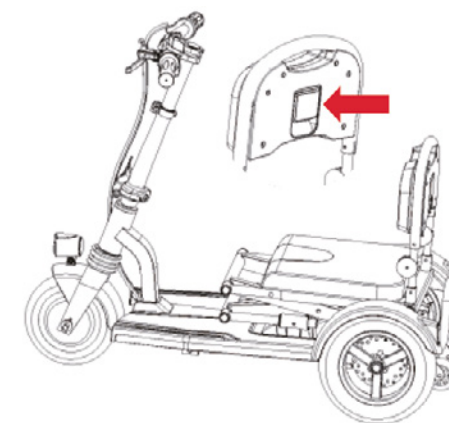
- Press down firmly on backrest to release it from locked position  
\*YOU WILL BE UNABLE TO FLIP THE BACKREST INTO POSITION UNLESS YOU PRESS DOWN FIRST TO RELEASE THE LOCKING MECHANISM, DO NOT FORCE THE BACKREST INTO POSITION\*  
• Lift the backrest to a 90 degree position for sitting


- Unlock the seat by pulling the latch located on the back of the backrest and lift the seat vertically up into seating position

# GETTING STARTED

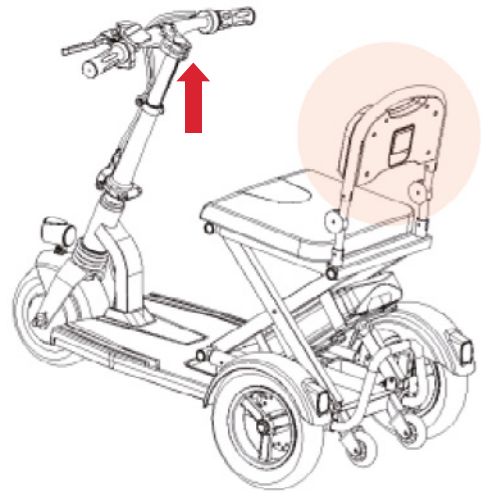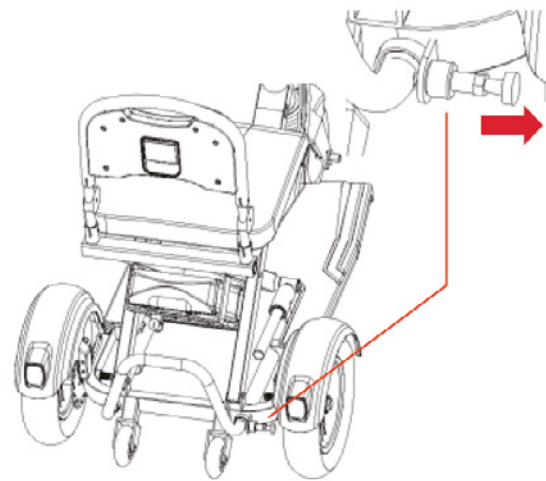- Raise the seat until there is a \*click\* indicating the seat has reached its fully extended locked position then press down on the backrest latch to lock it back into position.
- Adjust the height of the steering column as required

- To utilize the luggage carrier, loosen the locking pin and extend the carrier bar to required length. Ensure locking pin fastens into locked position when luggage rack extended

# GETTING STARTED

- Locate the rear fenders shipped with the CRUZ mobility scooter
- Next, remove the loosely installed two screws and one bolt from the fender position at the rear wheel

- Slide the rear fender on the fender holder and into position
- Ensure the reflector is facing the rear of the CRUZ
- Fasten the screws and bolt to secure the fender
- Repeat for second rear fender

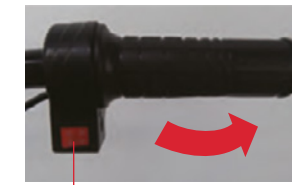
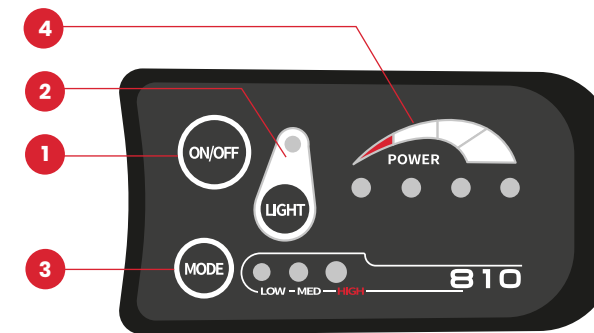
# GETTING STARTED

- To utilize the grab bar, power OFF your CRUZ mobility scooter
- Lift up the front wheel to gain access beneath the scooter
- Release the grab bar by pulling out the release pin located underneath the scooter
- Extend the grab bar until it is locked into position with a “click”
- Utilize the grab bar to pull the scooter similar to a suitcase

# GETTING STARTED

5

Drive. Rotate handle grip to accelerate.

6

Brake. Squeeze hand brake to brake.

-  ON/OFF System Power. Press to turn ON. When power turned ON, all lights will flash immediately. To turn OFF, press and hold for 2-seconds.
-  LED Headlight. When power is ON, press the LIGHT switch to turn on the headlight. Press again to turn OFF. Only works when power to scooter is on.
-   Power Level. Three unique power levels. LOW (first light), MEDIUM (second light), HIGH (third light). Corresponds to 4, 8, 12 kph at top speeds. Choose HIGH for climbing steep hills.
-  Battery Life Indicator. Four lights indicates greater than 75% battery life remaining, three lights indicates 50% remaining, two lights indicates 25% remaining, one light means low battery life remaining, charge immediately.
-  Drive Gear. D means drive is in forward mode. R means drive is in reverse mode.
-  Parking Brake. Depress to lock the brakes in place.

**NOTICE:** You may leave the grab bar extended while driving, however, be cautious as it reduces the ground clearance of the CRUZ and hitting obstacles with grab bar extended may bring CRUZ to a sudden stop

# OPERATING

- While sitting comfortably in the CRUZ scooter, press the controller button ON to initiate power
- Next select MODE to choose speed, we recommend beginning on the lowest speed level
- Check that there is enough battery power on the POWER indicator to reach your destination
- Ensure the CRUZ is in drive mode (D)
- Rotate right hand grip slowly to accelerate the scooter smoothly
- Engage hand brake on left side to slow the scooters movement

# LOADING & TRANSPORTING

TO LOAD THE SCOOTER INTO A VEHICLE WE RECOMMEND THE FOLLOWING STEPS:

- Power OFF your Cruz mobility scooter
- Release the grab bar beneath the scooter (image 1)
- Pull the scooter near to the vehicle with the front wheel facing the rear of the vehicle
- Squeeze the hand brake and engage the park brake to lock the rear wheels (image 2), check that the rear wheels are locked and parking brake is truly engaged

- Extend the luggage bar at the back of the scooter to help leverage the CRUZ into the vehicle (image 3)
- Lift the front wheel into the back of the vehicle, check to make sure it is secure and will not roll backwards. We recommend placing a blanket down over the back of the vehicle to avoid scratches (image 4)
- Grab the luggage bar at the back of the scooter and lift it into the vehicle while rolling the front wheel into position (image 5)

**WARNING:** Failure to engage the park brake for the rear wheels and remove the luggage rack may cause the scooter to roll back unintentionally when you are trying to load in the suggested manner

# BATTERY MAINTENANCE

Lithium-Ion rechargeable batteries require routine maintenance and care in their use and handling. Read and follow the guidelines in this document to safely use Lithium-Ion batteries and achieve the maximum battery life span.

## OVERVIEW

Do not leave batteries unused for extended periods of time, either in the product or in storage. When a battery has been unused for 6 months, check the charge status and charge or dispose of the battery as appropriate.

The typical estimated life of a Lithium-Ion battery is about three years or ~500 charge cycles, whichever occurs first. One charge cycle is a period of use from fully charged, to fully discharged, and fully recharged again. Use a three-year life expectancy for batteries that do not run through complete charge cycles.

Rechargeable Lithium-Ion batteries have a limited life and will gradually lose their capacity to hold a charge. This loss of capacity (aging) is irreversible. As the battery loses capacity, the length of time it will power the product (run time) decreases.

Lithium-Ion batteries continue to slowly discharge (self-discharge) when not in use or while in storage. Routinely check the battery's charge status.

**Use only the provided battery(s) in your product.**

**Use only the provided charger to charge your battery(s).**

## BATTERY MAINTENANCE

Observe and note the run time that a new fully-charged battery provides for powering your product. Use this new battery run time as a basis to compare run times for older batteries. The run time of your battery will vary depending on the product's configuration and the applications that you run.

Routinely check the battery's charge status.

Carefully monitor batteries that are approaching the end of their estimated life.

Consider replacing the battery with a new one if you note either of the following conditions:

- The battery run time drops below about 80% of the original run time.
- The battery charge time increases significantly.

If a battery is stored or otherwise unused for an extended period, be sure to follow the storage instructions in this document. If you do not follow the instructions, and the battery has no charge remaining when you check it, consider it to be damaged. Do not attempt to recharge it or to use it. Replace it with a new battery.

## CHARGING

Always follow the charging instructions provided with your product. Refer to your product's user manual and/or online help for detailed information about charging its battery.

**Use only the provided charger to charge your battery.**

# BATTERY MAINTENANCE

## STORAGE

- Charge or discharge the battery to approximately 50% of capacity before storage.
- Charge the battery to approximately 50% of capacity at least once every six months.
- Remove the battery and store it separately from the product.
- Store the battery at temperatures between 5 °C and 20 °C (41 °F and 68 °F).

## HANDLING PRECAUTIONS

- Do not disassemble, crush, or puncture a battery.
- Do not short the external contacts on a battery.
- Do not expose a battery to fire or water.
- Do not expose a battery to temperatures above 60 °C (140 °F).
- Keep the battery away from children.
- Avoid exposing the battery to excessive shock or vibration.
- Do not use a damaged battery.
- If a battery pack has leaking fluids, do not touch any fluids.
- Dispose of a leaking battery pack (see Disposal and Recycling in this document).
- In case of eye contact with fluid, do not rub eyes. Immediately flush eyes thoroughly with water for at least 15 minutes, lifting upper

and lower lids, until no evidence of the fluid remains. Seek medical attention.

- Immediately discontinue use of the battery if, while using, charging or storing the battery, the battery emits an unusual smell, feels hot, changes color, or shape, or appears abnormal in any way. Contact Travel Buggy if any of these problems are observed

## TRANSPORTATION

- Always check all applicable local, national, and international regulations before transporting a Lithium-Ion battery.
- Transporting an end-of-life, damaged, or recalled battery may, in certain cases, be specifically limited or prohibited.

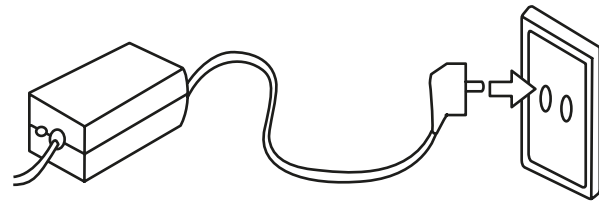
## DISPOSAL AND RECYCLING

- Lithium-Ion batteries are subject to disposal and recycling regulations that vary by country and region. Always check and follow your applicable regulations before disposing of any battery. Contact Rechargeable Battery Recycling Corporation ([www.rbrc.org](http://www.rbrc.org)) for U.S.A. and Canada, or your local battery recycling organization.
- Many countries prohibit the disposal of waste electronic equipment in standard waste receptacles.
- Place only discharged batteries in a battery collection container. Use electrical tape or other approved covering over the battery connection points to prevent short circuits.

**NOTE:** When you troubleshoot battery issues for dual battery configurations, test one battery and one battery slot at a time. A defective battery can prevent the battery in the opposite slot from charging, leaving you with two uncharged batteries.

**NOTE:** The battery self-discharges during storage. Higher temperatures (above 20 °C or 68 °F) reduce the battery storage life.

# CHARGING

- Fully Charged
- Charging

- We recommend removing the battery from your CRUZ foldable mobility scooter in order to charge it, rather than charging while in the scooter
- To remove the battery, utilize the provided key to unlock the battery from its holster
- Please ensure the charger and battery are on a flat surface and keep in a well ventilated area
- Connect the output plug of the charger into the charging socket of the battery, second connect the input plug of charger with your local power socket
- Ensure the plugs and power socket are safe
- When charging, the indicator light on the power block will be red. After 4 - 6 hours of charging (depending on battery level), the indicator light will change from red to green, indicating the battery is now fully charged

# MAINTENANCE & CARE

- Keep your CRUZ foldable mobility scooter clean from debris, moisture and dirt. Wipe clean often
- Visually inspect for worn, damaged or broken components. Special attention should be paid to the wheels and axles
- Check the condition of the warning labels. They are for the safety of the operator. REPLACE them if necessary.

## LUBRICATION

- According to use condition, please clean or lubricate the front axle, rear axle, front fork and rotating parts after 180 days
- Molybdenum disulfide grease is recommended
- Your CRUZ foldable mobility scooter has electric hub motors that do not require lubrication, please do not lubricate the motors

**⚠ WARNING:** TO REDUCE THE RISK OF INJURY, CHARGE LITHIUM-ION PROVIDED BATTERIES ONLY WITH PROVIDED LITHIUM-ION BATTERY CHARGER. Other types of chargers may cause the battery to generate heat, explode or ignite potentially causing personal injury or damage.

**⚠ WARNING:** Do not leave plugged in more than 12 hours as it will effect the longevity of the battery pack.

**⚠ WARNING:** Do not turn the scooter on if charging battery in scooter.

**⚠ WARNING:** Use of different chargers may cause serious injury to you and may damage the battery pack. Any consequences caused by this are the sole responsibility of the user.

# LIMITED WARRANTY

---

The purchaser of this product is entitled to a limited warranty as offered by Travel Buggy and its affiliates on the following components and timeframes from the date of purchase:

**2 YEAR WARRANTY** – Main Frame

**1 YEAR WARRANTY** – Motor, Control System (CPU)

**6 MONTHS WARRANTY** – Battery/Cables

**90 DAY LIMITED WARRANTY** – On the wearing parts, such as the rubber tires, armrest padding, joystick button covers, controls knobs

**30 DAY LIMITED WARRANTY** – Soft goods, such as seat cushion and backrest padding.

This warranty is valid for the replacement of dysfunctional parts only. Any parts under warranty will be replaced and shipped to your door. Any service and labor fees, if applicable, to replace parts under warranty must be paid by the user.

Due to its straightforward design, most parts can be easily exchanged by the end user without a professional service tech required. However, it is always recommended you seek professional help for maintenance and service, to make sure the work is done properly.

The warranty only covers NON-DELIBERATE Damage.

**The warranty does not cover:**

- Products damaged by user negligence.
- Products damaged accidentally.
- Products damaged intentionally.
- Products that have been subjected to negligence.
- Products that have been subjected to abuse.
- Products that have been improperly stored.
- Products that have been improperly handled.
- Products that have been improperly operated.
- Products that have experienced general misuse.
- Products that have been modified in an unauthorized, unapproved way.

Warranty is non-transferable and only valid for the original wheelchair purchaser.

Travel Buggy will, at its sole discretion, replace any warrantable item that fails within the warranty period.

# CONTACT & SUPPORT

---

You will find the operation and care of your CRUZ very simple and easy, but should you have any questions please contact Travel Buggy.

**OUR CUSTOMER HOTLINE**

1.888.56.BUGGY (Canada)

1.855.56.BUGGY (USA)

**EMAIL**

contact@travelbuggy.com

Additional images and videos are available on our website.

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