DASH

by Travel Buggy

Ultra Lite Power Wheelchair

TABLE OF CONTENTS

NOTICES	3	FOLDING	16
SAFETY INFORMATION	4	BATTERY MAINTENANCE	18
BATTERY SAFETY INFORMATION	5	CHARGING	20
SPECIFICATIONS	7	MAINTENANCE & CARE	21
GETTING STARTED	9	LIMITED WARRANTY	22
OPERATING	14	CONTACT & SUPPORT	23
FREEWHEEL (MANUAL) OPERATION	15		

NOTICES

Thank you for choosing DASH Ultra Lite Power Wheelchair by Travel Buggy, the World's lightest power wheelchair.

It is important to read through this manual before operating your DASH to become familiar with its features as well as its limitations and safety information

You will find the operation and care of your DASH very simple and easy, but should you have any questions please contact Travel Buggy

OUR CUSTOMER HOTLINE

1.888.56.BUGGY (Canada)

1.855.56.BUGGY (USA)

EMAIL

contact@travelbuggy.com

DISCLAIMER:

The notes, warnings, and cautions contained within the manual and marked by the triangular Caution Symbol  should be given special care. Users should also pay special attention to information marked in this manual beginning with **NOTICE**.

Travel Buggy, its affiliates and partners cannot be held responsible for personal injury or property damage resulting from the unsafe or improper operation or maintenance of the DASH Ultra Lite Power Wheelchair.

As part of our ongoing product improvement initiative, Travel Buggy reserves the right to change specifications and design without notice. As a result, there may be minor differences between your DASH and the photos, illustrations and instructions in this manual

SAFETY INFORMATION

- DO make certain that the seat is in a locked position when you are operating your DASH
- DO reduce your DASH's speed when you are driving around a corner
- DO proceed with caution as you approach the downgrade of a ramp or other incline and when driving near a raised surface or unprotected ledges and drop offs (curbs, porches, stairs, etc.)
- DO avoid exposure to rain, snow, ice, salt, or standing water whenever possible
- DO maintain and store in a clean and dry condition
- DO keep your hands away from the tires when driving. Be aware that loose fitting clothing can become caught in drive tires
- DO keep tools and other metal objects away from battery terminals. Contact with tools can cause electrical shock
- DO follow the recommended weight capacity (220 lbs / 100 kg)
- DO follow the recommended incline slope rating (12 degrees / 20% slope)
- DO check the battery charge to ensure there is enough charge to get to your destination before using.
- DO NOT drive your DASH across the side of a hill or diagonally up or down a hill, and do not stop (if possible) while driving up or down an incline
- DO NOT operate your new DASH for the first time without completely reading and understanding the owner's manual
- DO NOT carry additional passengers on your DASH
- DO NOT use your DASH as a seat in a moving vehicle
- DO NOT allow unsupervised children to play near the power wheelchair
- DO NOT operate your DASH while under the influence of alcohol, as this may impair your ability to drive safely
- ALL servicing must be performed by qualified personnel only. Unauthorized modifications to power wheelchair, its power system or its control system may compromise the performance and safety of the system. UNDER NO CIRCUMSTANCES should you attempt any repair or service that is not covered in this manual.
- ALWAYS ensure all safety warning labels are in place and legible.

BATTERY SAFETY INFORMATION

WHEN USING THE BATTERY

1. Lithium-ion battery packs may get hot, explode or ignite and cause serious injury if exposed to abuse conditions. Be sure to follow the safety warnings listed below:
 - DO NOT place the battery near fire or heat the battery
 - DO NOT install the battery backwards so the polarity is reversed
 - DO NOT connect the positive and negative terminal of the battery to each other with any metal object (such as a wire)
 - DO NOT carry or store the battery together with necklaces, hairpins, or other metal objects
 - DO NOT pierce the battery with nails strike the battery with a hammer, step on the battery or otherwise subject it to strong impacts or shocks
 - DO NOT solder directly onto the battery
 - DO NOT expose battery to water or salt water, or allow the battery to get wet
2. DO NOT disassemble or modify the battery. The battery contains safety and protection devices, which if damaged, may cause the battery to generate heat, explode or ignite
3. DO NOT place the battery in or near fire, on stoves or other high temperature locations. Do not place the battery in direct sunlight, or use or store batteries inside cars in hot weather. Doing so may cause the battery to generate heat, explode or ignite. Using the battery in this manner may also result in a loss of performance and a shortened life expectancy.
4. If the device is to be used by children, the caregiver should explain the contents of this document to the children and provide adequate supervision to ensure the device is being used appropriately
5. When the battery is worn out, insulate the terminals with adhesive tape or similar materials before disposal
6. Immediately discontinue use of the battery if, while using, charging or storing the battery, the battery emits an unusual smell, feels hot, changes color, or shape, or appears abnormal in any way. Contact Travel Buggy if any of these problems are observed
7. DO NOT place battery in microwave oven, high pressure containers or on induction cookware

 **WARNING:** The operator should read all of the warnings signs and instructions here and on the power wheelchair before use.

 **WARNING:** Always protect batteries from freezing temperatures and never charge a frozen battery. This will damage the battery and may cause personal injury. Attempting to charge a battery in freezing conditions does not prevent a battery from freezing.

 **WARNING:** You should not operate your scooter on public streets and roadways. Be aware that it may be difficult for traffic to see you when you are seated on your power wheelchair. Obey all local pedestrian traffic rules. Wait until your path is clear of traffic, and then proceed with extreme caution.

BATTERY SAFETY INFORMATION


CHARGING PRECAUTIONS

- BE SURE to follow the warnings listed below while charging the battery. Failure to do so may cause the battery to become hot, explode or ignite and cause serious injury.
 - When charging the battery, ONLY USE THE SPECIFIED CHARGER PROVIDED WITH THE PRODUCT
 - DO NOT attach the battery to a power supply plug or directly to a car's cigarette lighter
 - DO NOT place the battery in or near fire, or in direct sunlight. Heating the battery can damage the safety circuitry, which can cause additional heating, rupture or ignition of the battery
- DO NOT continue charging the battery if it does not recharge in the timelines specified in the operating manual
- THE TEMPERATURE range over which a battery can be charged is 0°C to 45°C (32°F to 113°F). Charging the battery at temperatures outside of that range may cause severe damage to the battery or reduce battery life expectancy

WHEN DISCHARGING THE BATTERY

- DO NOT discharge the battery using any other device except for the specified device. When the battery is used in devices other than the specified device, it may damage the battery or reduce its life expectancy. If the device causes abnormal current flow, it may cause the battery to become hot, explode or ignite and cause serious injury.
- THE TEMPERATURE range over which the battery can be discharged is -20°C to 60°C (-4°F to 140°F). Use of the battery outside this temperature range may damage the performance of the battery or may reduce its life expectancy.

SPECIFICATIONS

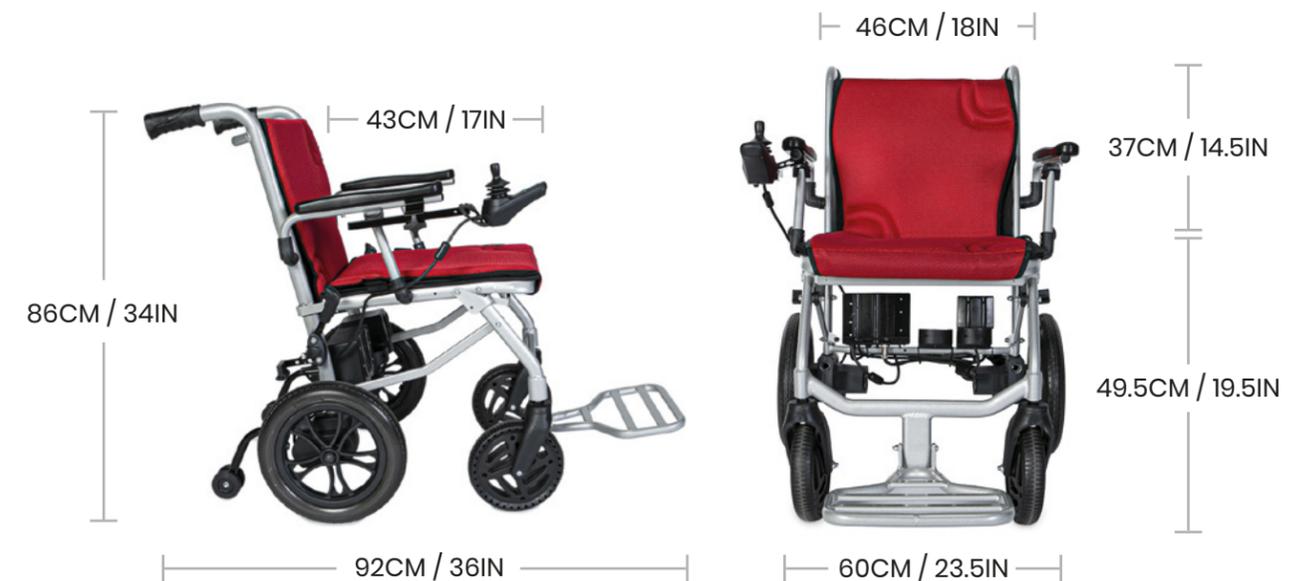SPECIFICATIONS

PERFORMANCE SPECIFICATIONS

DRIVING RANGE: 12.5 MI / 20 KM

MAXIMUM SPEED: 4 MPH / 6 KPH

DRIVE SYSTEM: TWO REAR MOTOR DRIVE

MOTOR TYPE: 250 WATT BRUSHLESS

INCLINE CAPABILITY: 12° / 20%

TURNING RADIUS: 37"

STOPPING DISTANCE: 5 FT (AT FULL SPEED)

OBSTACLE CLEARANCE: 1.5"

BRAKING SYSTEM: ELECTROMAGNETIC

MAX WEIGHT CAPACITY: 220 LBS

DIMENSIONAL SPECIFICATIONS

UNFOLDED DIMENSIONS: 36"L X 23.5"W X 34"H

CHAIR GROSS WEIGHT: 32.5 LBS (SINGLE),
35 LBS (DUAL)

CHAIR NET WEIGHT: 30 LBS (NO BATTERIES)

SEATING WIDTH: 18"

SEAT CUSHION SIZE: 17" (W) X 16.5" (D)

FOLDED DIMENSIONS: 23"L X 13"W X 36"H

FRONT WHEEL DIAMETER: 8"

REAR WHEEL DIAMETER: 12"

FLOOR TO SEAT HEIGHT: 19.5"

BACK SUPPORT HEIGHT: 14.5"

OTHER SPECIFICATIONS

BATTERY TYPE: LITHIUM-ION (24V 5.2AH)

BATTERIES PROVIDED: ONE (1), TWO (2)
OPTIONAL

BATTERY WATT HOUR: 125 WH (AIRLINE
APPROVED)

BATTERY CHARGE CYCLES: ~1,000

TIRES: NON-PUNCTURE

CHARGING TIME: 2-3 HOURS

CHARGER INPUT: 100-240V, 50/60HZ
(INTERNATIONAL)

CHARGER OUTPUT: 29.4V 2.0A

LAP BELT: PROVIDED

HORN: BUILT-IN

GETTING STARTED

- Unbox your new DASH Ultra Lite Power Wheelchair
- Remove all packaging and protective wrap
- Locate the components provided with the power wheelchair

INCLUDED:

- A. DASH power wheelchair
- B. Lithium-ion battery pack
- C. Battery charger
- D. Anti-tip wheels
- E. Joystick controller
- F. Controller attachment
- G. Cargo net
- H. Seat cushion
- I. Back rest cushion

GETTING STARTED

ASSEMBLY

- Unfold the power wheelchair until there is a “CLICK” indicating it is in the fully extended and locked seating position
- Attach the joystick controller by first locating the joystick attachment and loosely fastening it into place
- Insert joystick controller to desired length and fasten tight with the corkscrew knob. Note, the joystick length can be adjusted as required. Loop the joystick cable into the designated cable holder on the side of the power wheelchair
- Connect the joystick controller to the power source, ensure the connection is locked by rotating the coupling ring to secure the connection.


NOTICE: the Joystick Controller can be attached on the left or right side as preferred. Simply move the joystick attachment to the alternative side as desired.

GETTING STARTED

- Locate the smaller cushion of the two, the back rest cushion “I”, and adhere to Velcro at the bottom and around the top of the back rest. Widest part of the backrest cushion should be at the top of the back rest
- Next attach the seat cushion “H”. Be sure to fold the seat cushion under the front of the seat to secure it in place. The widest part of the seat cushion is nearest to the front of the chair.

GETTING STARTED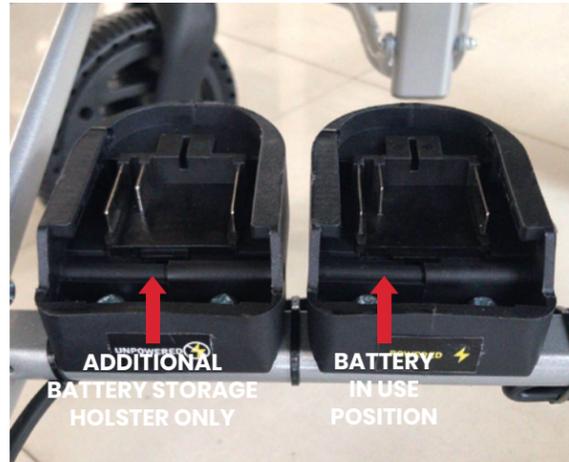

- Insert the battery into the right holster position on the back of the power wheelchair. Listen for a *CLICK* to ensure it was inserted fully.

- For additional safety, attach the anti-tip stabilizers at the rear of the wheelchair by depressing the button on the anti-tip wheels "D" and inserting into position. Ensure they are locked into the appropriate position.

NOTICE: Only the left holster utilizes the battery power. If an additional battery was purchased, it can be held on the right holster, but must be put onto left holster to be put in-use to power the wheelchair

GETTING STARTED

- Your DASH power wheelchair may have come with an optional cargo net for storage of personal items. The cargo net can be fixed beneath the seat or at the back of the chair as indicated in the photos. Attach the hooks on all four sides and ensure tension is created in the cargo net once fastened.

OPERATING

- First, turn on the systems power by locating the CPU at the rear of the power chair and pushing the switch to the ON position

○ Indicates the power to the unit is OFF
 | Indicates the power to the unit is ON

- Your battery is fully charged when all lights appear. As the battery is used, lights will disappear indicating the remaining charge life

●●●●● 80% to FULLY CHARGED
 ●●●●● 60% to 80%
 ●●●●● 40% to 60%
 ●●●●● 20% to 40%
 ●●●●● DEAD to 20%

- There are 3 speed settings as indicated on the controller


●●● FAST ●● MODERATE ● SLOW

FREEWHEEL (MANUAL) OPERATION

- DASH power wheelchair can be operated as a manual push chair as required
- First, ensure the CPU at the back of the chair is ON
- Second, ensure the joystick controller is OFF (no lights on joystick controller)
- Press the Freewheel Button at the back of the CPU as indicated in the image on the left side
- To turn back to power mode, switch the CPU power OFF then ON to reset it

FOLDING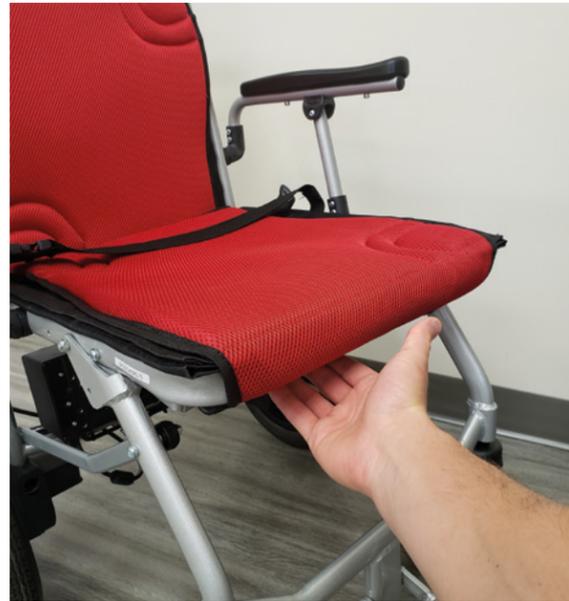

- To release the chair from the seated position for folding, reach beneath the front of the seat and push up on the release bar to unlock.


PRESS UP TO
RELEASE FOR
FOLDING

FOLDING

- Slowly fold the power chair on to the rear wheels. Ensure front wheels are facing forward to collapse fully.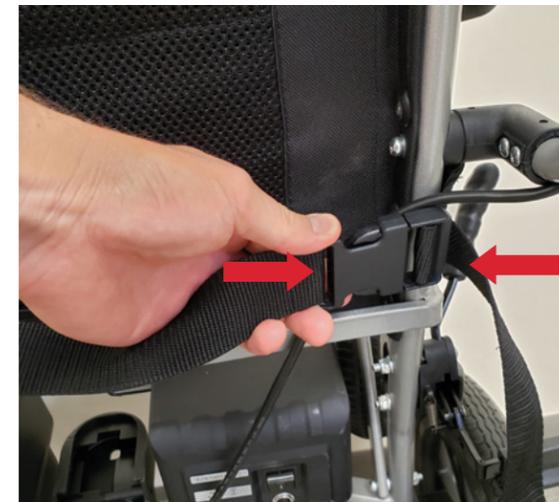

- Using the provided lap belt attached to the power chair, wrap it around the back of the chair and fasten it tight to lock the chair in the folded position for safe transport.

- Once folded, grasp the handles at the back and tilt the chair on to the small anti-tip wheels to move the chair while in folded position.

BATTERY MAINTENANCE

Lithium-Ion rechargeable batteries require routine maintenance and care in their use and handling. Read and follow the guidelines in this document to safely use Lithium-Ion batteries and achieve the maximum battery life span.

OVERVIEW

Do not leave batteries unused for extended periods of time, either in the product or in storage. When a battery has been unused for 6 months, check the charge status and charge or dispose of the battery as appropriate.

The typical estimated life of a Lithium-Ion battery is about three years or ~500 charge cycles, whichever occurs first. One charge cycle is a period of use from fully charged, to fully discharged, and fully recharged again. Use a three-year life expectancy for batteries that do not run through complete charge cycles.

Rechargeable Lithium-Ion batteries have a limited life and will gradually lose their capacity to hold a charge. This loss of capacity (aging) is irreversible. As the battery loses capacity, the length of time it will power the product (run time) decreases.

Lithium-Ion batteries continue to slowly discharge (self-discharge) when not in use or while in storage. Routinely check the battery's charge status.

Use only the provided battery(s) in your product.

Use only the provided charger to charge your battery(s).

BATTERY MAINTENANCE

Observe and note the run time that a new fully-charged battery provides for powering your product. Use this new battery run time as a basis to compare run times for older batteries. The run time of your battery will vary depending on the product's configuration and the applications that you run.

Routinely check the battery's charge status.

Carefully monitor batteries that are approaching the end of their estimated life.

Consider replacing the battery with a new one if you note either of the following conditions:

- The battery run time drops below about 80% of the original run time.
- The battery charge time increases significantly.

If a battery is stored or otherwise unused for an extended period, be sure to follow the storage instructions in this document. If you do not follow the instructions, and the battery has no charge remaining when you check it, consider it to be damaged. Do not attempt to recharge it or to use it. Replace it with a new battery.

CHARGING

Always follow the charging instructions provided with your product. Refer to your product's user manual and/or online help for detailed information about charging its battery.

Use only the provided charger to charge your battery.

BATTERY MAINTENANCE

STORAGE

- Charge or discharge the battery to approximately 50% of capacity before storage.
- Charge the battery to approximately 50% of capacity at least once every six months.
- Remove the battery and store it separately from the product.
- Store the battery at temperatures between 5 °C and 20 °C (41 °F and 68 °F).

HANDLING PRECAUTIONS

- Do not disassemble, crush, or puncture a battery.
- Do not short the external contacts on a battery.
- Do not expose a battery to fire or water.
- Do not expose a battery to temperatures above 60 °C (140 °F).
- Keep the battery away from children.
- Avoid exposing the battery to excessive shock or vibration.
- Do not use a damaged battery.
- If a battery pack has leaking fluids, do not touch any fluids.
- Dispose of a leaking battery pack (see Disposal and Recycling in this document).
- In case of eye contact with fluid, do not rub eyes. Immediately flush eyes thoroughly with water for at least 15 minutes, lifting upper

and lower lids, until no evidence of the fluid remains. Seek medical attention.

- Immediately discontinue use of the battery if, while using, charging or storing the battery, the battery emits an unusual smell, feels hot, changes color, or shape, or appears abnormal in any way. Contact Travel Buggy if any of these problems are observed

TRANSPORTATION

- Always check all applicable local, national, and international regulations before transporting a Lithium-Ion battery.
- Transporting an end-of-life, damaged, or recalled battery may, in certain cases, be specifically limited or prohibited.


DISPOSAL AND RECYCLING

- Lithium-Ion batteries are subject to disposal and recycling regulations that vary by country and region. Always check and follow your applicable regulations before disposing of any battery. Contact Rechargeable Battery Recycling Corporation (www.rbrc.org) for U.S.A. and Canada, or your local battery recycling organization.
- Many countries prohibit the disposal of waste electronic equipment in standard waste receptacles.
- Place only discharged batteries in a battery collection container. Use electrical tape or other approved covering over the battery connection points to prevent short circuits.

NOTE: When you troubleshoot battery issues for dual battery configurations, test one battery and one battery slot at a time. A defective battery can prevent the battery in the opposite slot from charging, leaving you with two uncharged batteries.

NOTE: The battery self-discharges during storage. Higher temperatures (above 20 °C or 68 °F) reduce the battery storage life.

CHARGING

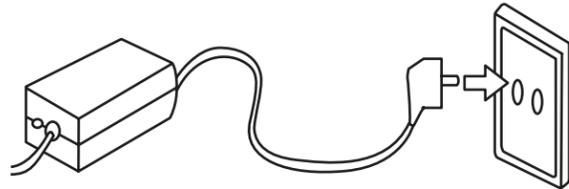- Fully Charged
- Charging

- We recommend that you remove the battery from your DASH power wheelchair before charging
- Remove the battery from the power wheelchair by depressing the red switch and pulling it off the holster
- Connect the output plug of the charger into the charging socket of the battery, second connect the input plug of charger with your local power socket
- Please ensure the charger and battery are on a flat surface and keep in a well ventilated area
- Ensure the plugs and power socket are safe.
- When charging, the indicator lights are red. After 2-3 hours charging (depending on the remain electric quantity, the indicator lights change from red to green, which means the battery is fully charged

MAINTENANCE & CARE

- Keep your DASH ultra lite power wheelchair clean from debris, moisture and dirt. Wipe clean often
- Visually inspect for worn, damaged or broken components. Special attention should be paid to the wheels and motors
- Check the condition of the warning labels. They are for the safety of the operator. REPLACE them if necessary.

LUBRICATION

- According to use condition, please clean or lubricate the front axle, rear axle, front fork and rotating parts after 180 days
- Molybdenum disulfide grease is recommended
- Your DASH ultra lite power wheelchair has electric hub motors that do not require lubrication, please do not lubricate the motors

⚠ **WARNING:** TO REDUCE THE RISK OF INJURY, CHARGE LITHIUM-ION PROVIDED BATTERIES ONLY WITH PROVIDED LITHIUM-ION BATTERY CHARGER. Other types of chargers may cause the battery to generate heat, explode or ignite potentially causing personal injury or damage.

⚠ **WARNING:** Do not leave plugged in more than 12 hours as it will effect the longevity of the battery pack.

⚠ **WARNING:** Do not turn the power wheelchair on if charging battery in power wheelchair.

⚠ **WARNING:** Use of different chargers may cause serious injury to you and may damage the battery pack. Any consequences caused by this are the sole responsibility of the user.

LIMITED WARRANTY

The purchaser of this product is entitled to a limited warranty as offered by Travel Buggy and its affiliates on the following components and timeframes from the date of purchase:

2 YEAR WARRANTY – Main Frame

1 YEAR WARRANTY – Motor, Control System (CPU)

6 MONTHS WARRANTY – Battery/Cables

90 DAY LIMITED WARRANTY – On the wearing parts, such as the tires, armrest padding, joystick button covers, controls knobs

30 DAY LIMITED WARRANTY – Soft goods, such as seat cushion and backrest padding.

This warranty is valid for the replacement of dysfunctional parts only. Any parts under warranty will be replaced and shipped to your door. Any service and labor fees, if applicable, to replace parts under warranty must be paid by the user.

Due to its straightforward design, most parts can be easily exchanged by the end user without a professional service tech required. However, it is always recommended you seek professional help for maintenance and service, to make sure the work is done properly.

The warranty only covers NON-DELIBERATE Damage.

The warranty does not cover:

- Products damaged by user negligence.
- Products damaged accidentally.
- Products damaged intentionally.
- Products that have been subjected to negligence.
- Products that have been subjected to abuse.
- Products that have been improperly stored.
- Products that have been improperly handled.
- Products that have been improperly operated.
- Products that have experienced general misuse.
- Products that have been modified in an unauthorized, unapproved way.

Warranty is non-transferable and only valid for the original wheelchair purchaser.

Travel Buggy will, at its sole discretion, replace any warrantable item that fails within the warranty period.

CONTACT & SUPPORT

You will find the operation and care of your DASH very simple and easy, but should you have any questions please contact Travel Buggy.

OUR CUSTOMER HOTLINE

1.888.56.BUGGY (Canada)

1.855.56.BUGGY (USA)

EMAIL

contact@travelbuggy.com

Additional images and videos are available on our website.

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