# DASH

by Travel Buggy

Ultra Lite Power Wheelchair

# TABLE OF CONTENTS

---

<b>NOTICES</b>	3	<b>FOLDING</b>	15
<b>SAFETY INFORMATION</b>	4	<b>BATTERY MAINTENANCE</b>	17
<b>BATTERY SAFETY INFORMATION</b>	5	<b>CHARGING</b>	19
<b>SPECIFICATIONS</b>	7	<b>MAINTENANCE &amp; CARE</b>	20
<b>GETTING STARTED</b>	9	<b>LIMITED WARRANTY</b>	21
<b>OPERATING</b>	13	<b>CONTACT &amp; SUPPORT</b>	22
<b>FREEWHEEL (MANUAL) OPERATION</b>	14		

# NOTICES

---

Thank you for choosing DASH Ultra Lite Power Wheelchair by Travel Buggy, the World's lightest power wheelchair.

It is important to read through this manual before operating your DASH to become familiar with its features as well as its limitations and safety information

You will find the operation and care of your DASH very simple and easy, but should you have any questions please contact Travel Buggy

## OUR CUSTOMER HOTLINE

1.888.56.BUGGY (Canada)

1.855.56.BUGGY (USA)

## EMAIL

[contact@travelbuggy.com](mailto:contact@travelbuggy.com)

## DISCLAIMER:

The notes, warnings, and cautions contained within the manual and marked by the triangular Caution Symbol  should be given special care. Users should also pay special attention to information marked in this manual beginning with **NOTICE**.

Travel Buggy, its affiliates and partners cannot be held responsible for personal injury or property damage resulting from the unsafe or improper operation or maintenance of the DASH Ultra Lite Power Wheelchair.

As part of our ongoing product improvement initiative, Travel Buggy reserves the right to change specifications and design without notice. As a result, there may be minor differences between your DASH and the photos, illustrations and instructions in this manual

# SAFETY INFORMATION

---

- DO make certain that the seat is in a locked position when you are operating your DASH
- DO reduce your DASH's speed when you are driving around a corner
- DO proceed with caution as you approach the downgrade of a ramp or other incline and when driving near a raised surface or unprotected ledges and drop offs (curbs, porches, stairs, etc.)
- DO avoid exposure to rain, snow, ice, salt, or standing water whenever possible
- DO maintain and store in a clean and dry condition
- DO keep your hands away from the tires when driving. Be aware that loose fitting clothing can become caught in drive tires
- DO keep tools and other metal objects away from battery terminals. Contact with tools can cause electrical shock
- DO follow the recommended weight capacity (220 lbs / 100 kg)
- DO follow the recommended incline slope rating (12 degrees / 20% slope)
- DO check the battery charge to ensure there is enough charge to get to your destination before using.
- DO NOT drive your DASH across the side of a hill or diagonally up or down a hill, and do not stop (if possible) while driving up or down an incline
- DO NOT operate your new DASH for the first time without completely reading and understanding the owner's manual
- DO NOT carry additional passengers on your DASH
- DO NOT use your DASH as a seat in a moving vehicle
- DO NOT allow unsupervised children to play near the power wheelchair
- DO NOT operate your DASH while under the influence of alcohol, as this may impair your ability to drive safely
- ALL servicing must be performed by qualified personnel only. Unauthorized modifications to power wheelchair, its power system or its control system may compromise the performance and safety of the system. UNDER NO CIRCUMSTANCES should you attempt any repair or service that is not covered in this manual.
- ALWAYS ensure all safety warning labels are in place and legible.

 **WARNING:** The operator should read all of the warnings signs and instructions here and on the power wheelchair before use.

 **WARNING:** Always protect batteries from freezing temperatures and never charge a frozen battery. This will damage the battery and may cause personal injury. Attempting to charge a battery in freezing conditions does not prevent a battery from freezing.

 **WARNING:** You should not operate your scooter on public streets and roadways. Be aware that it may be difficult for traffic to see you when you are seated on your power wheelchair. Obey all local pedestrian traffic rules. Wait until your path is clear of traffic, and then proceed with extreme caution.

# BATTERY SAFETY INFORMATION

---

## WHEN USING THE BATTERY

1. Lithium-ion battery packs may get hot, explode or ignite and cause serious injury if exposed to abuse conditions. Be sure to follow the safety warnings listed below:
  - DO NOT place the battery near fire or heat the battery
  - DO NOT install the battery backwards so the polarity is reversed
  - DO NOT connect the positive and negative terminal of the battery to each other with any metal object (such as a wire)
  - DO NOT carry or store the battery together with necklaces, hairpins, or other metal objects
  - DO NOT pierce the battery with nails strike the battery with a hammer, step on the battery or otherwise subject it to strong impacts or shocks
  - DO NOT solder directly onto the battery
  - DO NOT expose battery to water or salt water, or allow the battery to get wet
2. DO NOT disassemble or modify the battery. The battery contains safety and protection devices, which if damaged, may cause the battery to generate heat, explode or ignite
3. DO NOT place the battery in or near fire, on stoves or other high temperature locations. Do not place the battery in direct sunlight, or use or store batteries inside cars in hot weather. Doing so may cause the battery to generate heat, explode or ignite. Using the battery in this manner may also result in a loss of performance and a shortened life expectancy.
4. If the device is to be used by children, the caregiver should explain the contents of this document to the children and provide adequate supervision to ensure the device is being used appropriately
5. When the battery is worn out, insulate the terminals with adhesive tape or similar materials before disposal
6. Immediately discontinue use of the battery if, while using, charging or storing the battery, the battery emits an unusual smell, feels hot, changes color, or shape, or appears abnormal in any way. Contact Travel Buggy if any of these problems are observed
7. DO NOT place battery in microwave oven, high pressure containers or on induction cookware

# BATTERY SAFETY INFORMATION

---

## CHARGING PRECAUTIONS

1. BE SURE to follow the warnings listed below while charging the battery. Failure to do so may cause the battery to become hot, explode or ignite and cause serious injury.
  - When charging the battery, ONLY USE THE SPECIFIED CHARGER PROVIDED WITH THE PRODUCT
  - DO NOT attach the battery to a power supply plug or directly to a car's cigarette lighter
  - DO NOT place the battery in or near fire, or in direct sunlight. Heating the battery can damage the safety circuitry, which can cause additional heating, rupture or ignition of the battery
2. DO NOT continue charging the battery if it does not recharge in the timelines specified in the operating manual
3. THE TEMPERATURE range over which a battery can be charged is 0°C to 45°C (32°F to 113°F). Charging the battery at temperatures outside of that range may cause severe damage to the battery or reduce battery life expectancy

## WHEN DISCHARGING THE BATTERY

1. DO NOT discharge the battery using any other device except for the specified device. When the battery is used in devices other than the specified device, it may damage the battery or reduce its life expectancy. If the device causes abnormal current flow, it may cause the battery to become hot, explode or ignite and cause serious injury.
2. THE TEMPERATURE range over which the battery can be discharged is -20°C to 60°C (-4°F to 140°F). Use of the battery outside this temperature range may damage the performance of the battery or may reduce its life expectancy.

# SPECIFICATIONS

# SPECIFICATIONS

---

## PERFORMANCE SPECIFICATIONS

**DRIVING RANGE:** 12.5 MI / 20 KM

---

**MAXIMUM SPEED:** 4 MPH / 6 KPH

---

**DRIVE SYSTEM:** TWO REAR MOTOR DRIVE

---

**MOTOR TYPE:** 250 WATT BRUSHLESS

---

**INCLINE CAPABILITY:** 12° / 20%

---

**TURNING RADIUS:** 37"

---

**STOPPING DISTANCE:** 5 FT (AT FULL SPEED)

---

**OBSTACLE CLEARANCE:** 1.5"

---

**BRAKING SYSTEM:** ELECTROMAGNETIC

---

**MAX WEIGHT CAPACITY:** 220 LBS

---

## DIMENSIONAL SPECIFICATIONS

**UNFOLDED DIMENSIONS:** 36"L X 23.5"W X 34"H

---

**CHAIR GROSS WEIGHT:** 32.5 LBS (SINGLE),  
35 LBS (DUAL)

---

**CHAIR NET WEIGHT:** 30 LBS (NO BATTERIES)

---

**SEATING WIDTH:** 18"

---

**SEAT CUSHION SIZE:** 17" (W) X 16.5" (D)

---

**FOLDED DIMENSIONS:** 23"L X 13"W X 36"H

---

**FRONT WHEEL DIAMETER:** 8"

---

**REAR WHEEL DIAMETER:** 12"

---

**FLOOR TO SEAT HEIGHT:** 19.5"

---

**BACK SUPPORT HEIGHT:** 14.5"

---

## OTHER SPECIFICATIONS

**BATTERY TYPE:** LITHIUM-ION (24V 5.2AH)

---

**BATTERIES PROVIDED:** ONE (1), TWO (2)  
OPTIONAL

---

**BATTERY WATT HOUR:** 125 WH (AIRLINE  
APPROVED)

---

**BATTERY CHARGE CYCLES:** ~1,000

---

**TIRES:** NON-PUNCTURE

---

**CHARGING TIME:** 2-3 HOURS

---

**CHARGER INPUT:** 100-240V, 50/60HZ  
(INTERNATIONAL)

---

**CHARGER OUTPUT:** 29.4V 2.0A

---

**LAP BELT:** PROVIDED

---

**HORN:** BUILT-IN

---

# GETTING STARTED

---

- Unbox your new DASH Ultra Lite Power Wheelchair
- Remove all packaging and protective wrap
- Locate the components provided with the power wheelchair

## INCLUDED:

- A. DASH power wheelchair
- B. Lithium-ion battery pack
- C. Battery charger
- D. Anti-tip wheels
- E. Joystick controller
- F. Back rest cushion
- G. Seat cushion

# GETTING STARTED

---

- Unfold the power wheelchair like a lawn chair until there is a “CLICK” indicating it is in the fully extended and locked seating position
- Ensure the footplate does not get stuck under the chair as you begin to unfold it

- Next, begin to raise the arm rests from behind the chair
- As they are rotated towards the front of the chair, you will hear a “CLICK” indicating they’ve been placed into the locked position.

# GETTING STARTED

- Align the steel profile (C), to the existing mount assembly already attached to the right arm rest of your DASH in order to create a square opening as shown in image 1
- Next place the bolt (A) through the profile to secure it to the existing mount, see image 2
- Lastly, loosely fasten the corkscrew (B) to the thread of the bolt (A) to secure the profile into place to create the joystick mount

- Insert the joystick into the joystick mount created above, adjust for desired length
- Fasten the corkscrew (B) to secure the joystick into position
- Next, attach the joystick connection cable by aligning the arrows on the cable and pressing together
- Secure the cable connection by rotating the coupling rings on each end of the cable

# GETTING STARTED

---

- Locate the seat cushions provided with you DASH
- Secure the TOP cushion to the Velcro on the bottom of the back rest and around to the back of the chair
- Secure the BOTTOM cushion to the Velcro at the back of the seat and under of chair

- DASH can be safely entered from the sides of the chair by raising up the arm rest
- Locate the lock mechanism at the back of the arm rest, press up to release the arm rest

# OPERATING

- First, turn on the systems power by locating the CPU at the rear of the power chair and pushing the switch to the ON position

○ Indicates the power to the unit is OFF

| Indicates the power to the unit is ON

## BATTERY INDICATOR

- Your battery is fully charged when all lights appear. As the battery is used, lights will disappear indicating the remaining charge life

● ● ● ● ● 80% to FULLY CHARGED

● ● ● ● 60% to 80%

● ● ● 40% to 60%

● ● 20% to 40%

● DEAD to 20%

- There are 3 speed settings as indicated on the controller

⚠ **NOTICE:** Battery life are estimates only. Be sure there is more than enough charge to get you to your final destination before leaving.

⚠ **NOTICE:** Batteries may not be charged upon arrival.

# FREEWHEEL (MANUAL) OPERATION

---

- DASH power wheelchair can be operated as a manual push chair as required
- Turn OFF the CPU at the back of the chair, the wheels should now roll freely with a slight bit of pressure applied
- To turn back to power mode, switch the CPU ON to reset it


# FOLDING

---

- To fold the power chair we first recommend removing the joystick and placing it on the seat of the chair (image 1)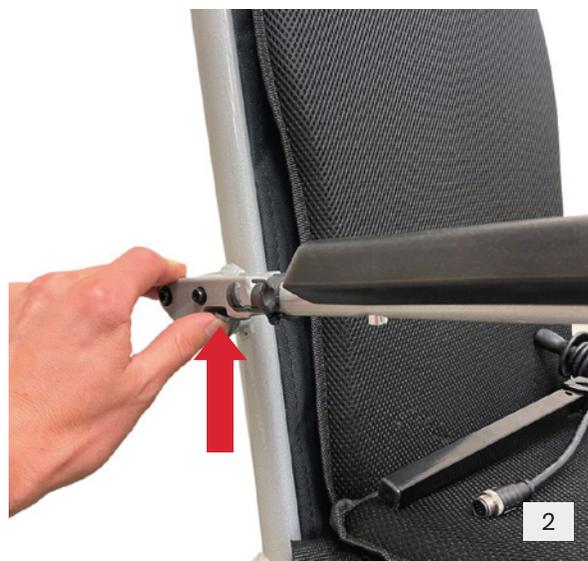

- Next, release the arm rests by pressing up on the locking mechanism at the back of the arm rest (image 2)

- Lift and rotate both arm rests all the way around to the back of the chair (image 3)

# FOLDING

- To release the chair from the seated position for folding, reach beneath the front of the seat and push up on the release bar to unlock.
- Slowly fold the power chair on to the rear wheels. Ensure front wheels are facing forward to collapse fully.

- Using the provided lap belt attached to the power chair, wrap it around the back of the chair and fasten it tight to lock the chair in the folded position for safe transport (image 5)

- Once folded, grasp the handles at the back and tilt the chair on to the small anti-tip wheels to move the chair while in folded position (image 6)

**NOTE:** IN ORDER TO FOLD THE CHAIR PROPERLY MAKE SURE THE FRONT WHEELS ARE FACING FORWARD AND THE FOOTREST IS FOLDED LAST

# BATTERY MAINTENANCE

---

Lithium-Ion rechargeable batteries require routine maintenance and care in their use and handling. Read and follow the guidelines in this document to safely use Lithium-Ion batteries and achieve the maximum battery life span.

## OVERVIEW

Do not leave batteries unused for extended periods of time, either in the product or in storage. When a battery has been unused for 6 months, check the charge status and charge or dispose of the battery as appropriate.

The typical estimated life of a Lithium-Ion battery is about three years or ~500 charge cycles, whichever occurs first. One charge cycle is a period of use from fully charged, to fully discharged, and fully recharged again. Use a three-year life expectancy for batteries that do not run through complete charge cycles.

Rechargeable Lithium-Ion batteries have a limited life and will gradually lose their capacity to hold a charge. This loss of capacity (aging) is irreversible. As the battery loses capacity, the length of time it will power the product (run time) decreases.

Lithium-Ion batteries continue to slowly discharge (self-discharge) when not in use or while in storage. Routinely check the battery's charge status.

**Use only the provided battery(s) in your product.**

**Use only the provided charger to charge your battery(s).**

## BATTERY MAINTENANCE

Observe and note the run time that a new fully-charged battery provides for powering your product. Use this new battery run time as a basis to compare run times for older batteries. The run time of your battery will vary depending on the product's configuration and the applications that you run.

Routinely check the battery's charge status.

Carefully monitor batteries that are approaching the end of their estimated life.

Consider replacing the battery with a new one if you note either of the following conditions:

- The battery run time drops below about 80% of the original run time.
- The battery charge time increases significantly.

If a battery is stored or otherwise unused for an extended period, be sure to follow the storage instructions in this document. If you do not follow the instructions, and the battery has no charge remaining when you check it, consider it to be damaged. Do not attempt to recharge it or to use it. Replace it with a new battery.

## CHARGING

Always follow the charging instructions provided with your product. Refer to your product's user manual and/or online help for detailed information about charging its battery.

**Use only the provided charger to charge your battery.**

# BATTERY MAINTENANCE

---

## STORAGE

- Charge or discharge the battery to approximately 50% of capacity before storage.
- Charge the battery to approximately 50% of capacity at least once every six months.
- Remove the battery and store it separately from the product.
- Store the battery at temperatures between 5 °C and 20 °C (41 °F and 68 °F).

## HANDLING PRECAUTIONS

- Do not disassemble, crush, or puncture a battery.
- Do not short the external contacts on a battery.
- Do not expose a battery to fire or water.
- Do not expose a battery to temperatures above 60 °C (140 °F).
- Keep the battery away from children.
- Avoid exposing the battery to excessive shock or vibration.
- Do not use a damaged battery.
- If a battery pack has leaking fluids, do not touch any fluids.
- Dispose of a leaking battery pack (see Disposal and Recycling in this document).
- In case of eye contact with fluid, do not rub eyes. Immediately flush eyes thoroughly with water for at least 15 minutes, lifting upper

and lower lids, until no evidence of the fluid remains. Seek medical attention.

- Immediately discontinue use of the battery if, while using, charging or storing the battery, the battery emits an unusual smell, feels hot, changes color, or shape, or appears abnormal in any way. Contact Travel Buggy if any of these problems are observed


## TRANSPORTATION

- Always check all applicable local, national, and international regulations before transporting a Lithium-Ion battery.
- Transporting an end-of-life, damaged, or recalled battery may, in certain cases, be specifically limited or prohibited.

## DISPOSAL AND RECYCLING

- Lithium-Ion batteries are subject to disposal and recycling regulations that vary by country and region. Always check and follow your applicable regulations before disposing of any battery. Contact Rechargeable Battery Recycling Corporation ([www.rbrc.org](http://www.rbrc.org)) for U.S.A. and Canada, or your local battery recycling organization.
- Many countries prohibit the disposal of waste electronic equipment in standard waste receptacles.
- Place only discharged batteries in a battery collection container. Use electrical tape or other approved covering over the battery connection points to prevent short circuits.

# CHARGING

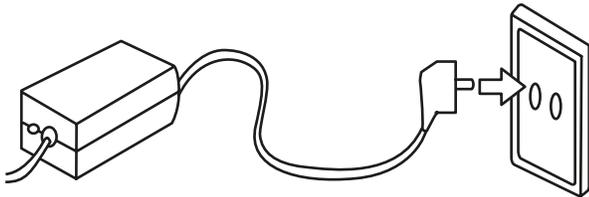- Fully Charged
- Charging

- We recommend that you remove the battery from your DASH power wheelchair before charging
- Remove the battery from the power wheelchair by depressing the red switch and pulling it off the holster
- Connect the output plug of the charger into the charging socket of the battery, second connect the input plug of charger with your local power socket
- Please ensure the charger and battery are on a flat surface and keep in a well ventilated area
- Ensure the plugs and power socket are safe.
- When charging, the indicator lights are red. After 2-3 hours charging (depending on the remain electric quantity, the indicator lights change from red to green, which means the battery is fully charged

**⚠ WARNING:** TO REDUCE THE RISK OF INJURY, CHARGE LITHIUM-ION PROVIDED BATTERIES ONLY WITH PROVIDED LITHIUM-ION BATTERY CHARGER. Other types of chargers may cause the battery to generate heat, explode or ignite potentially causing personal injury or damage.

**⚠ WARNING:** Do not leave plugged in more than 12 hours as it will effect the longevity of the battery pack.

**⚠ WARNING:** Do not turn the power wheelchair on if charging battery in power wheelchair.

**⚠ WARNING:** Use of different chargers may cause serious injury to you and may damage the battery pack. Any consequences caused by this are the sole responsibility of the user.

# MAINTENANCE & CARE

---

- Keep your DASH ultra lite power wheelchair clean from debris, moisture and dirt. Wipe clean often
- Visually inspect for worn, damaged or broken components. Special attention should be paid to the wheels and motors
- Check the condition of the warning labels. They are for the safety of the operator. REPLACE them if necessary.

## **LUBRICATION**

- According to use condition, please clean or lubricate the front axle, rear axle, front fork and rotating parts after 180 days
- Molybdenum disulfide grease is recommended
- Your DASH ultra lite power wheelchair has electric hub motors that do not require lubrication, please do not lubricate the motors

# LIMITED WARRANTY

---

The purchaser of this product is entitled to a limited warranty as offered by Travel Buggy and its affiliates on the following components and timeframes from the date of purchase:

**2 YEAR WARRANTY** - Main Frame

**1 YEAR WARRANTY** - Motor, Control System (CPU)

**6 MONTHS WARRANTY** - Battery/Cables

**90 DAY LIMITED WARRANTY** - On the wearing parts, such as the tires, armrest padding, joystick button covers, controls knobs

**30 DAY LIMITED WARRANTY** - Soft goods, such as seat cushion and backrest padding.

This warranty is valid for the replacement of dysfunctional parts only. Any parts under warranty will be replaced and shipped to your door. Any service and labor fees, if applicable, to replace parts under warranty must be paid by the user.

Due to its straightforward design, most parts can be easily exchanged by the end user without a professional service tech required. However, it is always recommended you seek professional help for maintenance and service, to make sure the work is done properly.

The warranty only covers NON-DELIBERATE Damage.

**The warranty does not cover:**

- Products damaged by user negligence.
- Products damaged accidentally.
- Products damaged intentionally.
- Products that have been subjected to negligence.
- Products that have been subjected to abuse.
- Products that have been improperly stored.
- Products that have been improperly handled.
- Products that have been improperly operated.
- Products that have experienced general misuse.
- Products that have been modified in an unauthorized, unapproved way.

Warranty is non-transferable and only valid for the original wheelchair purchaser.

Travel Buggy will, at its sole discretion, replace any warrantable item that fails within the warranty period.

# CONTACT & SUPPORT

---

You will find the operation and care of your DASH very simple and easy, but should you have any questions please contact Travel Buggy.

## **OUR CUSTOMER HOTLINE**

1.888.56.BUGGY (Canada)

1.855.56.BUGGY (USA)

## **EMAIL**

[contact@travelbuggy.com](mailto:contact@travelbuggy.com)

Additional images and videos are available on our website.

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