Health Care Pillow Instruction Sheet

10.29.2020 - FX

FOR CLINICAL USE ONLY • SOLO PARA USO CLÍNICO

Boppy® Healthcare Pillow

Durable • Wipeable • Comfortable

Usage Instructions:

To maintain cleanliness between patients, please wipe the entire pillow thoroughly with a hospital grade disinfectant cleanser.

Allow the pillow to air dry 2-3 minutes before applying the disposable slipcover or before storing.

Please dispose of used slipcovers in the waste receptacle, or as appropriate.

Please replace pillow every 6 months.

When using with infants, this pillow should be used for feeding only. Not for use in homes or child care centers.

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⚠️ WARNING

TO PREVENT SERIOUS INJURY OR DEATH

- NEVER leave baby unattended.
- Adult supervision is required.
- Improper use of this product could result in serious injury or death.
- ALWAYS support baby with your arms when holding baby.
- DO NOT use with baby while standing.

SUFFOCATION HAZARD

- NEVER allow baby to sleep on the pillow.
- DO NOT allow baby to lie face down on the pillow.
- DO NOT use in a crib, bassinet, playpen or play yard.
- NOT FOR USE on a bed, either with the baby alone or with a caregiver who may fall asleep while feeding the baby.

POSITIONAL ASPHYXIA HAZARD

- ALWAYS keep baby's airway open.
- DO NOT allow baby to be curled up in the center of the pillow area.
- DO NOT prop baby too high or too low on the pillow.

IMPORTANT: Remove all packaging and fasteners before use.

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www.boppy.com/healthcare

boppy. Approval Form:

Job Title: Health Care Pillow Instruction Sheet - FX

Date: 10.29.2020

Creator: Clarice

Clarice: _____

Teresa: _____

Cindi: _____

Beth: _____

Amy: _____

Renee: _____

External Proofread if new: yes no

Packaging Specifications:

Board Stock/Color: White

Board Stock Weight: 80lb cover

Board Stock Coating: _____

Flute Stock/Color: _____

Flute Stock Weight: _____

Flute Stock Direction: _____

Top Sheet Color: _____

Top Sheet Weight: _____

Top Sheet Coating: _____

Colors: black and PMS 152 orange

Flat size: 8.5 inches x 11 inches

21.6 cm x 28 cm

Plastic Handle/Hook: yes no

Glue: yes no

Acetate Window: yes no

Knock out window: yes no

Hang tag hole/grommet: yes no

PACKAGE NOTES/UPDATES:

10.29.2020 - new words