BLUETOOTH® BASALTHERMOMETER
INSTRUCTIONS FOR USE

 Read the instructions for use carefully, keep them for later use, and observe the information they contain.

Hello!

We are happy that you have chosen the *Ovy Bluetooth*[®] basal thermometer track to your menstrual cycle. Our mission is to support women around the globe to control their reproductive health. Ovy helps both - women who want to become pregnant and women who want to track their cycle naturally and hormone free. The Ovy App is most effective with the *Ovy Bluetooth*[®] basal thermometer and Ovy's product range - designed for the different needs during your menstrual cycle. All products are available online: www.ovyapp.com. Before you set up your new *Ovy Bluetooth*[®] basal thermometer, please install the free Ovy App from the Apple App Store (iOS) or Google Play Store (Android™) on your smartphone.

If you have any questions or feedback please feel free to contact the Ovy team at: hello@ovyapp.com.

Lots of love.
The Ovy team

Table of contents

| | |
|---|----|
| 1. Signs and symbols | 4 |
| 2. Intended use | 6 |
| 3. Important safety notes | 6 |
| 4. Instructions for use of the USB cable | 9 |
| 5. The Ovy <i>Bluetooth</i> [®] Basalthermometer | 10 |
| 6. Initial use | 11 |
| 7. Measuring your basal body temperature | 12 |
| 8. Notes on handling rechargeable batteries | 16 |
| 9. Cleaning the device | 16 |
| 10. Storage and disposal | 17 |
| 11. Error messages | 18 |
| 12. Technical data | 18 |
| 13. Ovy App | 21 |
| 14. Warranty and service | 23 |

Included in delivery

- 1x *Bluetooth*[®] basal thermometer
- 1x protection cap
- 1x USB cable
- 1x Instructions for use

1. Signs and symbols

The following symbols are used in these instructions for use, on the packaging and on the type plate of device.

| | |
|---|---|
|  | Attention |
|  | Note |
|  | Observe the instructions for use |
|  | Application part, type BF |
|  | Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive - WEEE |

| | |
|---|--|
|  | Manufacturer |
|  | European Authorised Representative for manufacturers of medical devices |
| IP 64 | Dust-proof and protected against splashes from all directions |
|  | Serial number |
|  | Permissible storage and transport temperature and humidity |
|  | Permissible operating temperature and humidity |
|  | Catalogue number |
|  | The CE labelling certifies that the product complies with the essential requirements of Directive 93/42/EEC on medical devices |
|  | Dispose of packaging in an environmentally friendly manner |

2. Intended use

The basal thermometer is a special digital thermometer designed to measure the basal temperature in women of childbearing age. It's used for daily measurement of the basal body temperature (BBT). The temperature is being transferred to the Ovy App automatically enabling the app to calculate the day of ovulation, the fertile window and the next period.

3. Important safety notes

To make optimum use of all the benefits of the thermometer, read the instructions for use carefully before initial use, retain them for future use and make them available to other users. The thermometer is intended for the measurement of the basal body temperature (BBT).

- The thermometer is designed for oral, vaginal or rectal measurement.
- Do not leave children unattended with the device.
- This thermometer is not a contraceptive.
- Check the thermometer for signs of damage or wear before each use. If the thermometer is damaged, it must no longer be used.
- The minimum duration of measurement (3 minutes) must always be observed until the acoustic signal.

- Please refer to the information field in chapter 6 how to set the signal at the end of measurement.
- The measurement is transferred directly to the Ovy App via *Bluetooth®* if *Bluetooth®* on your smartphone is turned on.
- Please only use the first measured basal temperature even though if the following ones differ from it.
- The thermometer contains delicate electronic components. It should therefore be protected against impacts, bending, high temperatures and direct sunlight.
- Prior to first use please fully charge the thermometer with the provided USB cable.
- The device is suitable for use in all environments listed in these instructions for use, including domestic environments.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the display/device.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.
- Keep portable RF communication devices (including peripheral equipment, such as antenna cables or external antennas) at least 30 cm away from all device parts,

including all cables included in delivery. Failure to comply with the above can impair the performance of the device.


- This thermometer features a flexible measuring tip and therefore provides more comfort and safety during measurement. Do not bend the tip more than 90°.
- The thermometer conducts a self-test when you switch it on. There is no need to check the measurement accuracy.
- Interfering factors such as alcohol consumption, a disrupted sleep, or sick days may distort your basal body temperature and make it unusable for evaluation. You can exclude such data in the Ovy App.
- If you obtain unusual temperature results, please contact your doctor straight away.
- This device meets the requirements of the EU Directive 93/42/EEC concerning medical devices, as well as those of the Medizinproduktegesetz (German Medical Devices Act), the European Standard EN 12470-3: Clinical thermometers – Part 3: Performance of compact
- Electrical thermometers (non-predictive and predictive) with maximum device and the European Standard EN 60601-1-2 (in accordance with IEC 61000-4-2, IEC 61000-4-8), and is subject to particular precautions with regard to electro-magnetic compatibility. The device is suitable for use in all environments listed

in these instructions for use, including domestic environments. Please note that portable and mobile HF communication systems may interfere with this device.

4. Instructions for use of the USB cable

- We suggest to fully charge the thermometer before the initial use.
- Put the USB cable into the micro USB interface on side of the device.
- Now connect the USB cable with a USB port on your laptop or computer.
- Once the charging process has successfully started CHArGE will appear on the display of the device.
- Once the thermometer is fully charged, FULL will appear on the display. Remove the USB cable.
- Use the thermometer as usual.

5. The Ovy Bluetooth® basal thermometer

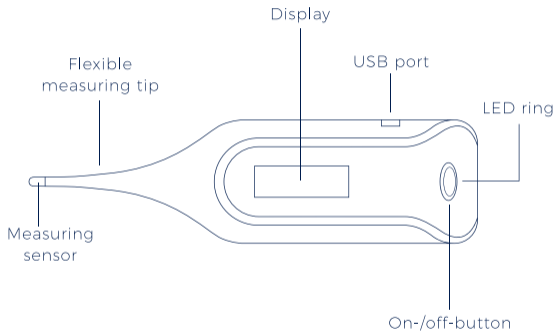6. Initial use

Within the scope of supply you will find a USB cable. Please fully charge the basal thermometer before first use. To do so please open the USB port on the device and connect the thermometer as described in chapter 4. Once the battery is fully charged FULL will appear on the display of the device.

Connecting the thermometer with a smartphone

Connect the thermometer to the Ovy App to start using the device. Download the Ovy App free of charge from the App Stores. In this manner, you can keep track of your entire cycle through your smartphone.

 Before initial use, you have to connect the thermometer with the Ovy App in order to synchronize the time. You can find further information about the Ovy App in chapter 13.

Proceed as follows:

1. Download the free Ovy App from the Apple App Store (iOS) or from Google Play (Android™).
2. Activate *Bluetooth*® on your smartphone in settings.
3. Open the Ovy App on your smartphone.
4. Press the on-/off-button on the thermometer in order to switch it on.

5. Press the on-/off-button again in order to access the measurement memory.
6. Follow the instructions in the Ovy App. Once the Ovy App is connected with the thermometer, a six-digit numerical code appears in the display of the thermometer.
7. Enter the six-digit numerical code in the smartphone. The thermometer is now connected with your smartphone.

 You can only set the temperature unit and thermometer signal via the Ovy App.

7. Measuring your basal body temperature

Take your temperature at the same time every morning, as soon as you wake up and before you get out of bed. Do not eat anything before the measurement and avoid physical exertion. If you sleep less than 4 hours, exclude the temperature on that day. Do the same on days when you are e.g. sick or had some alcohol. You will find a list with disturbing factors in the Ovy App. You can take oral, rectal or vaginal measurements, but you should always take the measurements in the same place in one cycle. We recommend taking the measurement under your tongue with your mouth closed. The thermometer should not be changed during the cycle. If you change the place of measurement, you need to track this in the app. For this cycle, a reliable evaluation is then no longer guaranteed. Your temperature is being transferred automatically after the measurement is done. The LED ring will light up blue. Your smartphone needs to be

close to the thermometer – a radius of 5 meters is recommended. Make sure your smartphone is connected to the internet and the flight mode is switched off.

Measuring

1. Remove the protective cap from the measuring tip. In order to switch on the thermometer, press the on-/off-button. A short acoustic signal confirms that the device is switched on. The thermometer first conducts a self-test for approx. 2 seconds. All segments of the display are visible during this time.

A digital display showing the number 888.888 in a seven-segment font, representing a self-test phase.

2. The last measurement then appears. Then the measurement symbol "°C" flashes and "Lo" is shown on the display. That means the thermometer is ready for use.

A digital display showing the temperature 36.50°C.A digital display showing the text "Lo°C", indicating the device is ready for use.

3. If you take your temperature orally: close your mouth and breathe calmly through your nose so that the air from your breathing does not influence the measurement. During the measuring process the current temperature is continuously shown and the "°C" symbol flashes. Measuring ends when a stable temperature

A digital display showing the temperature 36.67°C.

is reached. An acoustic signal sounds, the “°C” symbol no longer flashes and the measured temperature is displayed.

4. If *Bluetooth*® is activated on your smartphone and your smartphone is in range of at least 5 meters, the measurement is transferred to your smartphone immediately after completion of the measurement. During data transfer, the LED ring on the thermometer lights up blue. Once the data transfer has been successfully completed, the LED ring flashes twice.

 If you want to transfer the measurements at a later point in time, you have to switch to the thermometer's measurement memory.

5. The measurement time takes 3 minutes. As soon as the measurement process is done, an acoustic signal or vibration appears, which you can switch in the Ovy App as described in chapter 6. Please always wait until the end of the measurement before you remove the thermometer from the place of measurement. The last 30 measured values remain saved. The device switches off automatically around 30 seconds after the end of the measurement. You can also switch off the thermometer before this time by pressing the on-/off-button.

Memory

The thermometer has 30 memory spaces for measurements. The memory menu cannot be opened during data transfer.

1. In order to access the memory menu, press the on-/off-button to switch on the thermometer. After the self-test, briefly press the on-/off-button again. The thermometer now sends measurements to your smartphone. During data transfer, the LED ring on the thermometer lights up blue. Once the data transfer has been successfully completed, the LED ring flashes twice.
2. The measurement is displayed during data transfer. After data transfer, the memory space number first appears in the display and then the relevant measurement.
3. In order to navigate between the saved measurements, repeatedly press the on-/off-button briefly.
4. In order to leave the memory menu again, press and hold the on-/off-button for three seconds.

8. Notes on handling rechargeable batteries

- If your skin or eyes come into contact with fluid from the rechargeable battery cell, flush out the affected areas with water and seek medical assistance.
- Protect rechargeable batteries from excessive heat.
- Risk of explosion! Never throw rechargeable batteries into a fire.
- Prior to first use fully charge the rechargeable batteries as described in chapter 4.
- Only use USB cable provided in this packaging.
- Rechargeable batteries must be charged correctly prior to use as described in chapter 4.
- The instructions from the manufacturer and the specifications in these instructions for use regarding correct charging must be observed at all times.
- Fully charge the rechargeable batteries at least every 3 months.

9. Cleaning the device

- Clean the basal thermometer with a soft, clean cloth before and after each use.
- In the case of stubborn stains, you can clean the basal thermometer with a cloth dampened with water or a neutral cleaning agent. Then dry the basal thermometer with a soft, dry cloth.
- You can also disinfect the basal thermometer with 75% ethanol or isopropyl alcohol diluted with water. To disinfect, do not dip the measurement sensor (tip of the basal thermometer) in alcohol or hot water ($\geq 50^{\circ}\text{C}$ / 122°F).

- In order to prevent damage to the basal thermometer, do not use benzene, petrol or other strong thinners and solvents for cleaning.
- Do not sterilise the basal thermometer by boiling, with gas or with an autoclave.

10. Storage and disposal

Please store the thermometer in the original protective cap when it is not in use. The accuracy of this thermometer has been carefully checked and it has been developed with a long service life in mind. If the device is used for commercial medical purposes, it must be regularly tested for accuracy by appropriate means. Precise instructions for checking accuracy may be requested from the service address. For environmental reasons, do not dispose of the device in the household waste at the end of its service life. Dispose of the device at a suitable local collection or recycling point in your country. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

11. Error message

| | |
|---|---|
|  A rectangular box containing the text "Lo C" in a black, monospaced font. | The measured temperature is below 32.00°C and is therefore outside the measurement range. |
|  A rectangular box containing the text "Hi C" in a black, monospaced font. | The measured temperature is above 43.99°C and is therefore outside the measurement range. |
|  A rectangular box containing the text "Err" in a black, monospaced font. | System error. Switch the device off and wait one minute. Then switch the device on again. If the error message continues to appear, please contact the customer service department. |
|  A rectangular box containing the text "Lo BAt" in a black, monospaced font. | The battery is flat. Please charge the device. |

12. Technical data

| | |
|---------------------------------------|---|
| Type | OT35 |
| Measurement range | 32,00°C - 43,99°C |
| Measurement accuracy | ± 0,05°C between 35,00°C and 38,00°C at an ambient temperature of between 18°C - 28°C, ±0,1°C outside of this measurement and temperature range |
| Display | LED display with 5 digits |
| Storage and transportation conditions | Temperature -20°C - +55°C Rel. humidity 15% - 95% 700 - 1060 hPa ambient pressure |
| Operating conditions | Temperature 5°C - +40°C Rel. humidity 15% - 95% 700 - 1060 hPa ambient pressure |
| Battery | Lithium ion battery 3,7V/240mAh |
| Duration of measurement | 3 minutes |

| | |
|-----------------------------|---|
| Service life to be expected | 5 years |
| Battery service life | 60 measurements |
| Data transfer | 2402 MHz – 2480 MHz frequency band Max. 2.8 dBm transmission power The thermometer uses <i>Bluetooth</i> ® low energy technology Compatible with <i>Bluetooth</i> ® ≥ 4.0 Smartphones / Tablets |

The serial number is located on the device or in the battery compartment. We hereby guarantee that this product complies with the European RED Directive 2014/53/EU. The CE declaration of conformity for this product is available at www.beurer.com/web/we-landingpages/de/cedeclarationofconformity.php

13. Ovy App

The Ovy App is free for iOS and Android™. With the Ovy App and connected basal thermometer you can track your basal body temperature (BBT) and your reproductive system. Ovy helps women who want to become pregnant and women who want to track their cycle naturally. Download the Ovy App free of charge from the Apple App Store (iOS) or Google Play (Android™). The Ovy App is a medical device class 1.

System requirements

iOS ≥ 10.0

Android™ ≥ 5.0,

Bluetooth® ≥ 4.0

How does the Ovy App work?

The Ovy App predicts your menstrual cycle with the symptothermal method (NFP). Based on your basal body temperature and other body signals, Ovy calculates the day of ovulation, the fertile window and next period. For a reliable prediction of your menstrual cycle, you need to measure your waking temperature on a daily basis with the Ovy *Bluetooth*® basal thermometer and transfer the data to your smartphone. Ideally you start with the tracking on the first day of your new cycle, which is the first day of your next period. Otherwise you can start on any day during your cycle. Add other daily body signals such as cervical mucus, the state of your cervix, ovulation test results or your period, to name a few. The more data you track the

more accurate will be the calculation of your menstrual cycle. In case of interference factors (e.g. alcohol or sick days) please exclude the temperature on that day in the App. You also find a list in the Ovy App. You can enter up the basal body temperature, body signals, and interference factors later on, but make sure that measurement takes place on the same day. The Ovy App is based on a self-learning algorithm. This means, the more data you track the more accurate the calculation of your menstrual cycle.

Do you have questions about the Ovy App or Ovy products?

The Ovy team answers all questions regarding the app. Simply reach out to: hello@ovyapp.com or hit the contact button in your Ovy App profile. You can also find many explanations in the information fields of the Ovy App. Visit www.ovyapp.com to get to know Ovy's product range - designed for the different needs during your menstrual cycle.

Website: www.ovyapp.com

Email: hello@ovyapp.com

Facebook: Ovy

Instagram: @ovyapp

Twitter: @ovyapp

14. Warranty and service

In case of a claim under the warranty please contact your local dealer or the local representation mentioned at the end of these instructions for use. In case of returning the unit please add a copy of your receipt and a short report of the defect.

The following warranty terms shall apply:

- The warranty period for this product is either 5 years or – if longer – the country specific warranty period from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
- Repairs (complete unit or parts of the unit) do not extend the warranty period.
- The warranty shall not be valid for damages because of
 - a. improper treatment, e.g. nonobservance of the user instructions.
 - b. repairs or tampering by the customer or unauthorised third parties.
 - c. transport from the manufacturer to the consumer or during transport to the service centre.
 - d. The warranty shall not be valid for accessories which are subject to normal wear and tear (cuff, batteries etc.).

Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Ovy GmbH is under license. Other trademarks and trade names are those of their respective owners.

Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries. Google Play and the Google Play logo are trademarks of Google LLC. Android is a trademark of Google LLC.

In case of questions or feedback regarding the Ovy app please refer to:
hello@ovyapp.com

Distributed by: BEURER GmbH · Söflinger Str. 218 · 89077 Ulm (Germany)
Tel.: +49 (0) 731 / 39 89-144 · Fax: +49 (0) 731 / 39 89-255
www.beurer.de · Email: kd@beurer.de

Designed by: Ovy GmbH · Axel-Springer-Platz 3 · 20355 Hamburg (Germany)
www.ovyapp.com · Email: hello@ovyapp.com
(for warranty and services matters please refer to BEURER GmbH)

Medical Device Safety Service GmbH
Schiffgraben 41
30175 Hannover
Germany

JOYTECH Healthcare Co., Ltd.
No. 365, Wuzhou Road
Yuhang Economic Development Zone
Hangzhou City
311100 Zhejiang
P.R.C.

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The *Buetooth*[®] word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Ovy GmbH is under license. Other trademarks and trade names are those of their respective owners.
- (S)** Wellectronics AB