## EN Vitamin B12 Test

The cerascreen® Vitamin B12 Test is a send-in sampling kit to determine your vitamin B12 level. The test is used to analyse the concentration of the Vitamin B12 binding protein complex holotranscobalamin in human capillary blood. The sample is analysed in our specialised cerascreen® partner diagnostics laboratory.

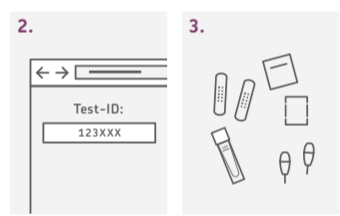
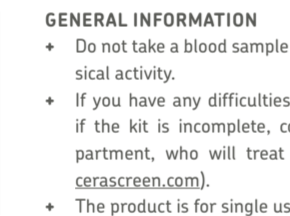
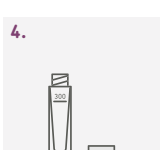
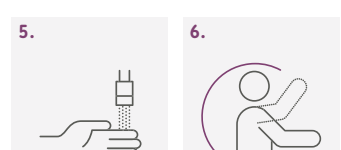
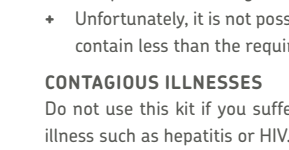
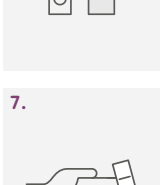
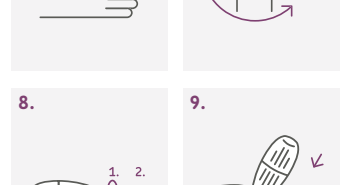
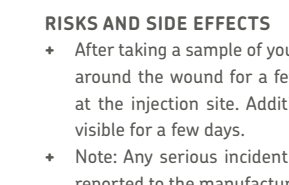
## Your test kit contains

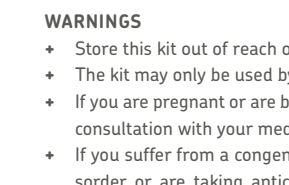
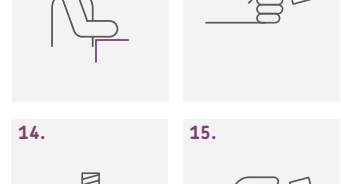
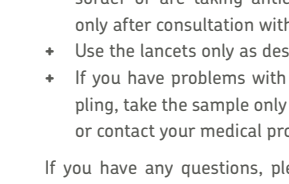
	
<b>A</b> 2 plasters C€ 1 swab C€ 1 alcohol wipe	<b>B</b> 1 transport tube with absorbent felt
	
<b>C</b> 1 microvette 300 µl (blood collection tube) C€	<b>D</b> 2 lancets C€ 0050
	
<b>E</b> 1 Return Carton	<b>F</b> 1 Adhesive point
	
<b>G</b> 1 test-ID card with sticker	

## Information and tips

- Read the instructions for use carefully before you begin and follow the listed steps.

IMPORTANT
<ul style="list-style-type: none"><li>The sample should only be taken between Sunday and Tuesday (at the latest) and sent off on the same day. Please take into account any public holidays.</li> <li>Please enter your test ID to activate the test before sending us your sample. Our laboratory needs this in order to identify your sample and send you the test result. You will find more detailed information in step 2+3.</li> <li>Please note that in the event of vitamin B12 injections, you should wait for at least four weeks after the injection before conducting this test. Supplements taken orally will not affect your test result.</li></ul>

<b>1.</b>	<b>2.</b>	<b>3.</b>
		
<b>4.</b>	<b>5.</b>	<b>6.</b>
		
<b>7.</b>	<b>8.</b>	<b>9.</b>
		

<b>10.</b>	<b>11.</b>	<b>12.</b>
		
<b>13.</b>	<b>14.</b>	<b>15.</b>
		

Do not take a blood sample immediately after strenuous physical activity.

- If you have any difficulties performing the blood sample or if the kit is incomplete, contact our customer service department, who will treat this matter confidentially ([help.cerascreen.com](mailto:help.cerascreen.com)).
- The product is for single use only.
- Unfortunately, it is not possible to analyse blood samples that contain less than the required amount of blood.

**CONTAGIOUS ILLNESSES**

- Do not use this kit if you suffer from a blood-borne contagious illness such as hepatitis or HIV.

**RISKS AND SIDE EFFECTS**

- After taking a sample of your blood, pressure sores may occur around the wound for a few days and/or a bruise may form at the injection site. Additionally, the puncture site may be visible for a few days.
- Note: Any serious incident related to the product should be reported to the manufacturer and to the competent authority of the Member State where the user is established.

**WARNINGS**

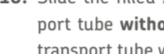
- Store this kit out of reach of children.
- The kit may only be used by adults aged 18 and older.
- If you are pregnant or are breastfeeding, use the kit only after consultation with your medical professional or pharmacist.
- If you suffer from a congenital or acquired blood-clotting disorder or are taking anticoagulant medication, use the kit only after consultation with your medical professional.
- Use the lancets only as described in the instructions for use.
- If you have problems with the sight of blood or blood sampling, take the sample only in the presence of another person or contact your medical professional or pharmacist.

If you have any questions, please visit our help website: [help.cerascreen.com](http://help.cerascreen.com)

<b>16.</b>	<b>17.</b>	<b>18.</b>
<b>19.</b>	<b>20.</b>	<b>21.</b>
<b>22.</b>	<b>23.</b>	<b>24.</b>

clean towel.

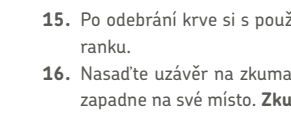
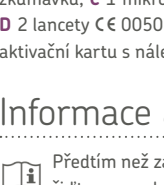
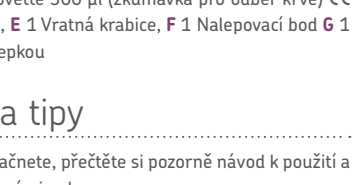
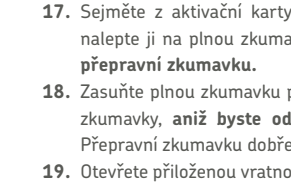
- Swing your preferred arm around in a circle 5 to 10 times, fully extended if possible, in large movements to stimulate blood circulation.
- Remove the alcohol wipe from its film and use it to disinfect the tip of the preferred finger. Wait until the finger is dry again (15 sec).
- Take the lancet. **(1)** Rotate the protective cover once around its axis. **(2)** Carefully remove the protective cover. **CAUTION:** Do not use a lancet with a loose or missing protective cover.
- Place your hand on a firm surface. Place the lancet diagonally on the tip of your finger as shown in the illustration. Only use the side of finger tip pad as the puncture site to minimize discomfort. Press the lancet firmly against the finger until it releases. **CAUTION:** The lancet can only be released once!
- Press the thumb of your other hand below the sampling spot and pull back on the skin slightly.
- Encourage blood flow by standing up and holding your hand below the heart. Gently massage your finger from the palm of your hand to your fingertip until a drop of blood forms.
- Before you start collecting the blood, take the swab (NOT the alcohol wipe) and wipe off the first drop of blood. Wait until a big drop has formed again.
- Hold your finger diagonally at the edge of the upright blood collection tube so that the blood can run into it. Continue massaging your finger to keep the blood flow going. This process may take several minutes.
- Only finish taking the sample when the blood volume reaches the 300 µl mark. Otherwise, the laboratory cannot analyse the sample.** Tap the tube gently numerous times against the table so the blood runs downwards.
- After taking the blood, use the swab and plaster to clean and cover the wound.
- Close the tube by firmly pressing the cap onto the opening until it clicks into place. **Shake the tube well.**
- Remove the square code adhesive label from the test-ID card and stick it on the filled blood collection tube—**NOT on the transport tube.**
- Slide the filled blood collection tube into the larger transport tube **without removing the absorbent felt.** Seal the transport tube well.
- Open the included return mailer box and set the clear plastic sticker aside.
- Set up the three central fastening tabs (locks) of the return box, pull them by the central tab and open the intermediate bottom. Place the blood collection tube in the intermediate bottom and close it again.
- Close the box and seal it using the clear plastic sticker included. The spot for the sticker is denoted with: "affix here". For return shipping with UPS, please go to: <https://www.cerascreen.com/return>. Check if the test has been activated and dispose of the used components in the household waste bin.

---

IMPORTANT
<ul style="list-style-type: none"><li>The sample should only be taken between Sunday and Tuesday (at the latest) and sent off on the same day. Please take into account any public holidays.</li> <li>Please enter your test ID to activate the test before sending us your sample. Our laboratory needs this in order to identify your sample and send you the test result. You will find more detailed information in step 2+3.</li> <li>Please note that in the event of vitamin B12 injections, you should wait for at least four weeks after the injection before conducting this test. Supplements taken orally will not affect your test result.</li></ul>

GENERAL INFORMATION

- Do not take a blood sample immediately after strenuous physical activity.
- If you have any difficulties performing the blood sample or if the kit is incomplete, contact our customer service department, who will treat this matter confidentially ([help.cerascreen.com](mailto:help.cerascreen.com)).
- The product is for single use only.
- Unfortunately, it is not possible to analyse blood samples that contain less than the required amount of blood.

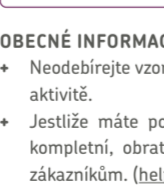
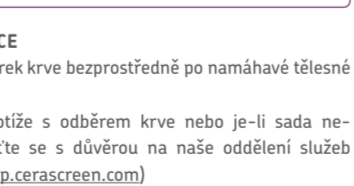
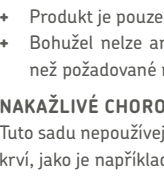
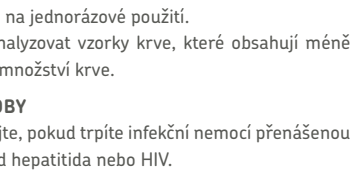
<b>4.</b>	<b>5.</b>	<b>6.</b>
		
<b>7.</b>	<b>8.</b>	<b>9.</b>
		

**CONTAGIOUS ILLNESSES**

- Do not use this kit if you suffer from a blood-borne contagious illness such as hepatitis or HIV.

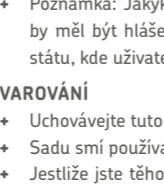
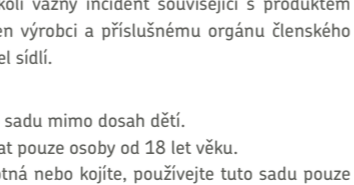
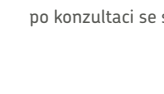
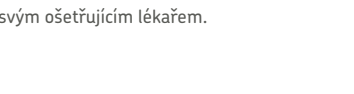
## RISKS AND SIDE EFFECTS

- After taking a sample of your blood, pressure sores may occur around the wound for a few days and/or a bruise may form at the injection site. Additionally, the puncture site may be visible for a few days.
- Note: Any serious incident related to the product should be reported to the manufacturer and to the competent authority of the Member State where the user is established.

<b>10.</b>	<b>11.</b>	<b>12.</b>
		
<b>13.</b>	<b>14.</b>	<b>15.</b>
		

Do not take a blood sample immediately after strenuous physical activity.

- If you have any difficulties performing the blood sample or if the kit is incomplete, contact our customer service department, who will treat this matter confidentially ([help.cerascreen.com](mailto:help.cerascreen.com)).
- The product is for single use only.
- Unfortunately, it is not possible to analyse blood samples that contain less than the required amount of blood.

<b>16.</b>	<b>17.</b>	<b>18.</b>
		
<b>19.</b>	<b>20.</b>	<b>21.</b>
		
<b>22.</b>	<b>23.</b>	<b>24.</b>

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