

PRONTOSAN® WOUND SPRAY

Cleansing and moistening of superficial wounds and burns as well as cleansing of entry ports for suprapubic urological catheters, PEG-/PEJ-tubes and peristomal skin areas.



The Prontosan® Wound Spray supports rapid wound healing by effective cleansing and moistening of superficial wounds and burns. Additionally, for the cleansing of entry ports such as transurethral or suprapubic urological catheters, PEG-/PEJ-tubes and peristomal skin areas.¹

INDICATIONS

For cleansing, irrigation and moistening of:

- Superficial acute and chronic wounds (e.g. skin lesions, bites, lacerations, abrasions, crush wounds, decubitus ulcers, ulcus cruris, diabetic ulcers)
- Superficial burns

Supports removal of encrusted bandages for:

- Urological catheters
- PEG-/PEJ-tubes
- Peristomal skin areas

HANDLING / APPLICATION TIP

Rinse off residues of soaps, ointments, oils or other wound cleansing substances thoroughly prior to the use of Prontosan® Wound Spray.

ADVANTAGES

- Prevents wound infection²
- Supports wound healing³
- Prevents Multi-Drug Resistant Organism contamination¹
- Prevents biofilm formation⁴
- Gentle application⁵



PRONTOSAN® WOUND SPRAY

Cleansing and moistening of superficial wounds and burns as well as cleansing of entry ports for suprapubic urological catheters, PEG-/ PEJ-tubes and peristomal skin areas.

APPLICATION

WOUND CARE:

Thoroughly spray Prontosan® Wound Spray onto the entire wound and the wound site. With this, the risk for infection is reduced and optimal healing conditions are generated. Afterwards, cover the wound with a dressing. Clotted or encrusted dressings can be moistened with Prontosan® Wound Spray in order to be easily removed and to avoid pain during removal.⁶

USE WITH TRANSURETHRAL CATHETERS:


Thoroughly moisten a sterile gauze and cleanse the catheter entry site.

PEG- / PEJ-TUBES AND SUPRAPUBICAL CATHETERS:

For cleansing of entry port, apply a sufficient amount of Prontosan® Wound Spray. For encrustations, leave on for at least 1 minute. Dry surrounding skin area by dabbing with a sterile gauze and apply a sterile cover afterwards.

STOMA CARE:

Thoroughly moisten a sufficient number of sterile compresses. Clean peristomal skin areas and dry afterwards. Subsequently, apply a new dressing. Prior to the use of Prontosan® Wound Spray, rinse off the residues of soaps, ointments, oils or other wound cleansing substances thoroughly.

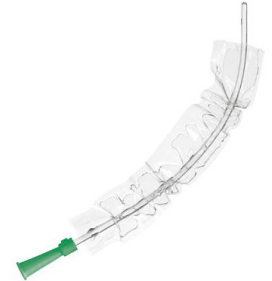

REMOVAL OF BIOFILM WAS **ONLY** DEMONSTRATED USING PRONTOSAN, COMPARED TO CURRENTLY USED WOUND IRRIGATION SOLUTIONS.^{7,8}

Prontosan®	Size	Pcs/Pack	Article number
Spray bottle	75 ml	20	400565

Composition: Purified Water, Betaine Surfactant, 0.1 % Polyaminopropyl Biguanide (Polihexanide)

FOR FURTHER INFORMATION CONTACT CUSTOMER SERVICE ON 1800 251 705

References:

1. Davis S, (2007) Determination of the effects of Prontosan® Irrigation Solution on MRSA and biofilms in a partial thickness porcine wound model.
2. Moller A, Nolte A, Kaehn K, (2004) Experiences with the use of PHMB-containing wound products in the management of chronic wounds.
3. Andriessen A, Eberlein T (2008) Assessment of a wound cleansing solution in the treatment of problem wounds.
4. Eberlein T, Assadian O. Clinical use of polihexanide on acute and chronic wounds for antisepsis and decontamination. *Skin Pharmacol Physiol* 2010;23 (Suppl 1):45-51.
5. Moeller A, Nolte A, Kaehn K. Experiences in using polihexanide containing wound products in the management of chronic wounds - results of a methodical and retrospective analysis of 953 cases. *Wound management* 2005;3:112-117
6. Instructions for use.
7. Seipp H M, Hofmann S, Hack A, et. al Efficacy of various Wound Irrigation Solutions Against Biofilms. *Zfw* 2005 4 (5) 160 - 163
8. Eberlein T, Assadian O. Clinical use of polyhexanide on acute and chronic wounds for antisepsis and decontamination. *Skin Pharmacol Physiol* 2010; 23 (suppl 1); 45-51

B. Braun Australia Pty Ltd | Bella Vista NSW | Tel.1800 251 705 | info.au@bbraun.com | www.bbraun.com.au
B. Braun New Zealand | Customer Care 0800 227 286 | Fax (09) 373 5601 | www.bbraun.co.nz

BANZ OM E0313 07/17 | © 2017 B. Braun Australia