




HOW TO USE Uro-Tainer® Twin

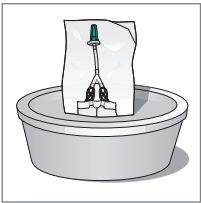


STEP 1
The Uro-Tainer® Twin sachet is supplied sterile and should not be removed from its overwrap until ready to use. The solution should be at body temperature, approximately 37 degrees. If the solution requires warming prior to use immerse the wrapped sachet into lukewarm water.

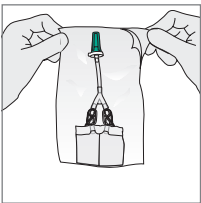


STEP 2
Peel open the plastic overwrap and remove the Uro-Tainer® Twin.

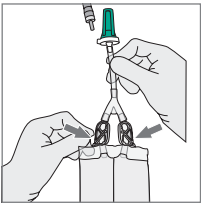


STEP 3
Close both tubes with the green and white clamps.

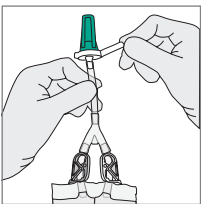


STEP 4
Remove the tear away closure ring.

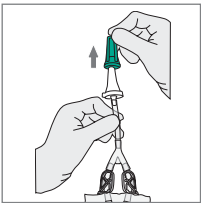


STEP 5
Twist and withdraw cap without touching sterile connector. Release the white clamp into the open position and allow a few drops of the solution to flow into the catheter to remove any air locks.

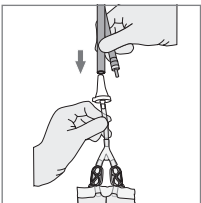


STEP 6
Insert connector into the catheter.

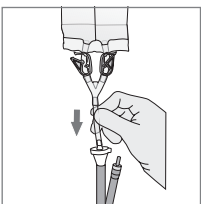


STEP 7
Holding Uro-Tainer Twin above the level of the bladder, allow the solution from the 1st chamber to flow into the bladder. To avoid any potential trauma to the bladder do not use force or agitation.

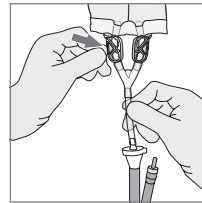


STEP 8
Close tube with white clamp.

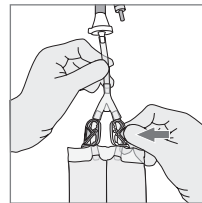


STEP 9
After 5 minutes release white clamp and hold bag below the level of the bladder, and drain the solution. Close tube with white clamp.

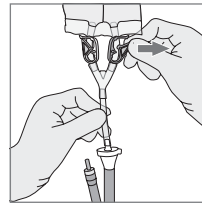


STEP 10
Now repeat the process using the 2nd chamber of solution. Hold Uro-Tainer® Twin above the level of the bladder and release the green clamp. Allow the solution to flow into the catheter. Again, do not use force or agitation.

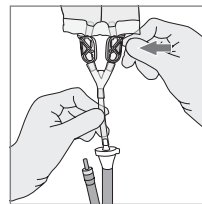

STEP 11
Close both tubes with the green and white clamps.

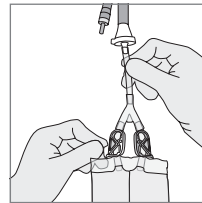

STEP 12
After 5 minutes release green clamp and hold bag below the level of the bladder, and drain the solution. Close tube with green clamp. Ensure both clamps are now closed.

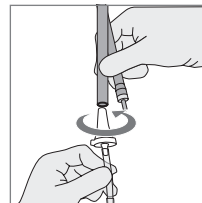

STEP 13
Remove the connector from the catheter by turning gently. Now connect the catheter to a sterile drainage bag. Discard after treatment. For single use only.