



Mepitac®

Soft silicone tape

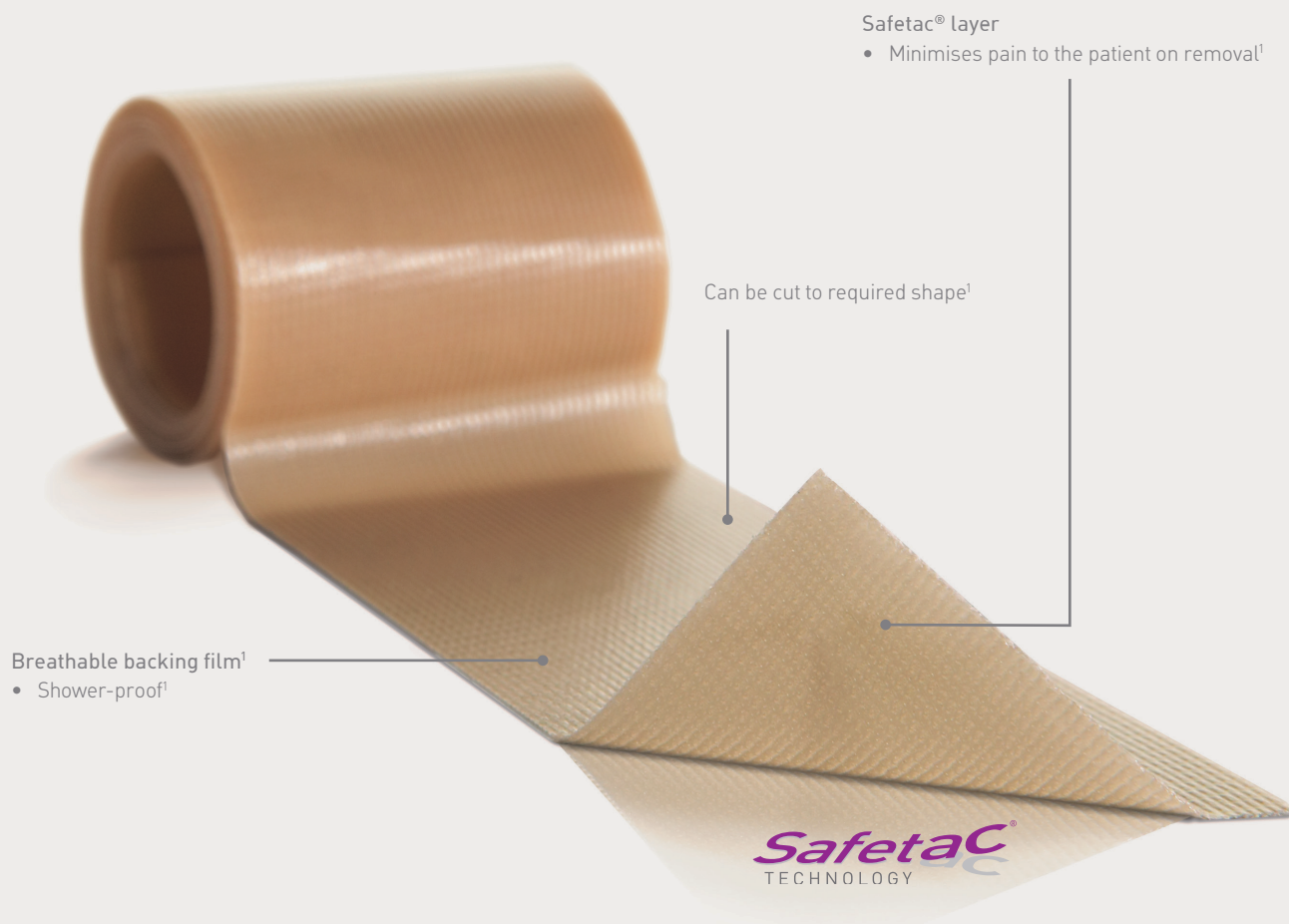

Safetac® technology. Less trauma. Less pain

In numerous randomised trials, dressings with Safetac® are clinically demonstrated to minimise damage to the wound and skin at removal. By sealing the wound margins, they help prevent maceration. With less damage to the wound and skin, pain at dressing change is minimised. Therefore, several randomised trials associate dressings with Safetac with faster healing and lower total treatment cost.¹

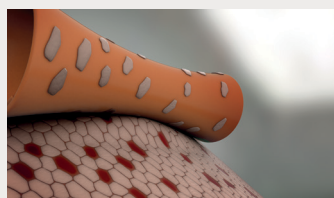

Skin stripping occurs with traditional adhesive




Safetac® minimises skin stripping


- Minimises pain at dressing change¹
- Offers gentle skin protection when used under devices, such as tubes¹

- Conforms well to body contours¹

How to use Mepitac®

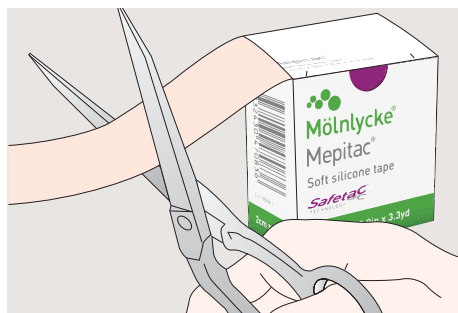

1. Make sure the skin is clean and dry and free from any ointment etc. Cut off the desired length of tape using scissors.

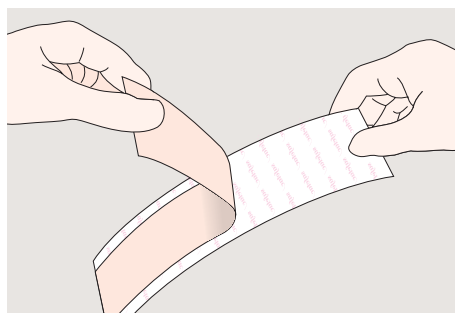

2. Remove the protective release film.



3. Apply Mepitac without stretching. If inspection or repositioning is necessary, Mepitac can be lifted and re-applied.

How Mepitac works

Mepitac is easy to use and provides reliable fixation of medical devices. Mepitac offers gentle skin protection when used under devices¹.

Mepitac can be lifted and adjusted without loss of adherent properties¹.

Benefits of Mepitac

- Minimises pain and trauma at dressing change¹
- Conforms well to body contours¹
- Minimal adhesive residue on the skin upon removal¹
- Shower-proof¹
- Breathable¹



Areas of use

Mepitac is designed for fixation of medical devices such as drains, tubes, probes, electrodes, IV cannulae and dressings. In addition, Mepitac offers gentle skin protection when used under devices e.g. tubes.



Ordering information (Non-sterile packed)

Product code	Size (cm)	Pcs/box
298300	2 × 300	1
298400	4 × 150	1

References: 1. Mölnlycke Health Care. Data on file.

www.molnlycke.com.au | www.molnlycke.co.nz

You will be required to input the following code **1FtYrV** or your **AHPRA registration number** to access our websites.

Mölnlycke, 12 Narabang Way, Level 4, Betrose, NSW, 2085. Phone 1800 005 231. New Zealand Orders & Enquiries 0800 005 231. www.molnlycke.co.nz. The Mölnlycke, Mepitac and Safetac trademarks, names and logos are registered globally to one or more of the Mölnlycke Health Care Group of Companies. ©2021 Mölnlycke Health Care. All rights reserved. AUWC_IM001860_V3


Mölnlycke®