


Mepitel® Film

SKIN PROTECTIVE FILM DRESSING

- Mepitel Film is a gentle, transparent breathable film dressing for skin protection. It can be used alone or in connection with other products.
- Mepitel Film minimises pain and trauma at dressing changes.^{1,2,3,4,5,6}
- Mepitel Film's application system is designed for ease-of-use.

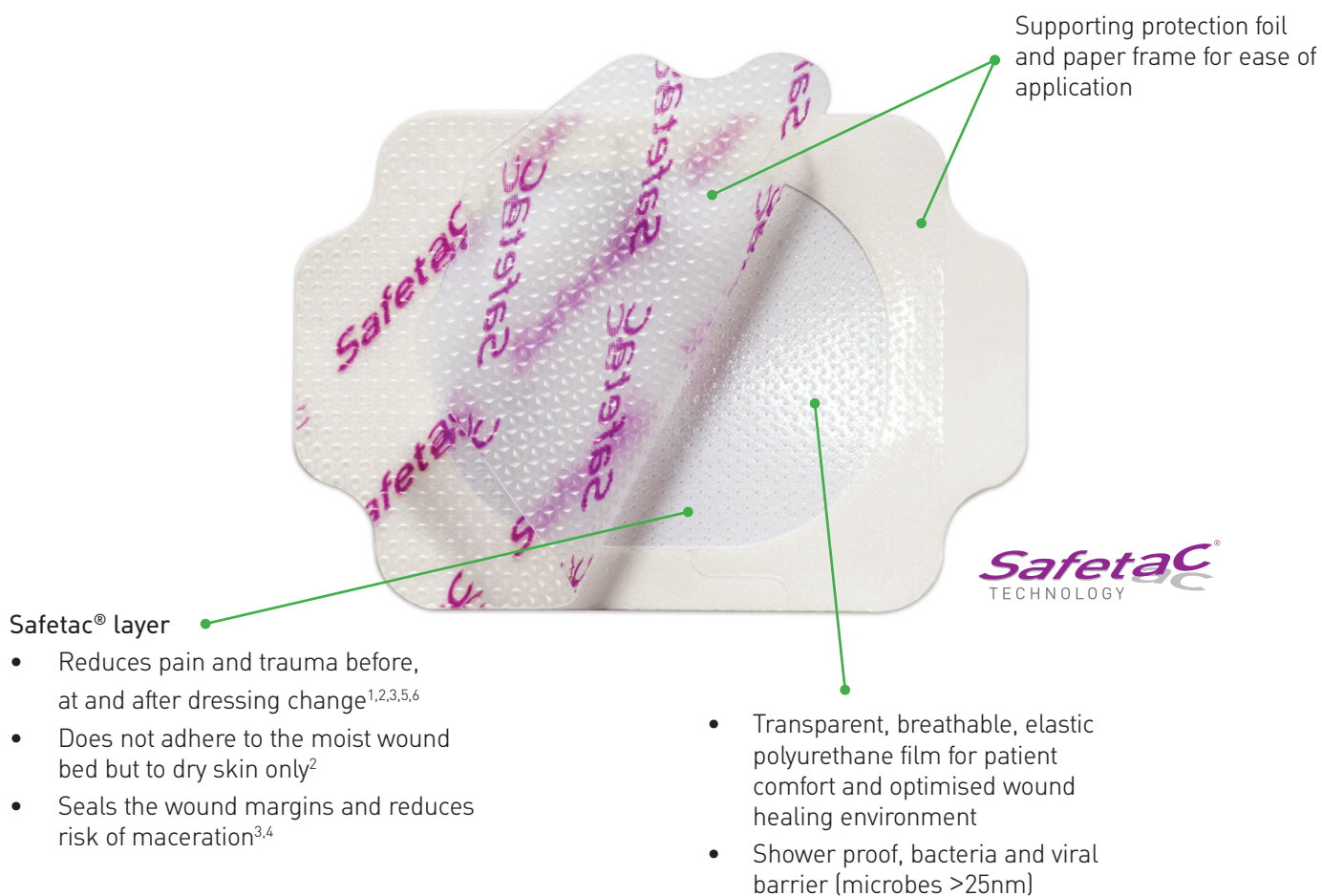


Safetac® technology


Safetac® technology. Less pain and less trauma.

Safetac is a unique proprietary adhesive technology that minimises pain to patients and trauma to wounds. Safetac technology is available exclusively on Mölnlycke Health Care dressings, including Mepilex® wound care dressings, Mepitel®, Mepiform® and Mepitac®.

For more information visit www.safetac.com

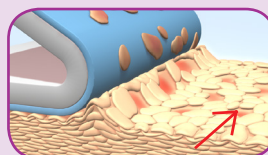

Skin stripping occurs with traditional adhesive

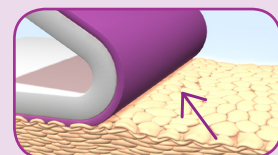

No skin stripping occurs with Safetac technology

How to use Mepitel® Film



Dry the skin/surrounding skin thoroughly. Remove the protection foil (printed Safetac®) to expose the adhesive.



Position the dressing. Mepitel Film can be repositioned as long as the paper frame is intact. Remove the white paper frame. Do not stretch the dressing when applying.



Firmly smoothen the dressing onto the skin. Mepitel Film can be removed without causing skin stripping and pain.

For best result, Mepitel Film shall overlap the dry surrounding skin by at least 1-2 cm for the sizes up to 10x12 cm and 5 cm for the larger sizes in order to fixate securely.

How Mepitel® Film works

Mepitel Film provides a flexible, transparent covering to protect the skin or wound from microbial contamination, fluid strike through and from other external contamination while conforming to surface irregularities and body contours. Mepitel® Film maintains a moist environment even though the vapour permeability allows excess moisture pass away from the skin. By the Safetac layer Mepitel Film provides instant tack adhesion that minimises the requirement of extra pressure to fixate well. The Safetac layer also ensures atraumatic dressing changes. The product does not leave residues and the adhesion level does not increase over time.

Frequency of change

Mepitel Film may be left in place for several days depending on the condition of the skin or wound, or as indicated by accepted clinical practice.

Benefits of Mepitel® Film

- Minimises pain and trauma at dressing changes
- Limits the risk for damage of newly formed tissue
- Transparent for easy inspection
- Conforms well to body contours, promoting patient comfort during wear
- Bacteria and viral barrier where microbes larger than 25nm
- Low potential for skin irritation and allergy
- Leaves no residues
- Fluid impermeable
- Shower proof

Areas of use

Mepitel Film is designed for a wide range of superficial wounds such as pressure injuries category 1 and 2, superficial skin injuries and superficial burns. Mepitel Film protects fragile and sensitive skin. Mepitel Film can also be used as a protective cover for open surgical wounds e.g abdomen, to fixate primary dressings and can be used in combination with gels and

ointments. Fixation tapes and devices can be placed on top of Mepitel Film, in this way the Mepitel Film is used as a protective landing zone for the fixation tape.

Precautions

- Mepitel Film is not a wound contact layer and does not allow wound exudate to pass through the dressing to a secondary layer.

Warnings

Mepitel Film can not be used as primary fixation for IV, cannulae, ports or other infusion and/or life sustaining devices.



MEPITEL FILM ASSORTMENT (sterile packed)

Code	Size (cm)	Pcs/shelf cont.	Pcs/transp cont.
296170	6 × 7	10	50
296270	10 × 12	10	70
296470	10 × 25	10	50
296670	15 × 20	10	70