


Mesorb[®]

Absorbent dressing with strike-through exudate barrier

Mesorb is a soft highly absorbent dressing that can retain large amounts of wound exudates. The fluid-repellent strike-through barrier helps preventing exudate to soil clothing and bed linen.

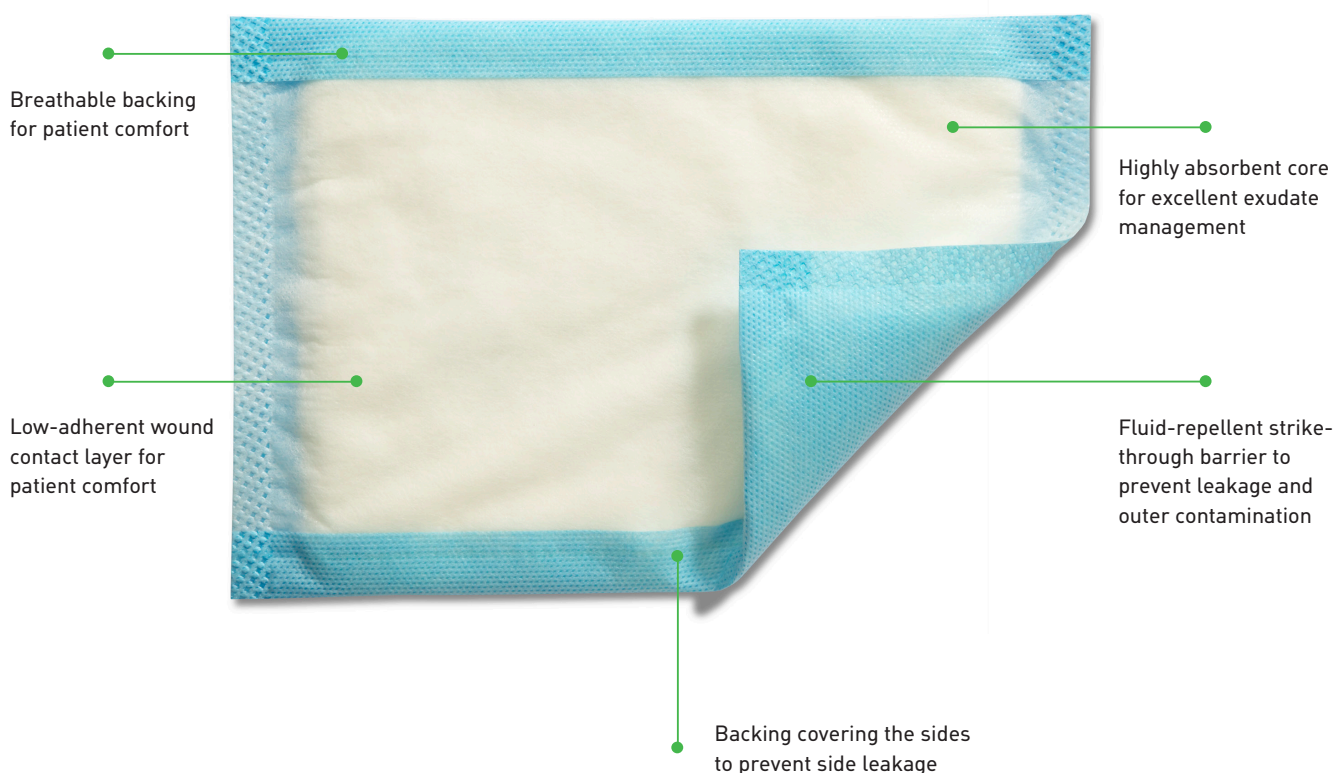

How Mesorb® works

Wound exudate is absorbed and transferred into Mesorb by the wound contact layer and the diffusion layer. The highly absorbent core absorbs exudate, while simultaneously providing a good protective and ventilating wound cushion.


The fluid-repellent backing prevents external contamination of the wound and protects clothes and bed linen from exudate strike-through. As the backing covers the sides of the dressing, it also prevents side leakage.

Areas of use


Mesorb is intended for use on moderately to heavily exuding wounds.

Mesorb can be used as a primary dressing (directly on the wound) or as a secondary dressing, where high absorbency is required (e.g. together with Mepitel® or Mepilex® Transfer).


How to use Mesorb

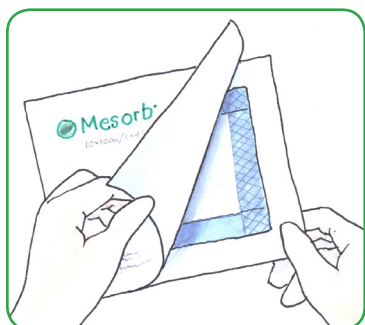

Open the package and remove the dressing.

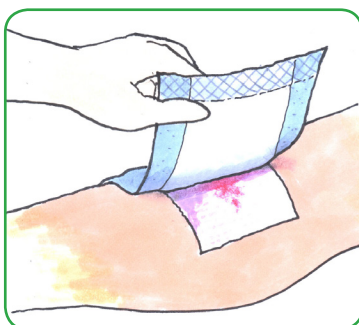

Apply Mesorb on the wound area or on the chosen primary dressing with at least 1 cm overlap on the surrounding skin.

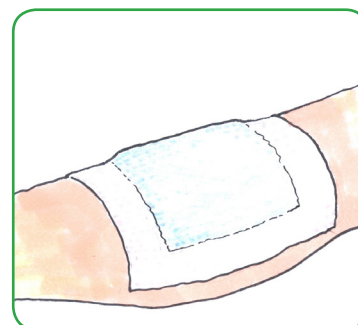

Fix in place with a suitable fixation device (e.g. Mefix®, Mepore® Film, or Tubifast™)

Frequency of change

Mesorb may be left in place for several days, depending on the wound and skin condition, or as indicated by a healthcare professional.

Benefits of Mesorb

- Highly absorbent
- Good protective cushioning
- Breathable
- Cost effective
- The unique strike-through exudate barrier reduces frequency of dressing changes and prevents soiling of clothing and bed linen

Precautions

Mesorb should not be used on infected wounds without consulting a health care professional.

Mesorb is packaged sterile in single packs

Code	Size cm	Pcs/ shelf cont.	Pcs/ transp. cont.
677001	10 x 13	50	400
677201	13 x 15	50	300
677401	10 x 23	50	300
677601	15 x 23	50	300
677701	23 x 25	30	120
677801	23 x 30	30	120



Find out more at www.molnlycke.com.au