



# ASKINA® BARRIER FILM


LASTING SKIN PROTECTION FOR STOMA AND WOUND CARE

B. Braun Wound Care. Right. From the start.

# Askina® Barrier Film

No sting, rapid drying, breathable skin barrier

## When to use Askina® Barrier Film?

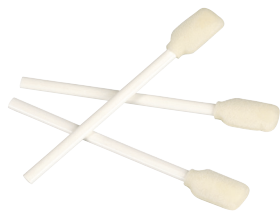

### 1 TO PREVENT skin breakdown caused by FRICTION or MOISTURE

- Protection of vulnerable areas: heels, elbows, toes, hips – from the first signs of reddening caused by friction.
- Protection of incontinent patients' skin.

### 2 TO PROTECT skin at risk from hyperhydration and/or MACERATION and EXCORIATION

- Periwound skin.
- Peristomal area.
- Around fixation sites for drainage tubes and external catheters.

### 3 TO MAINTAIN the integrity of newly healed skin to stop secondary breakdown



Skin at risk:  
incontinent patient



Incontinence dermatitis  
with skin breakdown



Peri-wound skin  
maceration




Tube insertion site

## INDICATIONS

Askina® Barrier Film is indicated for use on intact or damaged skin. Askina® Barrier Film acts as a protective barrier:

- Against irritation from bodily fluids
- For sensitive and fragile skin
- Under adhesive dressings on peri wound area to reduce disruption to newly healing tissue
- For damaged skin
- For skin tears with skin flap intact or category 1a, 1b, 2a, 2b
- To protect periwound and peristomal areas
- For small cuts and tears
- At fixation sites for drainage tubes and external catheters

## Completely sting free

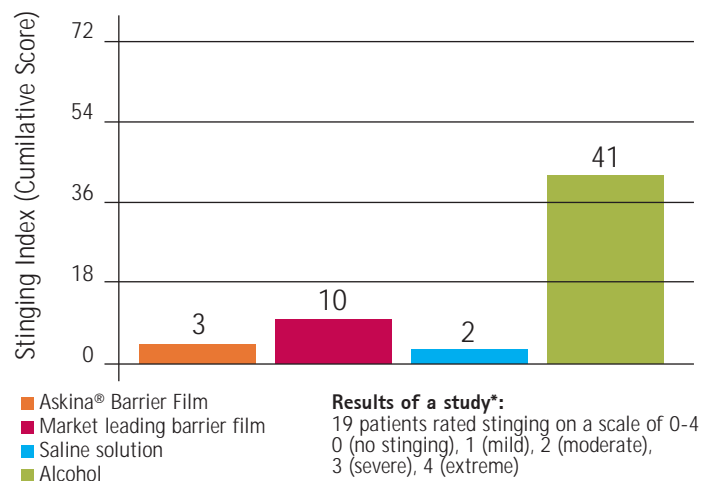

\* Data on file

# How to use Askina® Barrier Film?

## 1 Skin preparation

- Cleanse the skin with appropriate cleansing solution and dry thoroughly.

## 2 Application

- Holding the spray nozzle 10 - 15 cm from the skin, press the spray nozzle and apply a smooth coating of film over the necessary area.
- Allows the film to dry for 30 seconds.
  - If using the swab, apply to area requiring protection and allow 30 seconds to dry.

## 3 Removal

- If necessary, can be removed with Askina® Adhesive Remover or a commonly used medical adhesive remover.



## BENEFITS IN STOMACARE

Significant irritation of peristomal skin at day 1.




Signs of irritation have been almost entirely resolved at day 15.



After 15 days of treatment with Askina® Barrier Film, an improvement of the peristomal skin is evident.

Patient's main benefit in stomacare:  
Secure adhesion of pouch  
Less pain during pouch removal

## User opinion on skin protection efficacy<sup>1</sup>

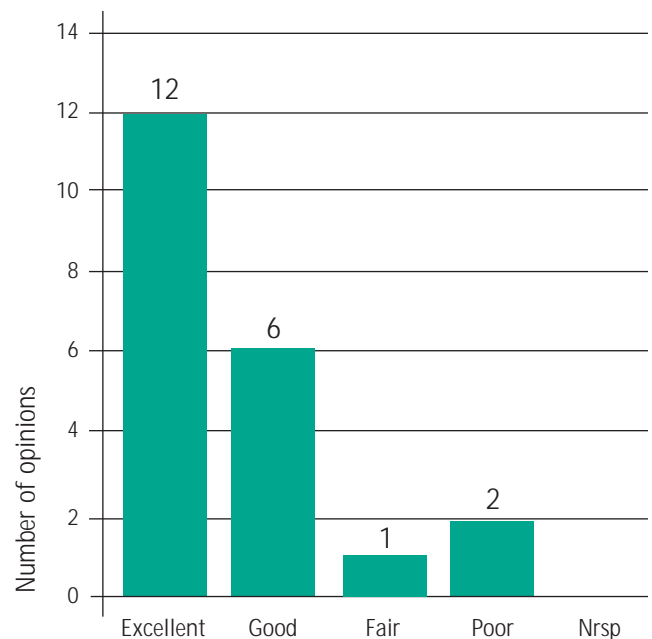

<sup>1</sup> 21 patients took part in the survey\*

In summary, 18 out of 21 patients rated the product as «Excellent» or «Good».

# Frequently Asked Questions

- **How often can Askina® Barrier Film be applied?**

The protective film will last between 48 and 72 hours, but re-application is recommended each time you change your stoma pouch. For IAD if you can still see the sheen, then there is no need to re-apply. Guidelines recommend to re-apply after every 3 washes.

- **How should I remove Askina® Barrier Film?**

Askina® Barrier Film can be removed using Askina® Adhesive Remover or a commonly used medical adhesive remover.

- **Can Askina® Barrier Film be used on damaged or sensitive skin?**

Yes, it contains no alcohol, and can be safely used to provide pain-free protection on damaged, sensitive and fragile skin.

- **Can Askina® Barrier Film be used on infants?**

Askina® Barrier Film can be used on infants over 1 month of age.

- **How long does the bottle last?**

On average, Askina® Barrier Film 28 ml spray will last for 140 pumps, which equates to one months usage.

- **Will Askina® Barrier Film dry my skin?**

Biological tests have been carried out on Askina® Barrier Film and there is scientific evidence that Askina® Barrier Film will not dry your skin. When applied to the skin it allows the skin to breath normally.

- **If my peristomal skin itches under my ostomy devices, could I use Askina® Barrier Film?**

Peristomal skin is damaged by frequent removal of ostomy devices, which can cause irritation. The use of Askina® Barrier Film will protect your skin and prevent any damage, therefore prevent irritation and itching.

## B. Braun Ordering Information

### Askina® Barrier Film

Size	Units/Box	Code
28 ml Spray Bottle	12	5036
1ml Foam Swab	25	5037

