

BREEZE & LIBERTY RAMP SAFETY & CARE

Your National Ramp Liberty or Breeze Ramp System is made in the USA. To ensure the safety of the ramp for years to come, please note the following guidelines:

- Caution should be used at all times. Proper maintenance and upkeep to the ramp surface is vital.
- **IMPORTANT:** Only use ramp system with a qualified helper.
- Always use your mobility device's lap belt.
- It's important that you refer to your equipment's Owners Guide for the proper degree of incline/decline and chair directions before attempting ramp use. Never exceed its recommendations.
- Periodically check and tighten any loose fasteners to ensure safety.
- Keep the ramp clear of dirt, leaves, and other debris that may accumulate on the surface. Sweeping or using a garden hose is recommended, but if needed a damp cloth or soft brush with soap and water can be used (avoid alkaloid detergents). Rinse well and use extra caution when the ramp surface is wet.
- If the ramp surface is covered with snow and/or ice, remove before use. Snow accumulation should be shoveled and the tread surface swept clean. Calcium chloride salt substitute may be used to melt snow and ice, however, once snow/ice has melted be sure to clean the tread surface with soap and water (avoid alkaloid detergents) to remove calcium chloride residue. Rinse well and use extra caution when the ramp surface is wet.
- Do not play on or near ramp.
- Ramp surface may become slippery when wet.
- Do not use handrails or ramp to support planters, decorations, etc.
- If any part of the ramp is damaged or loose, **DO NOT USE UNTIL REPAIRS CAN BE MADE BY A CERTIFIED INSTALLER OR OTHER QUALIFIED PERSON.**