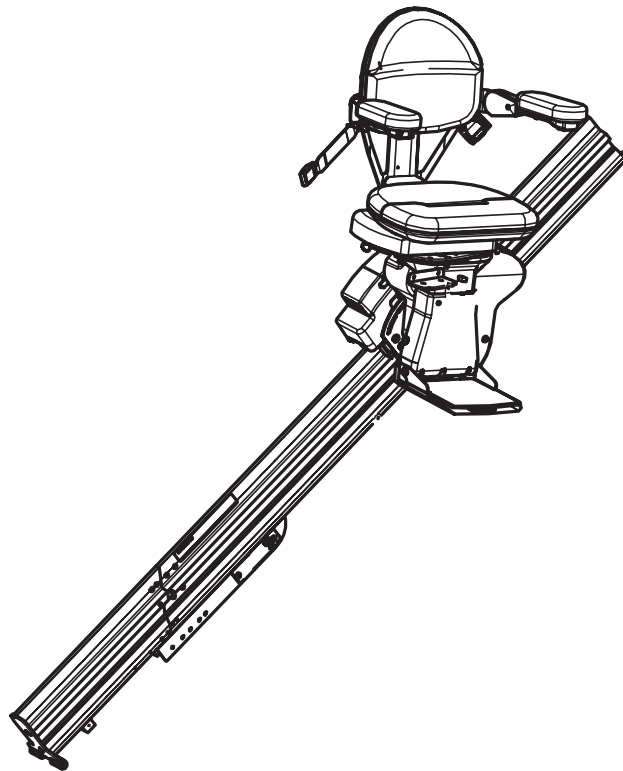
**SRE-2010**  
***ELITE***  
***Straight Rail Stairlift***

(Español 51-97)  
(Français 101-147)

**INSTALLATION MANUAL**  
**08 FEBRUARY 2019**  
**P/N 2010-I**

# REGULATORY INFORMATION AND IMPORTANT NOTES

---

## REGULATORY LISTINGS

Intertek  
4004689

Intertek  
4004689

This lift is intended to be installed in accordance with the current edition of:

- ASME A18.1: 2017 Safety Standard for Platform Lifts and Stairway Chairlifts - Units with overspeed assembly
- ASME A18.1: 2014 Safety Standard for Platform Lifts and Stairway Chairlifts - Units without overspeed assembly
- CAN/CSA-B613-00 (R2012) - Private Residence Lifts for Persons with Physical Disabilities
- ASME A17.5/CSA B44.1 - 2014 Elevator and Escalator Electrical Equipment
- NFPA 70: National Electrical Code®
- CSA C22.1 Canadian Electrical Code

Verify applicable local codes with the appropriate regulatory authorities.

## MATERIAL SAFETY DATA SHEETS

Material safety data sheets (MSDS) on materials used on this unit may be requested through the Bruno Technical Service Department.

## INSTALLING DEALER RESPONSIBILITIES

The installing dealer has responsibility to report any malfunctions, safety concerns or events to the manufacturer in a timely manner. The installing dealer also has the responsibility to ensure that the users are properly trained in the use and operation of the unit.

Bruno stairway elevators shall be put into service and regularly serviced by a Bruno authorized dealer in accordance with the information provided in installation documents provided with the device.

## FCC REGULATIONS (U.S.)

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 Subpart B of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the of the following measures:

- Reorient or relocate receiving antenna.
- Increase separation between equipment and receiver.
- Consult your dealer or an experienced radio/TV technician.

This stairlift is **FOR INDOOR USE ONLY**  
in enclosed heated locations **above 2°C (35°F)**.

The warranty for the Elite Straight Rail Stairlift is *rendered null and void* if unit is installed or maintained by anyone other than an authorized Bruno dealer.

409® is a registered trademark of the Clorox Company  
Fantastik® is a registered trademark of S.C. Johnson & Son, Inc.

## Introductory Pages

Regulatory Information and Important Notes . . . . .	2
Table of Contents . . . . .	3
Product Overview and Technical Specifications . . . . .	4
Tools, Torque Specifications, and Hardware Notes . . . . .	5

## Pre-Installation

Site Review and Preparation . . . . .	6
Rail Measurements . . . . .	6
Rail Cutting Guidelines . . . . .	7
Rail Support (Rail Bracket) Requirements . . . . .	7
Lower Rail Preparation . . . . .	8-9
Joint Prep . . . . .	9
Upper Rail Preparation . . . . .	10
Finish Preparation . . . . .	11-13

## Installation

Rail Installation . . . . .	14-17
Non-Folding Rail Installations . . . . .	14-15
Folding Rail Installations . . . . .	16-17
Carriage Installation . . . . .	18
Footrest Installation . . . . .	19
Completing Rail Installation . . . . .	20-22
Rail Cross-Pin Installation . . . . .	23-24
Make Charger Connections . . . . .	24-25
Non-Folding Rail Installations . . . . .	25
Folding Rail Installations . . . . .	26
Seat Installation . . . . .	26-28
Lubricate Gear Rack . . . . .	29
Battery Charger Installation . . . . .	30
Fit & Finish . . . . .	31
Customer Education . . . . .	32-33

## Electrical

Reprogramming Remote Transmitters . . . . .	34-35
Single-User vs. Multi-User Diagnostic Mode . . . . .	36
Battery Charger LED Information . . . . .	37

## Post-Installation Adjustments

Footrest Height Adjustment . . . . .	38
Drive Adjustment . . . . .	39
Carriage Angle Adjustment . . . . .	40
Seat Swivel Stop Adjustment . . . . .	41
Carriage Adjustment . . . . .	41
Arm Extension Adjustment . . . . .	42

## Troubleshooting

Diagnostic Codes . . . . .	44-45
Reset Folding Rail Joint . . . . .	46
Overspeed for SRE-2010C & SRE-2010OS Installations . . . . .	47

## Limited Warranty

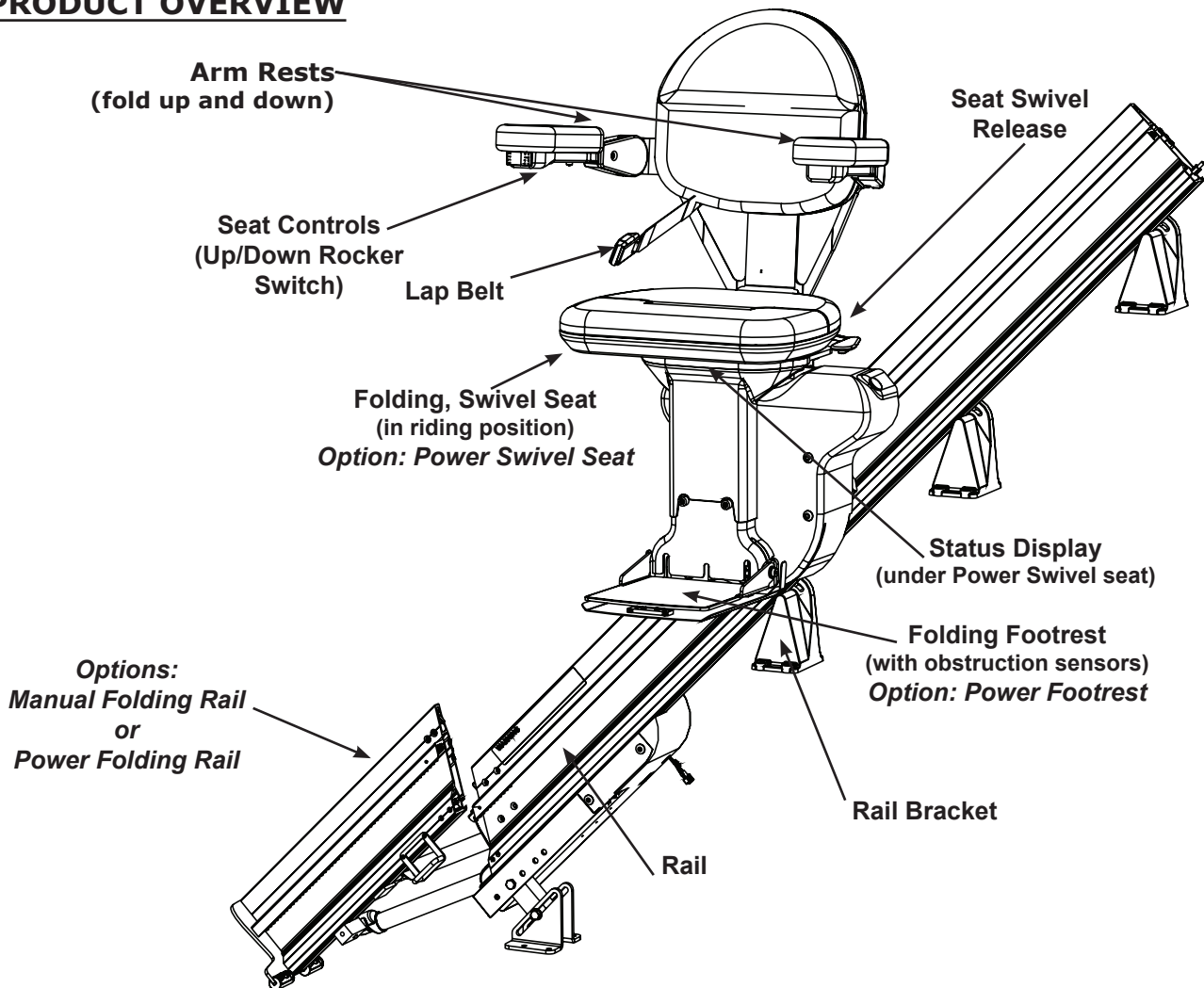
## Declaration of Conformity

## Wiring Schematics

## Exploded Views

# TECHNICAL SPECIFICATIONS

## PRODUCT OVERVIEW

## TECHNICAL SPECIFICATIONS

**Weight Capacity:** . . . . . 400 lbs. (181 kg)

**Speed:** . . . . . maximum 25 ft./min. (0 to 8.5 m/min)

**Power Source:** . . . . . two (2) 12-volt sealed, maintenance-free batteries with 24-volt continuous-duty charger

**Motor:** . . . . . 24 VDC, 2-pole, 0.68 hp

**Drive:** . . . . . self-locking gearbox, rack-and-pinion drive

**Control:** . . . . . constant pressure (armrest and 2 transmitters)

**Brake:** . . . . . electro-mechanical brake and self-locking worm gear

**Maximum incline:** . . . . 45°

**Rail:** . . . . . vertical aluminum extrusion with integral drive gear rack

**Seat Swivel:** . . . . . 0°, 60° and 90° at the top; 45° at the bottom

**Power Supply:** . . . . . 24 VDC battery charger powered by 120-volt wall outlet

# TOOLS, TORQUE SPECIFICATIONS, AND HARDWARE NOTES

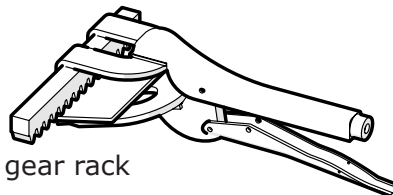
## **Tools**

- Drill driver with *MINIMUM* 18" extension
- Drill bits: appropriate for floor (eg. concrete, hardwood); 3/8", min. 6" long
- Cut-off saw (or other power saw with metal-cutting blade suitable for aluminum/steel)
- Metric open-end box wrench set
- **Adjustable wrench with 1-1/2" jaw** (for changing to opposite hand)
- Torque wrench (open end, and one that handles 50-80 ft.lb./68-108 Nm)
- Ratchet, 3/8" with 6" extension
- 1/2" SAE socket
- Metric sockets (10 mm, 13 mm, 17 mm)
- 10mm magnetic socket, 3/8" drive (Bruno recommends use with an 18" extension)
- Vise grips and C-clamp vise grips
- Needle-nose pliers
- Metric hex key set
- Magnetic Phillips screwdrivers
- Soft-faced or dead-blow hammer, or hard rubber or rawhide mallet
- Multimeter
- Extension cords
- Saw horses
- Safety glasses
- Torpedo level
- Tin snips
- Utility knife
- Wire stripper
- Wire crimper
- Metal file
- Scratch awl
- 20' tape measure
- Flashlight
- Threadlocker (i.e., Loctite®)

### **Next:**

- **Determine if the application requires installation of a manual or power folding rail.**

**Use FIX-00525 rail clamp tool (available from Bruno).**

OR Bar/Spring clamp with piece of gear rack

## **Torque Specs**

- Follow OEM torque recommendations for seat mounting hardware. If not available, refer to table on right indicating ISO and SAE recommended assembly torque specifications.

### **Torque Specifications GRADE 10.9 (Metric) (Rough equivalent to US Grade 8)**

M 6	.....	7 lb.-ft.....	9.49 Nm
M 8	.....	25 lb.-ft.....	33.8 Nm
M 10	.....	47 lb.-ft.....	63.7 Nm

**If it becomes necessary to replace any bolts or fasteners, use only metric GRADE 10.9 (hardened steel) hardware. Metric grade 10.9 bolts are identified by the number 10.9 on the head.**

**METRIC 10.9  
BOLT HEAD**

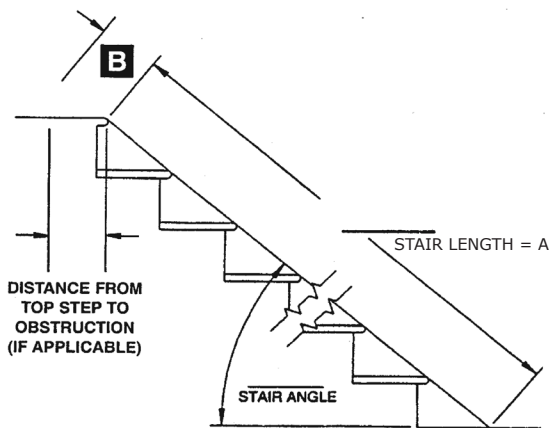
**Bolts without grade markings on the head are not strong enough to be safely used in this application.**

# PRE-INSTALLATION

## SITE REVIEW AND PREPARATION

1. Review site layout. Make sure to note all obstructions.
2. Panel off any open sections of the stairway prior to installation.
3. Determine if stairlift installation side (right or left) **as viewed from bottom of stairs**.

## RAIL MEASUREMENTS

1. Determine exact rail length.

**NOTE: The rail length will determine the seat height at the top of the stairs.**

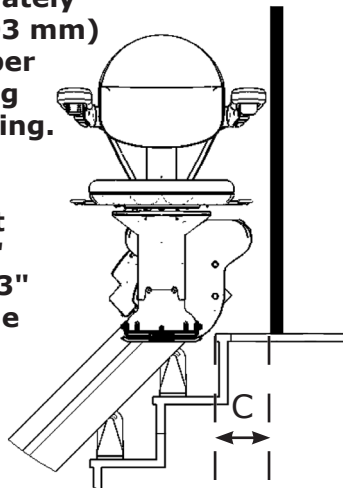
Measure length of stairway by placing a tape measure in a straight line on the stairs from the top step nose to the floor.

(A) Length of Stairway: \_\_\_\_\_

(B) Add 13" (330mm) for a 20" (508 mm) seat height: + \_\_\_\_\_  
 [Add 1.5" (38 mm) for each 1" (25 mm) seat height is increased.]

**NOTE: Chair armrest extends approximately 5 to 8" (127 to 203 mm) beyond rail at upper landing, depending on arm width setting.**

**NOTE: Seat height is adjustable in 1" increments up to 3" (76 mm). See page 38 for details.**

**! For obstacles at top of stairs: See Bruno's Brainshark "Ramping the Rail", and remember:**

**Nose-to-floor + nose-to-door + 2" = Overall Rail Length Required**

OR

Add Stair Length (A) to Measurement (C) (i.e., door) plus 2" (50mm): + \_\_\_\_\_

**Overall Rail Length Required = \_\_\_\_\_**

**NOTE: For units with overspeed brake, add 4" (102mm) to rail length.**

2. Mark *Overall Rail Length Required* on rail, measuring from top of rail downwards.

## RAIL CUTTING GUIDELINES

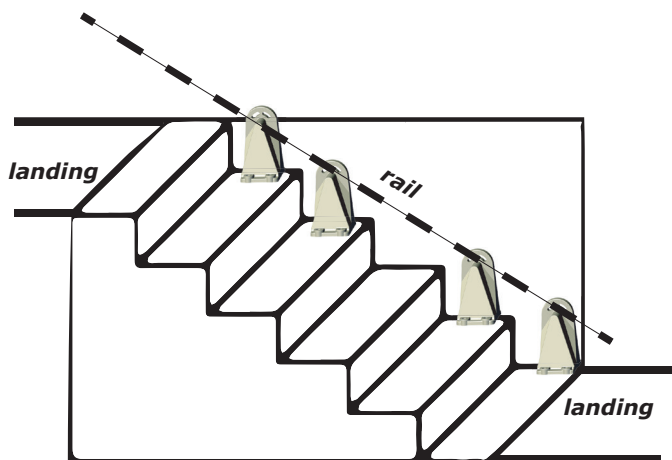
1. NEVER cut a rail shorter than 24" (610 mm). Rails shorter than 24" (610 mm) do not have room for the minimum two (2) rail brackets.

**NOTE: Folding rail assembly - NEVER cut a folding rail assembly shorter than 75" (1905mm).**

2. In two rail installations, if cutting only one rail would result in a length less than 24" (610 mm), cut from both rails as follows:
  - a. Cut at least 12" (305 mm) off one rail.
  - b. Then, calculate the second rail's length by subtracting the length of the newly cut rail from the required overall two-rail length (should be greater than 24"/610 mm).
3. Do not cut rail ends you plan to use for joining two sections together in order to preserve the factory fit at the joint.
4. For a precise and efficient cut, Bruno recommends the use of a cut-off saw

Equip cut-off saw (or other power saw) with a metal-cutting blade **suitable for aluminum and steel**.

## RAIL SUPPORT (Rail Bracket) REQUIREMENTS

1. In multiple-rail installations, use a minimum of two (2) rail brackets to support each rail section.
2. In single-rail installations, use a minimum of three (3) rail brackets to support rail. [This is typically rail-run lengths of ten feet (3 m) or less].
3. In folding rail installations, use a minimum of two (2) rail brackets to support any mid-rail section.

Do not include in this count, the fixed rail bracket (first tread) at the folding joint.

4. No more than three (3) steps in a row without a rail bracket.

# PRE-INSTALLATION

## LOWER RAIL PREPARATION

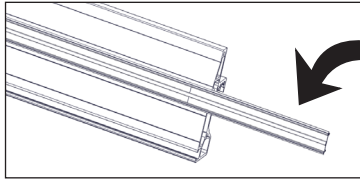
Fig. 1

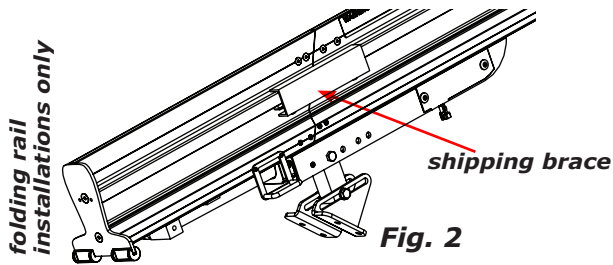
Fig. 2

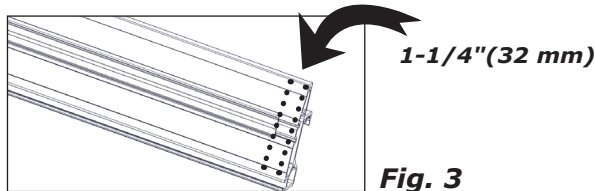
Fig. 3

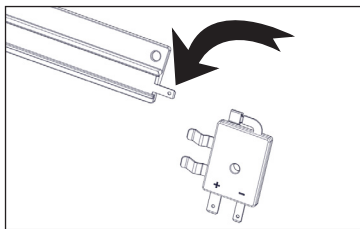
Fig. 4

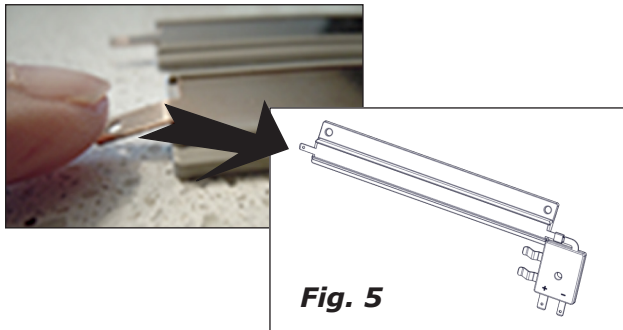
Fig. 5

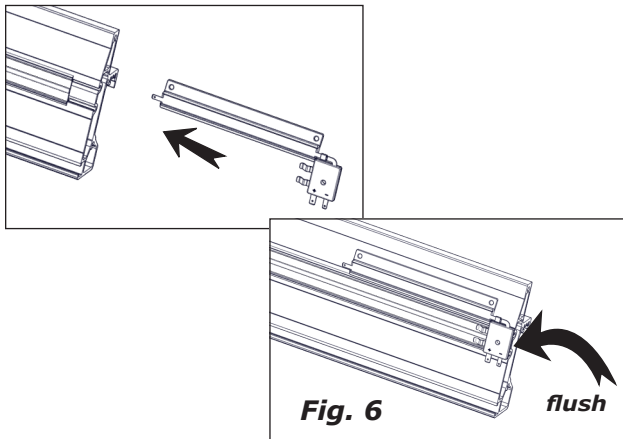
Fig. 6

1. Before cutting rail to length, remove charge strip carrier (Fig. 1). Set aside. Then, cut rail to length required for installation. (See page 6, **Rail Measurements** to calculate.)

**NOTE: Actual rail length for a 16' two-piece rail is 15'-9".**

2. Place flat side of rail on saw horses. Draw a line at desired finished rail length. Carefully align saw with marked cutting line.
3. Cut rail to length.
4. Once rail is cut, deburr cut end using a metal file or other appropriate tool.

**NOTE: Be sure to round edges that might abrade wiring at rail end routed to bumper.**

**For folding rail installations: Keep shipping brace on folding rail, maintaining a closed position (Fig. 2). Next, insert slow down strip into charge contact and fasten with two (2) self-drilling hex head screws. Then, feed charge lead through the hole in the rail out the cut end of the rail. Now, skip to Joint Preparation.**

5. With rail still on saw horses, slide charge strip carrier into back side of rail (Fig. 3). Inset carrier as noted in Fig. 3.
6. Attach charge contact to slow down (communications) strip (Fig. 4).
7. Bend slow down strip tab (at opposite end from charge contact) down slightly (Fig. 5). Do not let tab touch rail!
8. At bottom end of rail, slide charge contact assembly from Fig. 4 into back side of rail (Fig. 6). Assembly will fit snugly against charge strip carrier.

**NOTE: Carrier may be longer than rail at this point.**

## LOWER RAIL PREPARATION (continued)

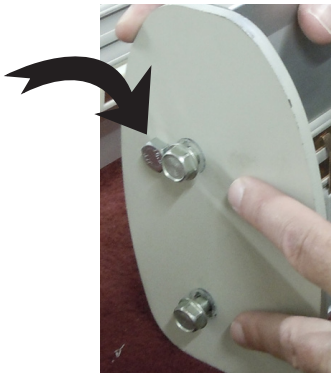
Fig. 6


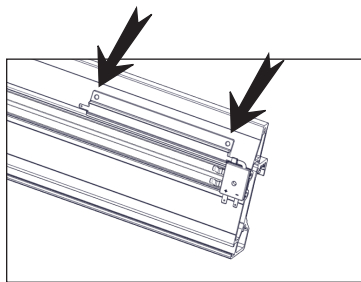
Fig. 7

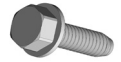- Use 1/2" socket to install two (2) 5/16" x 18 self-threading screws to mount the lower rail endplate (Fig. 6). A dab of grease will make installation easier.


**NOTE: Threaded hole in endplate goes toward front side of rail.**

**NOTE: DO NOT use an impact drill to install these screws.**

- Insert slow down strip into lower charge contact, keeping it parallel to and touching top edge of charge strip carrier.

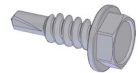Using two (2) self-drilling hex head screws, secure slow down strip to rail (Fig. 7).

**NOTE: Make sure slow down strip is parallel to charge strip carrier, and touching top edge of carrier.**

**FOR 1-PIECE RAIL INSTALLATIONS, SKIP TO UPPER RAIL PREPARATION.**

## JOINT PREPARATION

**NOTE: Lower charge contact is assembled at factory for folding rails.**


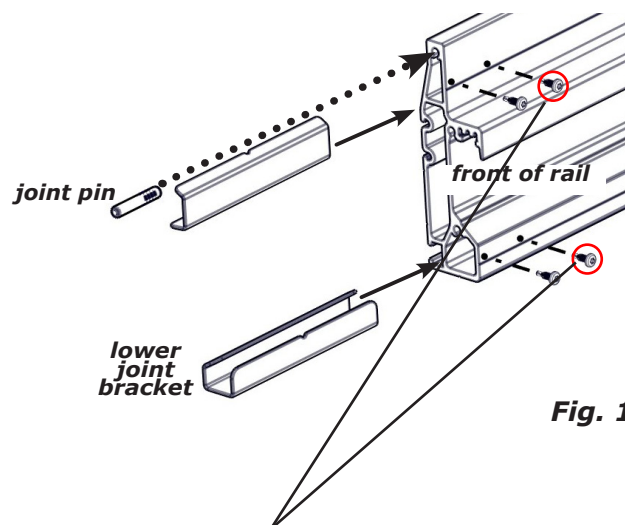
Fig. 1

**Wait to install second screw until after upper rail is installed.**

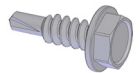- Pound barbed end of rail joint pin **halfway** into lower rail (Fig.1).
- Into upper part of lower rail section, insert an upper joint bracket halfway\* in.


Secure one (1) of two (2) self-drilling hex head screws (Fig. 1).

\* **Both upper and lower joint brackets are notched to indicate halfway point.**

- Into lower part of lower rail section, insert a lower joint bracket halfway\* in.

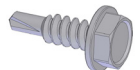Secure one (1) of (2) self-drilling hex head screws (Fig. 1).

# PRE-INSTALLATION

## UPPER RAIL PREPARATION

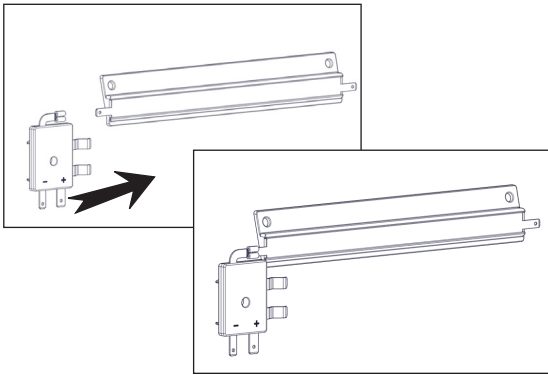
Fig. 1

1. Place upper rail section on saw horses.
2. Attach charge contact to the slow down strip (Fig. 1).
3. If previously removed, slide charge strip carrier into back side of rail.
4. Slide charge contact assembly from Fig. 1 into back side of rail. Assembly will fit snugly against charge strip carrier.

**NOTE: Carrier may be longer than joint end of rail at this point.**

**For standard installations, skip to Step 8.**

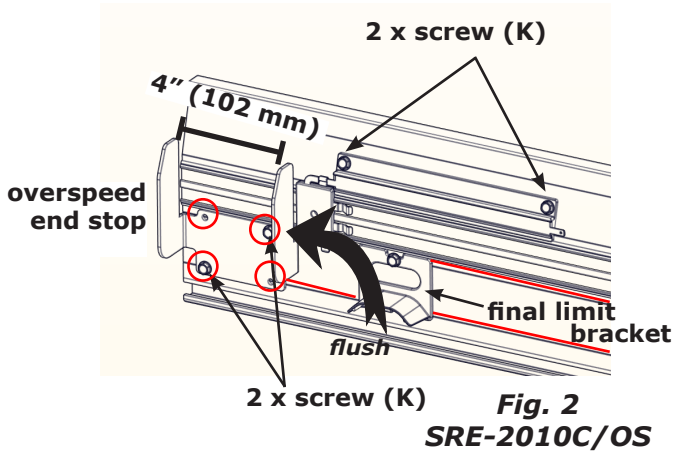
Fig. 2  
SRE-2010C/OS

5. (**SRE-2010C/OS**) Orient overspeed end stop ("U"-shaped bracket) so that four (4) holes circled in red line up with grooves etched in back side of rail as shown by red lines (Fig. 2).
6. (**SRE-2010C/OS**) Position upper leg of "U" flush with end of rail (Fig. 2).
7. (**SRE-2010C/OS**) Use two (2) self-threading screws to mount "U" bracket to rail, one in an upper hole and one in a lower hole (Fig. 2).

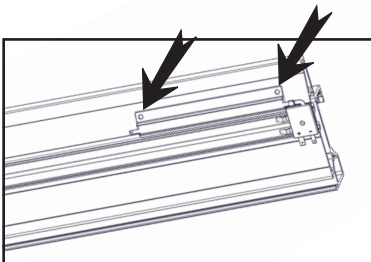
Fig. 3  
Standard

8. Using two (2) self-drilling hex head screws, secure slow down ramp to rail. See Fig. 3 for standard installations, and Fig. 2 for SRE-2010C/OS installations.

**NOTE: Confirm that slow down ramp is parallel to charge strip carrier, and touching top edge of carrier.**


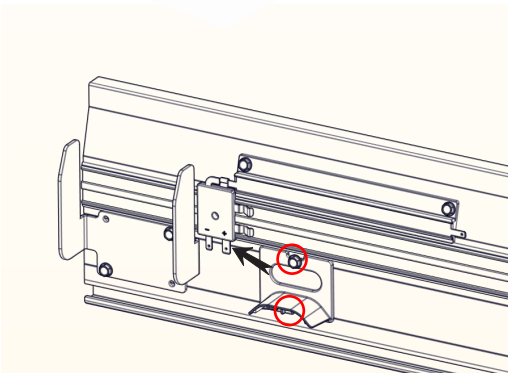
Fig. 4

9. Remove charge contact assembly from rail and save for later use.



Also remove overspeed end stop from rail if used in a SRE-2010C/OS installation.

10. Mount final limit bracket: Upper edge of final limit bracket should be flush with lower edge of charge contact. Insert a self-drilling hex head screw through each screw hole (in red circles, Fig. 4) on final limit bracket into etched lines on back of rail.

## FINISH PREPARATION

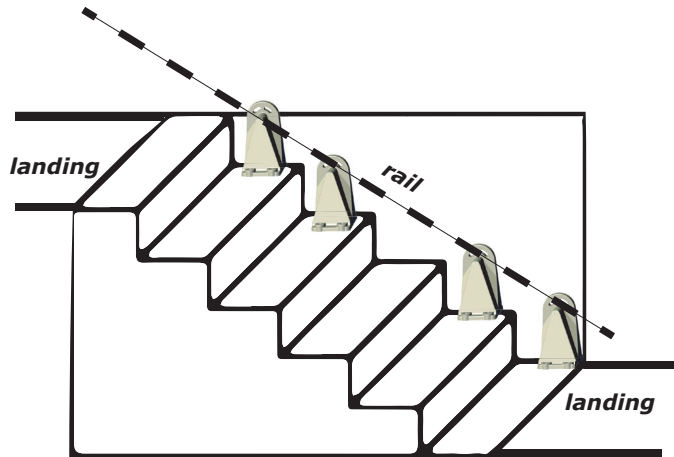*back side of rail*

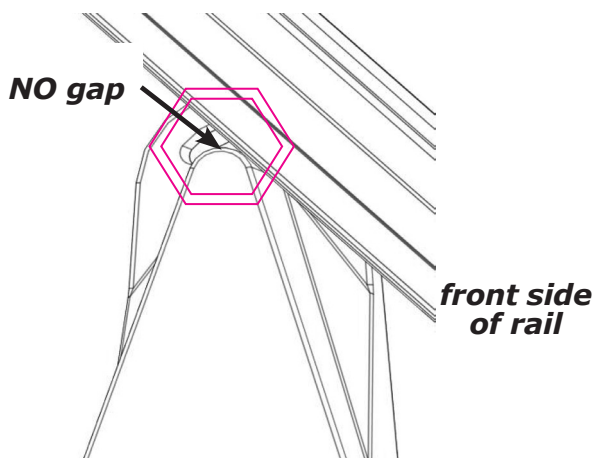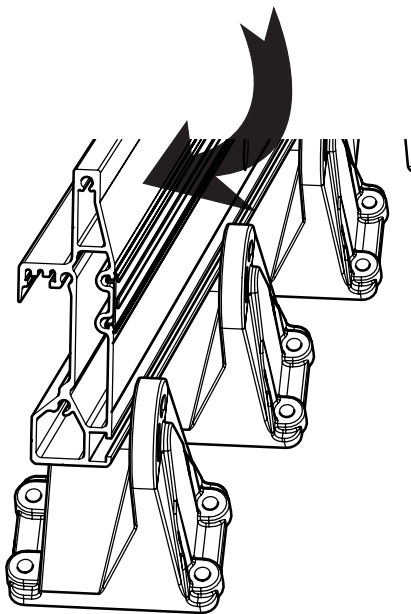1. **Determine rail bracket placement.**  
See Rail Support (Rail Bracket) Requirements on page 7 for guidelines.

Additional rail brackets may be required for longer installations.

Rail brackets may be purchased from Bruno.

2. Locate and open plastic bag marked "Stand feet hardware". These components are used to attach rail brackets to rail.
3. Pre-assemble hardware by inserting bolts into flat washers first and safety washers second.
4. Determine rail bracket-to-rail orientation, which **MUST** match the figures at left.

**Always install rail brackets with long nose facing the center of staircase.**

**NOTE: Failure to install rail brackets in correct orientation may lead to serious personal injury and/or property damage!**

**FOR 1-PIECE RAIL INSTALLATIONS,  
SKIP TO STEP 10 ON NEXT PAGE.**

# PRE-INSTALLATION

## FINISH PREPARATION (continued)

Fig. 1

5. **Join rails, if necessary.**  
Position upper rail section onto lower rail section. **Wiggle the upper section slightly** to make it align properly with lower section (Fig. 1).

**If installation involves a folding rail, feed charge harness through upper rail section before joining rails.**

6. Upper and lower rail sections should now be flush and snug.
7. Secure joint **brackets** to upper rail using two (2) self-drilling hex head screws per bracket, (i.e., etched lines on rail per Fig. 1).

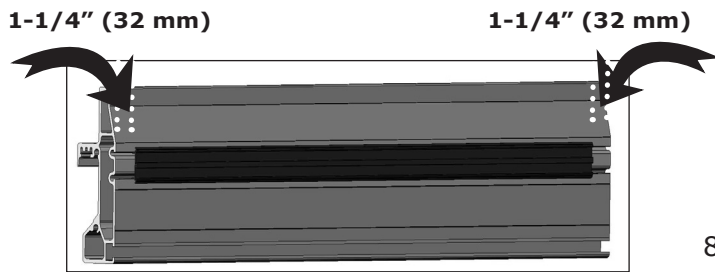
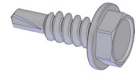
Fig. 2


**NOTE: Screws should be located no more than 2-3/4" (70 mm) from the joint if the bracket is installed correctly.**

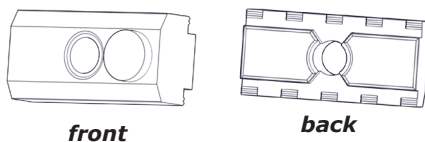8. On lower rail, install remaining screws (from Step 2 of Joint Preparation on page 9) in upper and lower brackets.
9. **Add nut bars (Fig. 3).**

With rail on a flat surface, slide one (1) nut bar for each rail bracket into lower rear slot of rail.

If joined rails, add one (1) additional nut bar.

**NOTE: There will be one (1) nut bar per rail bracket plus one (1) directly over the rail joint.**

front

back


nut bar

Fig. 3

10. **Trim charge strip carrier.**  
Trim 2-1/2" (63.5 mm) shorter than total rail length.

[6-1/2" (159 mm) for SRE-2010C/OS]

**If installation involves a folding rail, insert fin fastener (Fig. 4) into the mid-rail charge strip carrier at the joint. Trim 1.5" (38 mm) shorter than total rail length.**

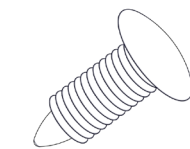fin fastener

Fig. 4

**NOTE: For SRE-2010C/OS installations, from leftover piece of carrier, cut a 4" (102 mm) "filler" piece to insert later.**

## **FINISH PREPARATION (continued)**

11. Slide charge strip carrier into back side of rail as shown (Fig 2).

12. ***Install gear rack pieces.***

Continuing to work on a flat surface, ***carefully\*\**** slide gear rack pieces into upper rail until:

(a) last piece extends beyond rail end

OR

(b) last piece is within 1/2" (13 mm) of rail end.

**\*\*Allowing gear rack pieces to "free fall" and land hard will knock joint out of adjustment.**

- **Maximum distance between last tooth of uppermost gear rack piece and rail end is 1-1/4" (32 mm).**

- If last piece extends beyond rail end, cut that piece to length. Mark piece even with rail end, slide it out, subtract 1/2" (13 mm) and cut.

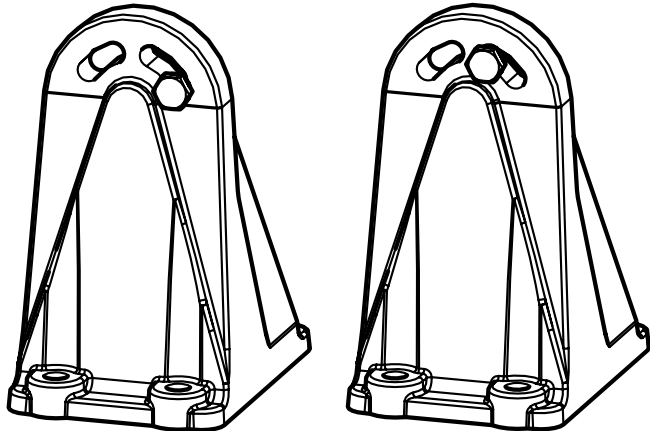
**NOTE: After insertion, remove last gear rack piece (or two) to facilitate carriage installation!**

**NOTE: In order to get the rail to the installation site, in some cases a two-piece rail may need to be separated and rejoined once on the staircase.**

**FOR FOLDING RAIL INSTALLATIONS,  
SKIP TO PAGE 16.**

# INSTALLATION

## RAIL INSTALLATION - NON-FOLDING RAIL


*bolt head correct*

*bolt head incorrect*

**Fig. 1**

**Fig. 2**

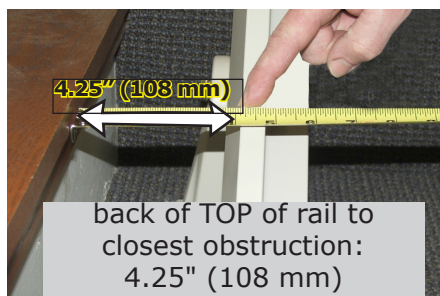**Fig. 3**

1. If not done previously, reconnect two-piece rail installations on staircase.
2. Place rail on stepnoses.

**NOTE: Place heavy toolbox at bottom of stairs to keep rail from sliding.**

**NOTE: Bottom of rail should be 1/8" (3 mm) off floor.**

3. Mount all rail brackets to rail.
  - Tap bolt down in rail bracket slot before tightening (Fig. 1).
  - Torque rail/rail bracket mounting hardware to 25 ft. lb. (34 Nm).
4. Slide rail out horizontally until bottom of rail is off stair stepnose by:
  - no carpet:** 2" (51 mm) *minimum*
  - carpet:** 2.5" (64 mm) *minimum*

**Measure every step** (Fig. 2).

5. Measure from wall side of TOP of rail to closest obstruction (i.e., wall, handrail, etc.):  
4.25" (108 mm) (See Fig. 3.)

**Measure at top, middle and bottom along the length of each rail section.**

6. Begin to anchor rail brackets, starting with rail bracket at top of stairs. Use only two (2) sheet metal screws, one (1) on the wall side first, then one (1) on the stairway side of rail.

**NOTE: Wait to anchor rest of rail brackets until running clearances are checked.**

**NOTE: If bottom of rail is closer than 2" (51 mm) to stair stepnose, remove screws, slide rail horizontally, and refasten rail bracket.**

## RAIL INSTALLATION - NON-FOLDING RAIL (continued)

7. ***Insert charge strips into rail.***

Locate two (2) charge strip coils. Hold one coil firmly to prevent uncoiling, and carefully undo tape securing coil.

8. Using tin snips, cut a 45° bevel on end of charge contact strips (see images at left).

**NOTE: This will make it MUCH easier to insert strip into carrier.**

9. Slide beveled end of charge contact strip into charge contact strip carrier until it sides under lower charge contact.


10. Cut off excess charge contact strip flush with carrier.

11. Repeat Steps 7-10 for the second charge contact strip coil.

**SKIP TO CARRIAGE INSTALLATION  
ON PAGE 18.**

# INSTALLATION

## RAIL INSTALLATION - FOLDING RAIL

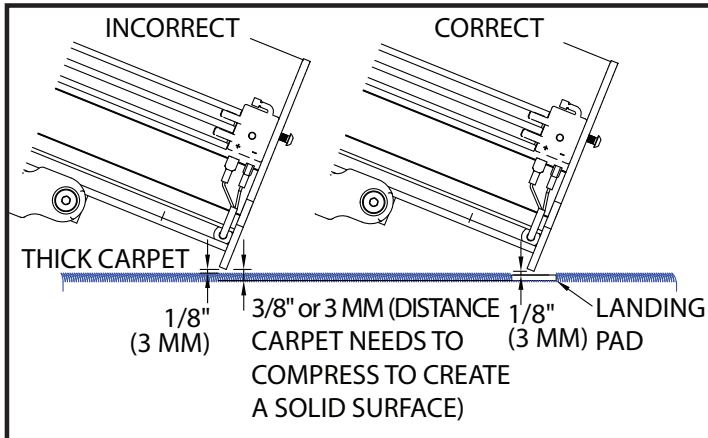**Fig. 1**

**adjustable post**

**Fig. 2**

**Fig. 3**

1. If not done previously, reconnect two-piece rail installations on staircase.
2. Place rail on stepnoses.

**NOTE: Place heavy toolbox at bottom of stairs to keep rail from sliding.**

**NOTE: Bottom of rail should be 1/8" (3 mm) off floor. If thick carpeting prevents this (Fig. 1), contact Bruno to purchase kit SRV-3000-038.**

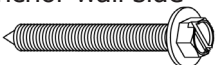
3. Mount upper rail bracket to rail, keeping upper end of rail 2" (51 mm) off stepnose and lower end tight against stepnose.
4. Install remaining rail brackets, including special folding rail bracket (Fig. 2).

5. Slide rail out horizontally until bottom of rail is off stair stepnose by:  
2-1/2" (64 mm) *minimum*

**Check clearance between bottom endplate and floor [1/8" (3 mm) off floor]. If too tight, rail will not close properly. Adjust by loosening bolts on all rail brackets, sliding rail up slightly, and retightening bolts.**

6. Measure from wall side of TOP of rail to closest obstruction (i.e., wall, handrail, etc.):  
4.25" (108 mm)

**Measure at top, middle and bottom along the length of each rail section.**

7. Fasten special folding rail bracket to stair tread using two (2) sheet metal screws  through the stairway side of rail bracket.
8. Remove M10 bolt (circled in red). Set rail off to side (Fig. 3).
9. Lower adjustable post on special rail bracket to set space between front and back bracket feet. Then, anchor wall side of bracket with two (2) sheet metal screws. 

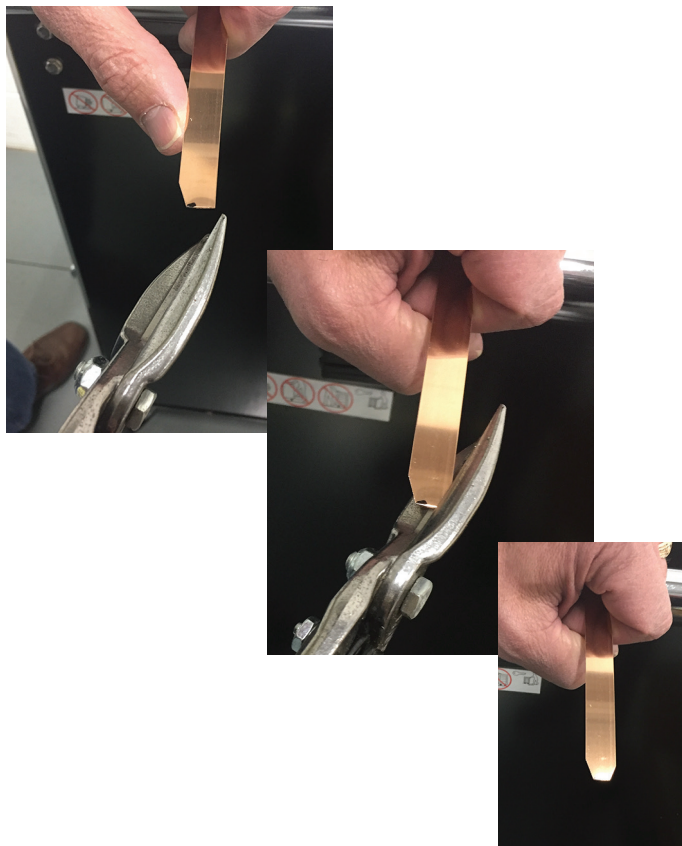
## **RAIL INSTALLATION - FOLDING RAIL (continued)**

10. Replace rail, seating firmly in special rail bracket.
11. Replace M10 bolt to secure rail to rail bracket. Finger tighten plus 1/4-turn.
12. Begin to anchor rest of rail brackets, starting with rail bracket at top of stairs. Use only two (2) sheet metal screws: one (1) on the wall side first, then one (1) on the stairway side of rail.

**NOTE: Wait to anchor rest of rail brackets until running clearances are checked.**

**NOTE: If bottom of rail is closer than 2" (51 mm) to stair stepnose, remove screws, slide rail out horizontally, and refasten rail bracket.**

13. ***Insert charge strips into rail.***  
Locate two (2) charge strip coils in rail installation kit\*. Hold one coil firmly to prevent uncoiling, and carefully undo tape securing coil.

\* Rail Installation Kits:  
SRE-K-2161 for 16' Non-Folding Rails  
SRE-K-2162 for 20' Non-Folding Rails  
SRE-K-2163 for 16'/18' Folding Rails

14. Using tin snips, cut a 45° bevel on end of charge contact strips (see images at left).





**NOTE: This will make it MUCH easier to insert strip into carrier.**

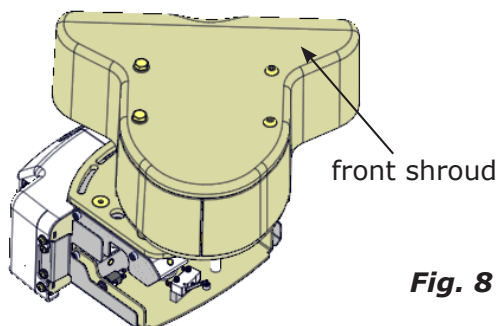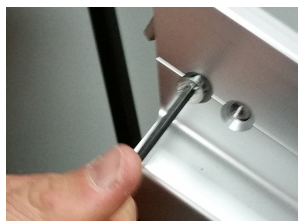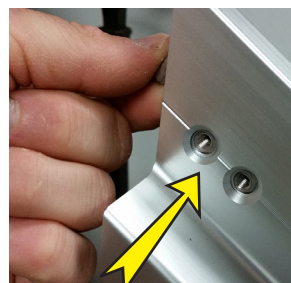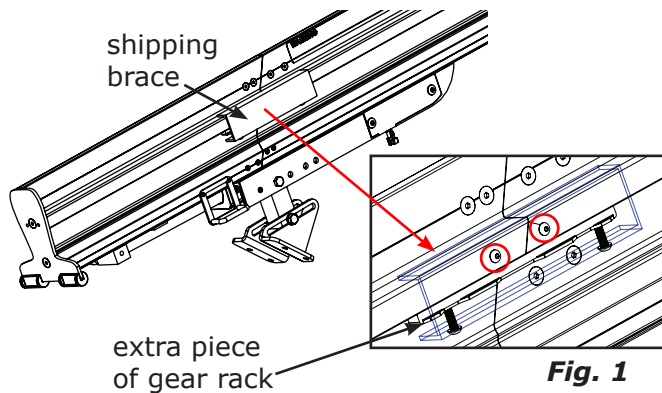15. Slide beveled end of charge contact strip into charge contact strip carrier until it slides under lower charge contact.
16. Cut off excess charge contact strip flush with carrier.
17. Repeat Steps 13-17 for the second charge contact strip coil.

**CONTINUE TO CARRIAGE INSTALLATION  
ON PAGE 18.**

# INSTALLATION

## CARRIAGE INSTALLATION

**NOTE:** Before installing carriage, check angle of stairs. If less than 25°, it is easier to adjust carriage with it off the rail. See chart on page 41.

### Prepare Rail

1. Turn OFF circuit breaker (Fig. 9).
2. Remove upper charge contact from rail.
3. Remove top two (2) gear rack segments to facilitate sliding carriage on rail.
4. Before installing carriage, use a 4mm hex key to remove two (2) button head screws from shipping brace on folding rail and extra piece of gear rack (Fig. 1).
5. Make sure distance between folding rail and ground is NO MORE THAN 1/8" (3 mm). The closer to the ground, the better (Fig. 2)!
6. Raise short rail to provide access to latch installation location (Fig. 3).
7. Insert latch, aligning holes (Fig. 4).

**NOTE:** Spacer plate should contact rail extrusion when installed.

8. Hold latch in position while inserting screws (Fig. 5).
9. Using 4 mm Allen wrench, tighten two screws (Fig. 6).

**NOTE:** Make sure the screws are tight and properly seated!

10. Lower short rail to ground (Fig. 7).
11. **Install carriage on rail.**

**Tip:** Remove front shroud (Fig. 8) to make it easier to transport the carriage, OR... lift and carry carriage as shown in Fig. 9. NEVER lift up by carriage switch housing!

- a. Slide carriage onto rail until it meets gear rack.
- b. Cut cable tie securing decal sheet and key to circuit breaker.
- c. Remove serial number decal from sheet and apply it to seat as instructed on decal sheet.

## CARRIAGE INSTALLATION (continued)

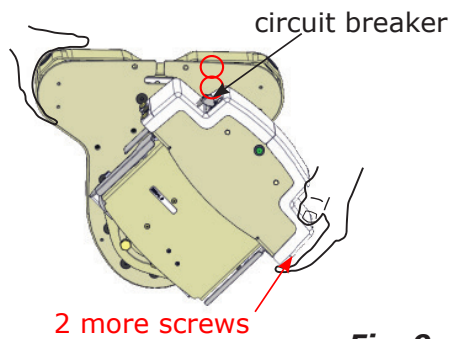
Fig. 9

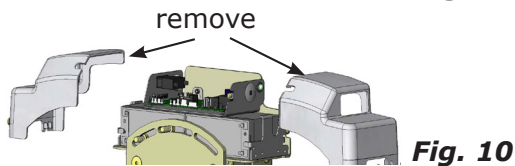
Fig. 10

- d. Remove four (4) Phillips screws (noted in red in Fig. 9). Then, remove two (2) halves of top carriage cover (Fig. 10).
- e. Turn circuit breaker ON, then carriage keyswitch.
- f. **(left-hand installation)**: Connect 3-pin and 6-pin jumpers.
- (right-hand installation)**: Connect 3-pin, 6 pin and 12-pin jumpers
- g. Using remote, power carriage onto gear rack approximately 6" (15 cm).

***If you power the carriage onto rail too far, you won't have easy access to hardware on back of carriage for angle adjustment.***

## FOOTREST INSTALLATION

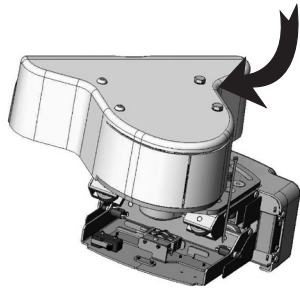
Fig. 1

Fig. 2

Fig. 3

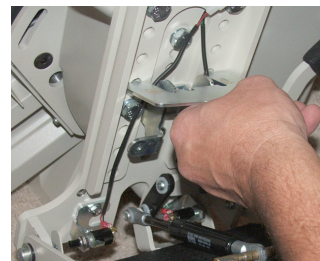
Fig. 4

Fig. 5

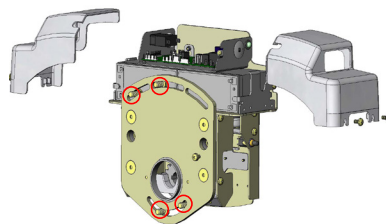
Fig. 6

*\*Some carriage components have been removed for illustration purposes.*

1. **Mount footrest to carriage.**
  - a. Remove two (2) hex head bolts from front of carriage (Figs. 1 and 2).
  - b. Secure footrest to carriage using two (2) hex head bolts.
  - Torque to 50 ft-lb (68 Nm).**
  - c. Insert pin in gas spring (Fig. 3).
  - d. Fold footrest to approximately 45°.
  - e. Rotating tool downward, gently push pin into notches on gas shock bracket (Fig. 4 and 5).
  - f. Remove pin insertion tool and set aside.
2. **Level footrest.**
  - a. Move carriage to top of rail.
  - b. Lower footrest.
  - c. Place a level on footrest pan.
3. Adjust carriage angle:
  - a. Loosen (**but do not remove**)\* four (4) bolts circled in red (Fig. 6).


**NOTE: For some installations, you WILL remove (and reinstall) two (2) top bolts. See Carriage Adjustment section at back of manual.**

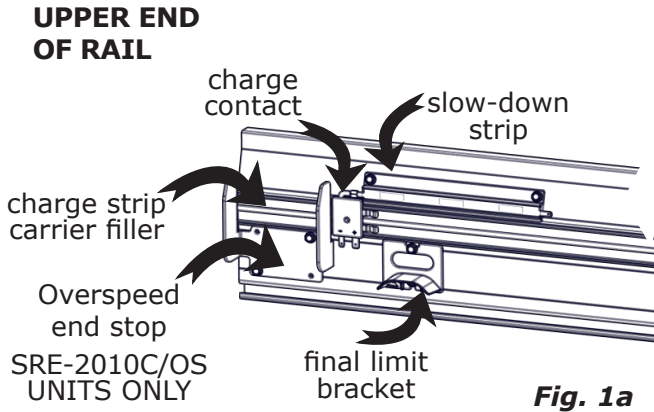  - b. Rotate carriage to correct angle.
  - c. Tighten four (4) previously-loosened bolts to a torque of 50 ft-lb (68 Nm).
4. Reinstall carriage covers (removed in Step 1) **before** installing seat.

# INSTALLATION

## COMPLETING RAIL INSTALLATION

### left-hand installation


1. Insert remaining gear rack pieces, cutting if necessary.
2. Add charge contact.
  - a. Slide charge contact into rail and connect to slow down contact (Figs. 1a and 1b).

### SRE-2010C/OS UNITS:

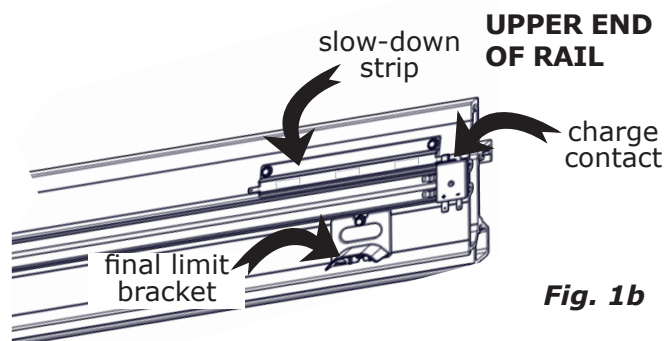Mount overspeed end stop and 4" (102 mm) charge strip carrier filler piece cut previously.

**CONTINUE TO PAGE 22 IF NO FOLDING RAIL IS INVOLVED.**


### right-hand installation

## COMPLETING RAIL INSTALLATION (continued)

**Fig. 3**

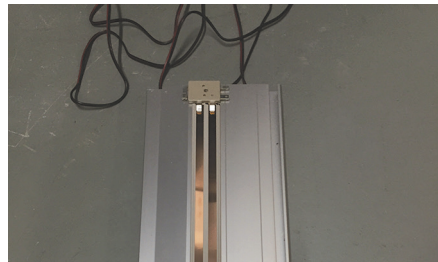**Fig. 4**

**Fig. 5**

**Fig. 6**

**Fig. 7**

**Fig. 8**

3. Insert last gear rack section into upper rail.

### ***For folding rail installations:***

**Fish black charge cable up through upper rail (see Figs. 3 and 4).**

**Cable wires will fit between spacer and end of rail.**

**Connect charge harness to charge contact and charger (Fig.5).**

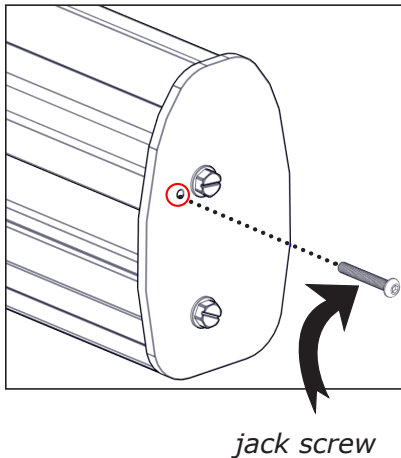
**Stuff remaining harness into end of rail (Fig. 6).**

**Mount provided spacer endplate (in kit SRE-K-3050) between top of endplate and end of rail to prevent damage to components (Fig. 7).**

**See Fig. 8 for a finished endplate example.**

# INSTALLATION

## COMPLETING RAIL INSTALLATION (continued)


#### 4. **Add top endplate.**

Use a 1/2" socket and ratchet to mount top endplate using two (2) 5-16" - 18x1" self-threading hex head screws.

#### 5. **Insert jack screw.**

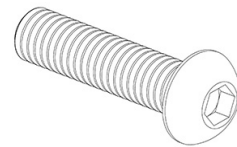Two (2) M6 socket button head cap screws (i.e., jack screws) are provided for pressure-loading the gear rack. One is 25 mm long and one is 40 mm long.

Use table below to determine which screw to use.

Distance from last full gear rack segment to end of rail	M6 x 25 mm screw	M6 x 40mm screw
1/2" (12mm) or less	X	
1/2" (12mm) to 1 1/4" (32mm)		X

Jack screw may be inserted into either top or bottom endplate.


**NOTE: Do NOT overtighten!**  
Finger tighten screw until screw contacts gear rack.  
Then, use hex wrench to tighten no more than one complete turn.

**NOTE: Jack screw will NOT be flush with endplate.**


6. Plug charger into a live, properly-grounded 120V AC outlet.

7. Turn charger switch ON.


## RAIL CROSS PIN INSTALLATION

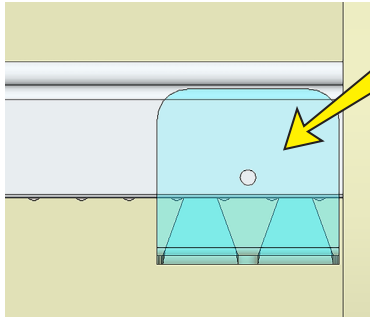
**Fig. 1**

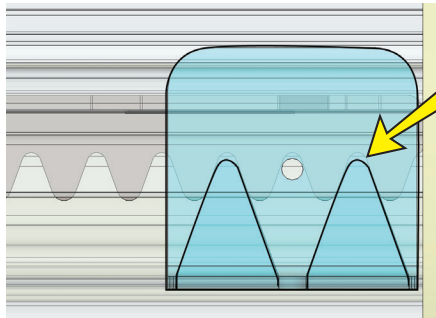
**Fig. 2**

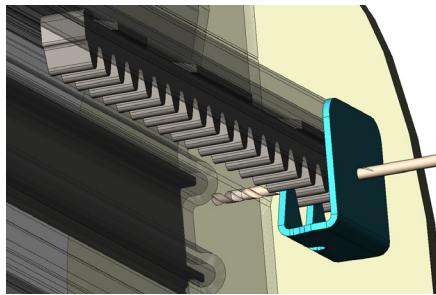**Fig. 3**

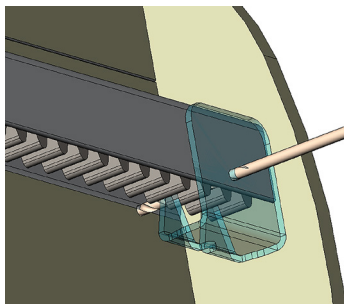**Fig. 4**

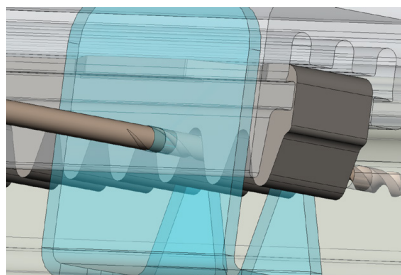**Fig. 5**

**NOTE: Install rail cross pin *after* jack screw has been properly tightened.**

1. At bottom end of rail (base of stairs), position SRE Cross Pin Drill Template after first tooth of first gear rack. Template hole should be up against the part of the extrusion covering the gear rack (Fig. 1). Template "teeth" should fit up against the gear rack valleys (Fig. 2).

2. Using a 1/8" (3.2mm) drill bit (Bruno P/N: DRB-13001, provided with kit SRV-3000-023), place a hole in 2 walls of the extrusion (Figs. 3 and 4). Do not drill through entire extrusion.


**NOTE: Drill bit will not make contact with gear rack (Fig. 5).**

# INSTALLATION


---

## **RAIL CROSS PIN INSTALLATION (continued)**

3. Once hole is drilled, place pin into rail extrusion (Fig. 6). Make sure pin is flush with outside of extrusion (Fig. 7).

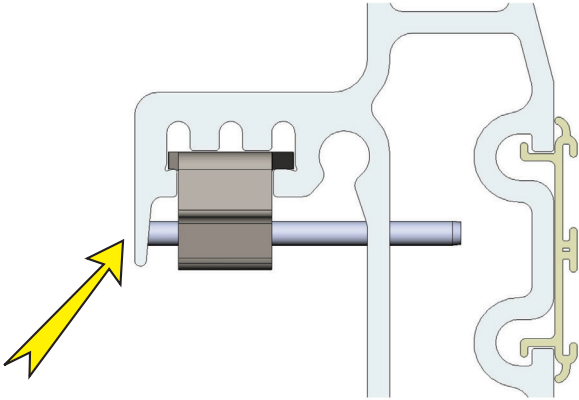**Fig. 6**

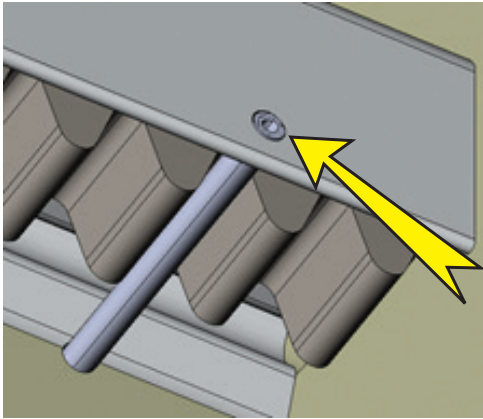**Fig. 7**

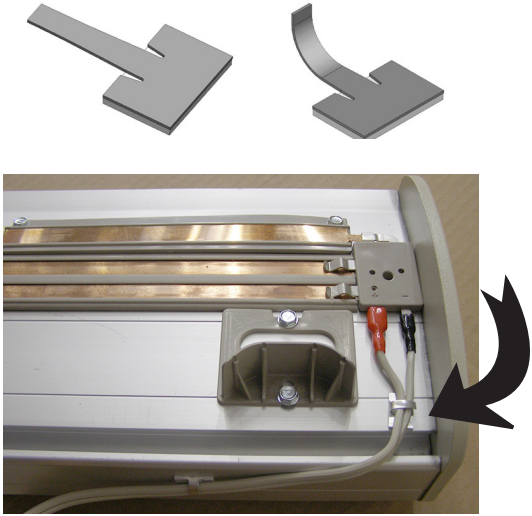
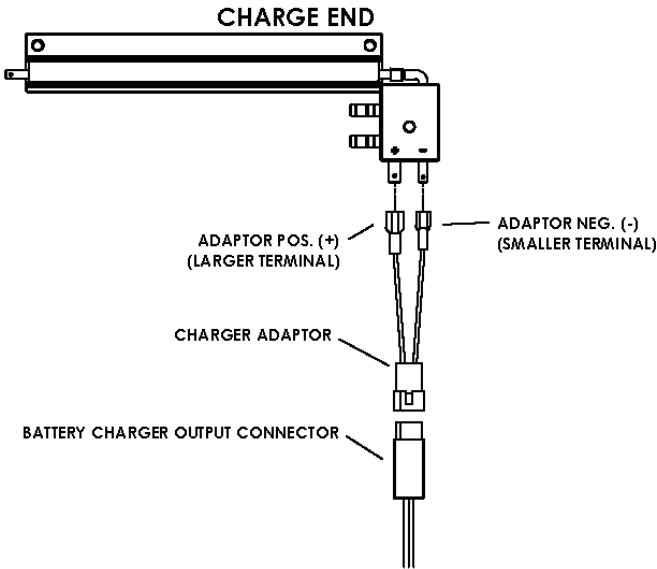
## MAKE CHARGER CONNECTIONS - NON-FOLDING RAIL

1. Connect wires from charger to charge contact assembly. (See figure at left.)
2. Attach adhesive wire clip to rail and fold tab over wires as shown below.

CONFIGURATION SHOWN:  
1) LEFT-HAND INSTALLATION WITH CHARGER AT BOTTOM.  
2) RIGHT-HAND INSTALLATION WITH CHARGER AT TOP.




### SRE-2010 (ELITE) CHARGER CONNECTION



# INSTALLATION

## MAKE CHARGER CONNECTIONS - FOLDING RAIL

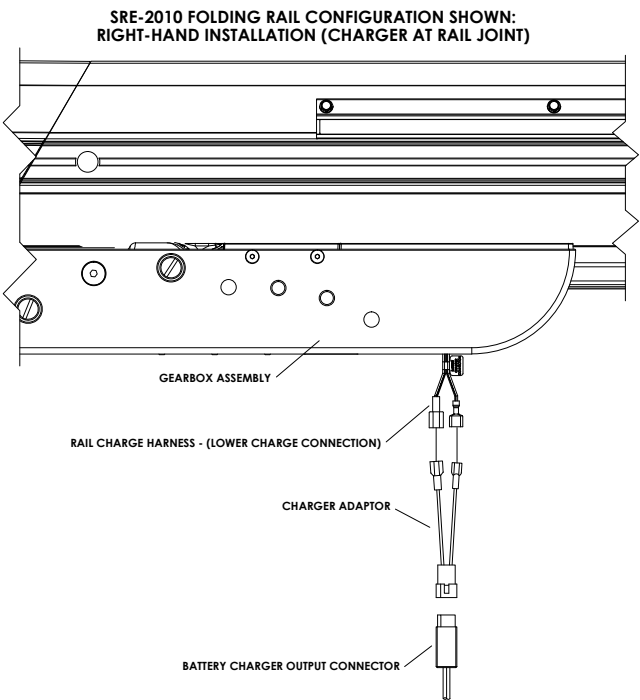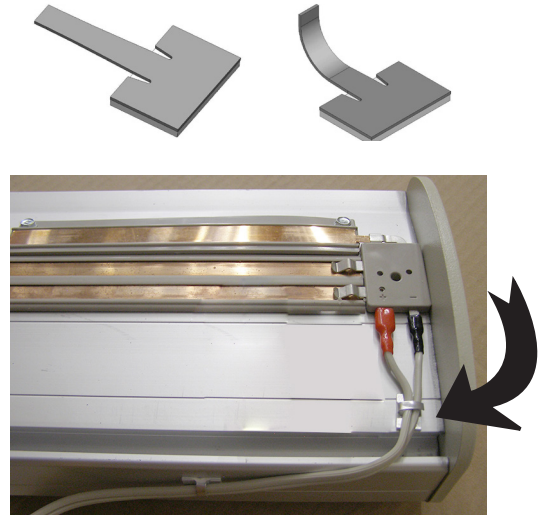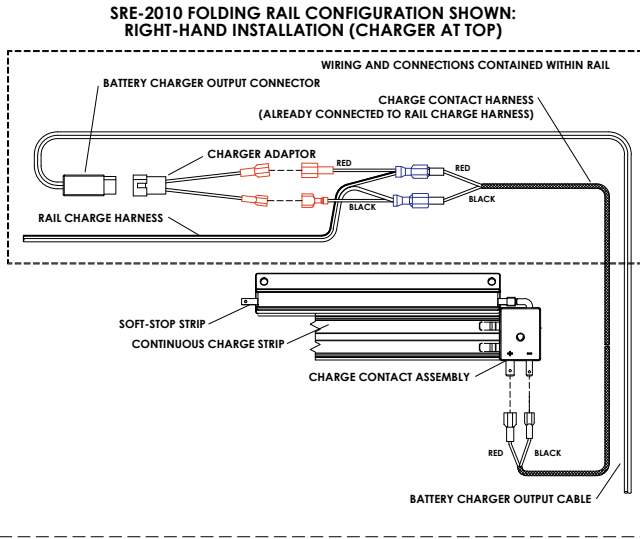1. Connect wires from charger to charge contact assembly. (See figures at left.)
2. Attach adhesive wire clip to rail and fold tab over wires as shown below.

## SEAT INSTALLATION

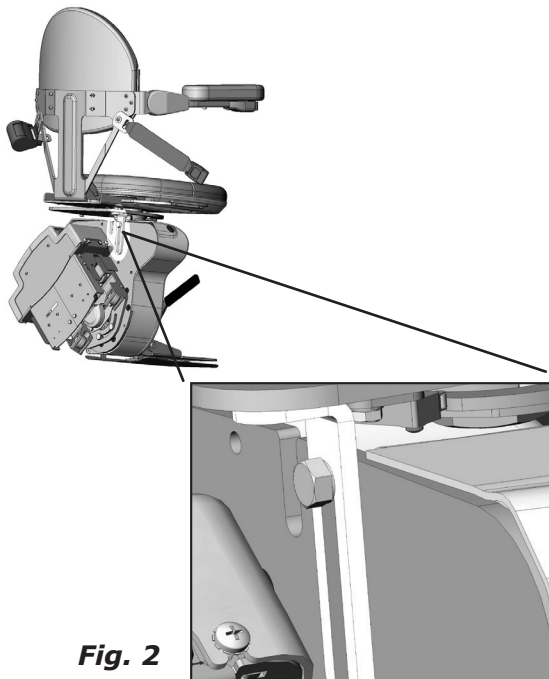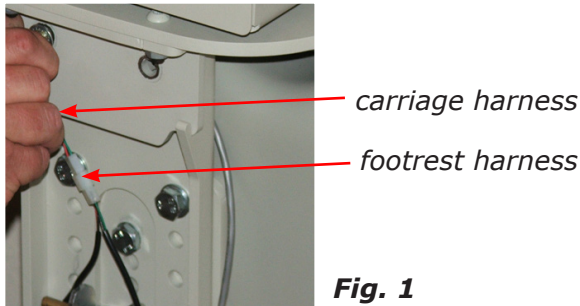1. Remove and save shipping jumpers from carriage harnesses.

Connect 2-pin connector from carriage to footrest harness (Fig. 1).

2. ***Mount rear seat support bracket.***

See Fig. 2 for location.

Use an M-10 bolt and safety lockwasher.

**Do not tighten at this time.**

3. ***Mount seat on carriage.***

Tilt seat forward slightly, and set into notches of adjustable seat bracket (Fig. 3).

4. Connect remaining two (2) wiring harnesses:


- 6-pin: seat swivel to rocker switch
- 12-pin: seat swivel to diagnostic board

5. ***Loosely secure seat.***

Insert two (2) hex head bolts and washers through front seat bracket holes (Fig. 4).


# INSTALLATION

## SEAT INSTALLATION (continued)

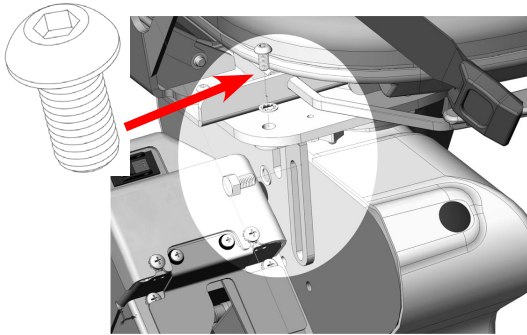
**Fig. 3**

6. Rotate seat to gain access to Allen head screw hole above seat support bracket (Fig. 3).
7. Insert and tighten Allen head screw.
8. While pulling seat back slightly forward, fully tighten M-10 seat support bracket bolt from Step 1.

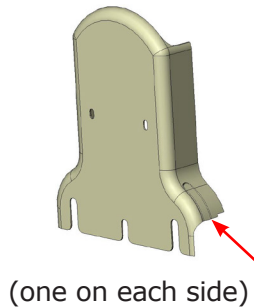
**Fig. 4**

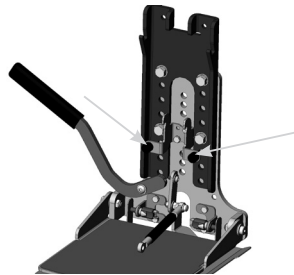**Fig. 5**

9. Tighten bolts inserted in Step 4.
10. **Mount footrest cover.**
  - remove cutout for footrest lever (Fig. 4);
  - remove two (2) footrest bumpers (Fig. 5);
  - place lower footrest cover against bracket;
  - reinsert bumpers.

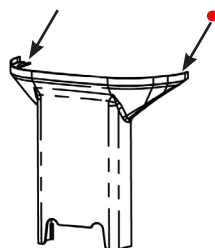**Fig. 6**

11. Loosen (**but do not remove**) two (2) top 10mm footrest top shroud nuts (Fig. 6).
12. Slide top footrest cover under nuts loosened in Step 10.
13. Tighten two (2) top nuts.

## SEAT INSTALLATION (continued)

 <b>WARNING</b>
<b>RISK OF INJURY!</b>
Do NOT ride the stairlift until all rail brackets are properly secured.
Failure to observe this warning could result in injury or damage to equipment.

14. Turn circuit breaker ON.

15. Turn key to ON.

16. Lower footrest.

17. Using arm rocker switch, run seat and carriage down rail to check for proper clearances:

*footrest-to-step nose* - approx. 1/2 - 3/4"  
(12-19 mm)

*seat-to-wall/obstruction* - 1/2" (12 mm)

Adjust as needed.

18. If the clearances are correct, firmly secure remaining rail bracket to stairs.

***Always secure back screws of rail brackets first!***

## LUBRICATE GEAR RACK

1. Run carriage up and down rail several times.

2. Apply a thin layer of white lithium grease (provided) **on gear rack ONLY**.

*Repeat often if unit is used very frequently or is used in a harsh environment.*

3. Regular cleaning will keep rail and rack free of dust and dirt.

# INSTALLATION

## BATTERY CHARGER INSTALLATION

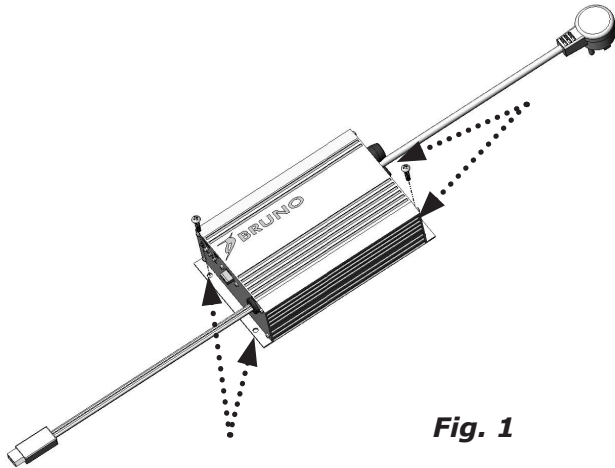
Fig. 1


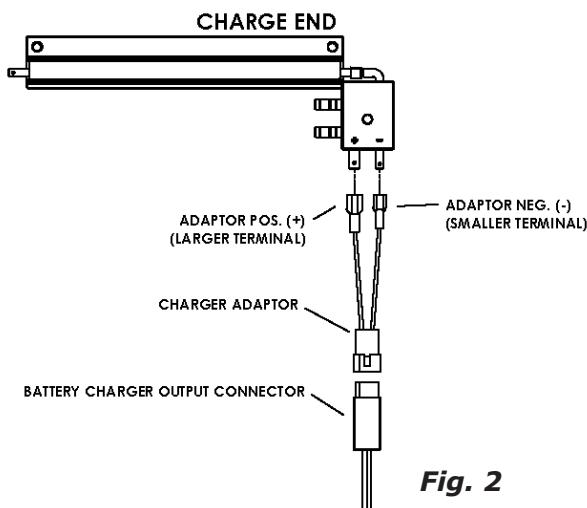
Fig. 2

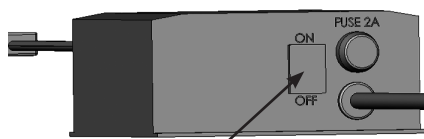charger power switch Fig. 3

1. Secure charger in chosen location by inserting two (2) provided screws through mounting holes in charger base (Fig. 1).

**NOTE: There are two (2) mounting holes on each side of charger base. Choose one of the two on each side.**

2. Connect power cord to nearest wall/floor outlet.

3. Connect battery charger output harness to rail charge connector using provided adaptor (Fig. 2).

4. Cable tie adapter wires to rail charge connector.

5. Bundle together loose wires.

6. Turn ON the charger power switch (Fig. 3).

**NOTES: If batteries become discharged, carriage will move more slowly until voltage drops to a point where controller will shut off.**

**If this happens, pause, then run carriage "down" until it engages charge contacts.**

**Partially charged batteries will run carriage "down", but not "up".**

**Confirm that charger is plugged into a "live" outlet and wait for batteries to recharge.**

## **FIT AND FINISH**

### ***Run carriage up and down rail to:***

- Recheck seat-to-wall clearance.
- Check correct operation of all components.

### ***Test main components:***

- Check proper operation of battery charger (see page 37).
- Check remote transmitters (see **Reprogramming Remote Transmitters** on page 34).
- Check on/off switch.
- Check footrest safety switches.
- Check that stairlift cannot operate with seat swiveled.
- Check carriage end safety switches.

### ***Test power folding rail (if applicable):***

- Check that when disable switch is turned off that rail cannot fold.
- Listen for continuous beep when folding rail is in motion. (This warns nearby people of rail operation.)
- Check that control box light flashes yellow.

### ***Make final adjustments (see Post-Installation Adjustments):***

- Adjust seat height.
- Adjust footrest.
- Adjust seat arm extension.

# INSTALLATION

## CUSTOMER EDUCATION

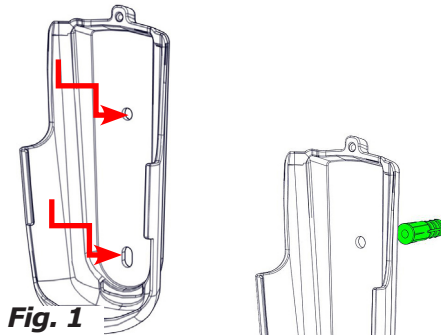
Fig. 1

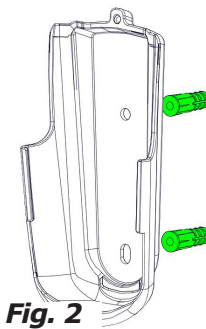
Fig. 2

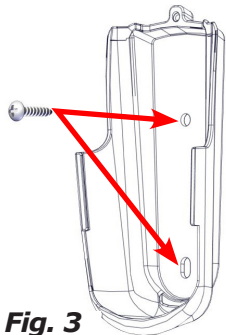
Fig. 3

Fig. 4

Fig. 5

1. Inform customer of location of Operator's Manual. Encourage them to become familiar with the contents.

2. **Mount Remote Transmitter with OPTIONAL TMR-00005 kit:**

- Discuss bracket location with customer before mounting to wall.
- Position bracket on wall. Mark top and bottom bracket holes (Fig. 1).
- Drill a pilot hole slightly smaller than anchors provided.
- Gently pound anchors into wall with rubber mallet.
- Position bracket on wall so top/bottom bracket holes align with anchors (Fig. 2).
- Secure bracket to wall using screws provided (Fig. 3).
- Slide transmitter into bracket (Fig. 4).
- *Optional:* Install additional screw provided to prevent removal of transmitter from bracket (Fig. 5).

3. **Test remote transmitters.**

- Use rocker switch on seat armrest to run unit up and down the rail.

**NOTE: Unit should travel noticeably faster going up than down.**

- Check for smooth operation and travel.
- Use remote transmitter to run unit up and down rail.
- Repeat Step 3 with second transmitter.
- Check for smooth operation and travel.

## CUSTOMER EDUCATION (continued)

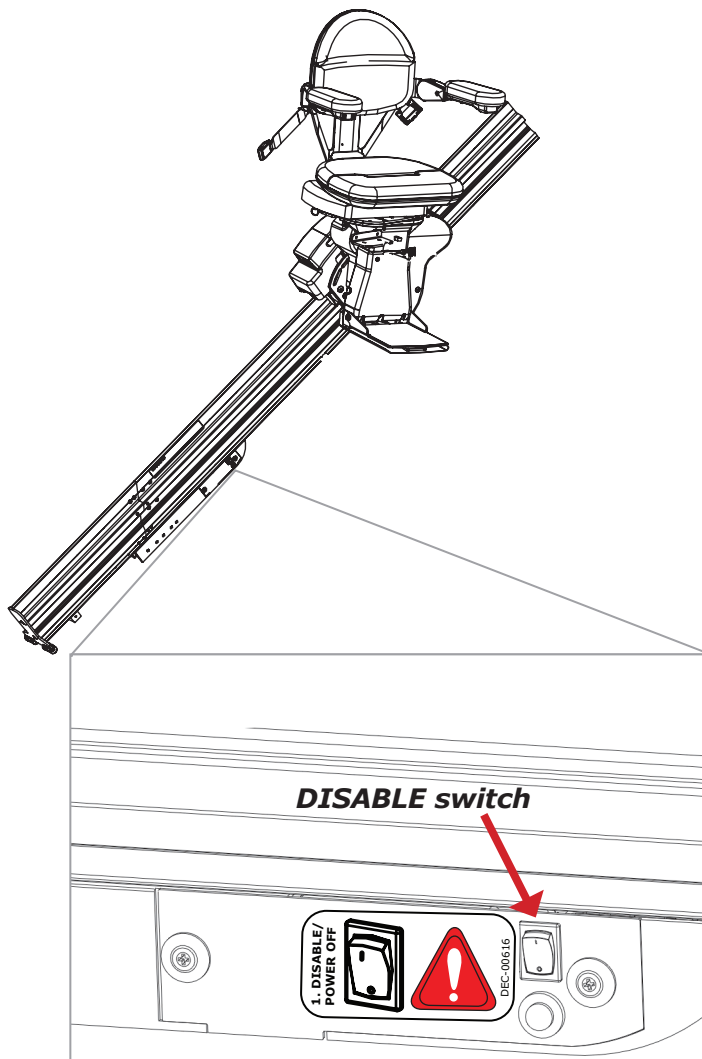
**⚠ WARNING**

**PINCH POINT!**

Make sure the areas above and below the folding rail are clear before operating the folding rail.

Keep children and pets at a safe distance.

Failure to observe this warning could result in injury or damage to equipment.


### 4. **Train customer to use stairlift correctly and safely.**

- Remind them to always buckle lap belt when using stairlift.
- Have them operate unit while you are there to answer questions and address concerns.
- **Manual Folding Rail:**  
When folding the rail UP, carriage **MUST** be at a position that allows rail to fully fold up!
- **Damage to carriage WILL occur** if carriage is below mid-park position when rail is folded up.
- **Power Folding Rail:**  
Will **only** work when carriage is at mid-park position.

### **Review disable switch.**

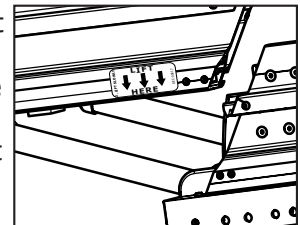- Control panel includes a disable switch. Press switch toward DISABLE to prevent rail from folding up. Use this switch if rail is to be kept folded down for an extended period of time.
- The disable switch also allows back drive of power in the unlikely event of an emergency.
- Make sure to lift slowly where indicated by the "LIFT HERE" decal on the short rail. Lifting the rail in a quick, sudden motion may result in damage to the folding rail gearbox.
- Press switch AWAY from DISABLE to allow rail to fold and unfold correctly under power.

**NOTE: If power folding rail stops, then slightly reverses direction, and there are NO OBSTRUCTIONS, call Bruno Technical Service.**

## REPROGRAMMING REMOTE TRANSMITTERS

**NOTE: Operating channel of two (2) transmitters included with SRE-2010 is pre-set at the Bruno factory.**

*Reasons for reprogramming transmitters:*

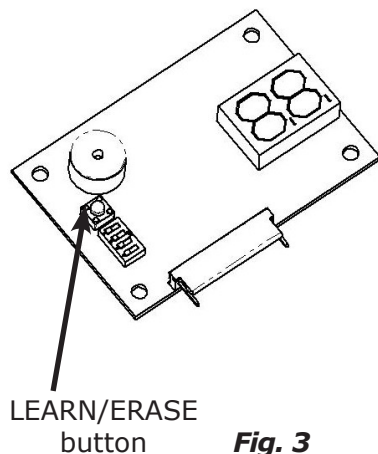
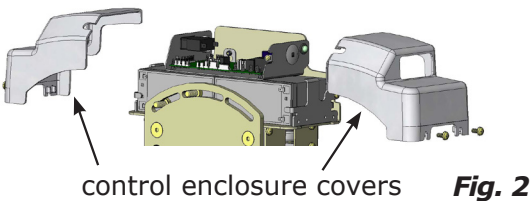
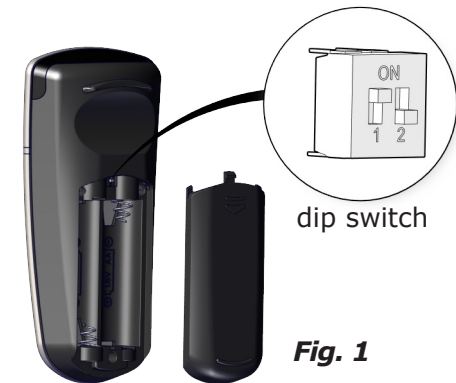
- multiple units in same location;
- transmitters need to be replaced.

**Reset operating channel of transmitters:**

1. Remove transmitter(s) battery access panel.
2. Locate DIP switch (Fig. 1).
3. Change BOTH transmitter DIP switches to one (1) of four (4) possible options.
  - **1 up, 2 down (default factory setting)**
  - **1 up, 2 up**
  - **1 down, 2 up**
  - **1 down, 2 down**

**Reprogramming transmitters:**

1. Turn circuit breaker OFF.
2. Remove seat hinge plate cover to expose circuit board.
3. For left-hand unit, remove lower side control enclosure cover (Fig. 2); for right-hand unit, remove upper side control enclosure cover to view LEARN/ERASE LED on circuit board.
4. Turn circuit breaker ON. Wait for "power-up" beep.
5. Locate LEARN/ERASE button on circuit board (Fig. 3).
6. **Clear memory.**
  - Hold down LEARN/ERASE button until the LED on circuit board goes out (approximately 12 seconds).

## **REPROGRAMMING REMOTE TRANSMITTERS (continued)**

7. Press and hold LEARN/ERASE button (LED on circuit board is on). As LEARN/ERASE button is pressed down, press either of the remote transmitter buttons until LED goes out (approximately 2-5 sec.).
8. Release both buttons. Remote is now reprogrammed.
9. ***Test transmitter operation*** by pressing either transmitter buttons.
  - *If carriage moves, new configuration has been accepted and transmitter is reprogrammed.*
  - *If carriage does not move, repeat Steps 6 through 9.*
10. Check that all transmitters are set to same DIP switch configuration.

**NOTE: It is only necessary to reprogram one transmitter if all transmitter DIP switches are set the same.**

11. Turn circuit breaker OFF.
12. Remount the control enclosure cover(s) and the seat hinge plate cover.
13. Turn circuit breaker ON. Verify operation of all functions.

## SINGLE-USER VS. MULTI-USER DIAGNOSTIC MODE

**NOTICE:**

Before touching anything inside carriage assembly, ground yourself by touching an unpainted metal surface on unit such as an exposed bolt, or one of the mounting screws on electrical panel.

While working, periodically touch an unpainted metal surface to dissipate any static electricity that could harm internal components.

**NOTES: Units ship in multi-user mode.**

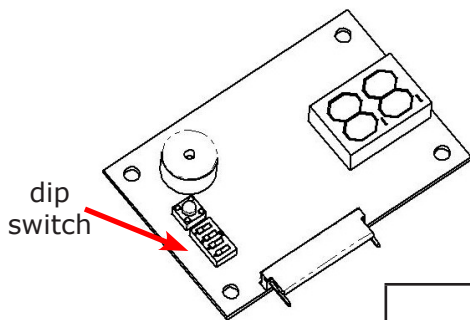
**Multi-user mode means that when seat is swiveled at top of rail, unit beeps once every three (3) seconds.**

**Single-user mode means that when seat is swiveled at top of rail, no beeps are heard.**

<b>AUDIO REFERENCE</b>	
Chirp	0.25 second
Short Beep	0.50 second
Long Beep	1.50 seconds
Pause	1.00 second

### ***Changing Multi-User to Single-User Mode***


1. Turn circuit breaker OFF.
2. Fold up seat. Remove screws securing seat hinge plate cover to provide access to interface.
3. Changes are made via the #1 and #2 positions on 4-Ganged DIP Switch (see chart below).
4. Remount the seat hinge plate cover.
5. Turn circuit breaker ON, wait for 'power-up' beep.

### **4 GANGED DIP SWITCH**

<b>DIAGNOSTIC MODE</b>	<b>DIP SWITCH POSITION</b>	
	<b>#1</b>	<b>#2</b>
Multi-User	OFF	OFF
Single-User	OFF	ON
Quiet	ON	OFF
Battery Warnings Only	ON	ON

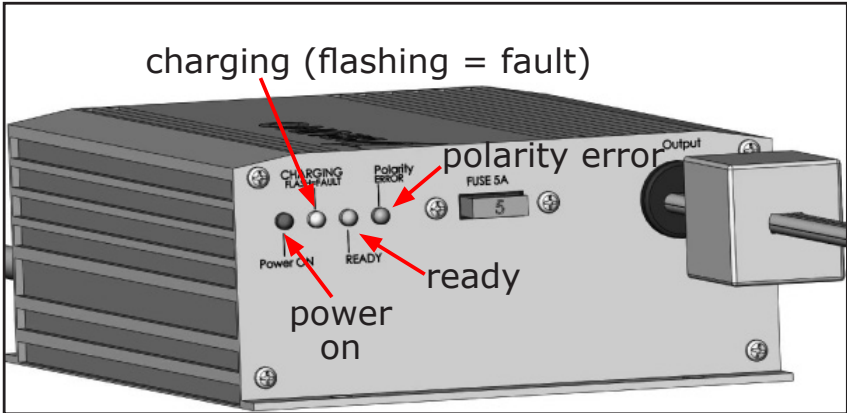**BATTERY CHARGER LED INFORMATION**

**6) LED indicators (status and color):**

STATUS	RED	YELLOW	GREEN	blue	COMMENT
AC Power On without any battery connected	ON	ON (< 1 sec.) OFF	OFF	OFF	Charger has not been connected to any batteries.
AC Power On	ON (< 5 sec.)	ON (< 5 sec.)	ON (< 5 sec.)	OFF	This feature detects any battery disconnect situation and has a 5 sec. delay until yellow & green are completely off.
Event battery disconnect	ON	OFF	BLINK (5 sec.)	OFF	
Bulk Charge (Constant current)	ON	ON	OFF	OFF	At this stage the charger is delivering max current (2.2 A)
Constant Voltage	ON	ON	OFF	OFF	At this stage the charger has reached maximum voltage and current continues to drop.
Float Charge	ON	OFF	ON	OFF	At this stage batteries are fully charged. The Yellow LED turns occasionally off.
Abnormal Conditions:				OFF	This feature indicates that the battery voltage is not climbing to acceptable level
Low Voltage Indication	ON	BLINKS ALTERNATELY	BLINKS ALTERNATELY		
Bad Battery/Sulfation	BLINKS ALTERNATELY	BLINKS ALTERNATELY	OFF		
Bad contact / Partial battery sulfation	ON	OFF	FAST BLINK		This feature indicates a partially open circuit. Bad contact or battery sulfation
Battery connector reverse				ON	

# POST-INSTALLATION ADJUSTMENTS

## FOOTREST HEIGHT ADJUSTMENT

1. Remove four (4) M10 x 16mm bolts.
2. Loosen seat support bracket bolt (M10) connecting seat to carriage.
3. Slide seat up or down.
4. Line up holes on seat post with holes on footrest mounting plate at desired height.
5. Reinsert M10 x 16mm bolts and corresponding washers.
6. Tighten.
7. Retighten seat support bracket bolt connecting seat to carriage.

### NOTES:


- **If you raise footrest and then wish to adjust seat height, it is not possible to mount seat in highest position. Seat will no longer properly engage seat support bracket. (See tables and picture at left.)**
- **If you wish to mount seat in one of the “~” positions, contact a Bruno Service Representative to order a large seat bottom bracket (SRE-20710).**

<i>Seat Height Position (total seat height in inches)</i>				
<i>Footrest Position</i>	0	1	2	3
0	21.75	22.75	23.75	24.75
0.63	22.38	23.38	24.38	~
1.26	23.01	24.01	~	~


*NOTE: 0,0 = factory setting  
~ = exceeds seat support bracket slot length*

<i>Seat Height Position with SRE-20710 (total seat height in inches)</i>				
<i>Footrest Position</i>	0	1	2	3
0	X	X	23.75	24.75
0.63	X	23.38	24.38	25.38
1.25	23.00	24.00	25.00	26.00

## DRIVE ADJUSTMENT

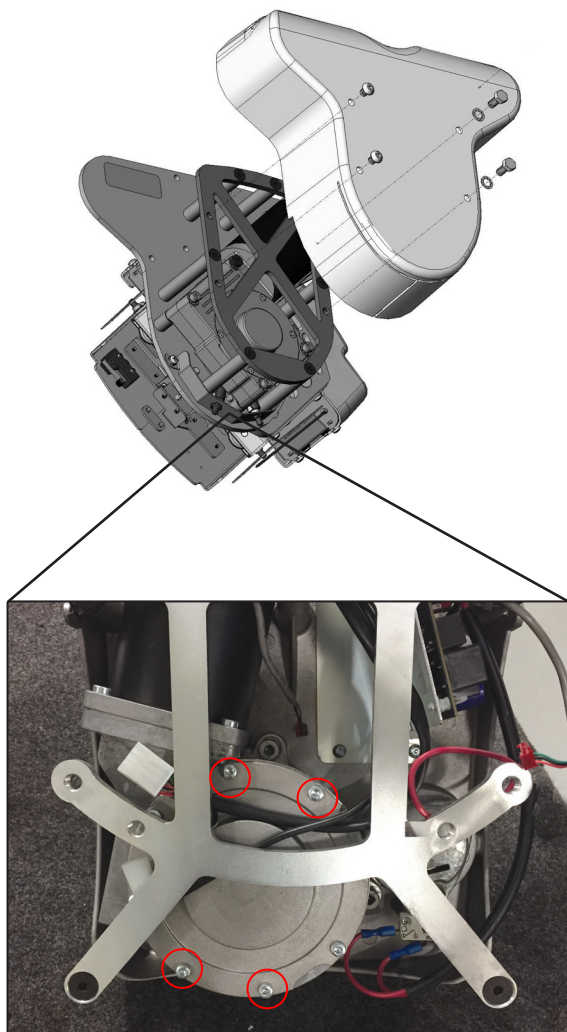**Fig. 1**

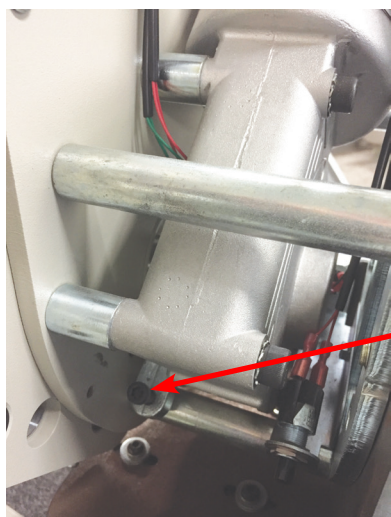**Fig. 2**


Even though each SRE-2010 is run on a test rail prior to shipment, there may be slight variations between the test rail and rail installed on site. These variations may result in a small but objectionable vibration of the carriage.

### ***Adjust rack and pinion gear to eliminate vibration.***

1. Make sure rack is properly greased.  
Inspect both gear rack and pinion for contaminants.
2. Remove seat and footrest assemblies.
3. With carriage installed on rail, and jumpers installed on carriage, run unit (loaded and unloaded) up and down rail over a few feet. Listen and watch for vibration.
  - a. If unit vibrates only in down direction, carriage and rail are fitted too loosely. The gear must be seated further into rack mesh.
  - b. If unit vibrates when running up and down, carriage and rail are fitted too tightly. The gear must be seated further out from rack mesh.
4. In both cases, loosen (but **DO NOT REMOVE**) four (4) drive mounting bolts (Fig. 1) 1-2 turns using 5mm and 6 mm Allen wrenches.
5. Tighten jack screw 1/8th turn (Fig. 2).
6. Run carriage on rail about 3' (1 m) in each direction with bolts still loose. This allows pinion gear to find the natural center of the gear rack.
  - If carriage runs smoothly, retighten drive mounting bolts.
  - If carriage does not run smoothly, repeat step 5.
7. *Making sure drive does not move as you tighten*, retighten four (4) drive mounting bolts.

# POST-INSTALLATION ADJUSTMENTS

## CARRIAGE ANGLE ADJUSTMENT

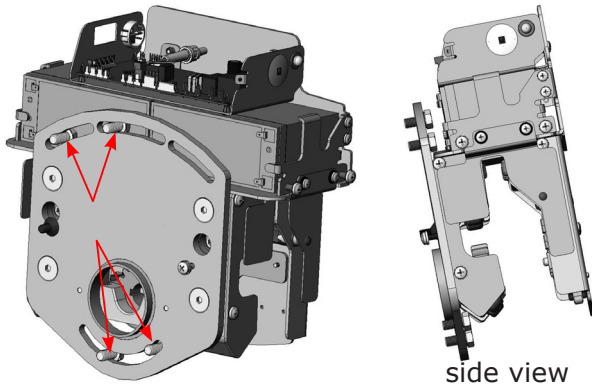
**Fig. 1**

1. Remove front and top carriage covers.
2. Remove batteries.

**NOTE: Do not ground battery posts to steel plates while removing.**

3. Loosen four (4) hex head bolts (Fig. 1).
4. Pivot carriage to desired angle.
5. Tighten four (4) hex head bolts.
6. Reinsert batteries and reconnect.
7. Remount and secure front and top carriage covers.

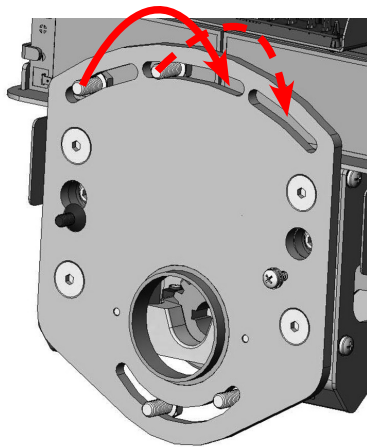**Fig. 2**

**NOTE: The angle adjustment plate (Fig. 2) is designed with multiple slots (rather than one continuous, open slot) for added safety.**

**Bolts are inserted at the factory into the slots to adapt to "normal" range of angle adjustments.**

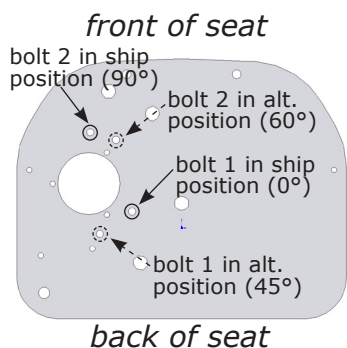
**However, your application may involve angles that are steeper or shallower than "normal".**

**In this case, remove bolts and reinsert them in adjoining slots.**






## SEAT SWIVEL STOP ADJUSTMENT

*left-hand orientation shown  
(top view)*

1. Remove lower seat hinge plate cover.
2. Remove bolts 1 and 2, ship position.
3. Pivot seat to expose holes for bolts 1 and 2, alternate position.
4. Apply Loctite 242® (blue) to bolts 1 and 2, insert and tighten to appropriate torque.
5. Remount lower seat hinge plate and secure with its two (2) screws.

## CARRIAGE ADJUSTMENT
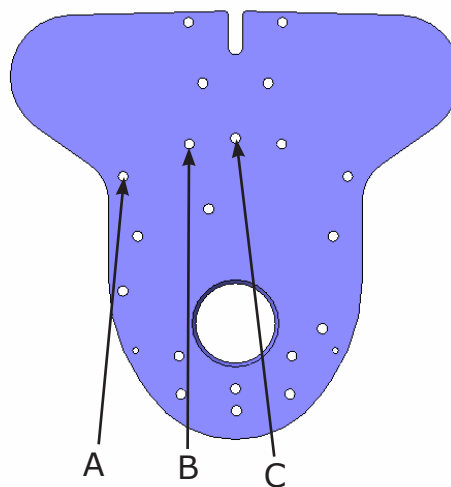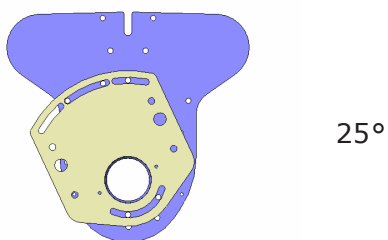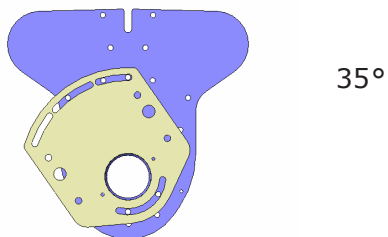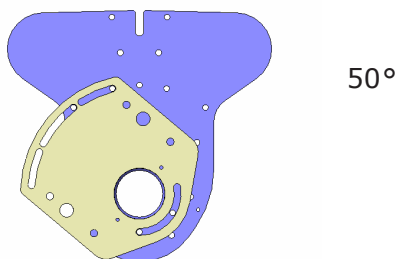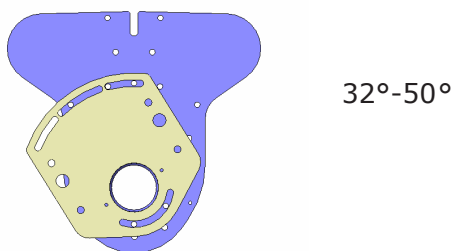
<b>Angle Range</b>	<b>Bolt Location</b>		
	<b>A</b>	<b>B</b>	<b>C</b>
32°-50°	Bolt 1	Bolt 2	None
25°-35°	Bolt 1	None	Bolt 2

Under 25°, use any two (2) of five top tapped holes that give maximum distance between holes for that location.

# POST-INSTALLATION ADJUSTMENTS

## ARMREST EXTENSION ADJUSTMENT

**Fig. 1**

**Fig. 2**

**Fig. 3**

**Fig. 4**

**Fig. 5**

1. Remove arm swivel cover (Fig. 1).
2. Cut cable tie (Fig. 2).
3. Remove three (3) arm bracket screws (Fig. 3).
4. Slide arm out to position that best fits user (Fig. 4).  
**NOTE: There are three (3) possible settings.**
5. Insert and tighten three (3) arm bracket screws.
6. Wind wire into a service loop (no cable tie needed).
7. Remount arm swivel cover and secure with its screw.
  - **Do not overtighten!**
  - **Make sure wire sits in cover slot (Fig. 5).**
8. Repeat these steps for opposite side (units without a keyswitch).

<b>UNIT DOES NOT OPERATE/ IS NOT CHARGING</b>	<ul style="list-style-type: none"> <li>• Check circuit breaker. <i>Reset if necessary.</i></li> </ul>
	<ul style="list-style-type: none"> <li>• Check battery connections.</li> </ul>
	<ul style="list-style-type: none"> <li>• Check footrest safety switches to see if one of these limit switches is depressed. <i>Sliding safety tray below footrest should slide freely and should not stick in a position which would depress one of the safety switches.</i></li> </ul>
	<ul style="list-style-type: none"> <li>• Check for discharged batteries. <i>Battery voltage should fall within the range of 16-28 VDC.</i></li> </ul>
	<ul style="list-style-type: none"> <li>• Check that keyswitch is ON.</li> </ul>
<b>UNIT OPERATES SLOWLY, LACKS POWER</b>	<ul style="list-style-type: none"> <li>• Check for discharged batteries.</li> </ul>
	<ul style="list-style-type: none"> <li>• Check for loose connections.</li> </ul>
	<ul style="list-style-type: none"> <li>• Check that charger is plugged in and working.</li> </ul>
<b>UNIT OPERATES BACKWARDS WITH REMOTE TRANSMITTER; UNIT GOES UP SLOWLY AND DOWN FASTER</b>	<ul style="list-style-type: none"> <li>• Unit has been installed on right-hand side of stairs and dip switch has not been set to right-hand and/or seat has not been installed.</li> </ul>
<b>UNIT OPERATES ERRATICALLY OR INTERMITTENTLY WITH REMOTE TRANSMITTER</b>	<ul style="list-style-type: none"> <li>• Check all safety mechanisms, including swivel seat safety switch.</li> </ul>
	<ul style="list-style-type: none"> <li>• Reorient transmitter to point directly at carriage.</li> </ul>
	<ul style="list-style-type: none"> <li>• Consult your dealer, an experienced technician or call the Bruno Technical Service Department.</li> </ul>
<b>UNIT OPERATES ERRATICALLY OR INTERMITTENTLY WITH A RIDER USING ARMREST MOUNTED CONTROL SWITCH</b>	<ul style="list-style-type: none"> <li>• Check swivel seat safety switch.</li> </ul>
	<ul style="list-style-type: none"> <li>• Check that footrest safety tray is not dragging on stair nosing or hitting debris on stairs.</li> </ul>
	<ul style="list-style-type: none"> <li>• Check rail for debris that may bump safety switches (footrest and carriage panels).</li> </ul>
<b>UNIT WILL NOT OPERATE UNLESS SEAT IS POSITIONED SO IT FACES OPEN SIDE OF STAIRS</b>	<ul style="list-style-type: none"> <li>• This is correct lift operation.</li> </ul>
	<ul style="list-style-type: none"> <li>• A safety switch in the seat swivel prevents unit from operating with seat "out of position".</li> </ul>
<b>UNIT WILL NOT OPERATE WITH REMOTE TRANSMITTER</b>	<ul style="list-style-type: none"> <li>• Check batteries in remote transmitter.</li> </ul>
	<ul style="list-style-type: none"> <li>• Aim transmitter at carriage.</li> </ul>
	<ul style="list-style-type: none"> <li>• Transmitters must be programmed.</li> </ul>
<b>UNIT DOES NOT SHUT OFF WHEN IT HITS BUMPER AT END OF RAIL</b>	<ul style="list-style-type: none"> <li>• Verify correct operation of limit switches in carriage assembly.</li> </ul>
<b>ROUGH OR CHATTERING RIDE</b>	<ul style="list-style-type: none"> <li>• Wipe down rail.</li> </ul>
	<ul style="list-style-type: none"> <li>• Apply a small amount of grease to <b><i>gear rack only.</i></b></li> </ul>

# TROUBLESHOOTING

## DIAGNOSTIC CODES

Cross reference the code displayed under the seat with the following table.

Readout/Diagnostic Group (Group Indicator)	S - Solid = 'Normal' Condition CF - Continuous Flash = Diagnostic Fault	VISUAL		AUDIBLE		VISUAL		AUDIBLE		AUDIBLE			
		Segment 1	Segment 2	Mode 1 S1 Off S2-Off	Mode 2 S1-Off S2-Off	Mode 3 S1-On S2-Off	Mode 4 S1-On S2-On	Segment 1	Segment 2	Mode 1 S1 Off S2-Off	Mode 2 S1-Off S2-On	Mode 3 S1-On S2-Off	Mode 4 S1-On S2-On
		CRE & SRE											
<b>Charge/Battery Condition Group (C)</b>													
Charging (Charger detected)	S	C	1	None	None	None	C	1	None	None	None	None	
Off-Charge (running w/rocker or remote)	S	C	2	None	None	None	C	2	None	None	None	None	
Off-Charge and Not-Parked (fault condition)	CF	C	3	BBBBL	BBBBL	None	NA	NA	NA	NA	NA	NA	
Parked and Charger Not Detected	CF	C	4	BBBBL	BBBBL	None	C	4	BBBBL	BBBBL	None	BBBBL	
Battery Voltage Critical (~16V)	CF	C	5	BBBBL	BBBBL	None	C	5	BBBBL	BBBBL	None	BBBBL	
Battery Voltage Low (<18V)	CF	C	6	None	None	None	C	6	None	None	None	None	
Motorized Swivel Moving (Prior to SW Rev. 2.7 only-see A6)	S	C	7	None	None	None	NA	NA	NA	NA	NA	NA	
Motorized Footrest Moving (Prior to SW Rev. 2.7 only-see A7)	S	C	8	None	None	None	NA	NA	NA	NA	NA	NA	
<b>Limit/Switch Input Group (L)</b>													
SSS open	S	L	1	S.M.C.P.P.P	S.M	None	L	1	S.M.C.P.P.P	S.M	None	None	
Up Limit Switch Open	S	L	2	S.M	S.M	None	L	2	S.M	S.M	None	None	
Down Limit Switch Open	S	L	3	S.M	S.M	None	L	3	S.M	S.M	None	None	
Keyswitch Closed (not currently used)	S	L	4	M.P.P.C.C	M	None	NA	NA	NA	NA	NA	NA	
Seat Fold Switch Open (not currently used)	S	L	5	M	M	None	NA	NA	NA	NA	NA	NA	
Mid-Rail Park Active	S	L	6	S	S	None	L	6	S	S	None	None	
Brake Input switch	S	L	7	M	M	None	NA	NA	NA	NA	NA	NA	
Soft Stop Switch	S	L	8	None	None	None	L	8	None	None	None	None	
<b>Operation Switch Group (O)</b>													
Rocker Up	S	O	1	None	None	None	O	1	None	None	None	None	
Rocker Down	S	O	2	None	None	None	O	2	None	None	None	None	
Learn Switch	S	O	3	None	None	None	O	3	None	None	None	None	
Remote Up	S	O	4	None	None	None	O	4	None	None	None	None	
Remote Down	S	O	5	None	None	None	O	5	None	None	None	None	
<b>Diagnostics Group (d)</b>													
Speed Pot Fail (open)	CF	d	1	None	None	None	d	1	None	None	None	None	
Transmitter ID Memory Full	CF	d	2	BLL	BLL	None	d	2	BLL	BLL	None	BLL	
Program Memory Bad CheckSum	CF	d	3	None	None	None	d	3	None	None	None	None	
Over Current Fault	CF	d	4	C.C.C.C	C.C.C.C	None	d	4	C.C.C.C	C.C.C.C	None	C.C.C.C	
Over Temperature Fault	CF	d	5	C.C.C	C.C.C	None	d	5	C.C.C	C.C.C	None	C.C.C	
Speed Over Ride on High Temperature.	CF	d	6	None	None	None	d	6	None	None	None	None	
Low Voltage Stop	CF	d	7	C.C.C.C.C	C.C.C.C.C	None	d	7	C.C.C.C.C	C.C.C.C.C	None	C.C.C.C.C	
Unlearned Remote Unit	CF	d	8	None	None	None	d	8	None	None	None	None	
<b>Relay FET Fault Group (F)</b>													
Drive FET Shorted	CF	F	1	None	None	None	F	1	None	None	None	None	
Safety Relay Shorted	CF	F	2	None	None	None	F	2	None	None	None	None	
H-Bridge Relay Fault	CF	F	3	None	None	None	F	3	None	None	None	None	
Safety Circuit Failed	CF	F	4	None	None	None	F	4	None	None	None	None	
Test Input High (Internal Failure, should never happen)	CF	F	5	None	None	None	F	5	None	None	None	None	
Test Input Low (Internal Failure, should never happen)	CF	F	6	None	None	None	F	6	None	None	None	None	
Aux Current Measurement Failure (Prior to SW Rev. 2.7 only-see A9)	CF	F	7	None	None	None	F	7	None	None	None	None	
Unknown Relay Fault (should never happen)	CF	F	8	None	None	None	F	8	None	None	None	None	

# TROUBLESHOOTING

## RESET FOLDING RAIL JOINT

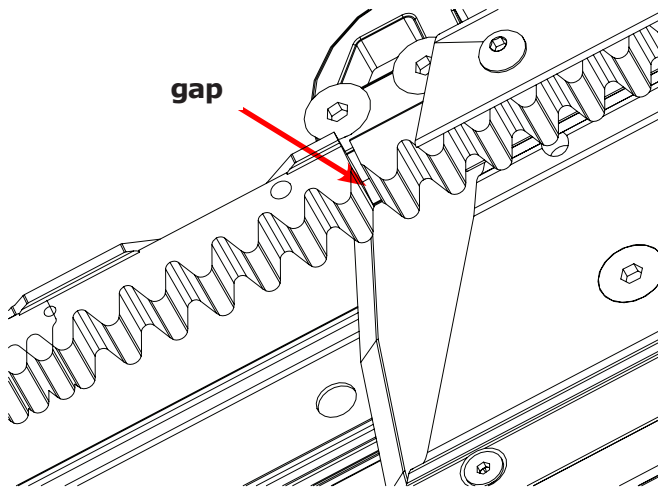
Fig. 1

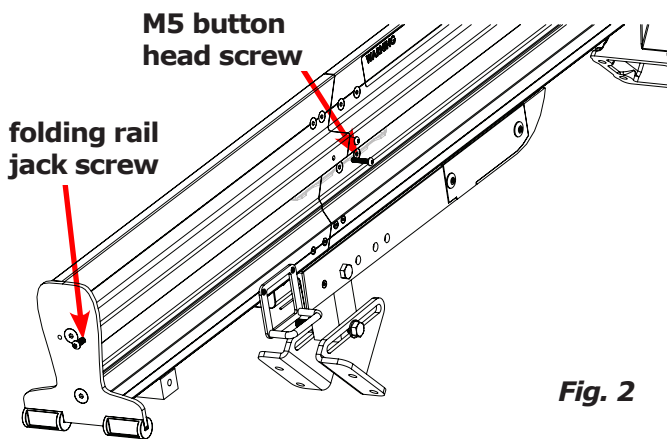
Fig. 2

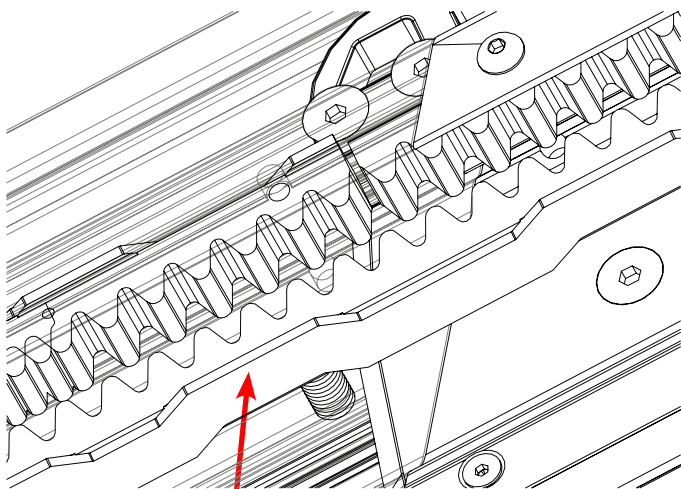
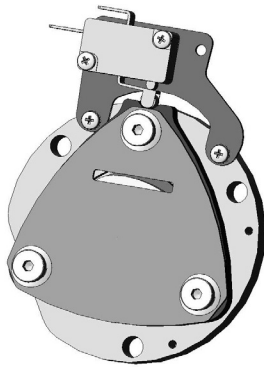
Fig. 3

1. Confirm factory-set gear rack gap (slight space between main rail and folding rail gear racks) is at folding joint (Fig. 1).
2. Loosen folding rail jack screw after jack screw in top endplate has been tightened.
3. Loosen M5 button head screw securing folding rail rack (Fig. 2).
4. Use a piece of spare gear rack to cover joint between main rail and folding rail (Fig. 3). This ensures pitch is maintained for safe travel of drive gear and overspeed gear (SRE-2010C/OS only) across joint.
5. Slowly tighten folding rail jack screw until loose gear rack piece falls away.
6. Tighten M5 button head screw to secure top piece of short folding rail.

**NOTE: NEVER remove main rail screw from rail! Carriage and gear rack segments will fall.**

## OVERSPEED FOR SRE-2010C & SRE-2010OS INSTALLATIONS

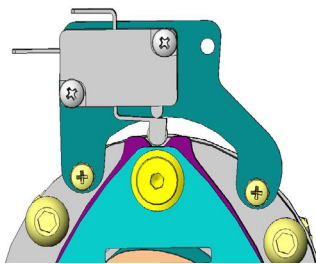
**Fig. 1**

The overspeed is a safety device designed to prevent operation of stairlift if descending travel speed becomes excessive.

When overspeed switch is tripped, wheel will not be in the center of notch (valley). See Fig. 1.

**You must reset the overspeed switch** to allow unit to operate (Fig. 2).

**Fig. 2**

### **Reset Overspeed Switch**


1. Remove access cap from motor side (Fig. 3).
2. Using a 5/16 socket tool, turn gear motor **clockwise** to move carriage UP. Continue turning until unit moves freely (Fig. 4).

**The overspeed switch should automatically reset.**

3. Check for damage to drive system.
4. If drive system is not damaged, use rocker switch to drive carriage off rail for a more complete inspection.

### **If Switch Does Not Reset**

1. Remove two (2) front covers.
2. Remove seat.
3. Remove footrest.
4. Remove drive cover.
5. Turn cam ring so switch roller is centered in cam valley.
6. Remove and inspect both drive system and carriage.
7. If possible, repair as needed and reassemble. Otherwise, return for factory service.

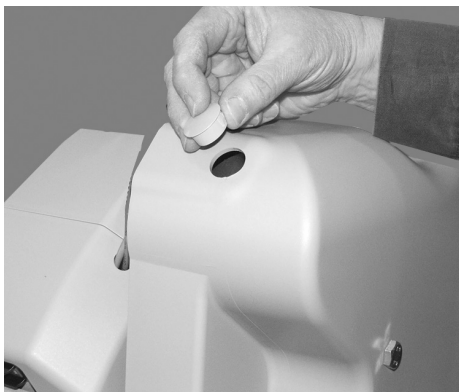**Fig. 3**

**Fig. 4**

## **BRUNO GOLD LIMITED WARRANTY FOR STAIRLIFT Limited Lifetime Major Components — 2 Year Parts**

Bruno Independent Living Aids, Inc. (Bruno) is confident in its USA-designed and built stairlift units and is pleased to provide the Original Purchaser of a Bruno Stairlift with the following "Industry Leading" warranty. The quality of parts used, along with the care of Bruno's employees in constructing Bruno's products, allow Bruno to provide the following warranty coverage to the Original Purchaser of the Bruno Stairlift:

- ◇ All Parts, other than Major Components, are/will be free of defects in material and workmanship for a period of two (2) years after the date of installation by a Bruno Authorized Dealer.
- ◇ All Major Components, including the motor, gearbox, drivetrain, and rail system of the Bruno Stairlift, are/will be free of defects in material and workmanship for the lifetime of the Bruno Stairlift and as long as the Original Purchaser owns the Stairlift.
- ◇ All labor, to be provided by a Bruno Authorized Dealer, to address any defect of the Bruno Stairlift is covered for the first thirty (30) days after installation thereof by a Bruno Authorized Dealer.
- ◇ All Bruno warranties begin on the date of installation of the Bruno Stairlift by a Bruno Authorized Dealer, but that date should be no later than 14 months after it was shipped from the factory.

Bruno Authorized Dealers are trained to evaluate your unique situation and then install the best Bruno Stairlift to meet your needs. The trained installation and service technicians employed by a Bruno Authorized Dealer have the appropriate equipment and resources needed to correctly install and maintain the Bruno Stairlift and should be the first contact for any warranty or service needs.

### **WHAT IS NOT WARRANTY COVERAGE:**

- ◇ Damage to units caused by misuse, abuse, accidents, neglect or modifications NOT made by a Bruno Authorized Dealer.
- ◇ Consumable items such as remote batteries
- ◇ All labor by a Bruno Authorized Dealer after the first thirty (30) days since installation.
- ◇ Reasonable wear and tear.

### **ADDITIONAL NOTES:**

- ◇ All Service and Warranty claims must be brought to the attention of a Bruno Authorized Dealer. To ensure warranty coverage, all maintenance and repair work on the Bruno Stairlift, other than maintenance provided by the Original Purchaser as called for in the Operator's Manual, must be performed by a Bruno Authorized Dealer.
- ◇ In no event shall Bruno be responsible for indirect, incidental or consequential damages, whether such damages arise based on claims based on contract, warranty, tort (including negligence, strict liability or product liability).
- ◇ Bruno, at its sole discretion, reserves the right to replace, repair, or, if not commercially practical and/or feasible, elect to refund the Original Purchaser the purchase price of the Bruno Stairlift.
- ◇ This warranty gives the Original Purchaser specific legal rights for which the Original Purchaser may also have other rights that vary from state to state. Bruno specifically does not authorize any person to extend the time or scope of this warranty.

For further information regarding this limited warranty, please contact Bruno using the contact information shown below:

In North America:

Bruno Independent Living Aids, Inc.®  
Attn: Service Department  
1780 Executive Drive  
Oconomowoc, Wisconsin 53066 United States  
Tel. 1-800-882-8183 (toll free)

In the UK and Europe

Bruno  
Unit 5, Millennium Court,  
Clayhill Industrial Park Neston,  
Wirral CH64 3UZ United Kingdom  
Tel. 0151 559 0732

# CE DECLARATION OF CONFORMITY CERTIFICATE

We:

**Bruno Independent Living Aids, Inc.**

Headquartered at:

1780 Executive Drive

Oconomowoc, WI 53066 USA

An ISO-9001:2000 registered company

Declare that under our sole responsibility, our manufactured product:

MODEL: **SRE-2010**

has been designed and tested to meet (where applicable) the following laws, directives, specifications and standards pertaining to the Machinery Directive or Powered Inclined Stairlifts for those who have physical impairments:

DIRECTIVES

STANDARDS

2004/108/EC Electromagnetic Compatibility Directive

EN 81-40:2008

EN 60204 Part 1:2006

EN 954 Part 1:1996

2006/42/EC Machinery Directive

EN 60335 Part 1:1994

EN ISO 12100-1:2003

2006/95/EC Low Voltage Directive

EN ISO 12100-2:2003

EN 349:2008

EN 294:1992

CISPR 11

EN 61000

When installed in accordance to the manufacturer's specifications and operated in accordance with manufacturer's operating instructions.

We hereby appoint:

Bruno Lifts, Ltd.

Unit 5, Millennium Court

Clayhill Industrial Park Neston

Wirral CH64 3UZ

United Kingdom

To act as Responsible Manufacturer's Representative.

Date of Issue: 02 April 2018

Location: Oconomowoc, WI USA

BY:

William W. Belson III  
Director of Engineering

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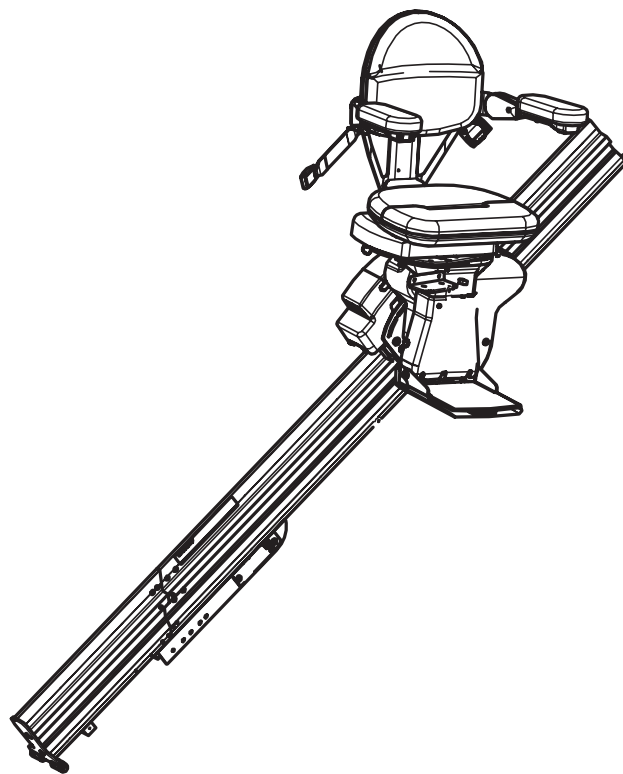
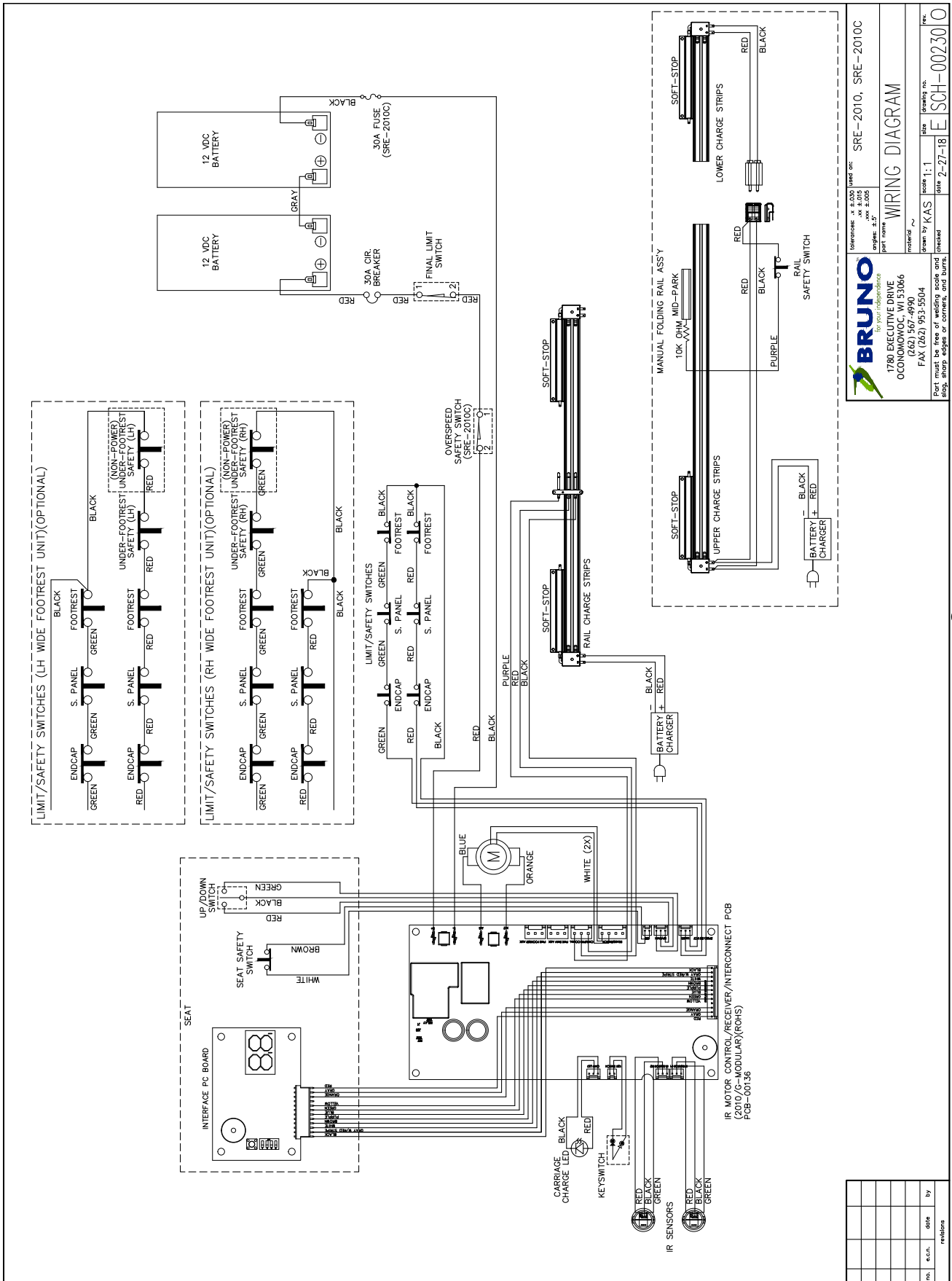
Part must be free of welding spalls and slag, sharp edges or burrs.

Scale: 1:1  
 Date: 2-27-18  
 Drawing No.: E SCH-0023010


Material: SRE-2010, SRE-2010C

WIRING DIAGRAM

Drawn by: KAS  
 Checked: [Signature]

©Bruno Independent Living Aids, Inc. SHEET 1 OF 2

# WIRING SCHEMATICS / ESQUEMA DE CABLEADO / SCHÉMAS DE CÂBLAGE

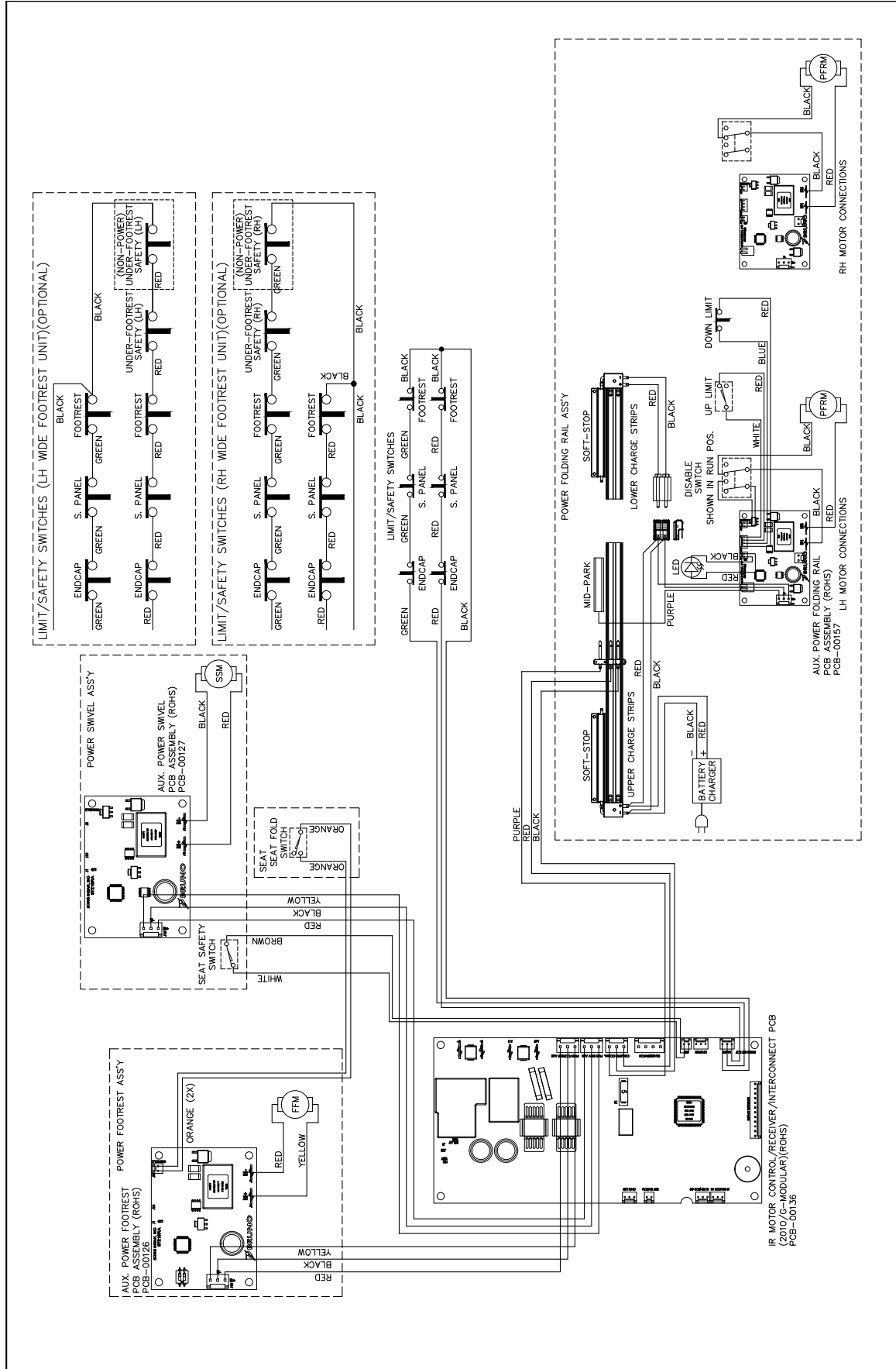**BRUNO**  
*For Your Independence*  
 1780 EXECUTIVE DRIVE  
 OCONOMOWOC, WI 53066  
 (262) 567-4990  
 FAX (262) 953-5504  
 Part must be free of welding scale and slag, sharp edges or corners, and burrs.

Material: **WIRING DIAGRAM-POWER OPTIONS**  
 Part name: **WIRING DIAGRAM-POWER OPTIONS**  
 Material: **WIRING DIAGRAM-POWER OPTIONS**  
 Part name: **WIRING DIAGRAM-POWER OPTIONS**  
 Material: **WIRING DIAGRAM-POWER OPTIONS**  
 Part name: **WIRING DIAGRAM-POWER OPTIONS**

Scale: 1:1  
 Date: 2-27-18  
 Drawing No.: SCH-00230  
 Rev: 0

Material: SRE-2010, SRE-2010C  
 Part name: SRE-2010, SRE-2010C  
 Material: SRE-2010, SRE-2010C  
 Part name: SRE-2010, SRE-2010C

©Bruno Independent Living Aids, Inc.

SHEET 2 OF 2

Rev.	Qty.	Date	By


This material is the property of Bruno Independent Living Aids, Inc. No rights are granted to use such material for any purpose other than (1) product evaluation or maintenance, or (2) the furnishing of services to Bruno Independent Living Aids, Inc.

# 'ELITE' STRAIGHT RAIL STAIRLIFT

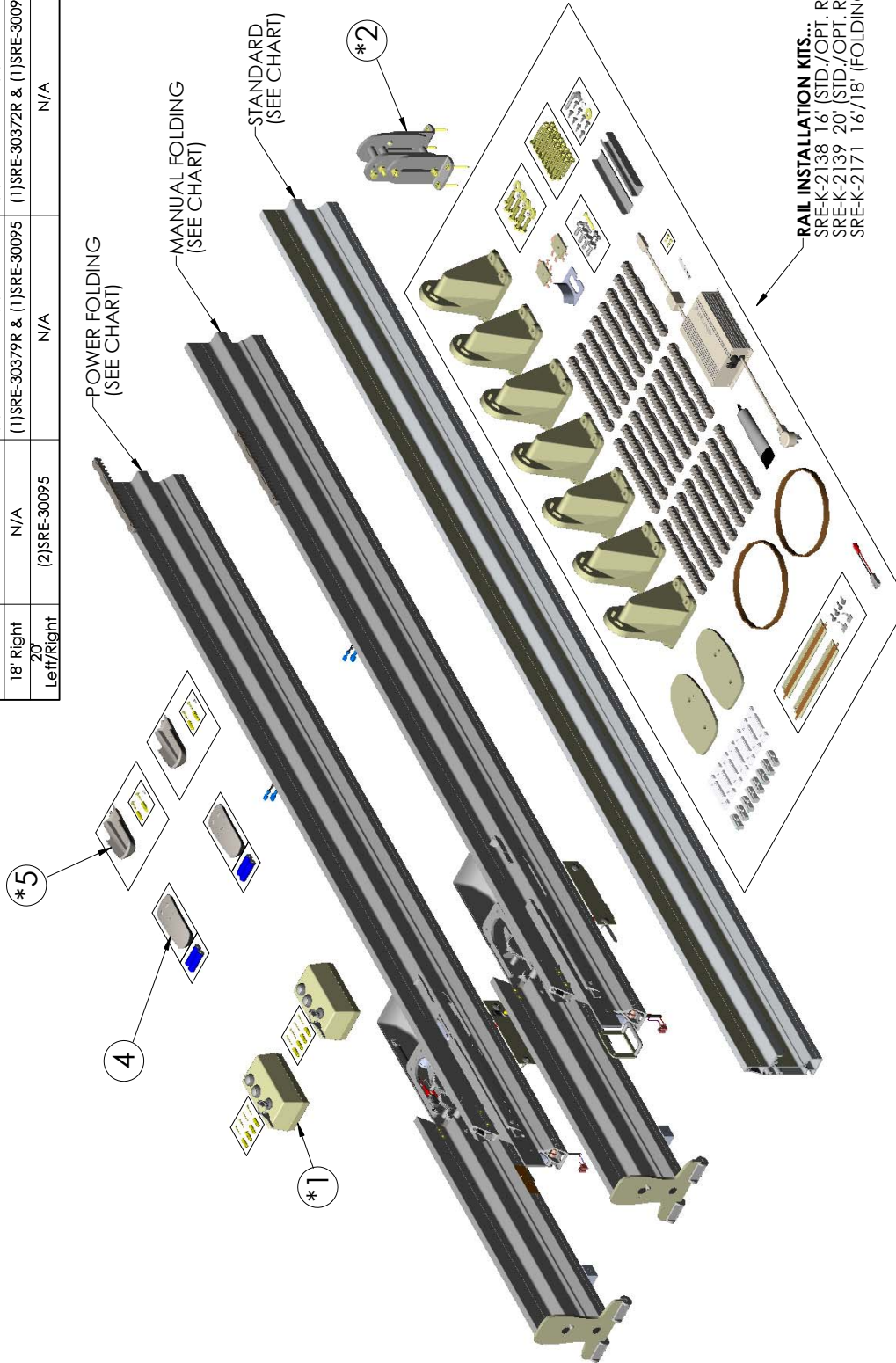ITEM NO.	QTY.	PART NO.	DESCRIPTION
*1	2	SRE-00963A	IR TRANSMITTER/BRACKET/HDW (KEYED)(ROHS)
*2	1	SRE-30150	ADJUSTABLE STAND FOOT ASSEMBLY
4	2	TMR-00004	IR TRANSMITTER ASSEMBLY W/ 2 'AA' BATTERIES
*5	2	TMR-00005	IR TRANSMITTER WALL MOUNT BRACKET W/ HARDWARE

\*OPTIONAL

RAIL MATRIX	Standard(Anodized)	Folding(Anodized)	Power Folding(Anodized)
16' Left/Right	(2)SRE-30094	N/A	N/A
16' Left	N/A	(1)SRE-30379L & (1)SRE-30094	(1)SRE-30372L & (1)SRE-30094
16' Right	N/A	(1)SRE-30379R & (1)SRE-30094	(1)SRE-30372R & (1)SRE-30094
18' Left	N/A	(1)SRE-30379L & (1)SRE-30095	(1)SRE-30372L & (1)SRE-30095
18' Right	N/A	(1)SRE-30379R & (1)SRE-30095	(1)SRE-30372R & (1)SRE-30095
20' Left/Right	(2)SRE-30095	N/A	N/A


**SRE-2010 Rail, Charger, & Call-Sends**  
**SHEET 1 OF 5**  
**REV. 32 (6395)(9/11/17)(RJH)**

# 'ELITE' STD. - NO POWER

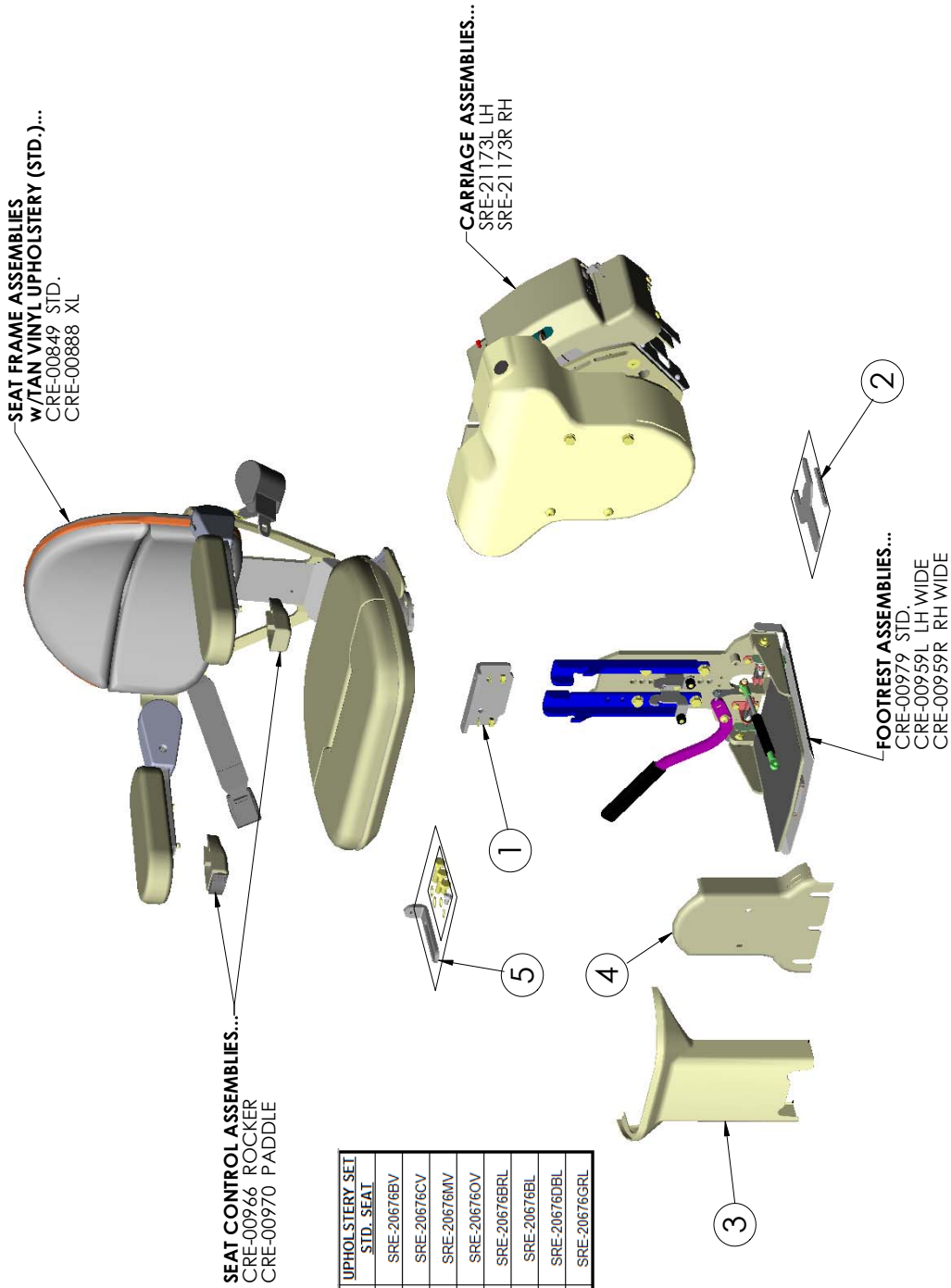ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	CRE-00895	SEAT MOUNTING BRACKET ASSEMBLY
2	1	CRE-K-2112	FOOTREST CYLINDER INSTALLATION KIT
3	1	SRE-20549	FRONT SEAT POST COVER
4	1	SRE-20402	FOOT REST COVER
5	1	SRE-K-2063	SEAT INSTALLATION KIT

SEAT CONTROL ASSEMBLIES...  
 CRE-00966 ROCKER  
 CRE-00970 PADDLE

SEAT FRAME ASSEMBLIES  
 W/TAN VINYL UPHOLSTERY (STD.)...  
 CRE-00849 STD.  
 CRE-00888 XL

CARRIAGE ASSEMBLIES...  
 SRE-21173L LH  
 SRE-21173R RH

FOOTREST ASSEMBLIES...  
 CRE-00979 STD.  
 CRE-00959L LH WIDE  
 CRE-00959R RH WIDE

ADDITIONAL SEAT COLORS/MATL	UPHOLSTERY SET STD. SEAT
LIGHT BROWN VINYL	SRE-20676BV
CHARCOAL VINYL	SRE-20676CV
MAROON VINYL	SRE-20676MV
OLIVE VINYL	SRE-20676OV
BROWN ULTRALEATHER	SRE-20676BRL
BURGUNDY ULTRALEATHER	SRE-20676BL
DARK BROWN ULTRALEATHER	SRE-20676DBL
GREY ULTRALEATHER	SRE-20676GRL


32

**SRE-2010 No Power**  
**SHEET 2 OF 5**  
**REV. 32 (6395)(9/11/17)(RJH)**

# 'ELITE' POWER SWIVEL & POWER FOOTREST

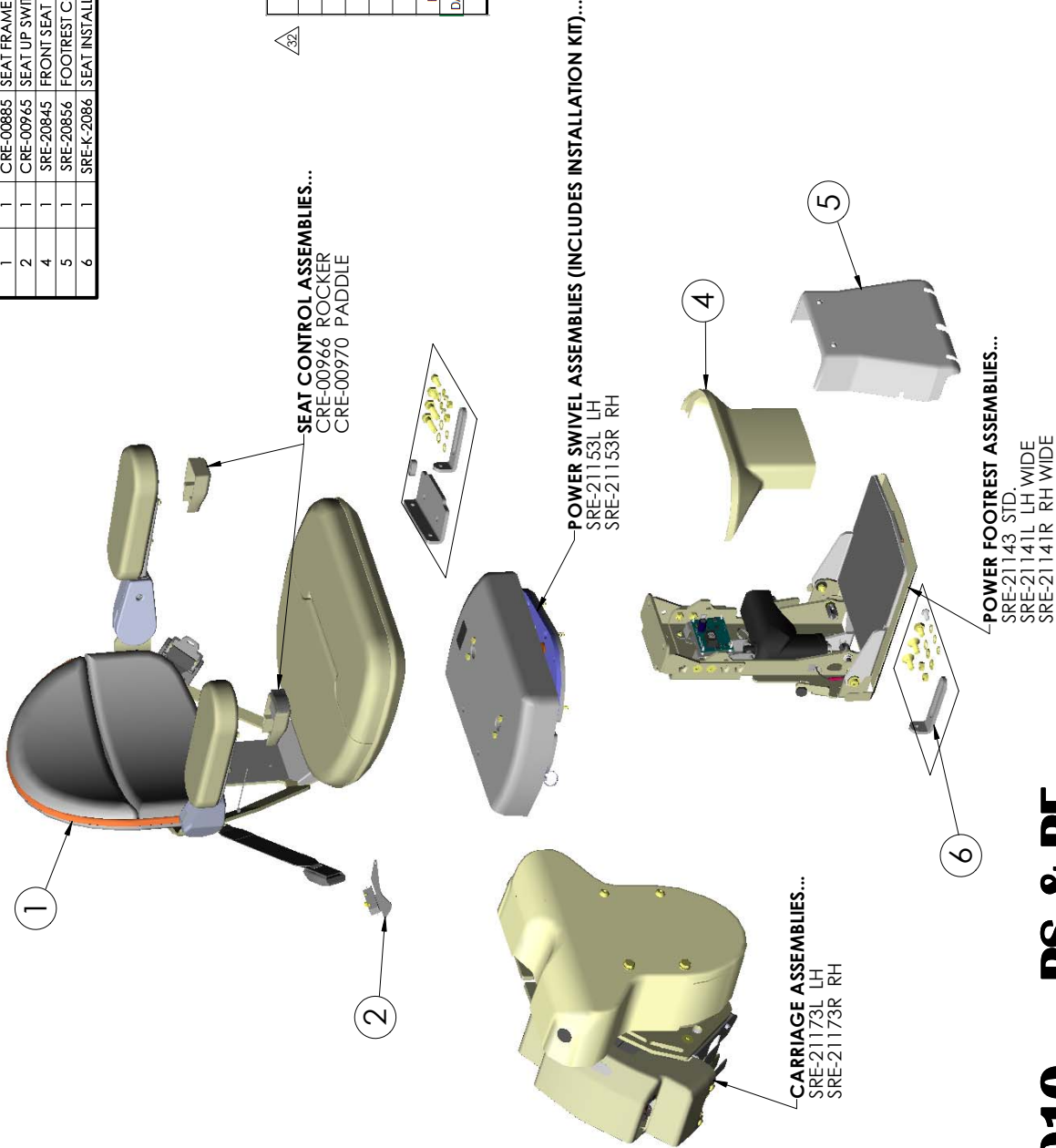ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	CRE-00885	SEAT FRAME ASSEMBLY w/TAN VINYL UPH - PWR SWIVL
2	1	CRE-00965	SEAT UP SWITCH ASSEMBLY-POWER SWIVEL & FOOTREST
4	1	SRE-20845	FRONT SEAT POST COVER (POWER FOOTREST)
5	1	SRE-20856	FOOTREST COVER ASSY (POWER FOOTREST)
6	1	SRE-K-2086	SEAT INSTALLATION KIT (POWER FOOTREST)

ADDITIONAL SEAT COLORS/MATL	UPHOLSTERY SET STD. SEAT
LIGHT BROWN VINYL	SRE-20676BV
CHARCOAL VINYL	SRE-20676CV
MAROON VINYL	SRE-20676MV
OLIVE VINYL	SRE-20676OV
BROWN ULTRALEATHER	SRE-20676BRL
BURGUNDY ULTRALEATHER	SRE-20676BL
DARK BROWN ULTRALEATHER	SRE-20676DBL
GREY ULTRALEATHER	SRE-20676GRL


**SRE-2010 PS & PF**  
**SHEET 3 OF 5**  
**REV. 32 (6395)(9/11/17)(RJH)**

# 'ELITE' POWER SWIVEL



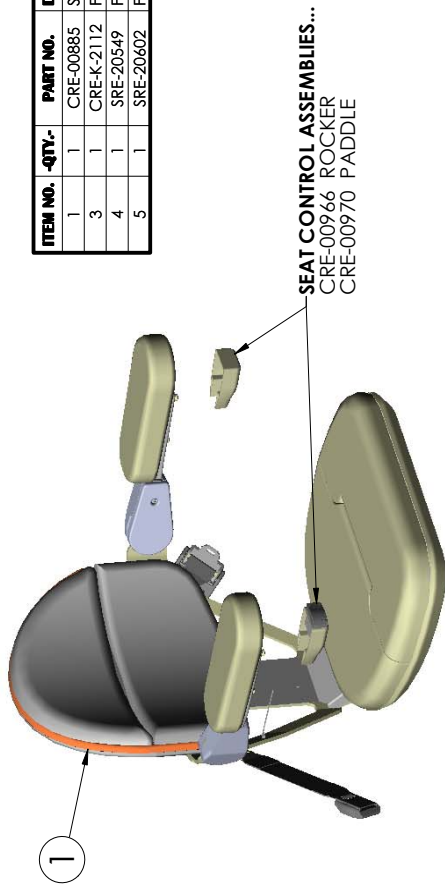
ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	CRE-00885	SEAT FRAME ASSEMBLY w/TAN VINYL UPH - PWR SWVL
3	1	CRE-K-2112	FOOTREST CYLINDER INSTALLATION KIT
4	1	SRE-20549	FRONT SEAT POST COVER
5	1	SRE-20602	FOOT REST COVER

ADDITIONAL SEAT COLORS/MATL	UPHOLSTERY SET STD. SEAT
LIGHT BROWN VINYL	SRE-20676BV
CHARCOAL VINYL	SRE-20676CV
MAROON VINYL	SRE-20676MV
OLIVE VINYL	SRE-20676OV
BROWN ULTRALEATHER	SRE-20676BRL
BURGUNDY ULTRALEATHER	SRE-20676BL
DARK BROWN ULTRALEATHER	SRE-20676DBL
GREY ULTRALEATHER	SRE-20676GRL

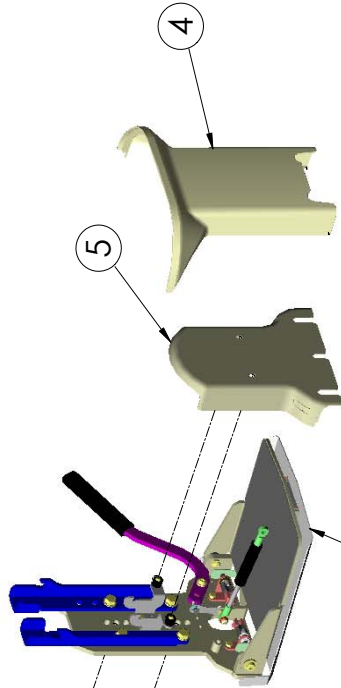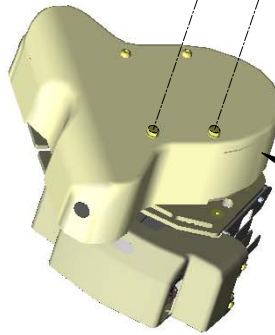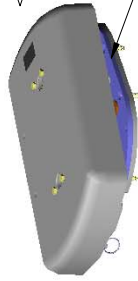
**POWER SWIVEL ASSEMBLIES (INCLUDES INSTALLATION KIT)...**  
 SRE-21153L LH  
 SRE-21153R RH

**SRE-2010 PS Only**  
**SHEET 4 OF 5**  
**REV. 32 (6395)(9/11/17)(RJH)**

# 'ELITE' POWER FOOTREST

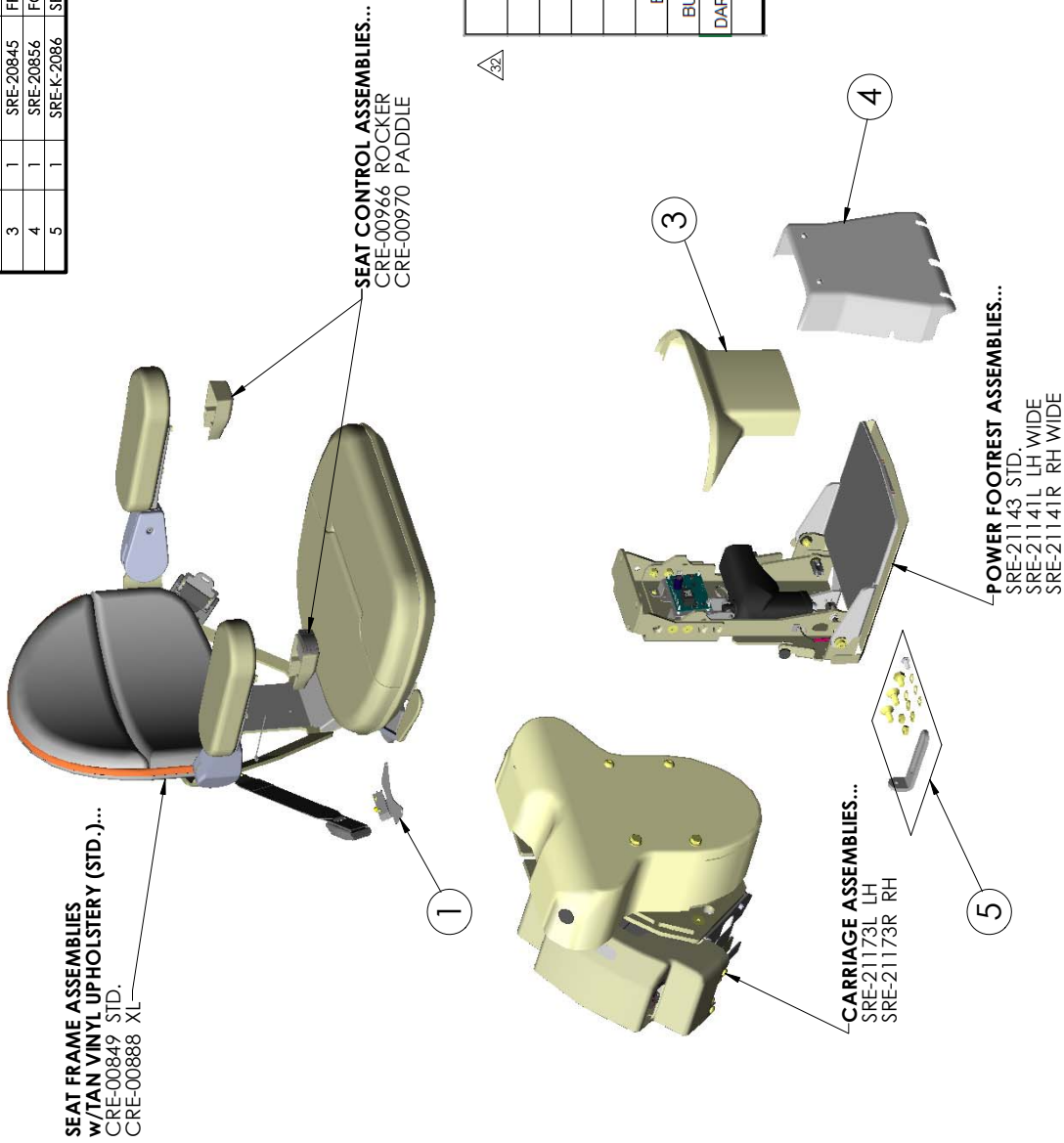ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	CRE-00965	SEAT UP SWITCH ASSEMBLY-POWER SWIVEL & FOOTREST
3	1	SRE-20845	FRONT SEAT POST COVER (POWER FOOTREST)
4	1	SRE-20856	FOOTREST COVER ASSY (POWER FOOTREST)
5	1	SRE-K-2086	SEAT INSTALLATION KIT (POWER FOOTREST)

ADDITIONAL SEAT COLORS/MATL	UPHOLSTERY SET STD. SEAT
LIGHT BROWN VINYL	SRE-20676BV
CHARCOAL VINYL	SRE-20676CV
MAROON VINYL	SRE-20676MV
OLIVE VINYL	SRE-20676OV
BROWN ULTRALEATHER	SRE-20676BRL
BURGUNDY ULTRALEATHER	SRE-20676BL
DARK BROWN ULTRALEATHER	SRE-20676DBL
GREY ULTRALEATHER	SRE-20676GRL


**SRE-2010 PF Only**  
**SHEET 5 OF 5**  
**REV. 32 (6395)(9/11/17)(RJH)**

**COMMERCIAL 'ELITE'**

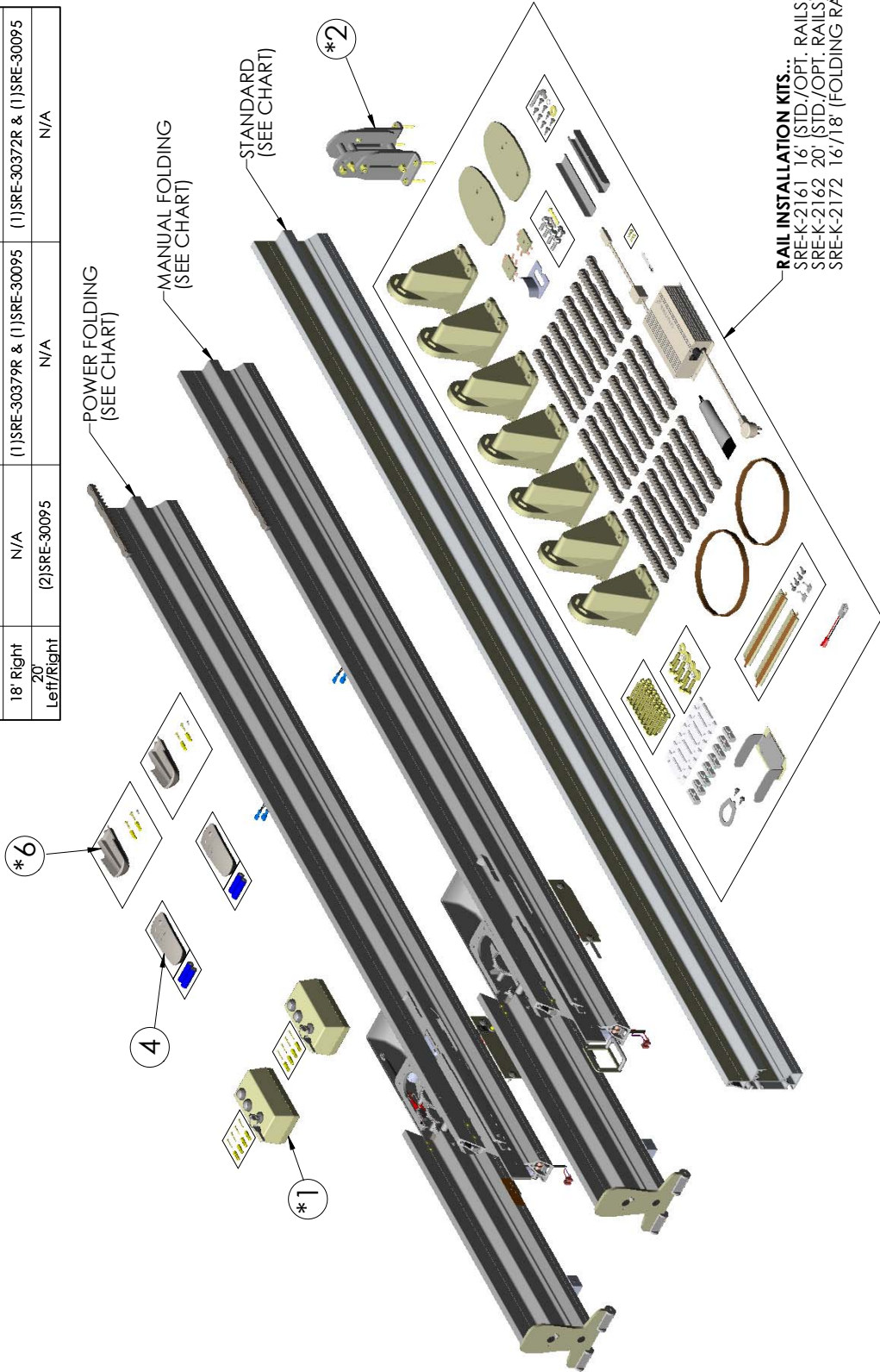ITEM NO.	QTY.	PART NO.	DESCRIPTION
*1	2	SRE-00963A	IR TRANSMITTER/BRACKET/HDWR (KEYED)(ROHS)
*2	1	SRE-30150	ADJUSTABLE STAND FOOT ASSEMBLY
4	2	TMR-00004	IR TRANSMITTER ASSEMBLY W/ 2 'AA' BATTERIES
*6	2	TMR-00005	IR TRANSMITTER WALL MOUNT BRACKET W/ HARDWARE

\*OPTIONAL

RAIL MATRIX	Standard(Anodized)	Folding(Anodized)	Power Folding(Anodized)
16' Left/Right	(2)SRE-30094	N/A	N/A
16' Left	N/A	(1)SRE-30379L & (1)SRE-30094	(1)SRE-30372L & (1)SRE-30094
16' Right	N/A	(1)SRE-30379R & (1)SRE-30094	(1)SRE-30372R & (1)SRE-30094
18' Left	N/A	(1)SRE-30379L & (1)SRE-30095	(1)SRE-30372L & (1)SRE-30095
18' Right	N/A	(1)SRE-30379R & (1)SRE-30095	(1)SRE-30372R & (1)SRE-30095
20' Left/Right	(2)SRE-30095	N/A	N/A


**SRE-2010C Rail, Charger, & Call-Sends**

**SHEET 1 OF 5**  
**REV. 30 (6395)(9/11/17)(RJH)**

# COMMERCIAL 'ELITE' STD. - NO POWER

ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	CRE-00895	SEAT MOUNTING BRACKET ASSEMBLY
2	1	CRE-K-2112	FOOIREST CYLINDER INSTALLATION KIT
3	1	SRE-20549	FRONT SEAT POST COVER
4	1	SRE-20602	FOOT REST COVER
5	1	SRE-K-2063	SEAT INSTALLATION KIT

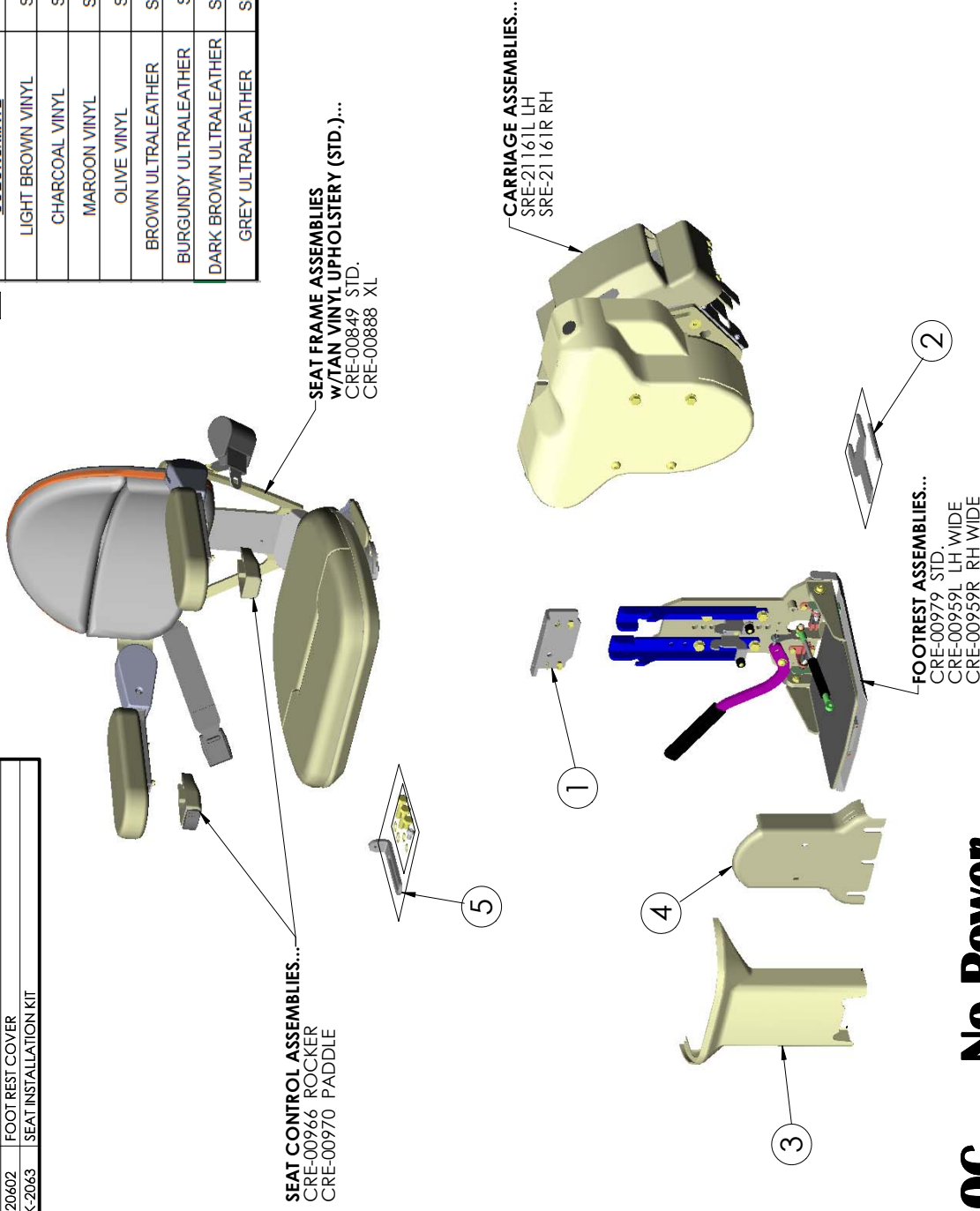ADDITIONAL SEAT COLORS/MATL	UPHOLSTERY SET STD. SEAT
LIGHT BROWN VINYL	SRE-20676BV
CHARCOAL VINYL	SRE-20676CV
MAROON VINYL	SRE-20676MV
OLIVE VINYL	SRE-20676OV
BROWN ULTRALEATHER	SRE-20676BRL
BURGUNDY ULTRALEATHER	SRE-20676BL
DARK BROWN ULTRALEATHER	SRE-20676DBL
GREY ULTRALEATHER	SRE-20676GRL


**SRE-2010C No Power**  
**SHEET 2 OF 5**  
**REV. 30 (6395)(9/11/17)(RJH)**

# COMMERCIAL 'ELITE' POWER SWIVEL & POWER FOOTREST

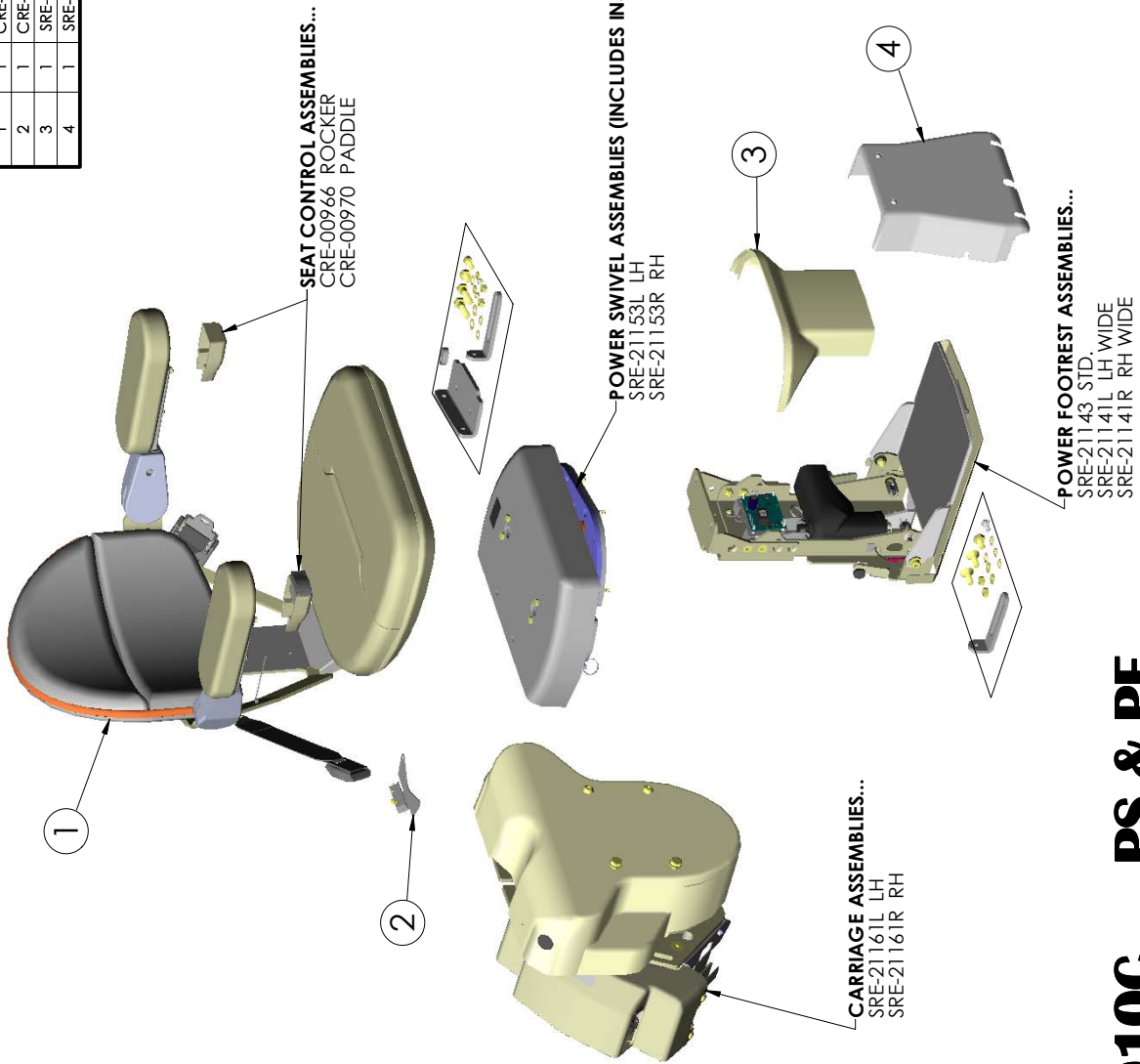ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	CRE-00885	SEAT FRAME ASSEMBLY w/TAN VINYL UPH - PWR SWVL
2	1	CRE-00965	SEAT UP SWITCH ASSEMBLY-POWER SWIVEL & FOOTREST
3	1	SRE-20845	FRONT SEAT POST COVER (POWER FOOTREST)
4	1	SRE-20856	FOOTREST COVER ASSY (POWER FOOTREST)


ADDITIONAL SEAT COLORS/MATL	UPHOLSTERY SET STD. SEAT
LIGHT BROWN VINYL	SRE-20676BV
CHARCOAL VINYL	SRE-20676CV
MAROON VINYL	SRE-20676MV
OLIVE VINYL	SRE-20676OV
BROWN ULTRALEATHER	SRE-20676BRL
BURGUNDY ULTRALEATHER	SRE-20676BL
DARK BROWN ULTRALEATHER	SRE-20676DBL
GREY ULTRALEATHER	SRE-20676GRL

**SRE-2010C PS & PF**  
**SHEET 3 OF 5**  
**REV. 30 (6395)(9/11/17)(RJH)**


# COMMERCIAL 'ELITE' POWER SWIVEL

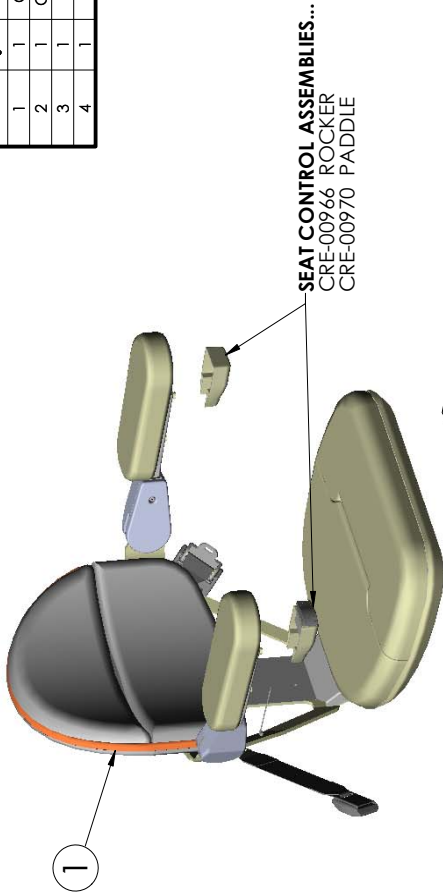ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	CRE-00885	SEAT FRAME ASSEMBLY w/TAN VINYL UPH. - PWR SWVL
2	1	CRE-K-2112	FOOTREST CYLINDER INSTALLATION KIT
3	1	SRE-20549	FRONT SEAT POST COVER
4	1	SRE-20602	FOOT REST COVER


SEAT CONTROL ASSEMBLIES...  
CRE-00966 ROCKER  
CRE-00970 PADDLE


ADDITIONAL SEAT COLORS/MATL	UPHOLSTERY SET STD. SEAT
LIGHT BROWN VINYL	SRE-20676BV
CHARCOAL VINYL	SRE-20676CV
MAROON VINYL	SRE-20676MV
OLIVE VINYL	SRE-20676OV
BROWN ULTRALEATHER	SRE-20676BRL
BURGUNDY ULTRALEATHER	SRE-20676BL
DARK BROWN ULTRALEATHER	SRE-20676DBL
GREY ULTRALEATHER	SRE-20676GRL

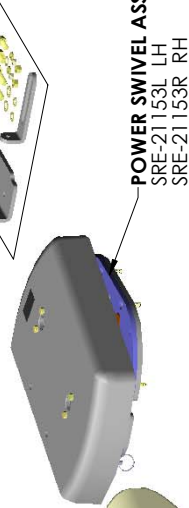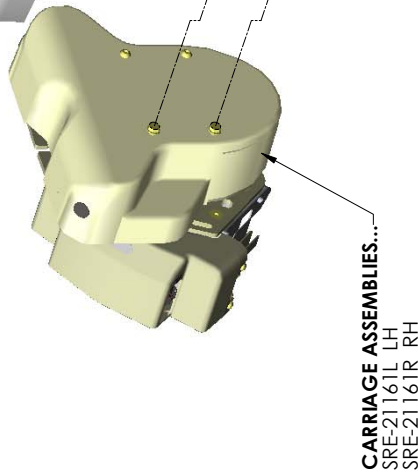POWER SWIVEL ASSEMBLIES (INCLUDES INSTALLATION KIT)...  
SRE-21153L LH  
SRE-21153R RH

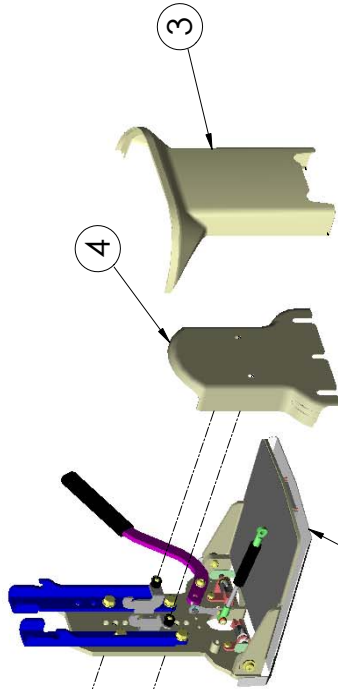CARRIAGE ASSEMBLIES...  
SRE-21161L LH  
SRE-21161R RH


FOOTREST ASSEMBLIES...  
CRE-00979 STD.  
CRE-00959L LH WIDE  
CRE-00959R RH WIDE

**SRE-2010C PS Only**  
**SHEET 4 OF 5**  
**REV. 30 (6395)(9/11/17)(RJH)**


# COMMERCIAL 'ELITE' POWER FOOTREST

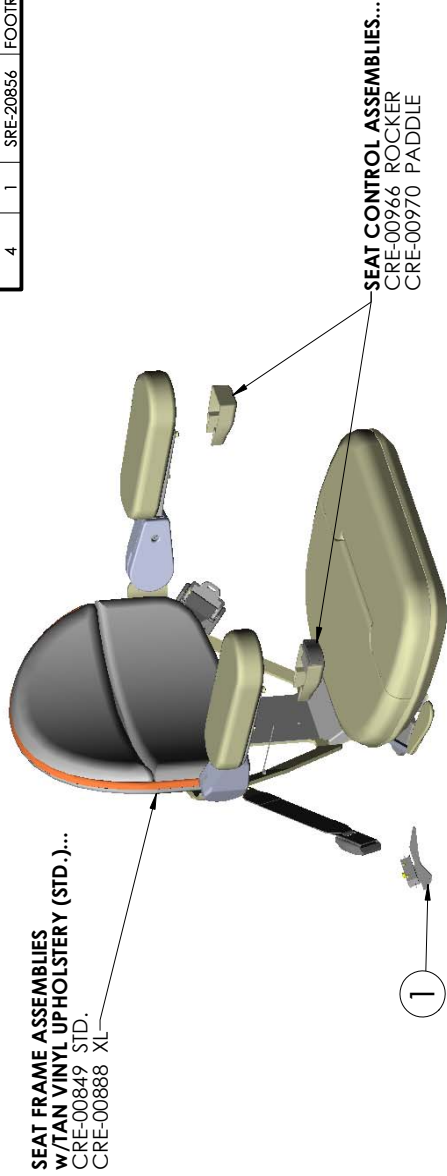ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	CRE-00945	SEAT UP SWITCH ASSEMBLY-POWER SWIVEL & FOOTREST
3	1	SRE-20845	FRONT SEAT POST COVER (POWER FOOTREST)
4	1	SRE-20856	FOOTREST COVER ASSY (POWER FOOTREST)

ADDITIONAL SEAT COLORS/OPTIONS	UPHOLSTERY SET STD. SEAT
LIGHT BROWN VINYL	SRE-20676BV
CHARCOAL VINYL	SRE-20676CV
MAROON VINYL	SRE-20676MV
OLIVE VINYL	SRE-20676OV
BROWN ULTRALEATHER	SRE-20676BRL
BURGUNDY ULTRALEATHER	SRE-20676BL
DARK BROWN ULTRALEATHER	SRE-20676DBL
GREY ULTRALEATHER	SRE-20676GRL

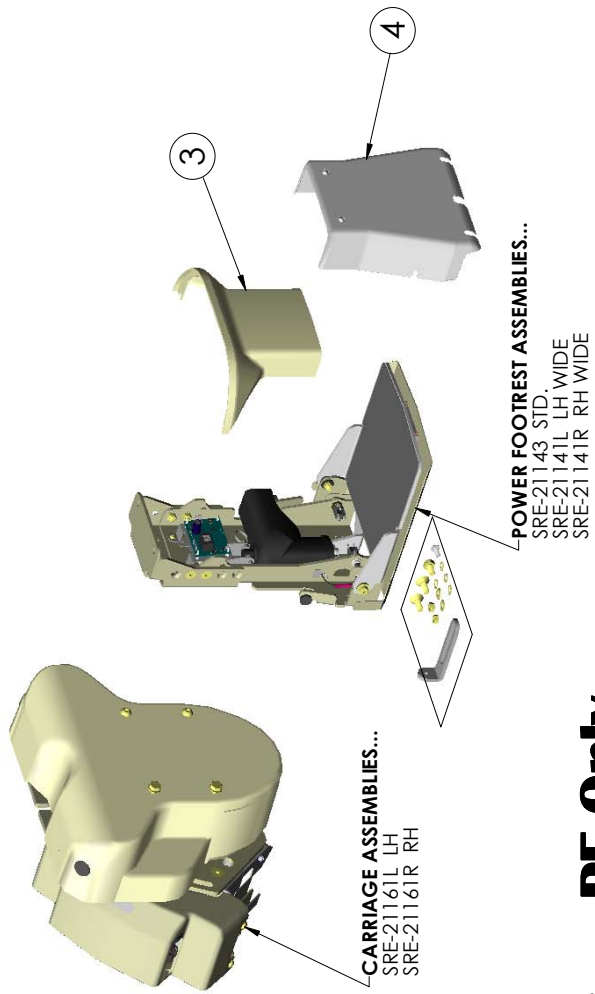
**SRE-2010C PF Only**  
**SHEET 5 OF 5**  
**REV. 30 (6395)(9/11/17)(RJH)**

# 'ELITE' STRAIGHT RAIL STAIRLIFT (OS)

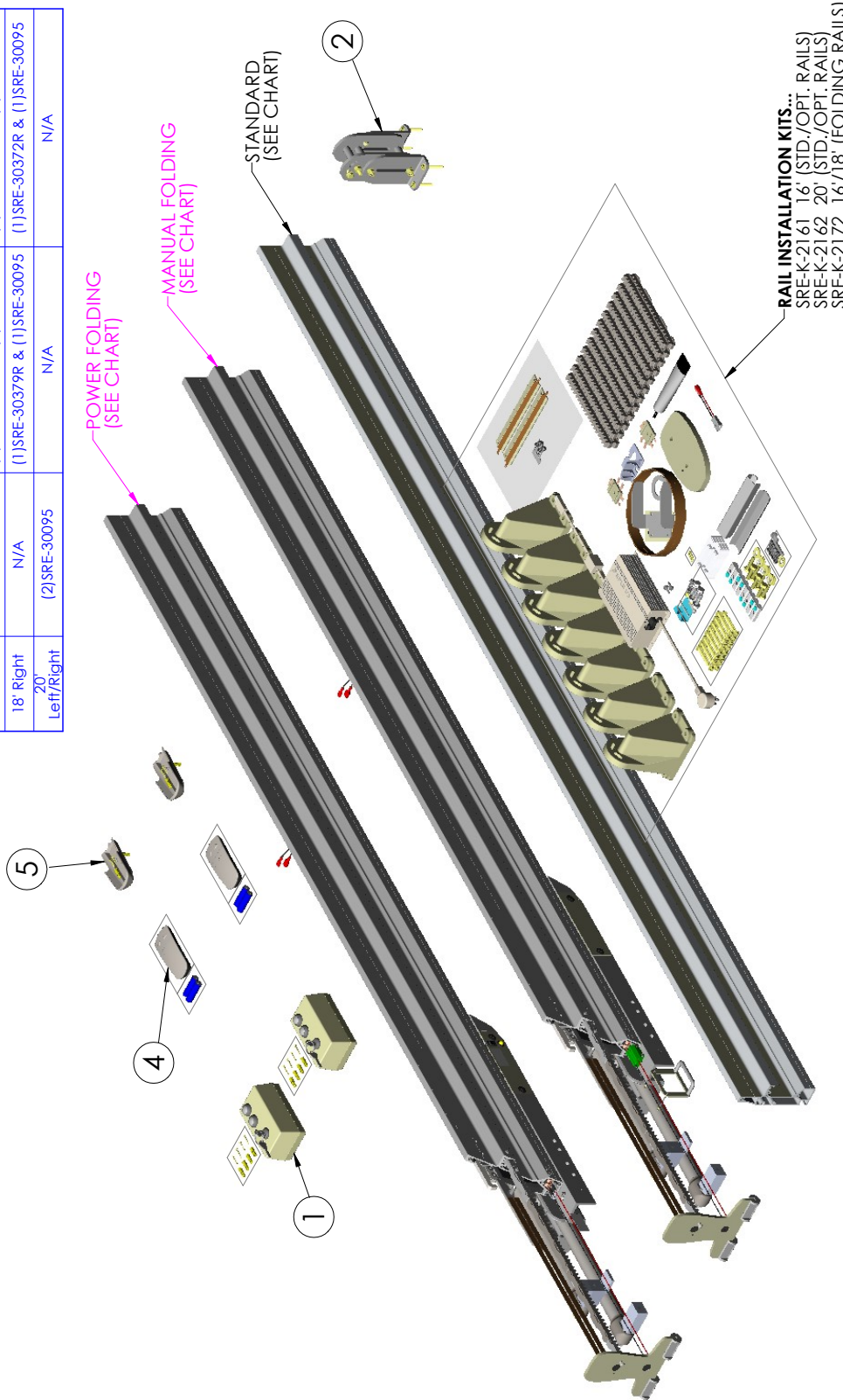ITEM NO.	QTY.	PART NO.	DESCRIPTION
*	1	SRE-00963A	IR TRANSMITTER/BACKET/HDWR (KEYED)(ROHS)
*	2	SRE-30150	ADJUSTABLE STAND FOOT ASSEMBLY
*	4	TMR-00004	IR TRANSMITTER ASSEMBLY W/ 2 'AA' BATTERIES
*	5	TMR-00005	IR TRANSMITTER WALL MOUNT BRACKET W/ HARDWARE

\*OPTIONAL

RAIL MATRIX	Standard(Anodized)	Folding(Anodized)	Power Folding(Anodized)
16' Left/Right	(2)SRE-30094	N/A	N/A
16' Left	N/A	(1)SRE-30379L & (1)SRE-30094	(1)SRE-30372L & (1)SRE-30094
16' Right	N/A	(1)SRE-30379R & (1)SRE-30094	(1)SRE-30372R & (1)SRE-30094
18' Left	N/A	(1)SRE-30379L & (1)SRE-30095	(1)SRE-30372L & (1)SRE-30095
18' Right	N/A	(1)SRE-30379R & (1)SRE-30095	(1)SRE-30372R & (1)SRE-30095
20' Left/Right	(2)SRE-30095	N/A	N/A


## SRE-2010 OS Rail, Charger, & Call-Sends

SHEET 1 OF 5  
REV. 0 (ISSUED)(1/11/19)(KAS)

# 'ELITE' STRAIGHT RAIL STAIRLIFT (OS) - NO POWER

ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	CRE-00895	SEAT MOUNTING BRACKET ASSEMBLY
2	1	CRE-K-2112	FOOTREST CYLINDER INSTALLATION KIT
3	1	SRE-20549	FRONT SEAT POST COVER
4	1	SRE-20602	FOOT REST COVER
5	1	SRE-K-2063	SEAT INSTALLATION KIT

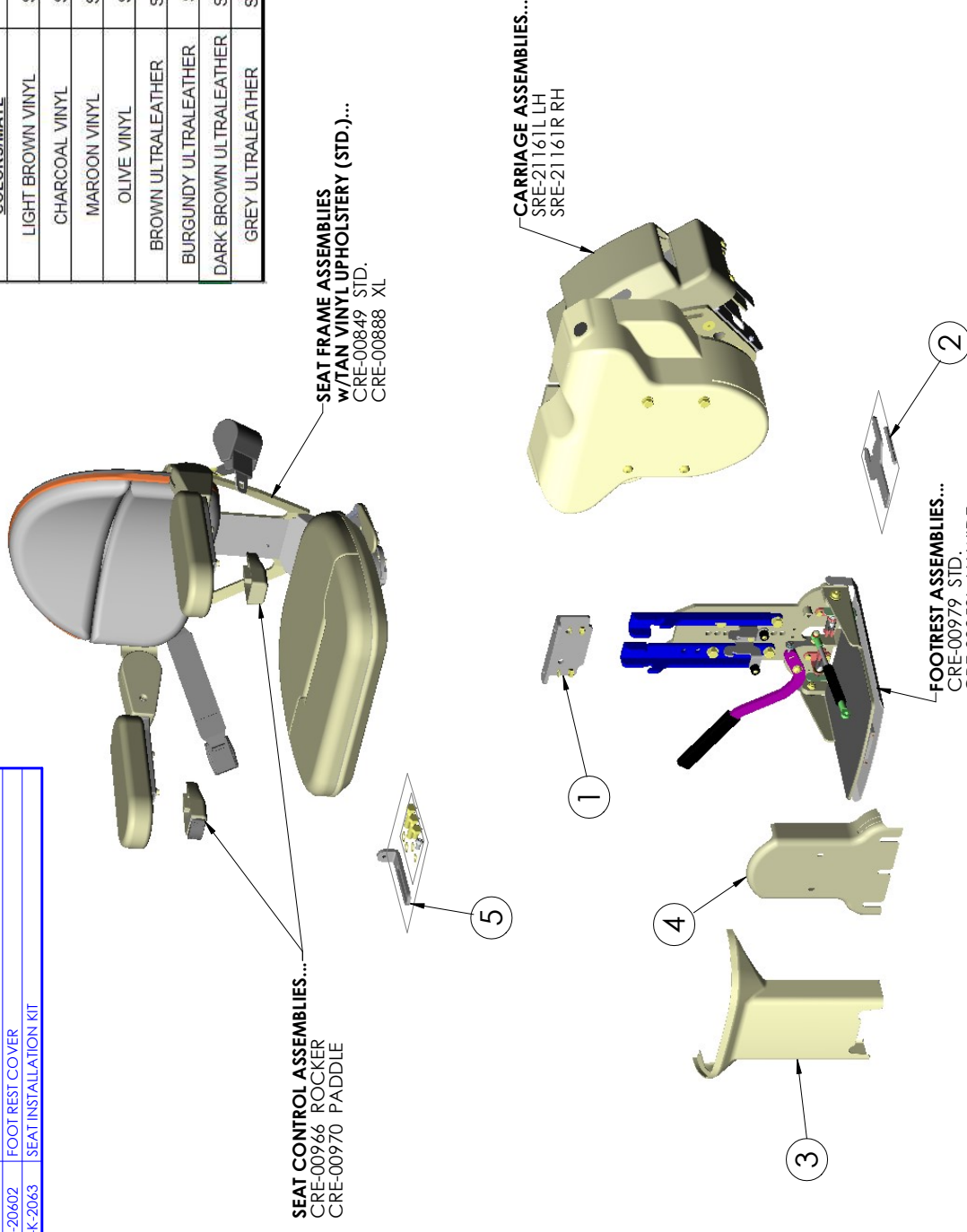ADDITIONAL SEAT COLORS/MATL	UPHOLSTERY SET STD. SEAT
LIGHT BROWN VINYL	SRE-20676BV
CHARCOAL VINYL	SRE-20676CV
MAROON VINYL	SRE-20676MV
OLIVE VINYL	SRE-20676OV
BROWN ULTRALEATHER	SRE-20676BRL
BURGUNDY ULTRALEATHER	SRE-20676BL
DARK BROWN ULTRALEATHER	SRE-20676DBL
GREY ULTRALEATHER	SRE-20676GRL

**SEAT CONTROL ASSEMBLIES...**  
 CRE-00966 ROCKER  
 CRE-00970 PADDLE

**SEAT FRAME ASSEMBLIES  
 W/TAN VINYL UPHOLSTERY (STD.)...**  
 CRE-00849 STD.  
 CRE-00888 XL


**CARRIAGE ASSEMBLIES...**  
 SRE-21161L LH  
 SRE-21161R RH

**FOOTREST ASSEMBLIES...**  
 CRE-00979 STD.  
 CRE-00959L LH WIDE  
 CRE-00959R RH WIDE

**SRE-2010 OS No Power**  
**SHEET 2 OF 5**  
**REV. 0 (ISSUED)(1/11/19)(KAS)**

'ELITE' STRAIGHT RAIL STAIRLIFT (OS) POWER SWIVEL & POWER FOOTREST

ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	CRE-00885	SEAT FRAME ASSEMBLY w/TAN VINYL UPH - PWR SWVL
2	1	CRE-00965	SEAT UP SWITCH ASSEMBLY-POWER SWIVEL & FOOTREST
3	1	SRE-20845	FRONT SEAT POST COVER (POWER FOOTREST)
4	1	SRE-20856	FOOTREST COVER ASSY (POWER FOOTREST)

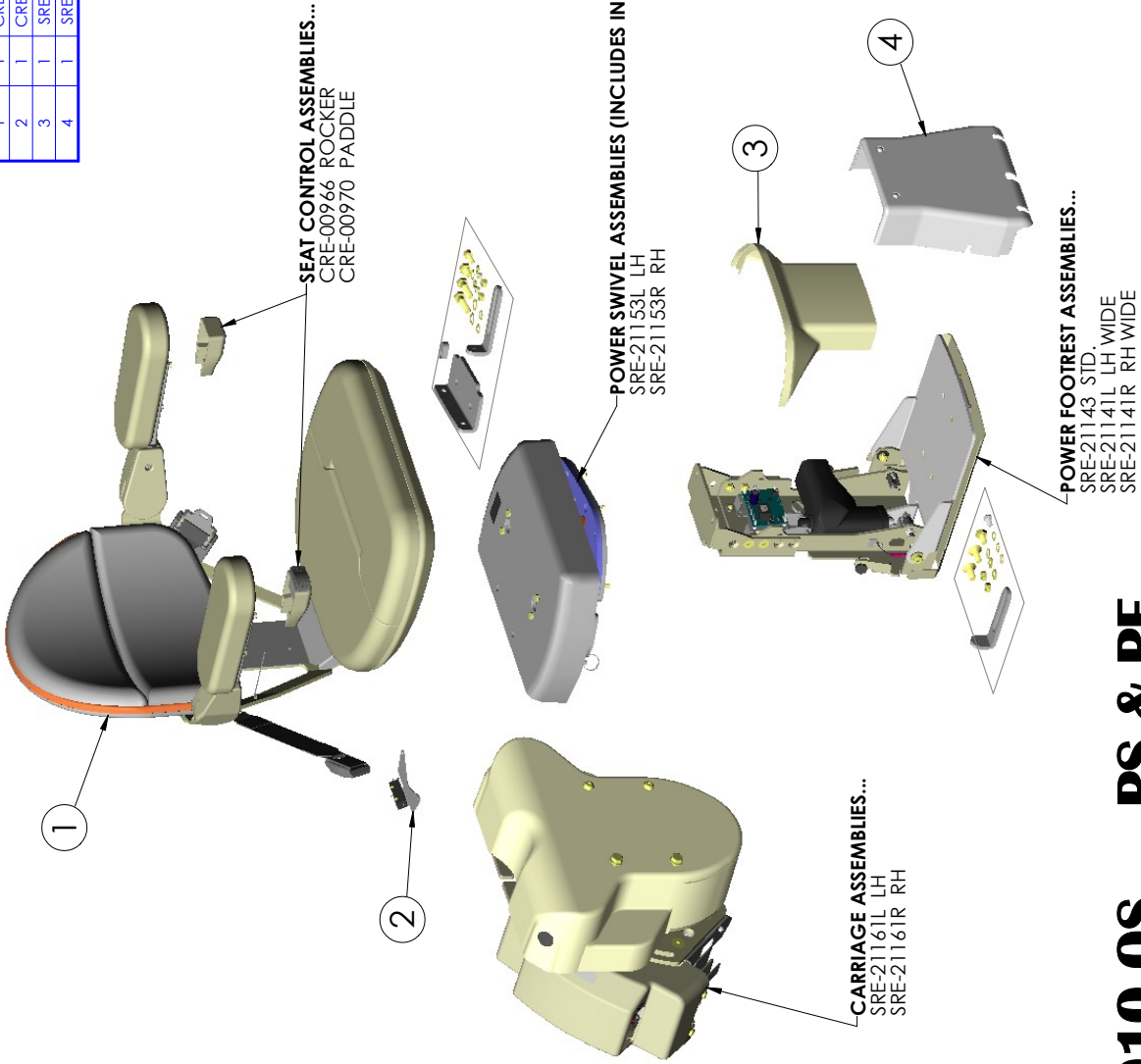
ADDITIONAL SEAT COLORS/MATL	UPHOLSTERY SET STD. SEAT
LIGHT BROWN VINYL	SRE-20676BV
CHARCOAL VINYL	SRE-20676CV
MAROON VINYL	SRE-20676MV
OLIVE VINYL	SRE-20676OV
BROWN ULTRALEATHER	SRE-20676BRL
BURGUNDY ULTRALEATHER	SRE-20676BL
DARK BROWN ULTRALEATHER	SRE-20676DBL
GREY ULTRALEATHER	SRE-20676GRL

**SRE-2010 OS PS & PF**  
**SHEET 3 OF 5**  
**REV. 0 (ISSUED)(1/11/19)(KAS)**

# 'ELITE' STRAIGHT RAIL STAIRLIFT (OS) POWER SWIVEL

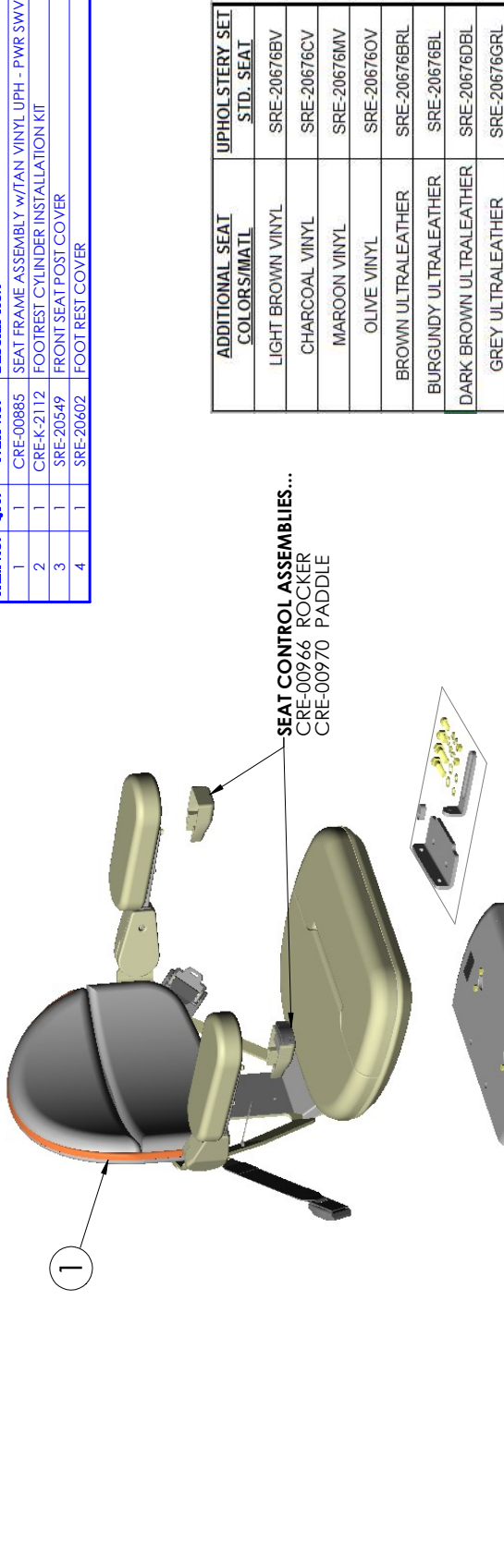
ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	CRE-00885	SEAT FRAME ASSEMBLY w/TAN VINYL UPH - PWR SWVL
2	1	CRE-K-2112	FOOTREST CYLINDER INSTALLATION KIT
3	1	SRE-20549	FRONT SEAT POST COVER
4	1	SRE-20602	FOOT REST COVER


**SRE-2010 OS PS Only**  
**SHEET 4 OF 5**  
**REV. 0 (ISSUED)(1/11/19)(KAS)**

# 'ELITE' STRAIGHT RAIL STAIRLIFT (OS) POWER FOOTREST

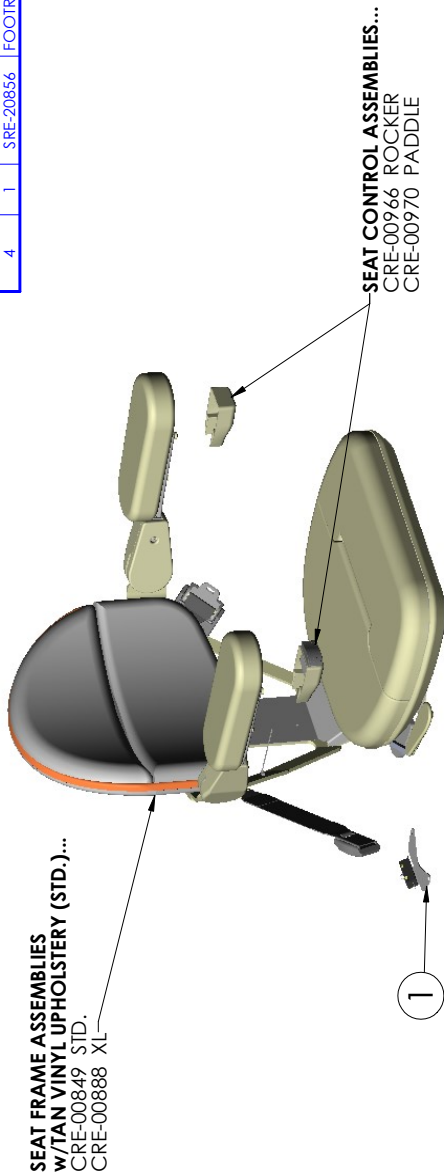ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	CRE-00965	SEAT UP SWITCH ASSEMBLY-POWER SWIVEL & FOOTREST
3	1	SRE-20845	FRONT SEAT POST COVER (POWER FOOTREST)
4	1	SRE-20856	FOOTREST COVER ASSY. (POWER FOOTREST)

ADDITIONAL SEAT COLORS/MATERIAL	UPHOLSTERY SET STD. SEAT
LIGHT BROWN VINYL	SRE-20676BV
CHARCOAL VINYL	SRE-20676CV
MAROON VINYL	SRE-20676MV
OLIVE VINYL	SRE-20676OV
BROWN ULTRALEATHER	SRE-20676BRL
BURGUNDY ULTRALEATHER	SRE-20676BL
DARK BROWN ULTRALEATHER	SRE-20676DBL
GREY ULTRALEATHER	SRE-20676GRL

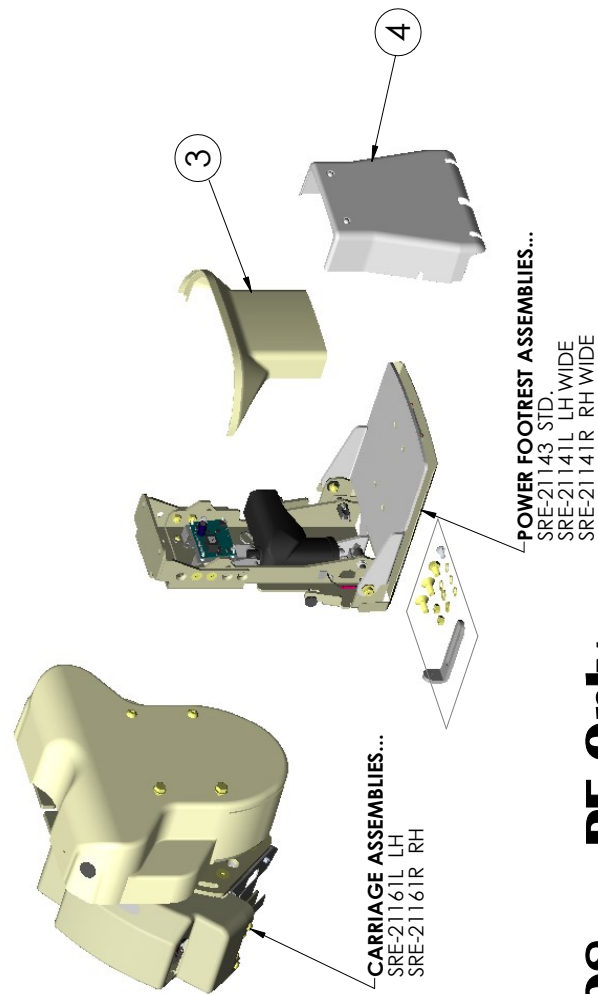**SRE-2010 OS PF Only**  
**SHEET 5 OF 5**  
**REV. 0 (ISSUED)(1/11/19)(KAS)**

---

# Innovative solutions

for your indoor and outdoor lifestyle

Indoor Stairlifts

Outdoor Stairlifts

Scooter/Powerchair Lifts

Vertical Platform Lifts

**Bruno Independent Living Aids, Inc.**  
1780 Executive Drive, Oconomowoc, WI 53066  
800-882-8183 • [www.bruno.com](http://www.bruno.com)

© Bruno Independent Living Aids, Inc. 2018

All illustrations and specifications in this brochure are based on the latest product information available at the time of publication.  
Bruno Independent Living Aids, Inc.® reserves the right to make changes at any time without notice. 2018-11-20-Manual-Back-Cover.indd