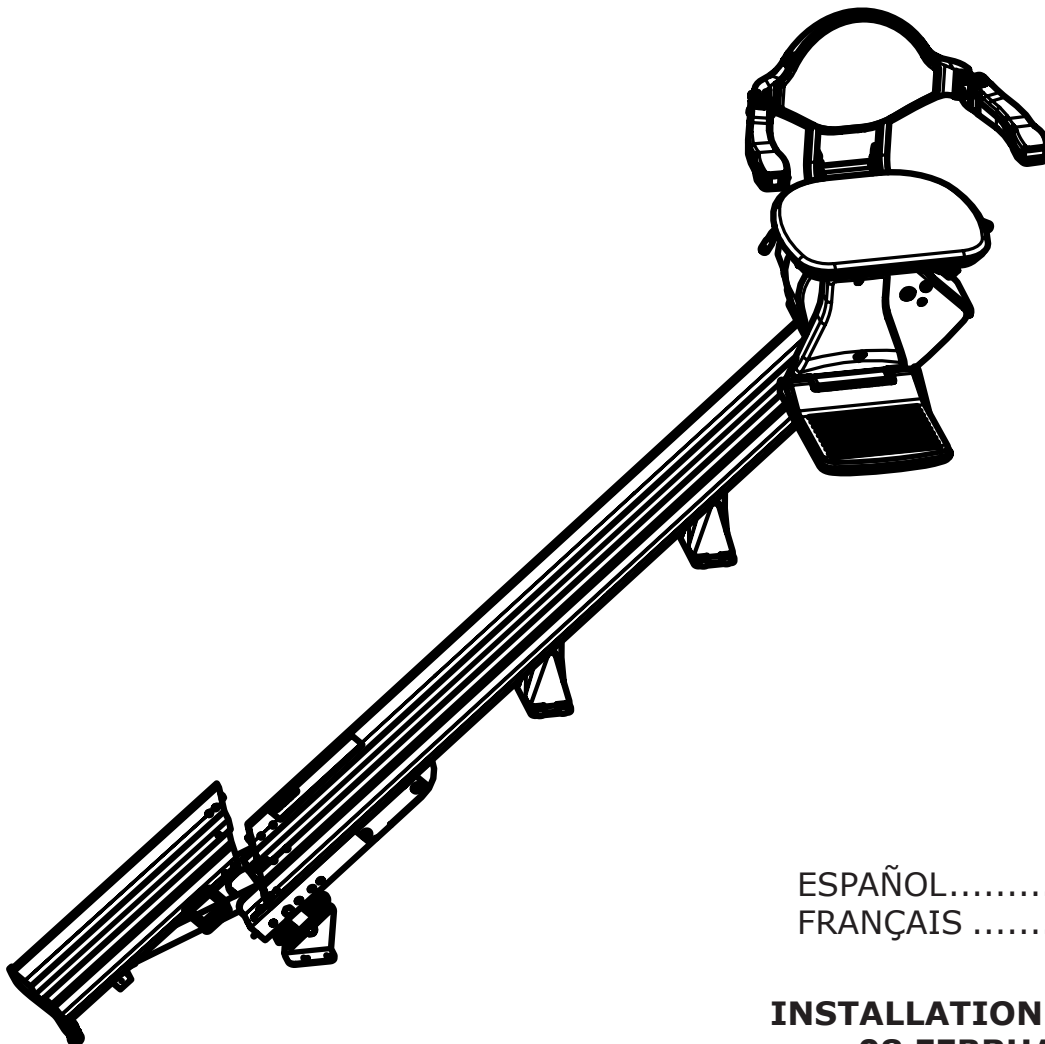
**SRE-3000**  
**ELAN**  
***Straight Rail Stairlift***

ESPAÑOL..... 46  
FRANÇAIS ..... 89

**INSTALLATION MANUAL**  
**08 FEBRUARY 2019**  
**P/N 3000-I**

# REGULATORY INFORMATION AND IMPORTANT NOTES

---

## REGULATORY LISTINGS

Intertek  
4004689

Intertek  
4004689

This lift is intended to be installed in accordance with the current edition of:

- ASME A18.1:2017 Safety Standard for Platform Lifts and Stairway Chairlifts
- CAN/CSA-B613-00 (R2012) - Private Residence Lifts for Persons with Physical Disabilities
- ASME A17.5/CSA B44.1 - 2014 Elevator and Escalator Electrical Equipment
- NFPA 70: National Electrical Code®
- CSA C22.1 Canadian Electrical Code

Verify applicable local codes with the appropriate regulatory authorities.

## MATERIAL SAFETY DATA SHEETS

Material safety data sheets (MSDS) on materials used on this unit may be requested through the Bruno Technical Service Department.

## INSTALLING DEALER RESPONSIBILITIES

The installing dealer has responsibility to report any malfunctions, safety concerns or events to the manufacturer in a timely manner. The installing dealer also has the responsibility to ensure that the users are properly trained in the use and operation of the unit.

Bruno stairway elevators shall be put into service and regularly serviced by a Bruno authorized dealer in accordance with the information provided in installation documents provided with the device.

## FCC REGULATIONS

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 Subpart B of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the of the following measures:

- Reorient or relocate receiving antenna.
- Increase separation between equipment and receiver.
- Consult your dealer or an experienced radio/TV technician.

This stairlift is **FOR INDOOR USE ONLY**  
in enclosed heated locations **above 35°F (2°C)**.

The warranty for the Elan Straight Rail Stairlift is *rendered null and void* if unit is installed or maintained by anyone other than an authorized Bruno dealer.

409® is a registered trademark of the Clorox Company  
Fantastik® is a registered trademark of S.C. Johnson & Son, Inc.

## Introductory Pages

Regulatory Information and Important Notes . . . . .	2
Table of Contents . . . . .	3
Product Overview and Technical Specifications. . . . .	4
Tools, Torque Specifications, and Hardware Notes . . . . .	5

## Pre-Installation

Site Review and Preparation . . . . .	6
Rail Measurements . . . . .	6
Rail Cutting Guidelines . . . . .	7
Rail Support (Rail Bracket) Requirements . . . . .	7
Lower Rail Preparation . . . . .	8-9
Joint Prep . . . . .	9
Upper Rail Preparation . . . . .	10
Finish Preparation . . . . .	11-13

## Installation

Rail Installation . . . . .	14-17
Non-Folding Rail Installations . . . . .	14-15
Folding Rail Installations . . . . .	16-17
Carriage Installation . . . . .	18-20
Footrest Installation . . . . .	20-21
Completing Rail Installation . . . . .	22-24
Rail Cross Pin Installation . . . . .	25-26
Make Charger Connections . . . . .	27-28
Non-Folding Rail Installations . . . . .	27
Folding Rail Installations . . . . .	28
Seat Installation. . . . .	29-31
Lubricate Gear Rack . . . . .	32
Battery Charger Installation . . . . .	32
Fit & Finish . . . . .	33
Customer Education . . . . .	34-35

## Electrical

Reprogramming Remote Transmitters. . . . .	36-37
Single-User vs. Multi-User Diagnostic Mode. . . . .	38
Battery Charger LED Information. . . . .	39

## Post-Installation Adjustments

Drive Adjustment. . . . .	40
---------------------------	----

## Troubleshooting

Reset Folding Rail Joint . . . . .	42
Overspeed . . . . .	43

## Limited Warranty

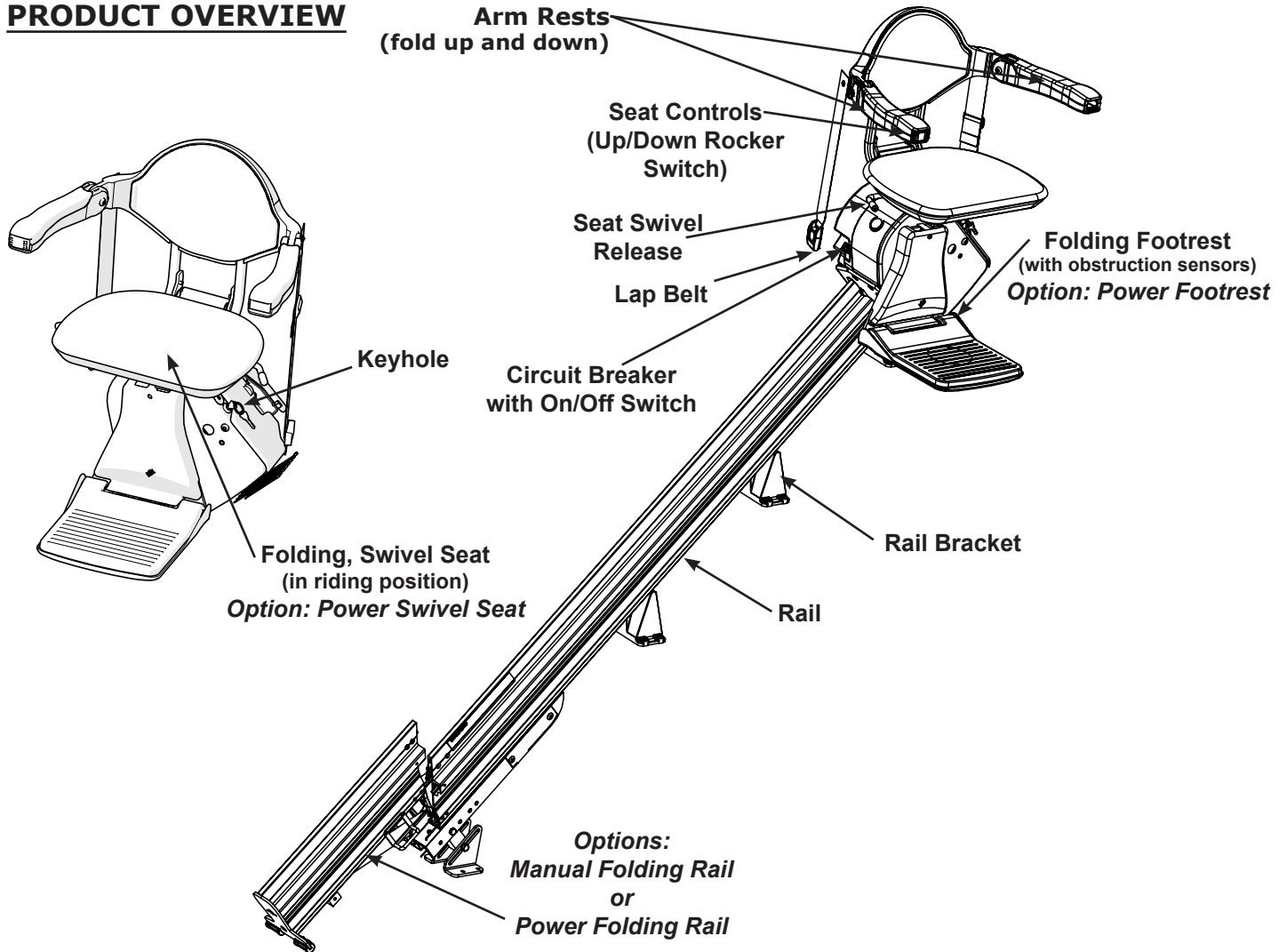
## Declaration of Conformity

## Wiring Schematics

## Exploded View

# TECHNICAL SPECIFICATIONS

## PRODUCT OVERVIEW

## TECHNICAL SPECIFICATIONS

**Weight Capacity:** . . . . . 300 lbs. (136 kg)

**Speed:** . . . . . maximum 18 ft./min. (0 to 5.5 m/min)

**Power Source:** . . . . . two (2) 12-volt sealed, maintenance-free batteries with 24-volt continuous-duty charger

**Motor:** . . . . . 24 VDC, 2-pole, 50 Nm torque and 16A minimum

**Drive:** . . . . . rack-and-pinion drive

**Control:** . . . . . constant pressure (armrest and 2 remote controls)

**Brake:** . . . . . 1 Nm electro-mechanical brake

**Incline:** . . . . . maximum 45°, minimum 22°, specials to 52°

**Rail:** . . . . . vertical aluminum extrusion with integral drive gear rack

**Seat Swivel:** . . . . . 0°, 67° and 90° at the top

**Power Supply:** . . . . . 24 VDC battery charger powered by 120-volt wall outlet

# TOOLS, TORQUE SPECIFICATIONS, AND HARDWARE NOTES

## **Tools**

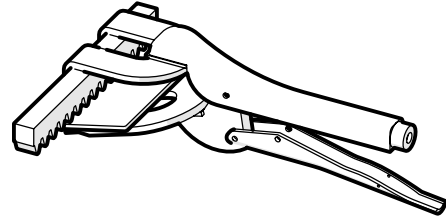
- Drill driver with *MINIMUM* 18" extension with 10mm magnetic socket
- Drill bits: appropriate for floor (eg. concrete, hardwood); 3/8", min. 6" long
- T-25 Torx® drive bit suitable for a power drill
- Cut-off saw (or other power saw with metal-cutting blade suitable for aluminum and steel)
- Metric open-end box wrench set
- Torque wrench (open end, and one that handles 15-80 ft.lb./20-108 Nm)
- Ratchet, 3/8" with 6" extension
- Metric sockets (13 mm and 17 mm) (no 10 mm?)
- 10mm magnetic socket, 3/8" drive (Bruno recommends use with an 18" extension)
- 5/16" and 3/8" SAE sockets
- Snap ring tool for heavy-duty snap ring
- Vise grips and C-clamp vise grips
- Needle-nose pliers
- Metric hex key set
- Magnetic Phillips screwdrivers
- Soft-faced or dead-blow hammer, or hard rubber or rawhide mallet
- Multimeter
- Extension cords
- Saw horses
- Safety goggles/glasses
- Brass hammer
- Tin snips
- Torpedo level
- Utility knife
- Edge utility cutter
- Wire stripper
- Wire crimper
- Metal file
- Scratch awl
- 20' tape measure
- Flashlight
- Threadlocker (i.e., Loctite®)

### **Next:**

- **Determine if application requires installation of a manual or power folding rail.**

**Use FIX-00525 rail clamp tool (available from Bruno)**

OR Bar/Spring clamp with piece of gear rack

## **Torque Specs**

- Follow OEM torque recommendations for seat mounting hardware. If not available, refer to table on right indicating ISO and SAE recommended assembly torque specifications.

### **Torque Specifications GRADE 10.9 (Metric) (Rough equivalent to US Grade 8)**

M 8	.....	25 lb.-ft.....	33.8 Nm
M 10	.....	47 lb.-ft.....	63.7 Nm
M 12	.....	83 lb.-ft...	112.5 Nm
M 14	.....	133 lb.-ft...	180.3 Nm

**If it becomes necessary to replace any bolts or fasteners, use only metric GRADE 10.9 (hardened steel) hardware. Metric grade 10.9 bolts are identified by the number 10.9 on the head.**

**METRIC 10.9  
BOLT HEAD**

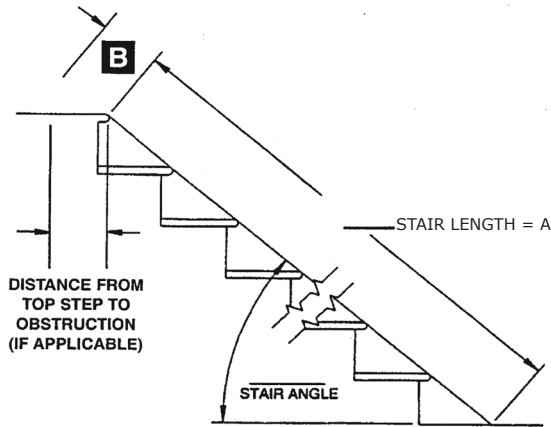
**Bolts without grade markings on the head are not strong enough to be safely used in this application.**

# PRE-INSTALLATION

## SITE REVIEW AND PREPARATION

1. Visually check out site, making sure all obstructions (i.e. windows, pipes, bulkheads, newel posts, etc.) have been taken into consideration.
2. If stairway is an "open" plan or stairs do not adjoin a solid wall, access to the stairs must be paneled off before installation may occur.
3. Determine if stairlift will be installed on the right or left side of stairs, **as viewed from bottom of the stairs.**

## RAIL MEASUREMENTS

1. Determine exact rail length.

**NOTE: The rail length will determine the seat height at the top of the stairs.**

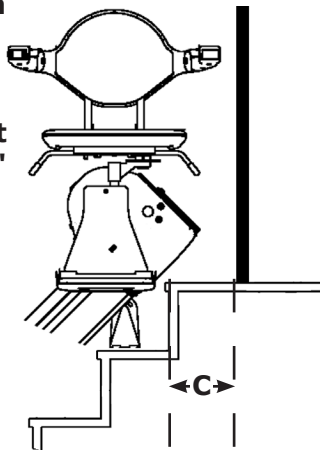
Measure length of stairway by placing a tape measure in a straight line on the stairs from the top step nose to the floor.

(A) Length of Stairway: \_\_\_\_\_

(B) Add 12" (305mm) for a 20" (508 mm) seat height: + \_\_\_\_\_  
 [Add 1.5" (38 mm) for each 1" (25 mm) seat height is increased.]

**NOTE: Chair armrest extends approximately 5 to 8" (127 to 203 mm) beyond rail at upper landing, depending on arm width setting.**

**NOTE: Seat height is adjustable in 1" increments up to 3" (76 mm). See page 27 for details.**

**! For obstacles at top of stairs:**  
**See Bruno's Brainshark "Ramping the Rail",**  
**and remember:**

**Nose-to-floor + nose-to-door + 3" (76mm) = Overall Rail Length Required**

Add Stair Length (A) to Measurement (C) (i.e., door) plus 3" (76mm): + \_\_\_\_\_

**Overall Rail Length Required = \_\_\_\_\_**

2. Mark *Overall Rail Length Required* on rail, measuring from top of rail downwards.

## RAIL CUTTING GUIDELINES

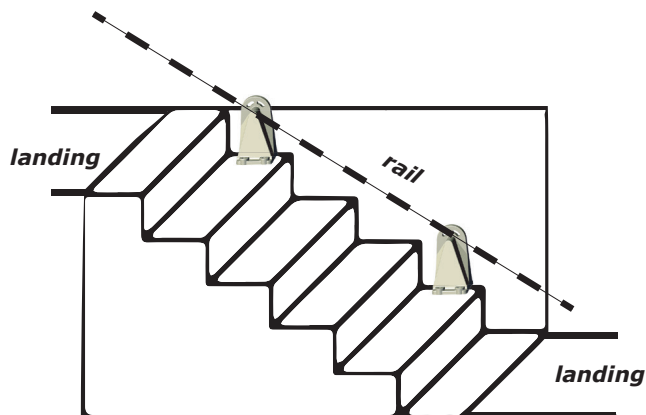
1. NEVER cut a rail shorter than 24" (610 mm). Rails shorter than 24" (610 mm) do not have room for the minimum two (2) rail brackets.

**NOTE: Folding rail assembly - NEVER cut a folding rail assembly shorter than 75" (1905mm).**

2. In two rail installations, if cutting only one rail would result in a length less than 24" (610 mm), cut from both rails as follows:
  - a. Cut at least 12" (305 mm) off one rail.
  - b. Then, calculate the second rail's length by subtracting the length of the newly cut rail from the required overall two-rail length (should be greater than 24"/610 mm).
3. Do not cut rail ends you plan to use for joining two sections together in order to preserve the factory fit at the joint.
4. For a precise and efficient cut, Bruno recommends the use of a cut-off saw.

Equip cut-off saw (or other power saw) with a metal-cutting blade **suitable for aluminum and steel**.

## RAIL SUPPORT (Rail Bracket) REQUIREMENTS

1. In multiple-rail installations, use a minimum of two (2) rail brackets to support each rail section.
2. In single-rail installations, use a minimum of three (3) rail brackets to support rail. [This is typically rail-run lengths of ten feet (3 m) or less].
3. In folding rail installations, use a minimum of two (2) rail brackets to support any mid-rail section.

Do not include in this count, the fixed rail bracket (first tread) at the folding joint.

4. No more than four (4) steps in a row without a rail bracket.
5. If rail extends horizontally from stop stepnose < 3" (76 mm), or diagonally extends past top stepnose > 12" (305 mm), one or more adjustable rail bracket sets (SRE-30150) are required.

# PRE-INSTALLATION

## LOWER RAIL PREPARATION

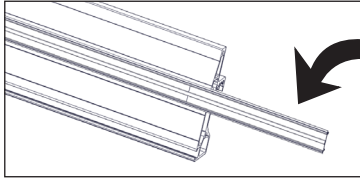
Fig. 1

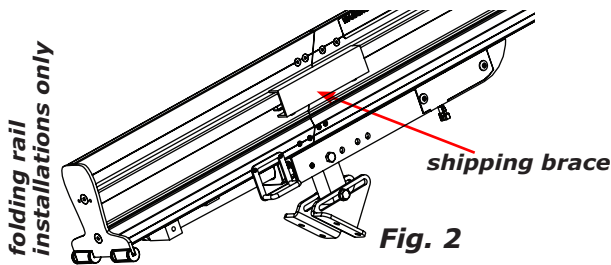
Fig. 2

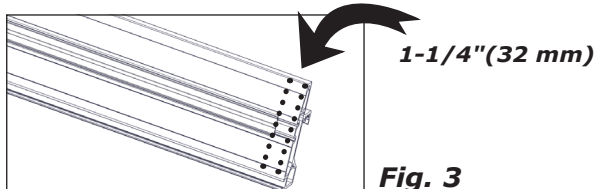
Fig. 3

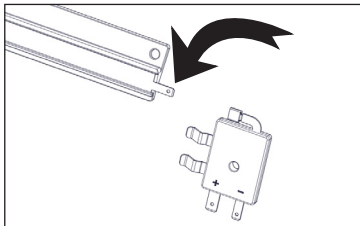
Fig. 4

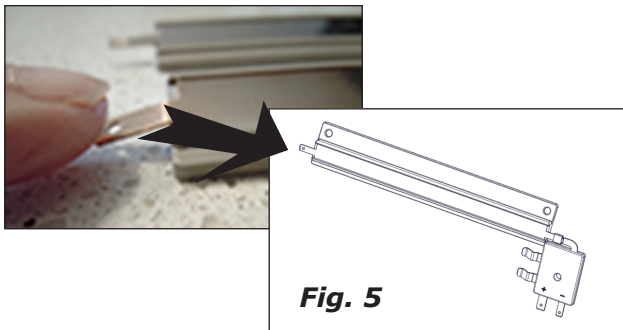
Fig. 5

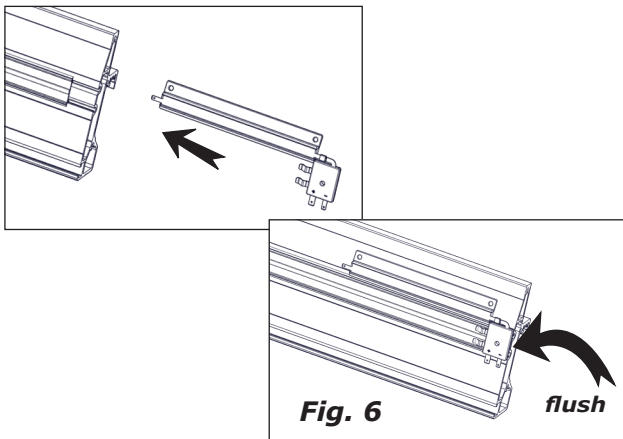
Fig. 6

1. Before cutting rail to length, remove charge strip carrier (Fig. 1). Set aside. Then, cut rail to length required for installation. (See page 6, **Rail Measurements** to calculate.)

**NOTE: Actual rail length for a 16' two-piece rail is 15'-9\"**

2. Place flat side of rail on saw horses. Draw a line at desired finished rail length. Carefully align saw with marked cutting line.
3. Cut rail to length.
4. Once rail is cut, deburr cut end using a metal file or other appropriate tool.

**NOTE: Be sure to round edges that might abrade wiring at rail end routed to bumper.**

**For folding rail installations: Keep shipping brace on folding rail, maintaining a closed position (Fig. 2). Next, insert slow down strip into charge contact and fasten with two (2) self-drilling hex head screws. Then, feed charge lead through the hole in the rail out the cut end of the rail.**

**Now, skip to Joint Preparation.**

5. With rail still on saw horses, slide charge strip carrier into back side of rail (Fig. 3). Inset carrier as noted in Fig. 3.
6. Attach charge contact to slow down (communications) strip (Fig. 4).
7. Bend slow down strip tab (at opposite end from charge contact) down slightly (Fig. 5). Do not let tab touch rail!
8. At bottom end of rail, slide charge contact assembly from Fig. 4 into back side of rail (Fig. 6). Assembly will fit snugly against charge strip carrier.

**NOTE: Carrier may be longer than rail at this point.**

## LOWER RAIL PREPARATION (continued)

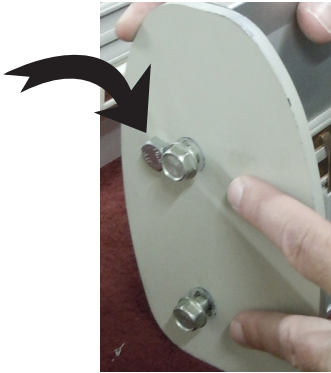
Fig. 6

- Use 1/2" socket to install two (2) 5/16" x 18 x 1" self-threading screws to mount the lower rail endplate (Fig. 6). A dab of grease will make installation easier.

**Torque to 22 ft-lb (30 Nm).**

**NOTE: Threaded hole in endplate goes toward front side of rail.**

**NOTE: DO NOT use an impact drill to install these screws.**


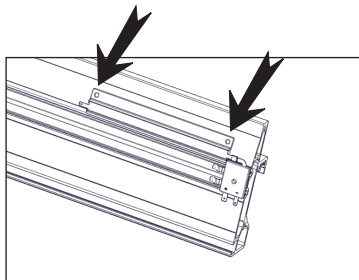
Fig. 7

- Insert slow down strip into lower charge contact, keeping it parallel to and touching top edge of charge strip carrier.

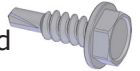Using two (2) self-drilling hex head screws, secure slow down strip to rail (Fig. 7). Do NOT overtighten!

**NOTE: Make sure slow down strip is parallel to charge strip carrier, and touching top edge of carrier.**

**FOR 1-PIECE RAIL INSTALLATIONS, SKIP TO UPPER RAIL PREPARATION.**

## JOINT PREPARATION

**NOTE: Lower charge contact is assembled at factory for folding rails.**


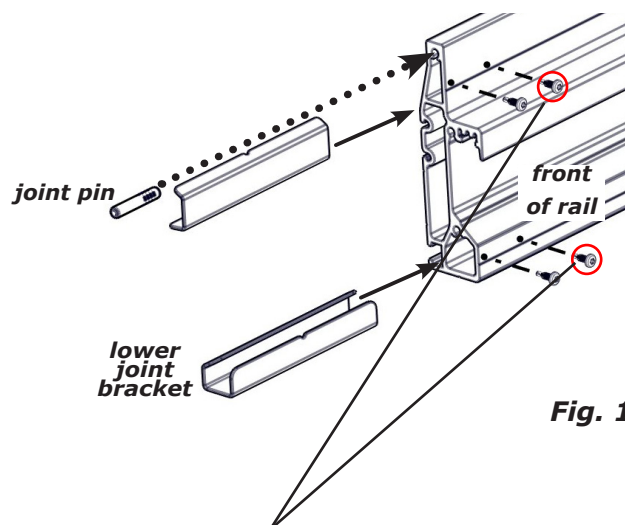
Fig. 1

- Pound barbed end of rail joint pin **halfway** into lower rail (Fig.1).
- Into upper part of lower rail section, insert an upper joint bracket halfway\* in.

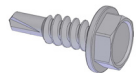
Secure one (1) of two (2) self-drilling hex head screws (Fig. 1).

\* **Both upper and lower joint brackets are notched to indicate halfway point.**

- Into lower part of lower rail section, insert a lower joint bracket halfway\* in.

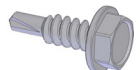Secure one (1) of (2) self-drilling hex head screws (Fig. 1).

**Wait to install second screw until after upper rail is installed.**

# PRE-INSTALLATION

## UPPER RAIL PREPARATION

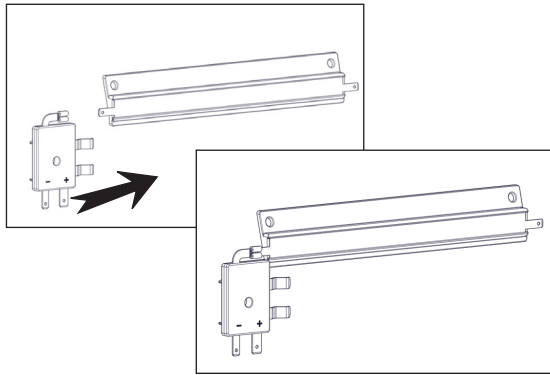
Fig. 1

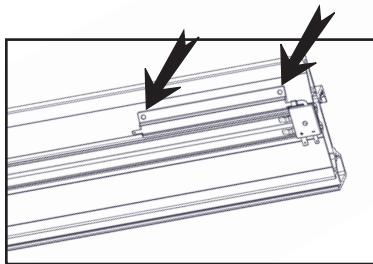
Fig. 2

### right-hand installation


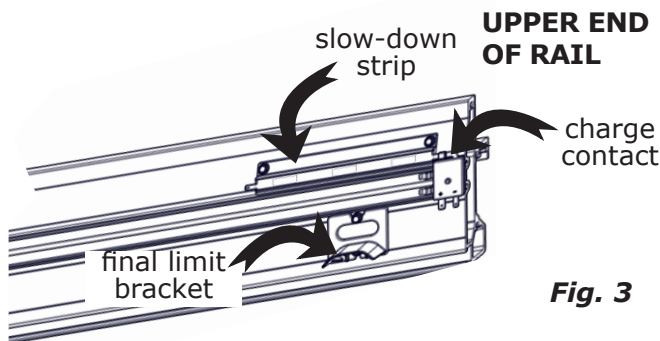
Fig. 3

1. Place upper rail section on saw horses.
2. Attach charge contact to the slow down strip (Fig. 1).
3. If previously removed, slide charge strip carrier into back side of rail.
4. Slide charge contact assembly from Fig. 1 into back side of rail. Assembly will fit snugly against charge strip carrier.

**NOTE: Carrier may be longer than joint end of rail at this point.**

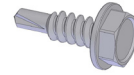5. Using two (2) self-drilling hex head screws, secure slow down ramp to rail. See Fig. 2.


**NOTE: Confirm that slow down ramp is parallel to charge strip carrier, and touching top edge of carrier.**



6. Remove charge contact assembly from rail and save for later use.
7. Mount final limit bracket: Upper edge of final limit bracket should be flush with lower edge of charge contact. Insert a self-drilling hex head screw through each screw hole on final limit bracket into etched lines on back of rail (Fig. 3)
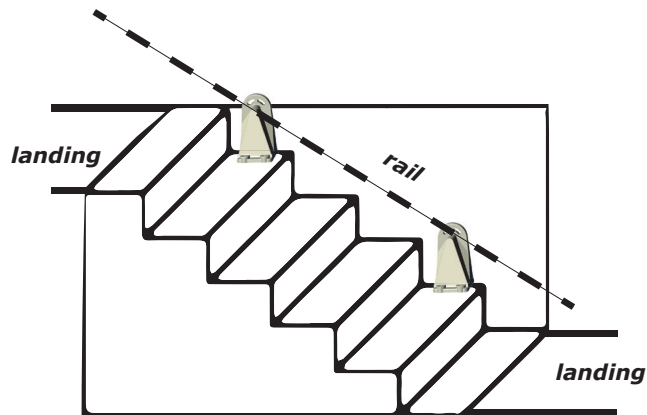
## FINISH PREPARATION

**back side of rail**

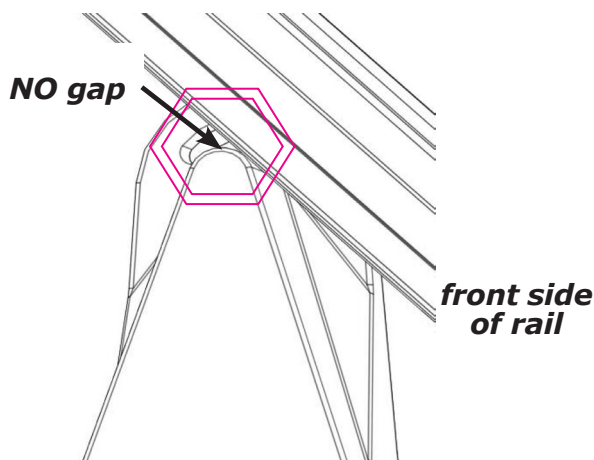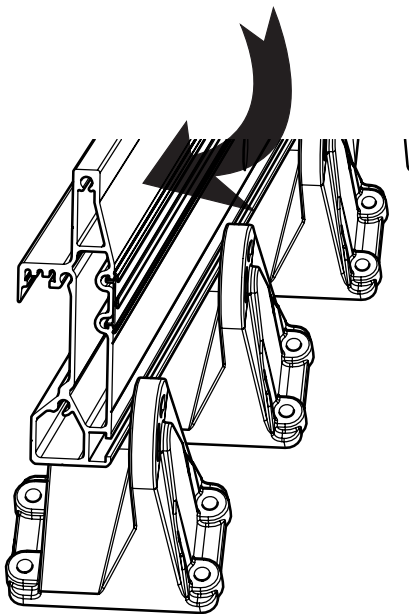**NOTE:** Additional rail brackets may be required on upper or lower landing if rider makes a "hard transfer" [dropping into seat by more than 4" (10 cm) from a standing position or while transferring from a mobility device]. These additional rail brackets prevent movement in rail during a "hard transfer".\*

1. **Determine rail bracket placement.** See *Rail Support (Rail Bracket) Requirements* on page 7 for guidelines.

\*Additional rail brackets may be required for longer installations.

Rail brackets may be purchased from Bruno.

2. Locate and open plastic bag marked "Stand feet hardware". These components are used to attach rail brackets to rail.
3. Pre-assemble hardware by inserting bolts into safety washers first and flat washers second.
4. Determine rail bracket-to-rail orientation, which **MUST** match the figures at left.

**Always install rail brackets with long nose facing the center of staircase.**

**NOTE:** Failure to install rail brackets in correct orientation may lead to serious personal injury and/or property damage!

**FOR 1-PIECE RAIL INSTALLATIONS,  
SKIP TO STEP 10 ON NEXT PAGE.**

# PRE-INSTALLATION

## FINISH PREPARATION (continued)

Fig. 1



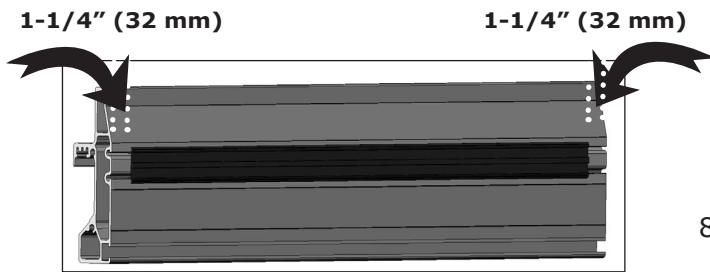
Fig. 2

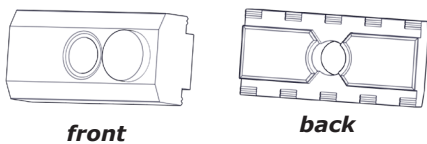nut bar

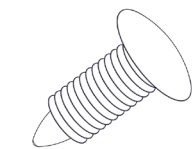Fig. 3


fin fastener

Fig. 4

5. **Join rails, if necessary.**  
Position upper rail section onto lower rail section. **Wiggle the upper section slightly** to make it align properly with lower section (Fig. 1).

**If installation involves a folding rail, feed charge harness through upper rail section before joining rails.**

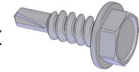6. Upper and lower rail sections should now be flush and snug.
7. Secure joint **brackets** to upper rail using two (2) self-drilling hex head screws per bracket, (i.e., etched lines on rail per Fig. 1).

**NOTE: Screws should be located no more than 2-3/4" (70 mm) from the joint if the bracket is installed correctly.**

8. On lower rail, install remaining screws (from Step 2 of Joint Preparation on page 9) in upper and lower brackets.

9. **Add nut bars (Fig. 3).**  
With rail on a flat surface, slide one (1) nut bar for each rail bracket into lower rear slot of rail.

If joined rails, add one (1) additional nut bar.

**NOTE: There will be one (1) nut bar per rail bracket plus one (1) directly over the rail joint.**

10. **Trim charge strip carrier.**  
Trim 2-1/2" (63.5 mm) shorter than total rail length.

**If installation involves a folding rail, insert fin fastener (Fig. 4) into the mid-rail charge strip carrier at the joint. Trim 1.5" (38 mm) shorter than total rail length.**

## **FINISH PREPARATION (continued)**

11. Slide charge strip carrier into back side of rail as shown (Fig 2).

12. ***Install gear rack pieces.***

Continuing to work on a flat surface, ***carefully\*\**** slide gear rack pieces into upper rail until:

(a) last piece extends beyond rail end

OR

(b) last piece is within 1/2" (13 mm) of rail end.

**\*\*Allowing gear rack pieces to "free fall" and land hard will knock joint out of adjustment.**

- **Maximum distance between last tooth of uppermost gear rack piece and rail end is 1-1/4" (32 mm).**

- If last piece extends beyond rail end, cut that piece to length. Mark piece even with rail end, slide it out, subtract 1/2" (13 mm) and cut.


**NOTE: After insertion, remove last gear rack piece (or two) to facilitate carriage installation!**

**NOTE: In order to get the rail to the installation site, in some cases a two-piece rail may need to be separated and rejoined once on the staircase.**


**FOR FOLDING RAIL INSTALLATIONS,  
SKIP TO PAGE 16.**

# INSTALLATION

## RAIL INSTALLATION - NON-FOLDING RAIL

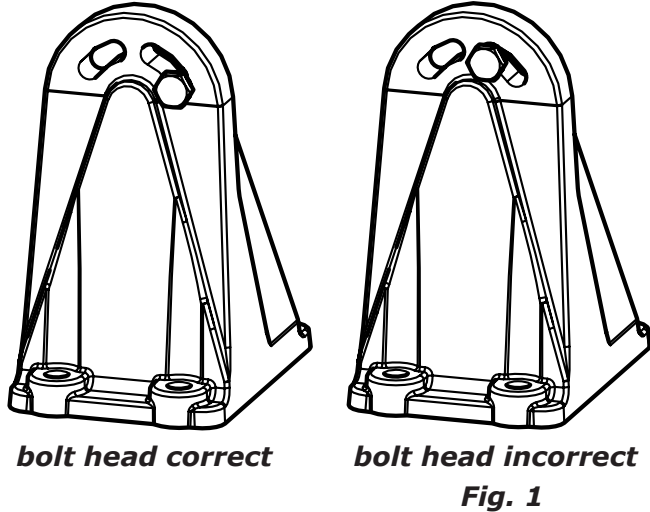**Fig. 2**

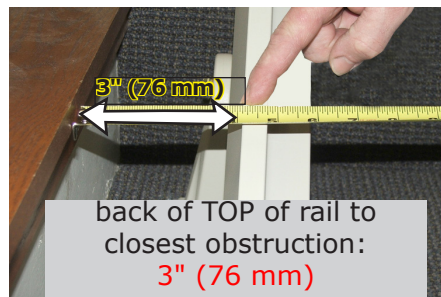**Fig. 3**

1. If not done previously, reconnect two-piece rail installations on staircase.
2. Place rail on stepnoses.  
**NOTE: Place heavy toolbox at bottom of stairs to keep rail from sliding.**  
**NOTE: Bottom of rail should be 1/8" (3 mm) off floor.**
3. Mount all rail brackets to rail.
  - Tap bolt down in rail bracket slot before tightening (Fig. 1).
  - Torque rail/rail bracket mounting hardware to 25 ft. lb. (34 Nm).
4. Slide rail out horizontally until bottom of rail is off stair stepnose by:  
**no carpet: 2" (51 mm) minimum**  
**carpet: 2.5" (64 mm) minimum**  
**Measure every step (Fig. 2).**
5. Measure from wall side of TOP of rail to closest obstruction (i.e., wall, handrail, etc.):  
3" (76 mm) (See Fig. 3.)  
**Measure at top, middle and bottom along the length of each rail section.**
6. Begin to anchor rail brackets, starting with rail bracket at top of stairs. Use only two (2) sheet metal screws, one (1) on the wall side first, then one (1) on the stairway side of rail.  
**NOTE: Wait to anchor rest of rail brackets until running clearances are checked.**  
**NOTE: If bottom of rail is closer than 2" (51 mm) to stair stepnose, remove screws, slide rail horizontally, and refasten rail bracket.**

## RAIL INSTALLATION - NON-FOLDING RAIL (continued)

7. ***Insert charge strips into rail.***

Locate two (2) charge strip coils. Hold one coil firmly to prevent uncoiling, and carefully undo tape securing coil.

8. Using tin snips, cut a 45° bevel on end of charge contact strips (see images at left).

**NOTE: This will make it MUCH easier to insert strip into carrier.**

9. Slide beveled end of charge contact strip into charge contact strip carrier until it sides under lower charge contact.

10. Cut off excess charge contact strip flush with carrier.

11. Repeat Steps 7-10 for the second charge contact strip coil.

**SKIP TO CARRIAGE INSTALLATION  
ON PAGE 18.**

# INSTALLATION

## RAIL INSTALLATION - FOLDING RAIL


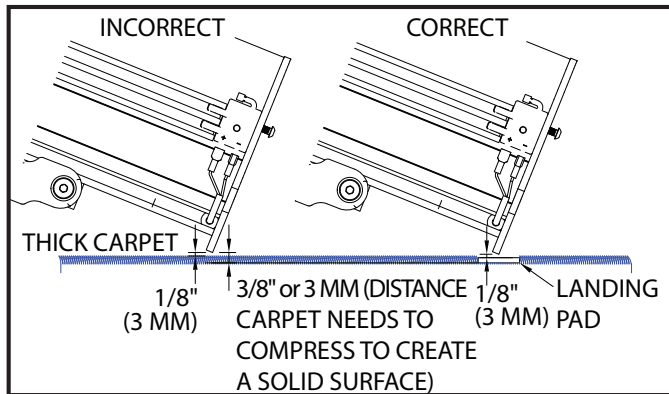
Fig. 1

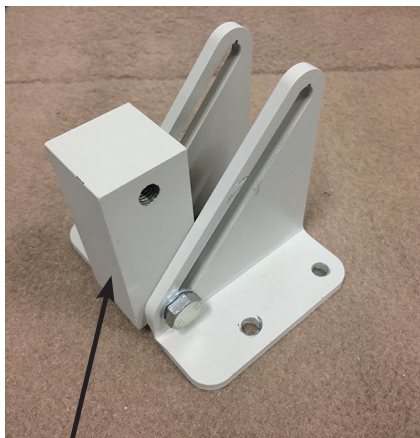adjustable post

Fig. 2

Fig. 3

1. If not done previously, reconnect two-piece rail installations on staircase.
2. Place rail on stepnoses.
 

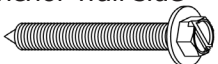
**NOTE: Place heavy toolbox at bottom of stairs to keep rail from sliding.**

**NOTE: Bottom of rail should be 1/8" (3 mm) off floor. If thick carpeting prevents this (Fig. 1), contact Bruno to purchase kit SRV-3000-038.**
3. Mount upper rail bracket to rail, keeping upper end of rail 2" (51 mm) off stepnose and lower end tight against stepnose.
4. Install remaining rail brackets, including special folding rail bracket (Fig. 2).
5. Slide rail out horizontally until bottom of rail is off stair stepnose by:
 

2-1/2" (64 mm) *minimum*

**Check clearance between bottom endplate and floor [1/8" (3 mm) off floor]. If too tight, rail will not close properly. Adjust by loosening bolts on all rail brackets, sliding rail up slightly, and retightening bolts.**
6. Measure from wall side of TOP of rail to closest obstruction (i.e., wall, handrail, etc.):
 

4.25" (108 mm)

**Measure at top, middle and bottom along the length of each rail section.**
7. Fasten special folding rail bracket to stair tread using two (2) sheet metal screws  through the stairway side of rail bracket.
8. Remove M10 bolt (circled in red). Set rail off to side (Fig. 3).
9. Lower adjustable post on special rail bracket to set space between front and back bracket feet. Then, anchor wall side of bracket with two (2) sheet metal screws. 

## **RAIL INSTALLATION - FOLDING RAIL (continued)**

10. Replace rail, seating firmly in special rail bracket.
11. Replace M10 bolt to secure rail to rail bracket. Finger tighten plus 1/4-turn.
12. Begin to anchor rest of rail brackets, starting with rail bracket at top of stairs. Use only two (2) sheet metal screws: one (1) on the wall side first, then one (1) on the stairway side of rail.

**NOTE: Wait to anchor rest of rail brackets until running clearances are checked.**

**NOTE: If bottom of rail is closer than 2" (51 mm) to stair stepnose, remove screws, slide rail out horizontally, and refasten rail bracket.**

13. ***Insert charge strips into rail.***  
Locate two (2) charge strip coils in rail installation kit\*. Hold one coil firmly to prevent uncoiling, and carefully undo tape securing coil.

\* Rail Installation Kits:  
SRE-K-3001 for 16' Non-Folding Rails  
SRE-K-3002 for 20' Non-Folding Rails  
SRE-K-3050 for 16'/18' Folding Rails

14. Using tin snips, cut a 45° bevel on end of charge contact strips (see images at left).

**NOTE: This will make it MUCH easier to insert strip into carrier.**

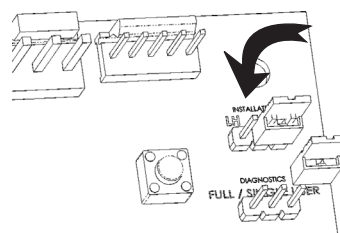
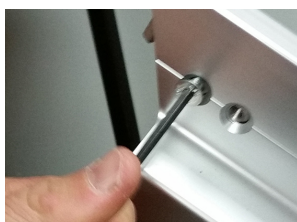
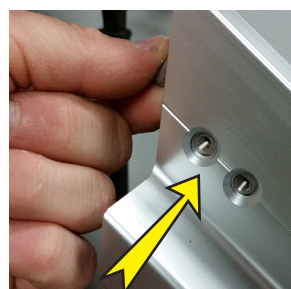
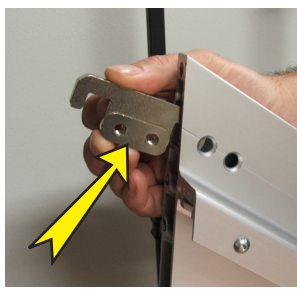
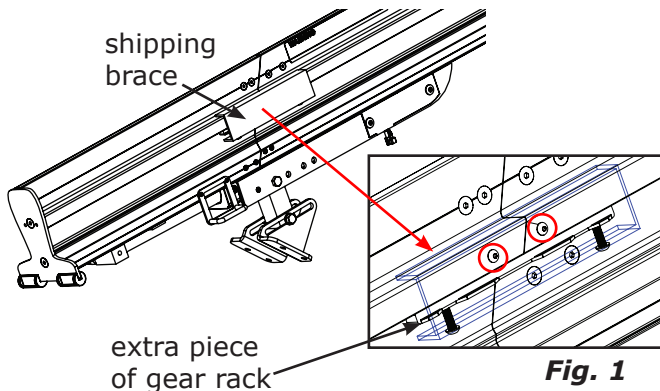
15. Slide beveled end of charge contact strip into charge contact strip carrier until it slides under lower charge contact.
16. Cut off excess charge contact strip flush with carrier.
17. Repeat Steps 13-17 for the second charge contact strip coil.

**CONTINUE TO CARRIAGE INSTALLATION  
ON PAGE 18.**

# INSTALLATION

## CARRIAGE INSTALLATION

**NOTES:** When handling carriage, take extra care to avoid damaging final limit bracket near battery housing!

**Do NOT turn on circuit breaker until carriage is on the rail!**

1. If not previously done, remove last gear section from upper rail.
2. Check that **circuit breaker is TURNED OFF.**
3. Before installing carriage, use a 4mm hex key to remove two (2) button head screws from shipping brace on folding rail and extra piece of gear rack (Fig. 1).
4. Make sure distance between folding rail and ground is NO MORE THAN 1/8" (3 mm). The closer to the ground, the better (Fig. 2)!
5. Raise short rail to provide access to latch installation location (Fig. 3).
6. Insert latch, aligning holes (Fig. 4).

**NOTE: Spacer plate should contact rail extrusion when installed.**

7. Hold latch in position while inserting screws (Fig. 5).

**NOTE: Failure to hold onto latch will allow it to slide to end of short rail. If this occurs, you must remove short rail end plate to retrieve latch.**

8. Using 4 mm Allen wrench, tighten two screws (Fig. 6).

**NOTE: Make sure the screws are tight and properly seated!**

9. Lower short rail to ground (Fig. 7).

**NOTE: The SRE-3000 is shipped from the factory ready for a left-hand installation.**


**! If installing a left-hand unit, skip to Step 11 on next page.**

10. Right-hand installations require these carriage changes BEFORE carriage is slid onto rail:
  - a. Remove top, center carriage cover. There are no screws! Pop it out (Fig. 8).
  - b. On circuit board, move LH/RH jumper to RH position (Fig. 9).


## CARRIAGE INSTALLATION (continued)

**Fig. 10**

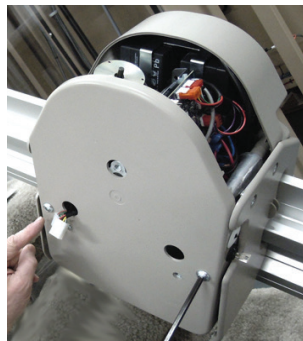
**Fig. 11**

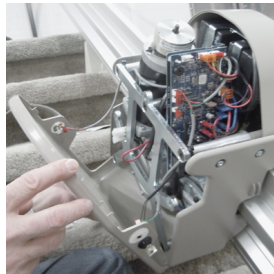
**Fig. 12**

**Fig. 13**

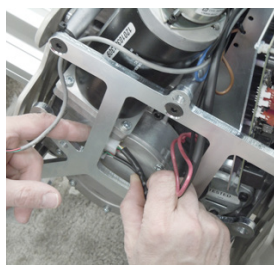**Fig. 14**

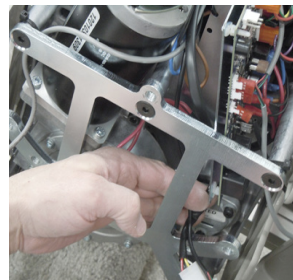**Fig. 15**

**Fig. 16**

**Fig. 17**

- c. Disconnect shipping jumper (Fig. 10).
  - d. Using a 6mm Allen wrench, remove two (2) button head screws securing front carriage cover (Fig. 11).
  - e. Pull front cover away from carriage. To provide full access to wiring harness, disconnect one sensor connector to allow carriage to hang to one side (Fig. 12).
- Note orientation of sensor connector!**
- f. Pull wiring harness from behind bracket and re-route as shown (Figs. 14 and 15).
  - g. Reconnect sensor lead (Fig. 13).
  - h. Route wiring harness through right-hand hole in carriage front cover (Fig. 16).
  - i. Remount and secure front carriage cover.
- NOTE: Two (2) button head screws will both be on "upstairs" side of the carriage.**
- j. Snap center carriage cover back in place. There are no screws!
  - k. Reconnect shipping jumper removed in Step 10c. This enables unit to run on rail.
  - l. Place plastic plug in open hole. Hand change is complete (Fig. 17).

11. If not previously done, SLOWLY slide carriage onto upper rail section until carriage spur gear rests against gear rack.

**NOTE: In right-hand installations, overspeed gear will contact gear rack before spur gear. Lower carriage slowly to avoid tripping overspeed gear.**

12. Remove serial number decal from backing (attached with cable tie to circuit breaker) and affix decal to window in data plate (bottom of seat).
13. Cut cable tie. **Do NOT** lose carriage key attached to cable tie.
14. Insert carriage key.
15. Turn carriage key to ON position.
16. Turn circuit breaker ON.

**NOTE: For multiple unit installations, remote transmitter and carriage for one unit will need to be re-programmed at this point. (See Reprogramming Remote Transmitters on page 32.)**

# INSTALLATION

## CARRIAGE INSTALLATION (continued)

Fig. 1a

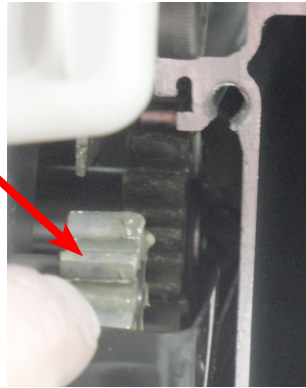
Fig. 1b

17. Using remote transmitter, power carriage approximately 12" (30 cm) onto gear rack.

**NOTES:** In left-hand installations, power carriage *slowly*, allowing overspeed gear to align with gear rack.

While jogging carriage on rail with remote, if overspeed gear "clunks" against gear rack, and carriage stops, manually ease carriage toward end of rail, just far enough to provide access to overspeed gear. Use a finger to turn overspeed gear past a few teeth. Try again to power carriage onto gear rack using remote (Figs. 1a, 1b).

## FOOTREST INSTALLATION


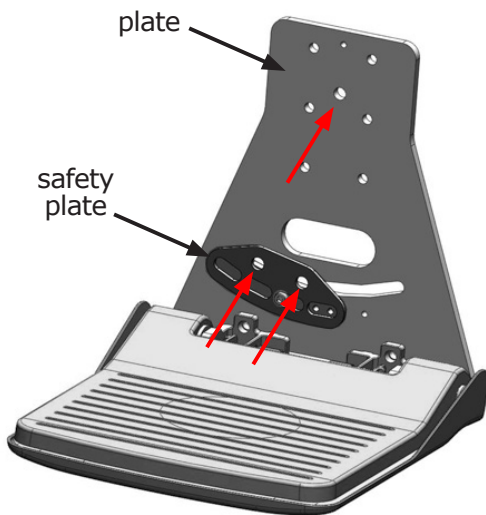
Fig. 2

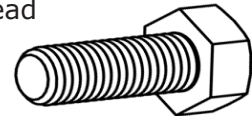1. **Before mounting footrest**, turn circuit breaker OFF.
2. Remove downside M10 button head screw securing front cover during shipping.
3. Insert this button head screw in open hole on upstairs side of front cover, next to its mate.
4. Mount footrest to carriage by inserting three (3) M10-25 hex head cap bolts and three (3) safety washers.

Insert one (1) washer through top of plate and two (2) through safety plate (Fig. 2).

**DO NOT TIGHTEN AT THIS POINT!**

## FOOTREST INSTALLATION (continued)

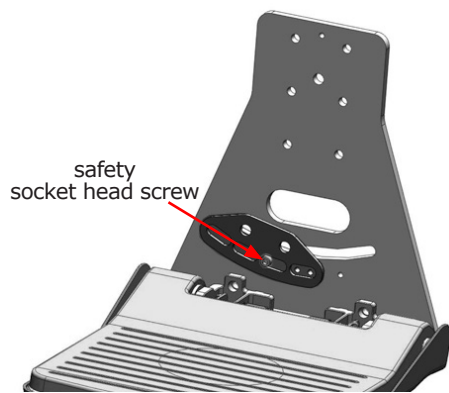
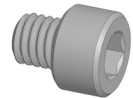
**Fig. 3**

5. Verify that footrest is level (Fig. 3).
  - Lower footrest.
  - Place a level on footrest pan.
6. Fully tighten nuts on three (3) footrest bolts inserted in Step 4.

***Torque to 50 ft-lb (68 Nm).***

7. Insert and tighten safety socket head screw (shear bolt) (Fig. 4).

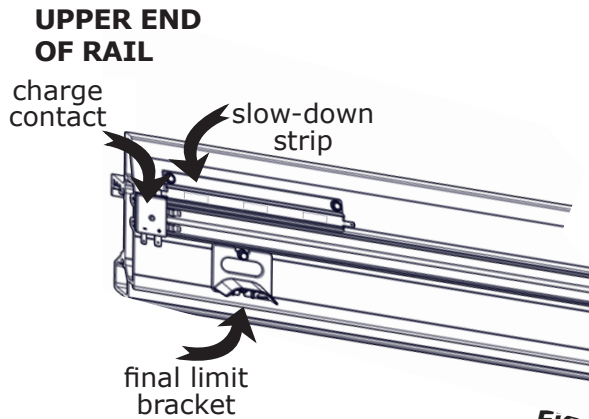
**Fig. 4**

# INSTALLATION

## COMPLETING RAIL INSTALLATION

### *left-hand installation*


**Fig. 1a**

**NOTE: Make sure circuit breaker is still turned OFF.**

1. Insert remaining gear rack pieces, cutting if necessary.
2. Add charge contact.
  - a. Slide charge contact into rail and connect to slow down contact (Figs. 1a and 1b).

**CONTINUE TO PAGE 24 IF NO FOLDING RAIL IS INVOLVED.**

### *right-hand installation*

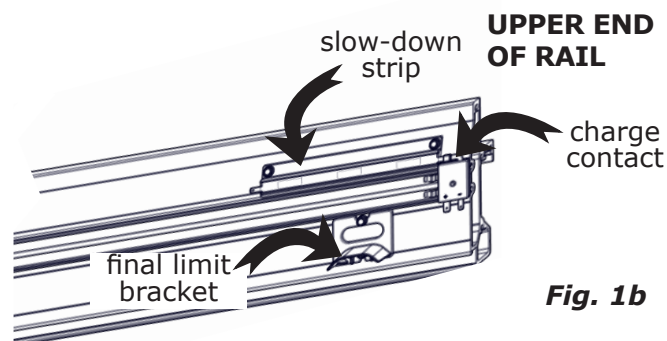
**Fig. 1b**


## COMPLETING RAIL INSTALLATION (continued)

**Fig. 3**

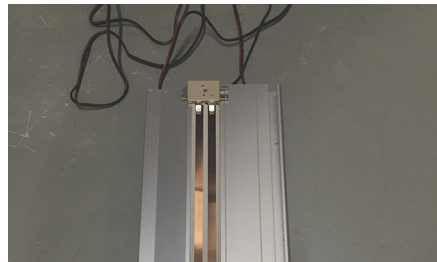**Fig. 4**

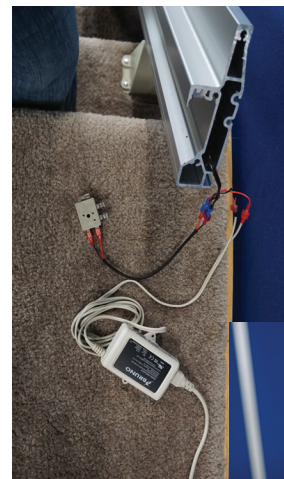**Fig. 5**

**Fig. 6**

**Fig. 7**

**Fig. 8**

3. Insert last gear rack section into upper rail.

### ***For folding rail installations:***

**Fish black charge cable up through upper rail (see Figs. 3 and 4).**

**Cable wires will fit between spacer and end of rail.**

**Connect charge harness to charge contact and charger (Fig.5).**

**Stuff remaining harness into end of rail (Fig. 6).**

**Mount provided spacer endplate (in kit SRE-K-3050) between top of endplate and end of rail to prevent damage to components (Fig. 7).**

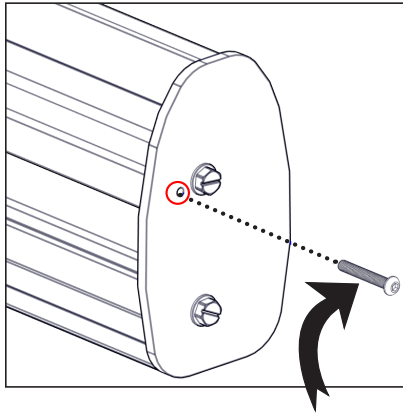
**See Fig. 8 for a finished endplate example.**

4. ***Add top endplate.***

Use a 1/2" socket and ratchet to mount top endplate using two (2) 5-16" - 18x1" self-threading hex head screws.

# INSTALLATION


## COMPLETING RAIL INSTALLATION (continued)

jack screw

### 5. **Insert jack screw.**

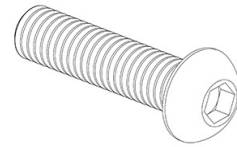Two (2) M6 socket button head cap screws (i.e., jack screws) are provided for pressure-loading the gear rack. One is 25 mm long and one is 40 mm long.

Use table below to determine which screw to use.


Distance from last full gear rack segment to end of rail	M6 x 25 mm screw	M6 x 40mm screw
1/2" (12mm) or less	X	
1/2" (12mm) to 1 1/4" (32mm)		X

Jack screw may be inserted into either top or bottom endplate.


**NOTE: Do NOT overtighten!**  
Finger tighten screw until screw contacts gear rack. Then, use hex wrench to tighten no more than one complete turn.

**NOTE: Jack screw will NOT be flush with endplate.**


## RAIL CROSS PIN INSTALLATION

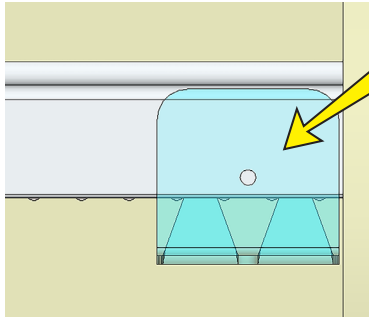
**Fig. 1**

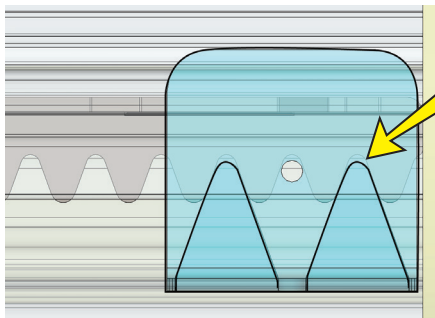
**Fig. 2**

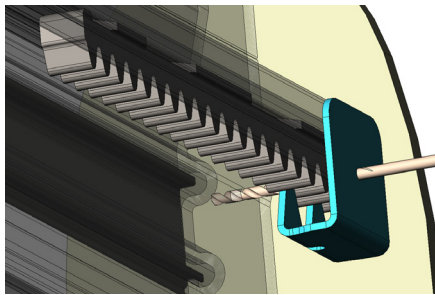**Fig. 3**

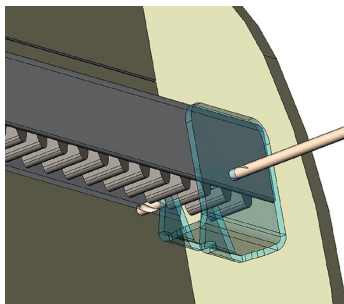**Fig. 4**

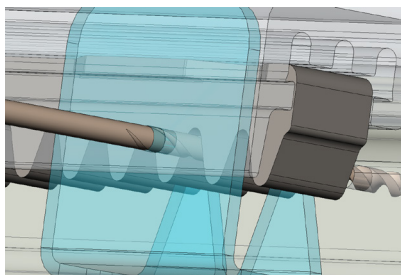**Fig. 5**

**NOTE: Install rail cross pin *after* jack screw has been properly tightened.**

1. At bottom end of rail (base of stairs), position SRE Cross Pin Drill Template after first tooth of first gear rack. Template hole should be up against the part of the extrusion covering the gear rack (Fig. 1). Template "teeth" should fit up against the gear rack valleys (Fig. 2).

2. Using a 1/8" (3.2mm) drill bit (Bruno P/N: DRB-13001, provided with kit SRV-3000-023), place a hole in 2 walls of the extrusion (Figs. 3 and 4). Do not drill through entire extrusion.


**NOTE: Drill bit will not make contact with gear rack (Fig. 5).**

# INSTALLATION


---

## **RAIL CROSS PIN INSTALLATION (continued)**

3. Once hole is drilled, place pin into rail extrusion (Fig. 6). Make sure pin is flush with outside of extrusion (Fig. 7).

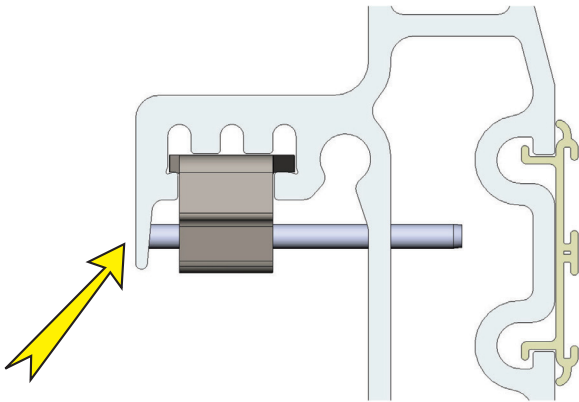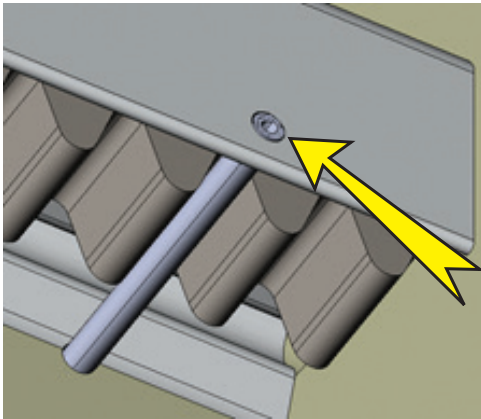**Fig. 6**


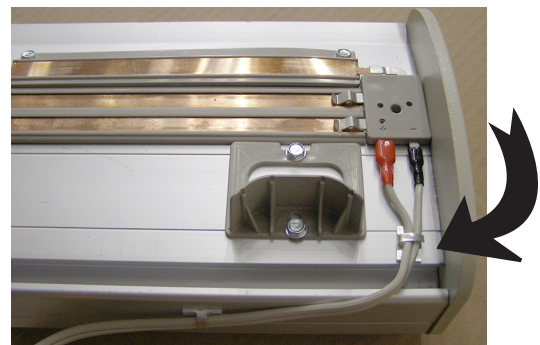
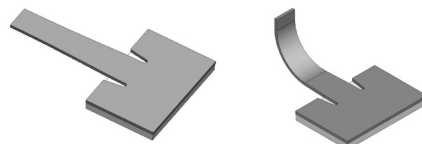
**Fig. 7**

## **MAKE CHARGER CONNECTIONS - NON-FOLDING RAIL**

1. Connect wires from charger to charge contact assembly. (See figure at left.)
2. Attach adhesive wire clip to rail and fold tab over wires as shown below.

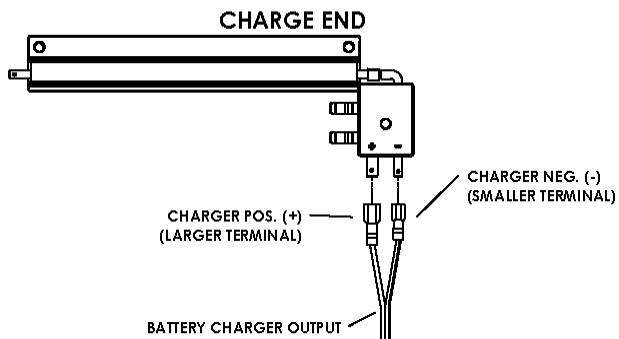- CONFIGURATION SHOWN:**  
1) LEFT-HAND INSTALLATION WITH CHARGER AT BOTTOM.  
2) RIGHT-HAND INSTALLATION WITH CHARGER AT TOP.

3. Plug charger into a live, properly-grounded 120V AC outlet.
4. Turn on circuit breaker.



# INSTALLATION

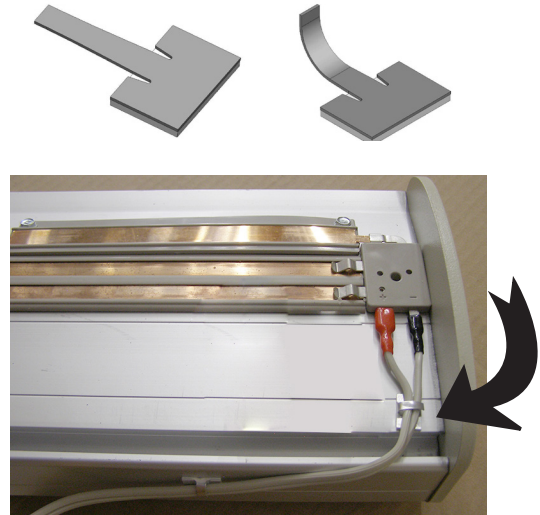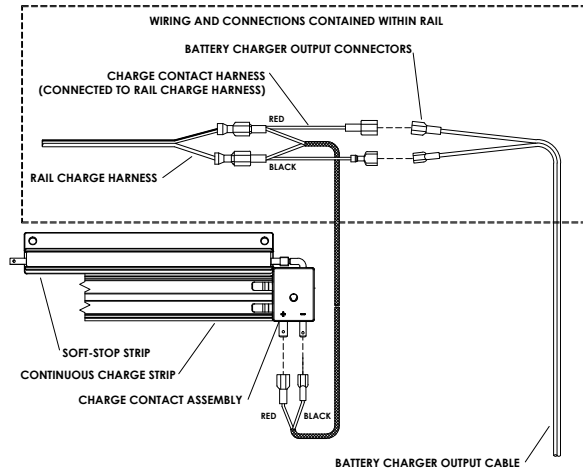
## MAKE CHARGER CONNECTIONS - FOLDING RAIL

1. Connect wires from charger to charge contact assembly. (See figures at left.)
2. Attach adhesive wire clip to rail and fold tab over wires as shown below.

SRE-3000 FOLDING RAIL CONFIGURATION SHOWN:  
RIGHT-HAND INSTALLATION (CHARGER AT TOP)

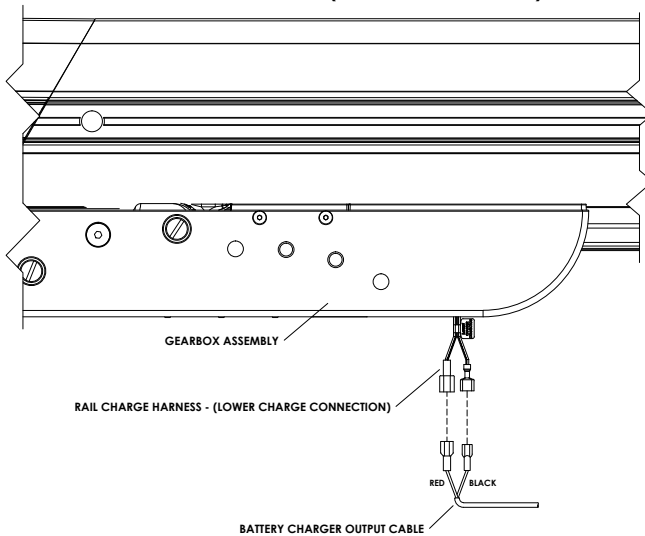
SRE-3000 FOLDING RAIL CONFIGURATION SHOWN:  
RIGHT-HAND INSTALLATION (CHARGER AT RAIL JOINT)


3. Plug charger into a live, properly-grounded 120V AC outlet.
4. Turn on circuit breaker.


## SEAT INSTALLATION

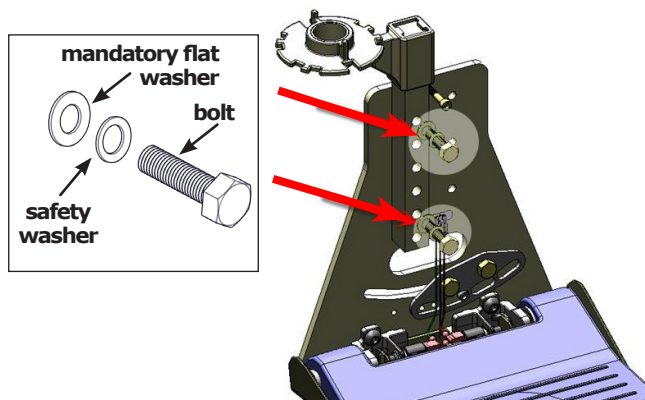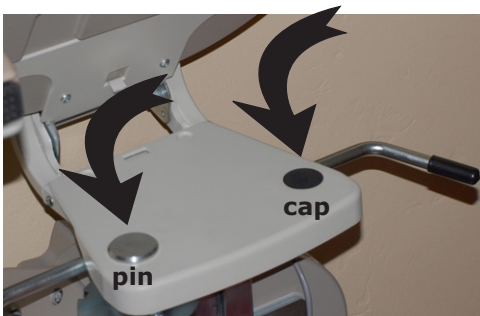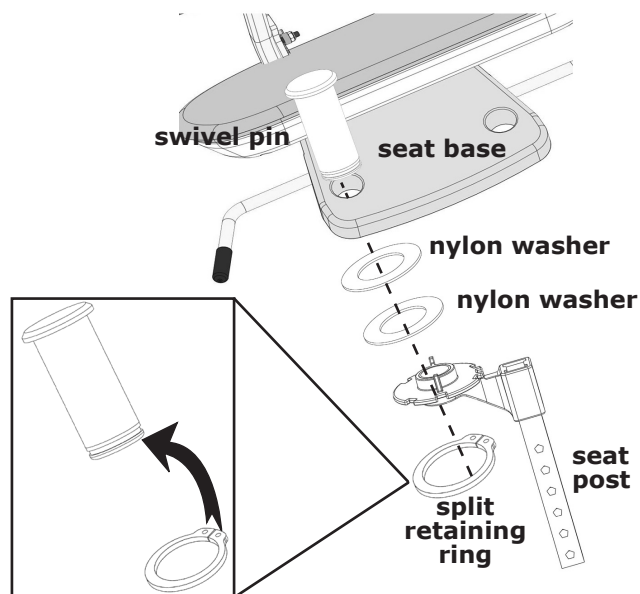seat post orientation: **Fig. 1**  
toward top of rail  
(right-hand installation shown)


pin and cap in seat base  
(right-hand installation shown)

retaining (snap) ring  
mounting to bottom  
of seat swivel pin

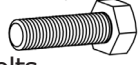**Fig. 3**

1. Mount and fully tighten seat post using:

- two (2) M10-35 hex head cap bolts, 
- two (2) safety lock washers, 
- two (2) MANDATORY flat washers. 

**NOTE: Finger tighten hardware three (3) turns, then torque to avoid cross threading!**

**Torque to 50 ft-lb (68 Nm).**

**NOTE: Offset seat post towards top of rail.**

2. Insert seat swivel pin in seat base, using "upstairs side" hole.
3. Slide two (2) nylon washers onto seat swivel pin, positioned between seat base and seat post.

**NOTE: These washers are shipped on the seat post, requiring disassembly before performing Step 3.**

4. With seat facing 45° off center to upstairs side, insert seat swivel pin into seat post to mount seat.

**NOTE: If necessary, use a dead blow hammer to tap pin into place.**

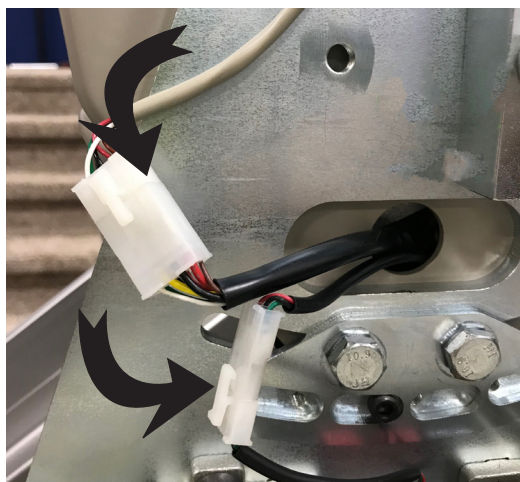
5. Using seat swivel lever, pivot seat to its locked position. Seat should drop into place.
6. Using heavy-duty snap ring pliers, install retaining (snap) ring onto bottom of seat swivel pin (Fig.3). 

7. Remove shipping jumper from carriage harness.

# INSTALLATION

---

## SEAT INSTALLATION (continued)

8. Connect 3-pin carriage harness to footrest harness.

Connect 6-pin carriage harness to seat harness.

9. **Add the footrest shroud.**

Determine which tab to remove for the seat post.

10. Using a utility knife, cut each side of tab and use finger to snap off.

## SEAT INSTALLATION (continued)

11. Mount shroud to unit using M6 socket head screw.


# INSTALLATION

## LUBRICATE GEAR RACK

1. Run carriage up and down rail several times.
2. Apply a thin layer of white lithium grease (provided) **on gear rack ONLY**.  
*Repeat often if unit is used very frequently or is used in a harsh environment.*
3. Regular cleaning will keep rail and rack free of dust and dirt.

## BATTERY CHARGER INSTALLATION

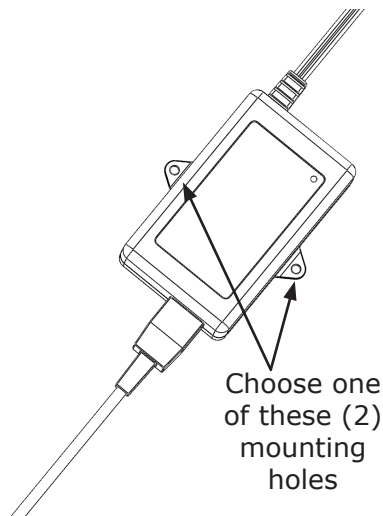**Fig. 1**

1. Connect battery charger to wall outlet and rail.  
**NOTE: Both wall and rail cords are each 6' (1.8 m) long.**
2. Position charger in a suitable, permanent location **where it will not create a tripping hazard**.
3. Secure charger in chosen location by inserting two (2) wood screws (provided) through mounting holes in charger base (Fig. 1).

**NOTES: If batteries become discharged, carriage will move more slowly until voltage drops to a point where controller will shut off.**

**If this happens, pause, then run carriage "down" until it engages charge contacts.**

**Partially charged batteries will run carriage "down", but not "up".**

**Confirm that charger is plugged into a "live" outlet and wait for batteries to recharge.**

## **FIT AND FINISH**

### ***Run carriage up and down rail to:***

- Recheck seat-to-wall clearance.
- Check correct operation of all components.

### ***Test main components:***

- Check proper operation of battery charger (see page 35).
- Check call/send transmitters (see **Reprogramming Remote Transmitters** on page 32).
- Check on/off switch.
- Check footrest safety switches.
- Check that stairlift cannot operate with seat swiveled.
- Check carriage end safety switches.

### ***Test power folding rail (if applicable):***

- Check that when disable switch is turned off that rail cannot fold.
- Listen for continuous beep when folding rail is in motion. (This warns nearby people of rail operation.)
- Check that control box light flashes yellow.

### ***Make final adjustments (see Post-Installation Adjustments):***

- Adjust seat height.
- Adjust footrest.
- Adjust seat arm extension.

# INSTALLATION

## CUSTOMER EDUCATION

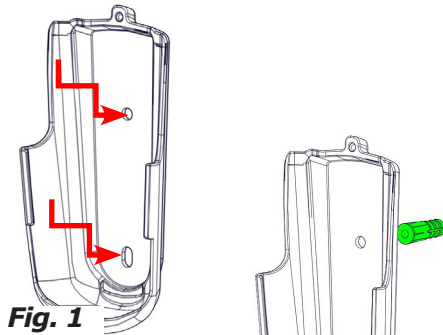
Fig. 1

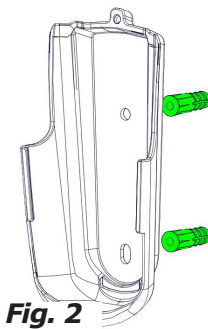
Fig. 2

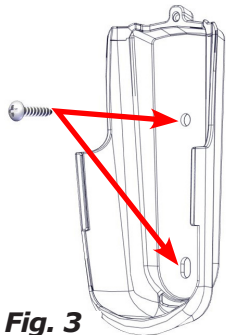
Fig. 3

Fig. 4

Fig. 5

1. Inform customer of location of Operator's Manual. Encourage them to become familiar with the contents.
2. **Mount Call/Send Transmitter with OPTIONAL TMR-00005 kit:**
  - Discuss bracket location with customer before mounting to wall.
  - Position bracket on wall. Mark top and bottom bracket holes (Fig. 1).
  - Drill a pilot hole slightly smaller than anchors provided.
  - Gently pound anchors into wall with rubber mallet.
  - Position bracket on wall so top/bottom bracket holes align with anchors (Fig. 2).
  - Secure bracket to wall using screws provided (Fig. 3).
  - Slide transmitter into bracket (Fig. 4).
  - *Optional:* Install additional screw provided to prevent removal of transmitter from bracket (Fig. 5).

### 3. **Test call/send transmitters.**

- Use rocker switch on seat armrest to run unit up and down the rail.

**NOTE: Unit should travel noticeably faster going up than down.**

- Check for smooth operation and travel.
- Use remote call/send transmitter to run unit up and down rail.
- Repeat Step 3 with second transmitter.
- Check for smooth operation and travel.

## CUSTOMER EDUCATION (continued)

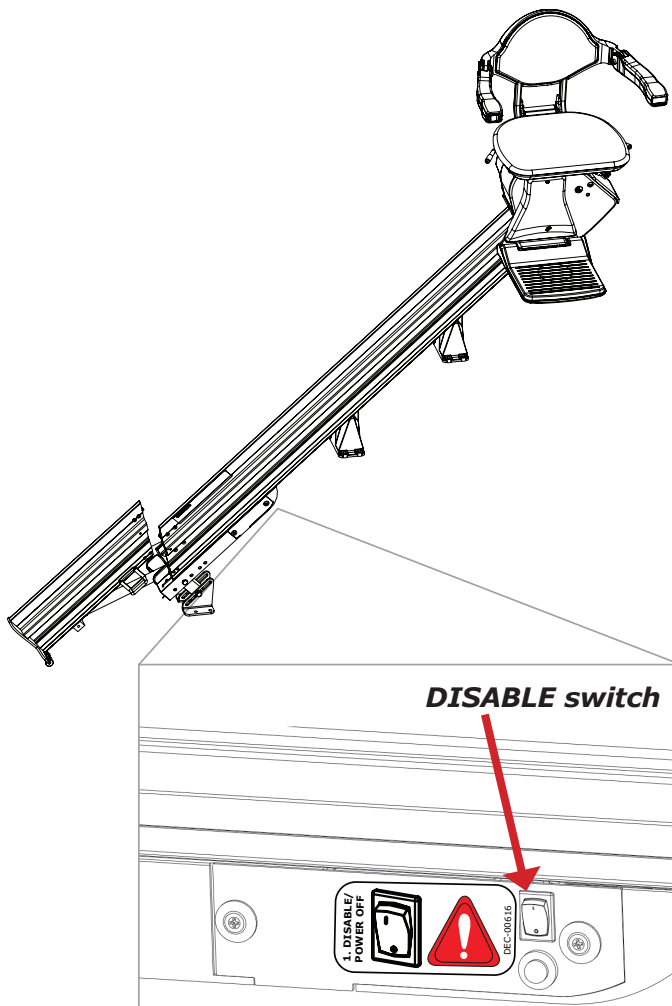
**⚠ WARNING**

**PINCH POINT!**

Make sure the areas above and below the folding rail are clear before operating the folding rail.

Keep children and pets at a safe distance.

Failure to observe this warning could result in injury or damage to equipment.

#### 4. *Train customer to use stairlift correctly and safely.*

- Remind them to always buckle lap belt when using stairlift.
- Have them operate unit while you are there to answer questions and address concerns.

#### **Manual Folding Rail:**

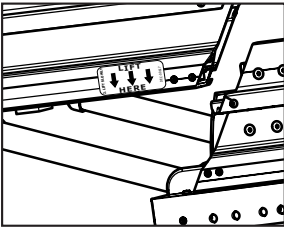
When folding the rail UP, carriage **MUST** be at a position that allows rail to fully fold up!

**Damage to carriage WILL occur** if carriage is below mid-park position when rail is folded up.

#### **Power Folding Rail:**

Will **only** work when carriage is at mid-park position.

#### **Review disable switch.**

- Control panel includes a disable switch. Press switch toward DISABLE to prevent rail from folding up. Use this switch if rail is to be kept folded down for an extended period of time.
- The disable switch also allows back drive of power in the unlikely event of an emergency.
- Make sure to lift slowly where indicated by the "LIFT HERE" decal on the short rail. Lifting the rail in a quick, sudden motion may result in damage to the folding rail gearbox. 
- Press switch AWAY from DISABLE to allow rail to fold and unfold correctly under power.

**NOTE: If power folding rail stops, then slightly reverses direction, and there are NO OBSTRUCTIONS, call BrunoTechnical Service.**

## REPROGRAMMING REMOTE TRANSMITTERS

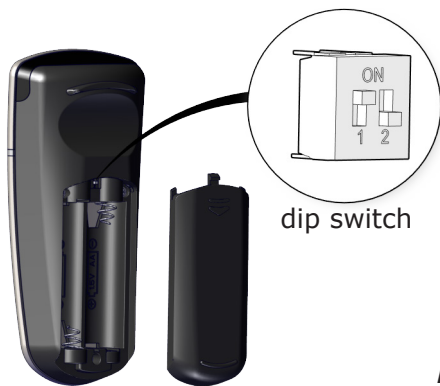
Fig. 1

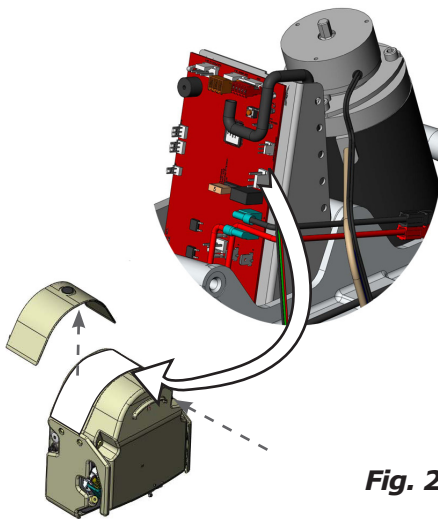
Fig. 2

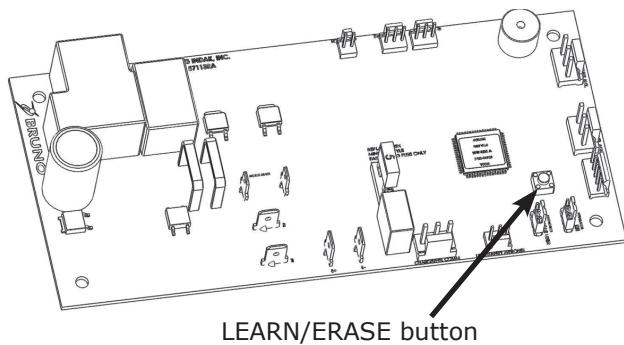
Fig. 3

**NOTE: Operating channel of the two (2) remote transmitters included with the SRE-3000 is pre-set at the Bruno factory.**

Reasons for reprogramming transmitters:

- multiple units in same location;
- remotes need to be replaced.

### **Reset operating channel of remotes:**

1. Remove remote(s) battery access panel.
2. Locate dip switch labeled 'SW3' (Fig. 1).
3. Change BOTH dip switch configurations to one (1) of four (4) possible options:
  - **1 up, 2 down (default factory setting)**
  - **1 up, 2 up**
  - **1 down, 2 up**
  - **1 down, 2 down**

### **To reprogram a transmitter:**

1. Turn circuit breaker OFF.
2. Remove top mid panel of control enclosure to expose circuit board (Fig. 2).
3. Locate LEARN/ERASE button on circuit board (Fig. 3).
4. Turn circuit breaker ON, and wait for "power-up" beep.
5. **Clear the memory:**
  - Hold down LEARN/ERASE button until LED on circuit board goes out (approx. 12 seconds).
6. Press and hold LEARN/ERASE button (LED on circuit board is on).

As you continue to press LEARN/ERASE, press either call/send button on remote until LEARN/ERASE LED goes out (approx. 2-5 sec.).

## **REPROGRAMMING REMOTE TRANSMITTERS (continued)**

7. Release both buttons (LEARN/ERASE and one on remote).

**The remote is now programmed.**

8. Make sure all remotes are set to same dip switch configuration.

**NOTE: It is only necessary to program one remote if all remote dip switches are set the same.**

9. ***Test transmitter operation:***
  - Press either button on remote.
  - If carriage moves, new settings been accepted and remote is programmed.
  - If carriage does not move, repeat Steps 1 through 8.
10. Remount remote (back on wall if approved by customer).
11. Turn circuit breaker OFF.
12. Remount and secure control enclosure covers.
13. Turn circuit breaker ON, and check operation of all functions.

## SINGLE-USER VS. MULTI-USER DIAGNOSTIC MODE

**NOTICE:**


Before touching anything inside carriage assembly, ground yourself by touching an unpainted metal surface on unit such as an exposed bolt, or one of the mounting screws on electrical panel. While working, periodically touch an unpainted metal surface to dissipate any static electricity that could harm internal components.

**NOTES:** Units ship in multi-user mode.

**Multi-user mode means that when seat is swiveled at top of rail, unit beeps once every three (3) seconds.**

**Single-user mode means that when seat is swiveled at top of rail, no beeps are heard.**


<b>AUDIO REFERENCE</b>	
Chirp	0.25 second
Short Beep	0.50 second
Long Beep	1.50 seconds
Pause	1.00 second

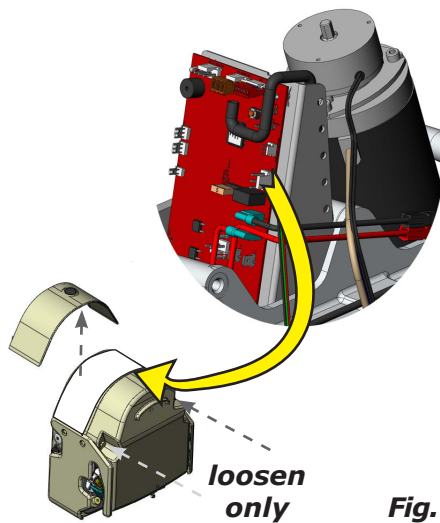**Fig. 1**

### Changing Multi-User to Single-User Mode

1. Turn circuit breaker on carriage to OFF.
2. **Loosen**, but do not remove top carriage cover screws.
3. Remove top carriage cover (Fig. 1).
4. Change diagnostic mode by pulling jumper off (Fig. 2) and pushing it on to two (2) right-hand pins.

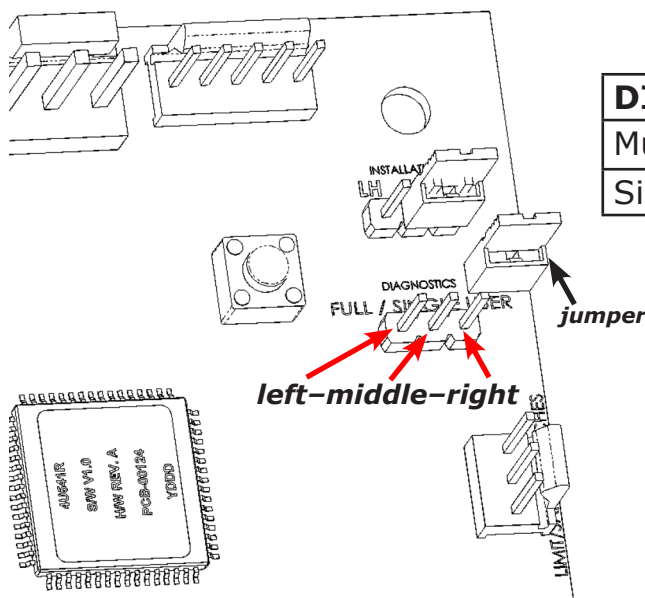**Fig. 2**

<b>DIAGNOSTIC MODE</b>	<b>pins under jumper</b>
Multi-User	left and middle
Single-User	middle and right

**BATTERY CHARGER LED INFORMATION**

<b><i>LED COLOR/FUNCTION</i></b>	<b><i>STATUS/PROBLEM</i></b>	<b><i>REMEDY</i></b>
red	AC power on	normal condition when connected to a live outlet
red (flashing)	defective batteries in unit	test batteries for damage or sulfation (load test)
yellow	charging	status OK
green	battery fully charged/float mode charging	status OK
green (flashing)	reverse polarity connection to unit	check wiring to rail charge strips for correct polarity connections

# POST-INSTALLATION ADJUSTMENTS

## DRIVE ADJUSTMENT

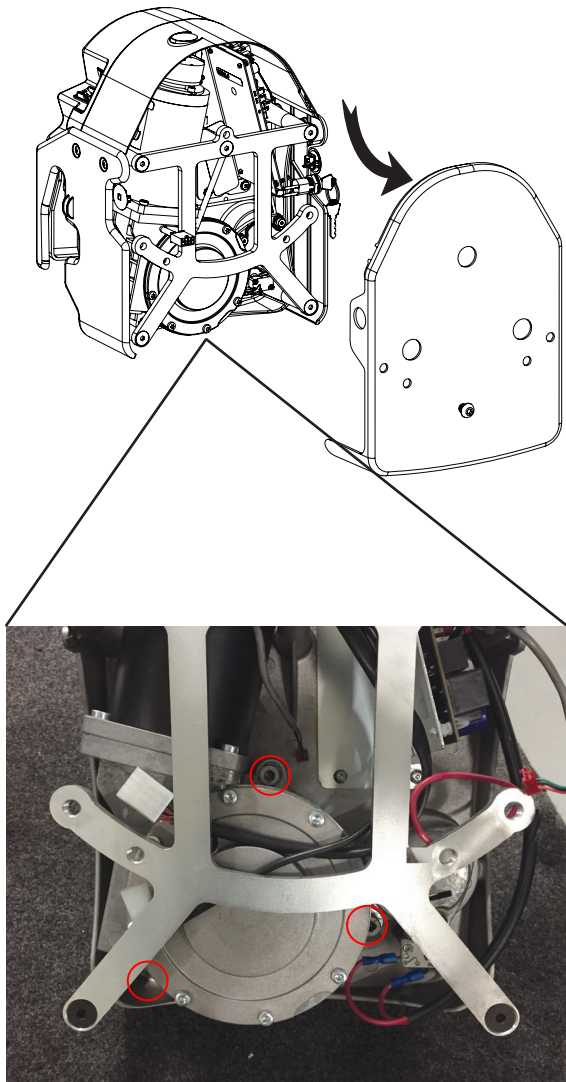
Fig. 1

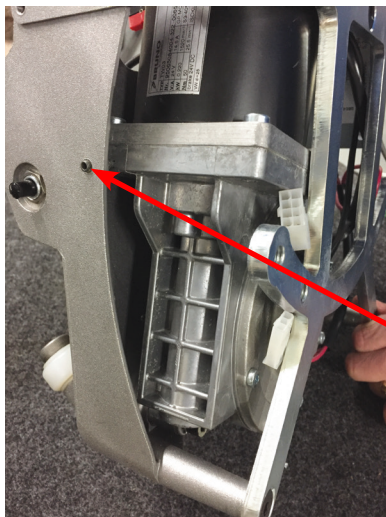
Fig. 2

Even though each SRE-3000 is run on a test rail prior to shipment, there may be slight variations between the test rail and rail installed on site. These variations may result in a small but objectionable vibration of the carriage.

### **Adjust rack and pinion gear to eliminate vibration.**

1. Make sure rack is properly greased.  
Inspect both gear rack and pinion for contaminants.
2. Remove seat and footrest assemblies.
3. With carriage installed on rail, and jumpers installed on carriage, run unit (loaded and unloaded) up and down rail over a few feet. Listen and watch for vibration.
  - a. If unit vibrates only in down direction, carriage and rail are fitted too loosely. The gear must be seated further into rack mesh.
  - b. If unit vibrates when running up and down, carriage and rail are fitted too tightly. The gear must be seated further out from rack mesh.
4. In both cases, loosen (but **DO NOT REMOVE**) three (3) drive mounting bolts (Fig. 1) 1-2 turns using 5mm and 6 mm Allen wrenches.
5. Tighten jack screw 1/8th turn (Fig. 2).
6. Run carriage on rail about 3' (1 m) in each direction with bolts still loose. This allows pinion gear to find the natural center of the gear rack.
  - If carriage runs smoothly, retighten drive mounting bolts.
  - If carriage does not run smoothly, repeat step 5.
7. *Making sure drive does not move as you tighten*, retighten three (3) drive mounting bolts.

<b>UNIT DOES NOT OPERATE/ IS NOT CHARGING</b>	<ul style="list-style-type: none"> <li>• Check circuit breaker. <i>Reset if necessary.</i></li> </ul>
	<ul style="list-style-type: none"> <li>• Check battery connections.</li> </ul>
	<ul style="list-style-type: none"> <li>• Check footrest safety switches to see if one of these limit switches is depressed. <i>Sliding safety tray below footrest should slide freely and should not stick in a position which would depress one of the safety switches.</i></li> </ul>
	<ul style="list-style-type: none"> <li>• Check for discharged batteries. <i>Battery voltage should fall within the range of 16-28 VDC.</i></li> </ul>
	<ul style="list-style-type: none"> <li>• Check that keyswitch is ON.</li> </ul>
<b>UNIT OPERATES SLOWLY, LACKS POWER</b>	<ul style="list-style-type: none"> <li>• Check for discharged batteries.</li> </ul>
	<ul style="list-style-type: none"> <li>• Check for loose connections.</li> </ul>
	<ul style="list-style-type: none"> <li>• Check that charger is plugged in and working.</li> </ul>
<b>REMOTE CALL/SENDS OPERATE BACKWARDS; UNIT GOES UP SLOWLY AND DOWN FASTER</b>	<ul style="list-style-type: none"> <li>• Unit has been installed on right-hand side of stairs and dip switch has not been set to right-hand and/or seat has not been installed (circuit board).</li> </ul>
<b>UNIT OPERATES ERRATICALLY OR INTERMITTENTLY WITH REMOTE CALL/SEND</b>	<ul style="list-style-type: none"> <li>• Check all safety mechanisms, including swivel seat safety switch.</li> </ul>
	<ul style="list-style-type: none"> <li>• Reorient transmitter to point directly at carriage.</li> </ul>
	<ul style="list-style-type: none"> <li>• Consult your dealer, an experienced technician or call the Bruno Technical Service Department.</li> </ul>
<b>UNIT OPERATES ERRATICALLY OR INTERMITTENTLY WITH A RIDER USING ARMREST MOUNTED CONTROL SWITCH</b>	<ul style="list-style-type: none"> <li>• Check swivel seat safety switch.</li> </ul>
	<ul style="list-style-type: none"> <li>• Check that footrest safety tray is not dragging on stair nosing or hitting debris on stairs.</li> </ul>
	<ul style="list-style-type: none"> <li>• Check rail for debris that may bump safety switches (footrest and carriage panels).</li> </ul>
<b>UNIT WILL NOT OPERATE UNLESS SEAT IS POSITIONED SO IT FACES OPEN SIDE OF STAIRS</b>	<ul style="list-style-type: none"> <li>• This is correct lift operation.</li> </ul>
	<ul style="list-style-type: none"> <li>• A safety switch in seat swivel prevents unit from operating with seat "out of position".</li> </ul>
<b>UNIT WILL NOT OPERATE WITH CALL/SEND REMOTE</b>	<ul style="list-style-type: none"> <li>• Check batteries in remote call/send unit.</li> </ul>
	<ul style="list-style-type: none"> <li>• Aim remote at carriage.</li> </ul>
	<ul style="list-style-type: none"> <li>• Remotes must be reprogrammed.</li> </ul>
<b>UNIT DOES NOT SHUT OFF WHEN IT HITS BUMPER AT END OF RAIL</b>	<ul style="list-style-type: none"> <li>• Verify correct operation of limit switches in carriage assembly.</li> </ul>
<b>ROUGH OR CHATTERING RIDE</b>	<ul style="list-style-type: none"> <li>• Wipe down rail.</li> </ul>
	<ul style="list-style-type: none"> <li>• Apply a small amount of grease to <b><i>gear rack only.</i></b></li> </ul>

# TROUBLESHOOTING

## RESET FOLDING RAIL JOINT

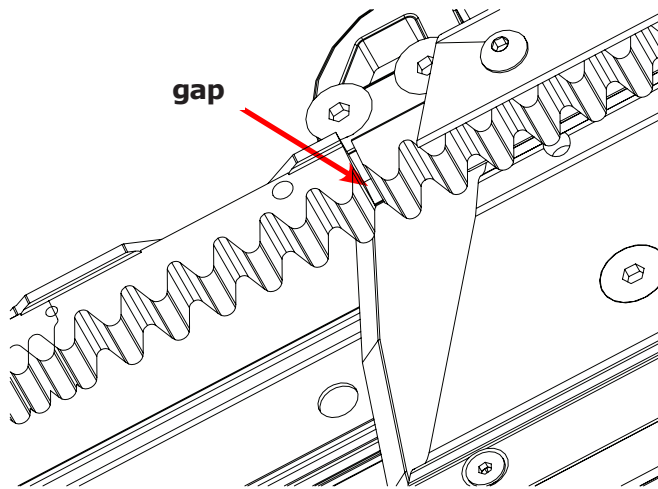
Fig. 1

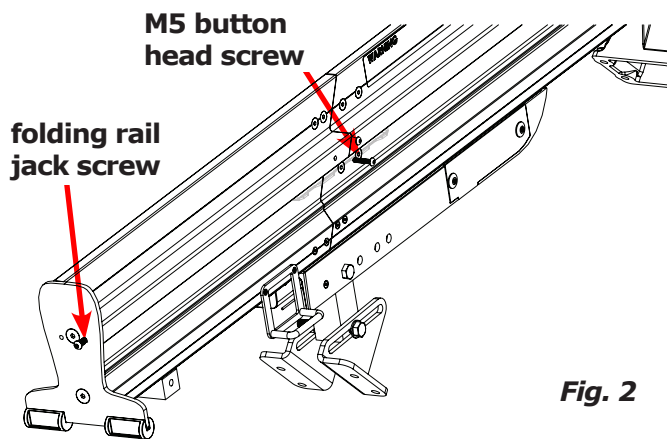
Fig. 2


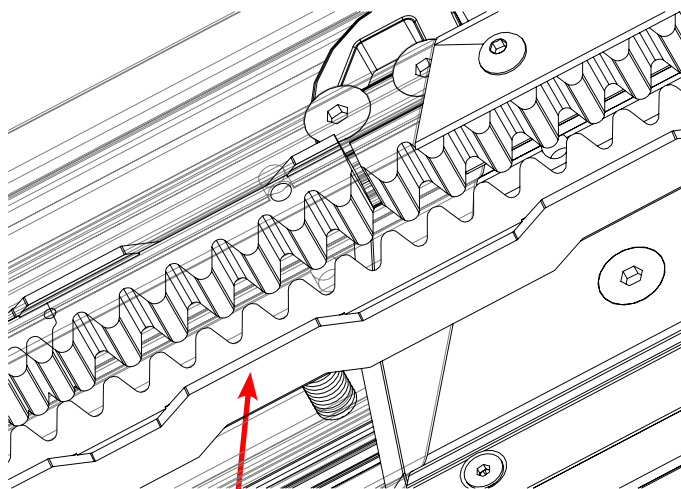
Fig. 3

1. Confirm factory-set gear rack gap (slight space between main rail and folding rail gear racks) is at folding joint (Fig. 1).
2. Loosen folding rail jack screw after jack screw in top endplate has been tightened.
3. Loosen M5 button head screw securing folding rail rack (Fig. 2).
4. Use a piece of spare gear rack to cover joint between main rail and folding rail (Fig. 3). This ensures pitch is maintained for safe travel of drive gear across joint.
5. Slowly tighten folding rail jack screw until loose gear rack piece falls away.
6. Tighten M5 button head screw to secure top piece of short folding rail.


**NOTE: NEVER remove main rail screw from rail! Carriage and gear rack segments will fall.**

## OVERSPEED

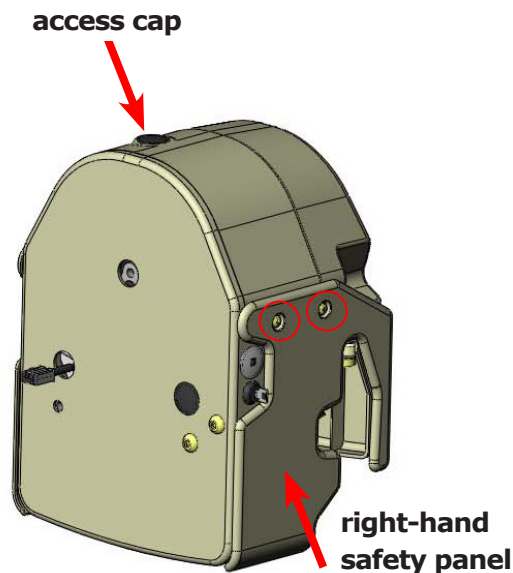**Fig. 1**

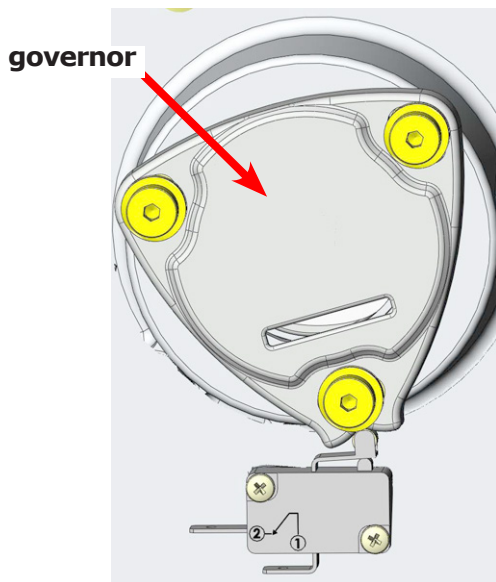**not tripped Fig. 2**

The overspeed is a safety device designed to prevent operation of stairlift if descending travel speed becomes excessive.

### **Reset Overspeed Switch**

When overspeed switch is tripped, wheel will not be in the center of notch (valley).

**You must reset the overspeed switch** to allow unit to operate.

1. Remove **right-hand safety panel**.

Loosen two (2) Phillips head screws (in red circles) in order to slide panel away from carriage.

2. Remove **access cap** from motor side (Fig. 1).
3. Rotate **governor** until it looks like Fig. 2.
4. If governor does not rotate, use a ¼" socket tool or manual back up tool available in kit SRV-3000-002 (available from Bruno), to turn motor shaft extension and move carriage upwards:
  - turn **clockwise** for a left-hand installation
  - turn **counterclockwise** for a right-hand installation
5. Check for damage to drive system.
6. If drive system is not damaged, use rocker switch to drive carriage off the rail for a more complete inspection.

## **BRUNO GOLD LIMITED WARRANTY FOR STAIRLIFT Limited Lifetime Major Components — 2 Year Parts**

Bruno Independent Living Aids, Inc. (Bruno) is confident in its USA-designed and built stairlift units and is pleased to provide the Original Purchaser of a Bruno Stairlift with the following "Industry Leading" warranty.

The quality of parts used, along with the care of Bruno's employees in constructing Bruno's products, allow Bruno to provide the following warranty coverage to the Original Purchaser of the Bruno Stairlift:

- ◇ All Parts, other than Major Components, are/will be free of defects in material and workmanship for a period of two (2) years after the date of installation by a Bruno Authorized Dealer.
- ◇ All Major Components, including the motor, gearbox, drivetrain, and rail system of the Bruno Stairlift, are/will be free of defects in material and workmanship for the lifetime of the Bruno Stairlift and as long as the Original Purchaser owns the Stairlift.
- ◇ All labor, to be provided by a Bruno Authorized Dealer, to address any defect of the Bruno Stairlift is covered for the first thirty (30) days after installation thereof by a Bruno Authorized Dealer.
- ◇ All Bruno warranties begin on the date of installation of the Bruno Stairlift by a Bruno Authorized Dealer, but that date should be no later than 14 months after it was shipped from the factory.

Bruno Authorized Dealers are trained to evaluate your unique situation and then install the best Bruno Stairlift to meet your needs. The trained installation and service technicians employed by a Bruno Authorized Dealer have the appropriate equipment and resources needed to correctly install and maintain the Bruno Stairlift and should be the first contact for any warranty or service needs.

### **WHAT IS NOT WARRANTY COVERAGE:**

- ◇ Damage to units caused by misuse, abuse, accidents, neglect or modifications NOT made by a Bruno Authorized Dealer.
- ◇ Consumable items such as remote batteries
- ◇ All labor by a Bruno Authorized Dealer after the first thirty (30) days since installation.
- ◇ Reasonable wear and tear.

### **ADDITIONAL NOTES:**

- ◇ All Service and Warranty claims must be brought to the attention of a Bruno Authorized Dealer. To ensure warranty coverage, all maintenance and repair work on the Bruno Stairlift, other than maintenance provided by the Original Purchaser as called for in the Operator's Manual, must be performed by a Bruno Authorized Dealer.
- ◇ In no event shall Bruno be responsible for indirect, incidental or consequential damages, whether such damages arise based on claims based on contract, warranty, tort (including negligence, strict liability or product liability).
- ◇ Bruno, at its sole discretion, reserves the right to replace, repair, or, if not commercially practical and/or feasible, elect to refund the Original Purchaser the purchase price of the Bruno Stairlift.
- ◇ This warranty gives the Original Purchaser specific legal rights for which the Original Purchaser may also have other rights that vary from state to state. Bruno specifically does not authorize any person to extend the time or scope of this warranty.

For further information regarding this limited warranty, please contact Bruno using the contact information shown below:

In North America:

Bruno Independent Living Aids, Inc.®  
Attn: Service Department  
1780 Executive Drive  
Oconomowoc, Wisconsin 53066 United States  
Tel. 1-800-882-8183 (toll free)

In the UK and Europe

Bruno  
Unit 5, Millennium Court,  
Clayhill Industrial Park Neston,  
Wirral CH64 3UZ United Kingdom  
Tel. 0151 559 0732

# CE DECLARATION OF CONFORMITY CERTIFICATE

We:

**Bruno Independent Living Aids, Inc.**

Headquartered at:

1780 Executive Drive  
Oconomowoc, WI 53066 USA

An ISO-9001:2008 registered company

Declare that under our sole responsibility, our manufactured product:

MODEL: **SRE-3000**

has been designed and tested to meet (where applicable) the following laws, directives, specifications and standards pertaining to the Machinery Directive or Powered Inclined Stairlifts for those who have physical impairments:

## DIRECTIVES

2004/108/EC Electromagnetic Compatibility Directive  
2006/42/EC Machinery Directive  
2006/95/EC Low Voltage Directive

## STANDARDS

EN 81-40:2008  
EN 60204 Part 1:2006  
EN 60335 Part 1:2002  
EN ISO 12100-2:2003  
EN ISO 13857  
EN 61000  
EN 954 Part 1:1996  
EN ISO 12100-1:2003  
EN 349:2008  
CISPR 11

When installed in accordance to the manufacturer's specifications and operated in accordance with manufacturer's operating instructions.

We hereby appoint:

Bruno Lifts, Ltd.  
Unit 5, Millennium Court  
Clayhill Industrial Park Neston  
Wirral CH64 3UZ  
United Kingdom

To act as Responsible Manufacturer's Representative.

Date of Issue: 02 April 2018

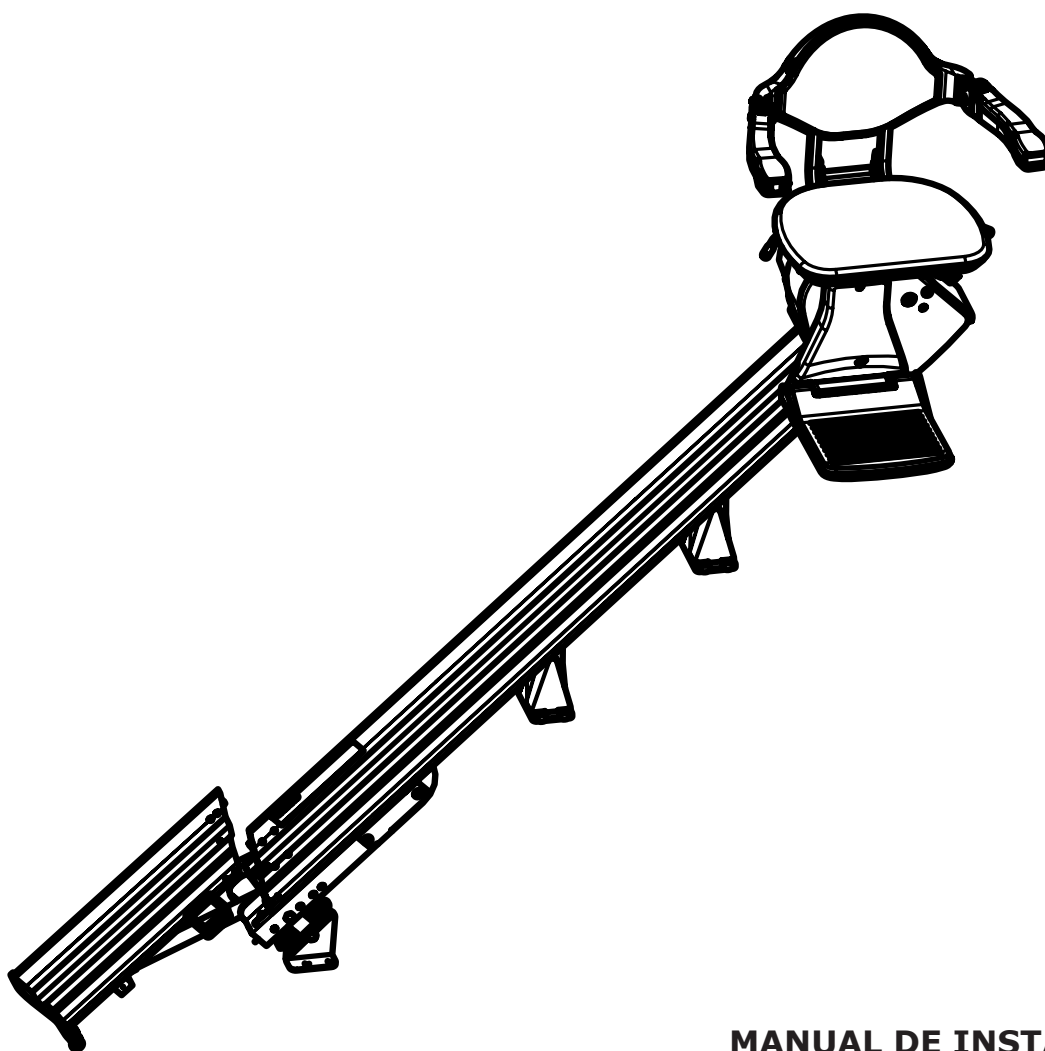
Location: Oconomowoc, WI USA

BY: William W. Belson III  
Director of Engineering

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# 'ELAN' STRAIGHT RAIL STAIRLIFT

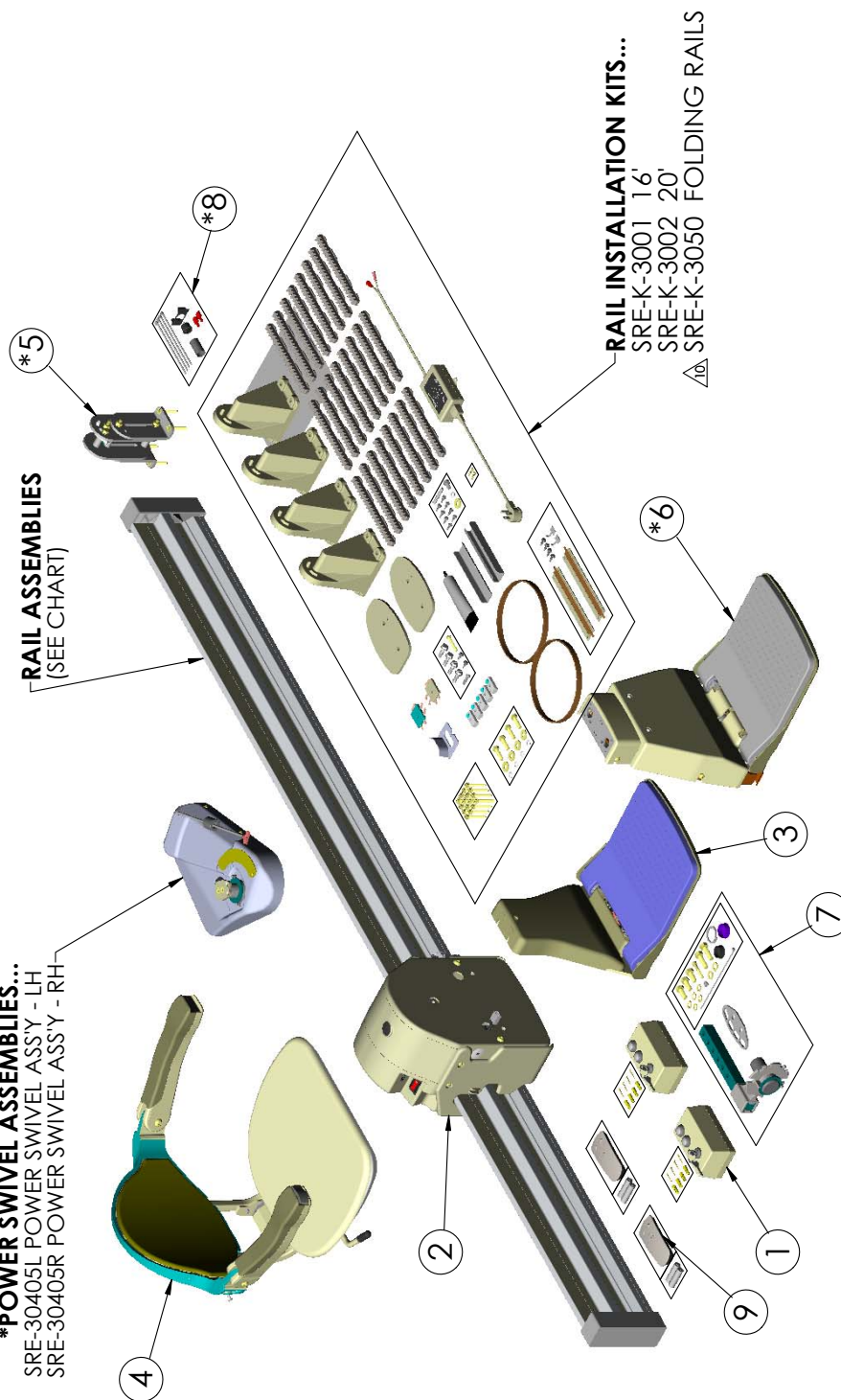
## EXPLODED VIEW

ITEM NO. - QTY.-	PART NO.	DESCRIPTION
1	SRE-00963A	IR TRANSMITTER/BRACKET/HDWR (KEYED)(ROHS)
2	SRE-30099	CARRIAGE ASSEMBLY
3	SRE-30100	FOOTREST ASSEMBLY
4	SRE-30102	SEAT ASSEMBLY
*5	SRE-30150	ADJUSTABLE STAND FOOT ASSEMBLY
*6	SRE-30212	POWER FOOTREST ASSEMBLY
7	SRE-K-3011	FOOTREST/SEAT INSTALLATION KIT
*8	SRE-K-3027	FOOTREST FOLD SWITCH KIT (ARMREST MOUNT)
9	TMR-00004	IR TRANSMITTER ASSEMBLY W/ 2 'AA' BATTERIES

\*OPTIONAL

**\*POWER SWIVEL ASSEMBLIES...**  
 SRE-30405L POWER SWIVEL ASSY - LH  
 SRE-30405R POWER SWIVEL ASSY - RH

RAIL MATRIX	Standard(Anodized)	Folding (Anodized)	Power Folding(Anodized)
16' Left/Right	(2)SRE-30094	N/A	N/A
16' Left	N/A	(1)SRE-30379L & (1)SRE-30094	(1)SRE-30372L & (1)SRE-30094
16' Right	N/A	(1)SRE-30379R & (1)SRE-30094	(1)SRE-30372R & (1)SRE-30094
18' Left	N/A	(1)SRE-30379L & (1)SRE-30095	(1)SRE-30372L & (1)SRE-30095
18' Right	N/A	(1)SRE-30379R & (1)SRE-30095	(1)SRE-30372R & (1)SRE-30095
20' Left/Right	(2)SRE-30095	N/A	N/A

**SRE-3000**  
**SHEET 1 OF 1**  
**REV. 11 (6180)(3/17/17)(DPG)**

**BRUNO**  
 for your independence  
 ©Bruno Independent Living Aids, Inc.

# Innovative solutions

for your indoor and outdoor lifestyle

Indoor Stairlifts

Outdoor Stairlifts

Scooter/Powerchair Lifts

Vertical Platform Lifts

**Bruno Independent Living Aids, Inc.**  
1780 Executive Drive, Oconomowoc, WI 53066  
800-882-8183 • [www.bruno.com](http://www.bruno.com)

© Bruno Independent Living Aids, Inc. 2018