# CLENARE™

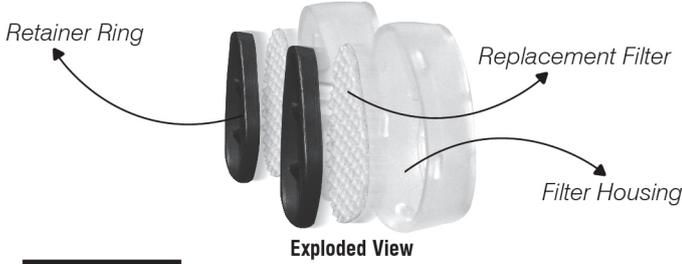
NASAL FILTERS

Website: [www.clenare.in](http://www.clenare.in)

Customer Support: [customercare@clenare.in](mailto:customercare@clenare.in)


Toll Free No. 1800 120 1828

Product Model	Filter Efficiency by Particle Size			Air Resistance* (85 L/MIN)	Disinfects
	10 Microns Pollen, Dust, Dander, Mold	2.5 Microns Germs, PM 2.5 Pollution	Gas Second-hand Smoke, TVOC		
SUPER DEFENSE FILTER	97%	90%	23%	3.0 pa	Yes
MAX DEFENSE FILTER	94%	80%	11%	2.8 pa	Yes

Exploded View

### HOW TO WEAR

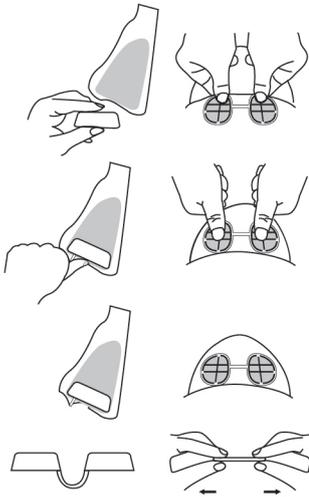
**Positioning** - Hold the frame of the nasal filter with the narrow ends pointing towards your nose tip, the wider end pointing towards your face, and the connector pointing down.

**Insert** - Using your thumbs, insert the filters into your nostrils until the bottom (open edge) of the frame is positioned just inside your nostrils.

You are ready to breathe clean air.

**Note on tightness** - If the connector is too tight for comfort, please remove the nasal filter and gently pull both the ends of the *connector* several times to stretch and loosen it. Avoid using excessive force to prevent breakage.

### HOW TO SET UP WITH REPLACEMENT FILTER PACK

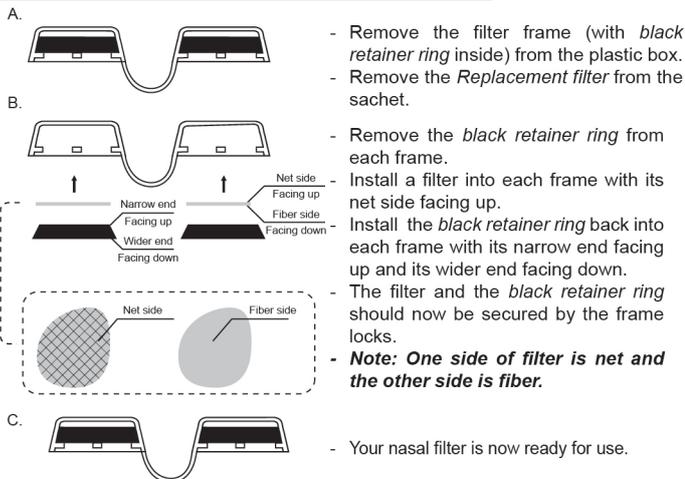- Remove the filter frame (with *black retainer ring* inside) from the plastic box.  
- Remove the *Replacement filter* from the sachet.

- Remove the *black retainer ring* from each frame.

- Install a filter into each frame with its net side facing up.  
- Install the *black retainer ring* back into each frame with its narrow end facing up and its wider end facing down.

- The filter and the *black retainer ring* should now be secured by the frame locks.


- **Note: One side of filter is net and the other side is fiber.**

- Your nasal filter is now ready for use.

### Important Points:-

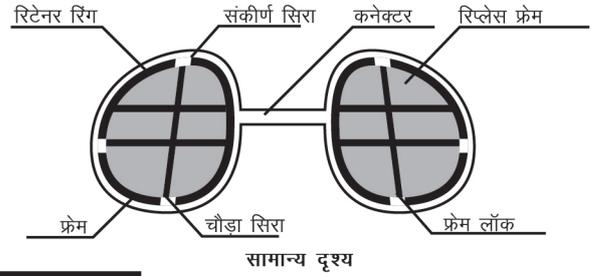- Service life** - The outer frame can be used long-term, and the removable filters should be replaced when they start turning black in color or in 15 days whichever is earlier.
- Clean** - Wash the frame every day with clean water or 70% / 30% water/alcohol solution for optimal results.
- Washing the disposable filter replacements is not recommended, as this may reduce the filter efficiency.
- The filters captures most of the germs but does not kill the bacteria directly. If disinfection is desired, please replace filters every day for maximum protection.

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