Recover Knee Brace

FITTING INSTRUCTIONS

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
MISE EN PLACE

INSTRUCCIONES DE AJUSTE

Breg, Inc.
2885 Loker Ave. East
Carlsbad, CA 92010 U.S.A.
P: 800-321-0607
F: 800-329-2734
www.breg.com
©2013 Breg, Inc.

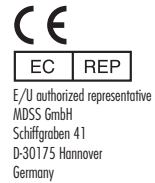
AW 1.00532 REV A 0313


E/U authorized representative
MDSS GmbH
Schiffgraben 41
D-30175 Hannover
Germany

Recover Knee Brace

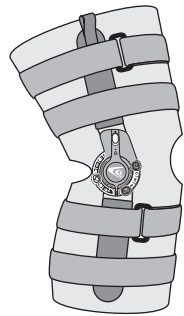Airmesh • Luftdurchlässiges Netzmaterial •
Rete traspirante • Mailles aérées • Tejido de malla


Long
Lang
Lungo
Longue
Larga

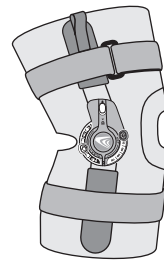
Short
Kurz
Corto
Courte
Corta

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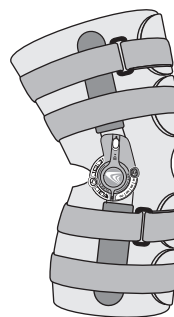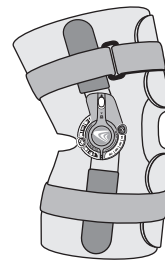Short
Kurz
Corto
Courte
Corta

WARNINGS

WARNING: CAREFULLY READ FITTING INSTRUCTIONS AND WARNINGS PRIOR TO USE. TO ENSURE PROPER PERFORMANCE OF THE BRACE, FOLLOW ALL INSTRUCTIONS.

WARNING: DO NOT REMOVE THE RECOVER KNEE BRACE UNLESS INSTRUCTED BY YOUR MEDICAL TREATMENT PROFESSIONAL. DO NOT CHANGE RANGE OF MOTION HINGE SETTINGS WITHOUT SUPERVISION BY A MEDICAL PROFESSIONAL.

WARNING: IF YOU EXPERIENCE INCREASED PAIN, SWELLING, SKIN IRRITATION, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.

WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM. CONSULT YOUR LICENSED HEALTH CARE PROFESSIONAL REGARDING SAFE AND APPROPRIATE ACTIVITY LEVEL WHILE WEARING THIS DEVICE.

CAUTION: FOLLOW REMOVAL AND CLEANING PROCESS DESCRIBED IN INSTRUCTIONS.

CAUTION: FOR SINGLE PATIENT USE ONLY.

CAUTION: FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PRACTITIONER.

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
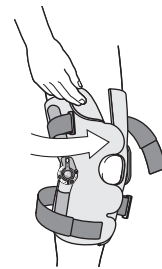
Application Instructions for Knee Braces:
The wrap-on brace is applied with the closures in the front of the leg and the kneecap cutout centered over the knee. Secure the closures firmly.

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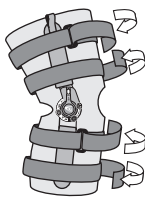The straps should be tightened firmly, starting with the two straps closest to the knee.

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Tips on Use and Care of Your Knee Brace:

After exposure to salt water or dirt, thoroughly rinse the brace with fresh water and allow to air dry.

A dry lubricant, such as teflon spray, may be used on the hinge occasionally. Do not use wet lubricants.

The hinge assemblies may be removed to allow easier washing of the sleeve and straps.

Medical Professional - The side bars may be bent to accommodate individual legs. Bend in equal amounts above and below the hinge.

Periodically inspect the sleeve, straps and hinges for excessive wear. Be sure all screws are tight and stops are in place.

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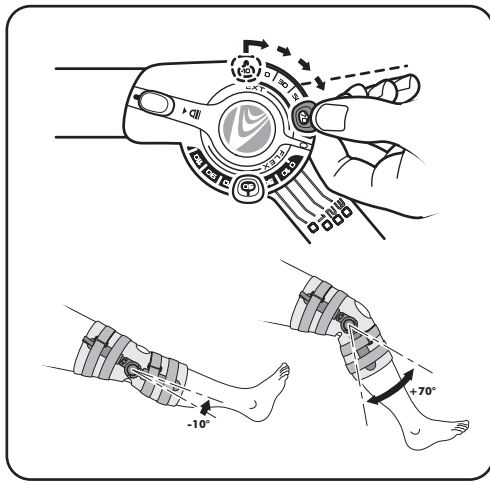
ROM Hinge Adjustments

Extension Limit settings may be selected between 10° of hyperextension and 70° by pulling the tab out and sliding to desired position.

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
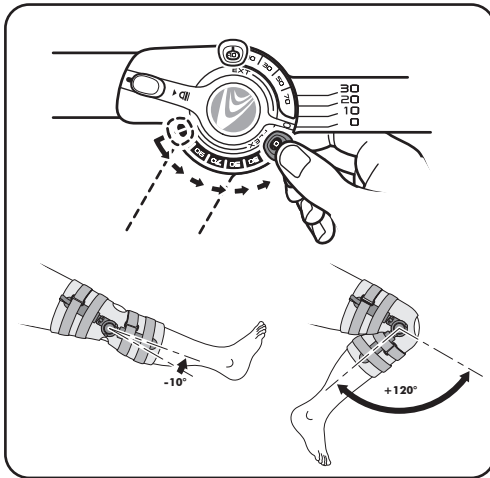
Flexion Limit settings may be selected between 10° of hyperextension and 120°.

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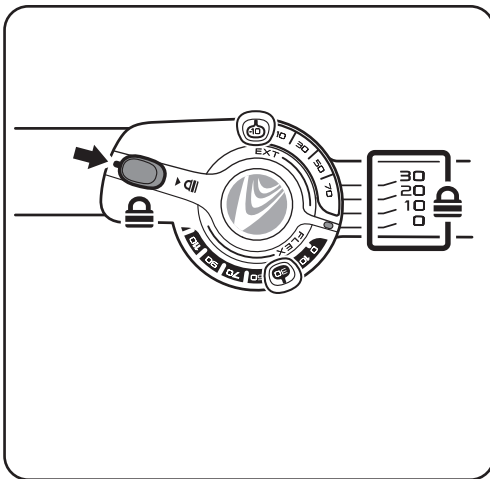The hinge may be locked by sliding the quick lock button into the locked position at any one of 5 positions : 10° of hyperextension, 0° (Neutral), 10°, 20°, 30°.

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NOTE: Due to soft tissue compression, we recommend the use of extension stops at least 10° greater than the desired limit, (e.g., if you desire a 10° extension limit, use the 20° extension stop).

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