

HOOF-LIFE

HOOF SUPPORT SYSTEM

"A Sound Solution"

ORTHO-PAK PUTTY

TWO FORMULAS – SOFT (BLUE COLOR) AND FIRM (PINK COLOR)

2-COMPONENT HOOF SUPPORT SYSTEM

Net Weight Each Component: 16 oz. (.455 kg.)

Distributed by VICTORY
1200 Rosedale Avenue
Baltimore, MD 21237
410.391.6600
www.victoryracingplate.com

DIRECTIONS: Read label DIRECTIONS and TIPS FOR USE before using this product. Trim hoof as usual making sure all dirt and disease are removed from bottom of hoof. Determine amount of material needed to fill sole area. Take ½ of desired amount from each container, making certain that two different colors are to be mixed. Mix materials thoroughly by hand until a uniform color is reached. Press mixture firmly into hoof cavity starting with deepest area to shallow areas. Allow horse to stand firmly on a flat surface such as a flat plastic pad until ORTHO-PAK PUTTY firmly sets (5 to 10 minutes) depending on ambient temperature.

TIPS FOR USE: Temperature affects time required for material to set. Be sure material is kept at a minimum of 55 degrees F. (13 degrees C.) ORTHO-PAK PUTTY sets faster at higher temperatures. Hoof must be clean and dry before use. Use a TRUE-FLITE Cushion Frog or Cushion Heel pad to provide maximum support. TO AVOID CROSS-CONTAMINATION AND PREMATURE SET-UP of container contents, be certain to use different scoops or different hands to take colored material from each container AND be certain that the same container lids (color coded) are replaced on their original containers.

SOME SUGGESTED USES: Redistribute weight-bearing load of hoof capsule; Stimulate blood flow; Support sole and frog; Reduce sole bruising; Treatment of laminitis or founder.