



SIMPLY RELAX

breo. iSee M

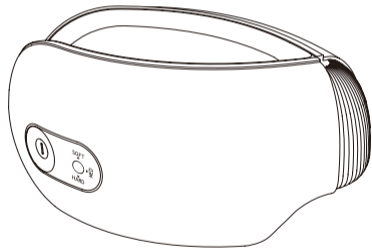

breo.iSee M

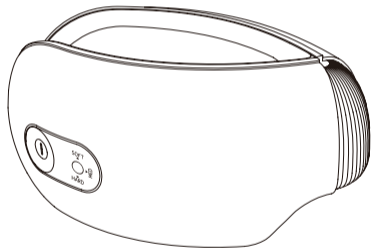

MENU

Attention	3
Indications for use	5
Product Structure	5
What's Included	6
How to Use	7
How to Connect Breo App	7
How to Wear	17
Charging	18
Caution	19
Symbol description	20
Cleaning and Maintenance	21
Troubleshooting	22
Guarantee	23
Product parameters	23
FCC Regulatory Compliance	24
Disposal	25
Customer Service	25



iSee M User Manual

Thank you for your purchasing Breo iSee M Eye Massager!
Hope it helps you relax your eye muscle.
Please read this user's manual carefully prior to use and,
keep it properly for future reference.

Attention

This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
Children shall not play with the appliance.
Cleaning and user maintenance shall not be made by children without supervision.
Please read and comprehend all the contents of this manual and operate the product according to the instructions.

1. DO NOT use if you have had an eye operation, or if you have any eye diseases such as glaucoma, cataract, and amotio retinae.
2. DO NOT clean the product with water; clean it with cotton soaked in alcohol.
3. The appliance has a heated surface. Persons insensitive to heat must be careful when using the appliance.
4. Children, Juveniles, disabled people, and the elderly should only use this product when properly supervised.
5. The product is intended for personal use; it is not intended to replace medical care and medical devices.
6. DO NOT use if you have any swelling in the eye area.
7. DO NOT disassemble the product, DO NOT impact or squeeze its interior lithium battery or expose it to fire.
8. DO NOT use if the battery has been immersed in water.
9. DO NOT use in high temperatures or humid environments. Avoid direct exposure to the sun.
10. Consult your doctor if you have sensitive skin, allergic dermatitis, or orthopedic biomaterials.
11. Remove contact lenses, hairpins, and hair accessories before using.
12. Start the eye massager after putting it on in order to avoid damage to the air bag due to over pressure.
13. It must only be supplied at safety extra low voltage corresponding to the marking on the appliance.
14. You must have your eyes closed while massaging.
15. DO NOT use sharp objects to scratch this product
16. This appliance contains batteries that are non-replaceable.



Indications for use:

Breo iSee M Eye Massager is intended to relax eye muscle and relieve minor eyes muscle aches and pains.

Operation principle:


Breo iSee M Eye Massager is used heat and pressure function help relax eye muscle and relieve minor eyes muscle aches and pains.

Product Structure

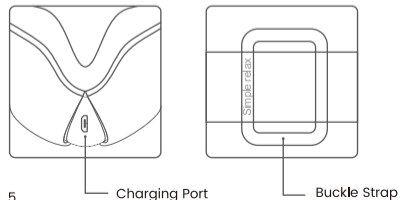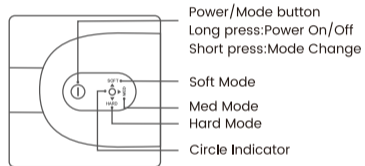

What's Included

- 1*breo iSee M Eye Massager
- 1*USB charging Cable
- 1*User Manual
- 1*Storage Bag

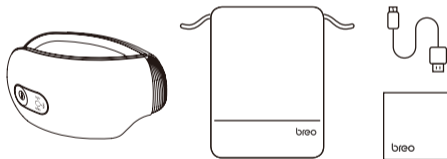

How to Use

1. Power On: Long press "Power/Mode button" for 3 seconds to power on the massager;
2. Mode Selection: Short press "Power/Mode button" to select between Soft Mode , Med Mode, Hard Mode.
Soft Mode: Heat Only
Med Mode: Middle intensity eye massage with heat
Hard Mode: High intensity eye massage with heat
3. Power Off: Long press "Power/Mode button" for 3 seconds to power off the massager.

How to Connect Breo App

1. Download the "breo" App


Search "breo" from Apple Store / Google Play to download the App.



* The "breo" App may appear slightly different due to continuous updates.


2. Add Device

- (1) Long press "Power/Mode button" for 3 seconds to turn on the product. The massager must be turned on to connect to the APP.
- (2) Click the 'settings', turn on the Bluetooth of the mobile phone and search for the product model (such as iSeeM_OOCE7D) to connect.
- (3) Open the breo APP, log in App through your phone number. No need to log in repeatedly when entering the APP next time.

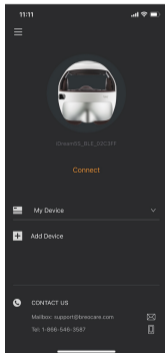

- (4) Select "Add Device" after logging in, click product model (such as iSeeM_OOCE7D) and connect the device.

3.Preset Mode

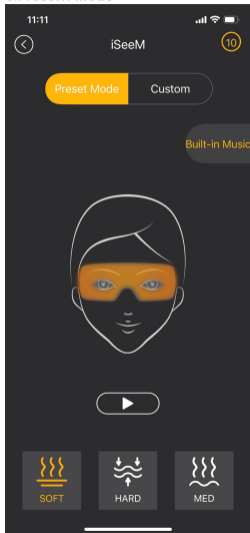

3.Preset Mode

Enter breo APP user interface and tap on "Preset Mode":


(1) There are 3 preset modes for selection per your preference.

Soft Mode: Heat Only

Med Mode: Middle intensity eye massage with heat

Hard Mode: High intensity eye massage with heat

Active Mode: Suitable for morning refreshment or those who like a strong power.

(2) Select the mode you like and click  to start.


(3) Click  to pause the preset mode.

4.Custom Mode

Custom Mode

Enter the breo APP user interface and tap on "Custom" Mode.

All functions are off by default, and you need to set each function before use.

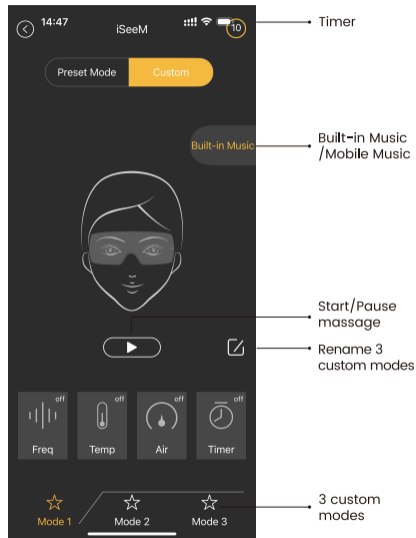

Timer


Built-in Music / Mobile Music

Start/Pause message


Rename 3 custom modes

3 custom modes


(1) Tap on "Freq" to switch frequency between high, middle and low per your preference.

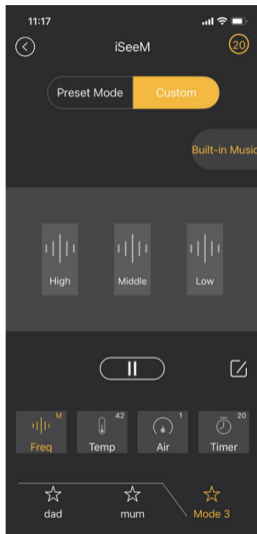


(2) Tap on "Temp" to switch temperature from 42°C to 47°C per your preference.

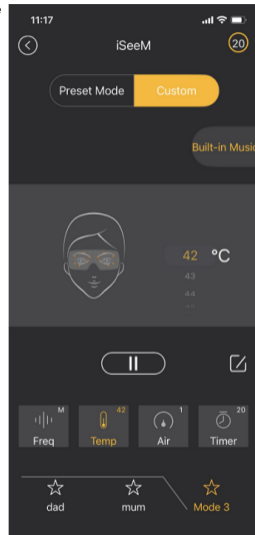



(3) Tap on "Pressure" to switch pressure from min to max per your preference.

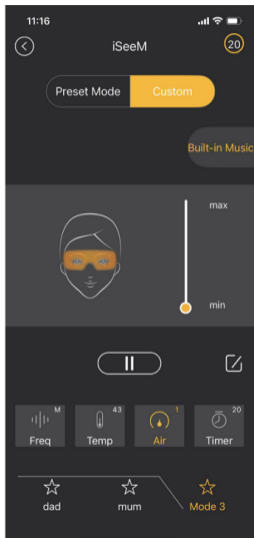


(4) Tap on "Timer" to switch time from 3 to 20mins per your preference.

(5) When you finish the settings , tap on  to start the massage.

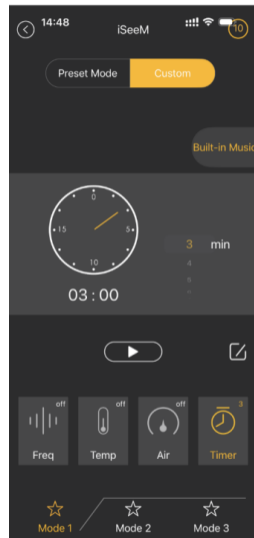


(6) Tap on  to rename the custom mode names. Edit the name and click "OK" to save it.

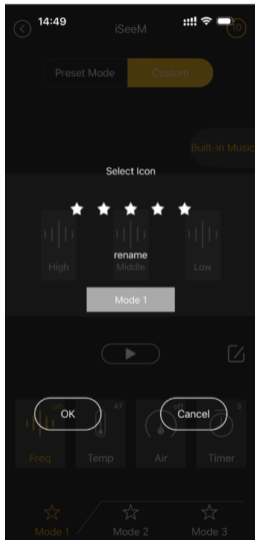


5. Built-in Music / Phone Music

(1) Click Built-in Music in the upper right corner.

Click  to play built-in music .

Click  to switch to the next built-in music song.


Here are 3 built-in music songs for selection.



Click  to pause music.

(2) Mobile Music:


① If you are in the built-in music state, click "Built-in Music" to switch to Mobile Music.

② When you enter mobile music state, it will automatically jump to your mobile phone music APP. Select a song and play it. If it do not jump to your mobile phone APP automatically, please go to your mobile phone music APP and select a song to play it.

③ Click  to switch to the next mobile music song on the Breo App page.

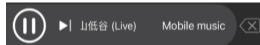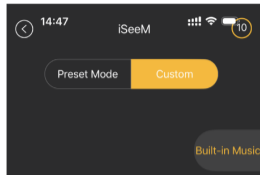

How to Wear

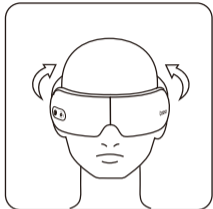

Align the cloth cover to your eyes and adjust the tightness of elastic band to the most comfortable position. Hold the power button for 3 seconds to turn on the eye massager.


Charging

1. Plug the USB charging cable into the socket of the charger.
2. Connect the charging cable into the charging port of the device.
3. Plug the charger into the power socket
4. iSee M eye massager operation and Indicators are as following:







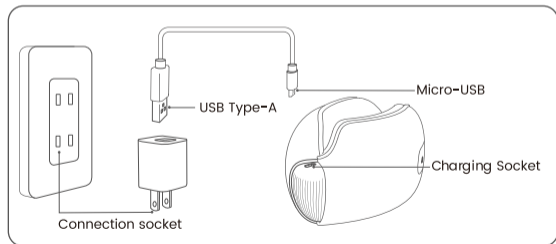



Operation	Indicator Light
Soft/Med/hard Mode Selected	Mode white light on
Charging	Circle Indicator white light flash
Charged full	Circle Indicator white light always on
Bluetooth not connected	Circle Indicator blue light flash
Bluetooth connected/APP controlled	All 3 Mode Indicator white lights always on
Low battery	Circle Indicator white light flash

Caution:

1. It needs 2-3 hours for charging.
2. During the charging process, the product cannot be turned on.
3. If the product is charged while it is working, the product will automatically stop working.
4. When the product is not used for a long time, the battery will discharge slowly. In order to avoid battery damage due to low voltage for a long time, please charge the device for every three months.
5. Don't try to replace the rechargeable battery by yourself. Please contact the manufacturer for replacement. Self-disassembly and replacement the battery may cause damage to the main unit and battery.



Symbol description

	Keep Dry		Applied part of type BF
	Refer to instruction manual		Serial number
	Manufacturer information: The manufacturer name and address		Disposal in accordance with Directive 2002/96/EC (WEEE)
	Medical device		The device complies with part 15 of the FCC rules
	This device has not been tested for use in an MR environment and should not be used exposed to MR environments while patients are wearing the device. Keep it outside the MRI scanner room.		
	The device complies with compliance with RoHS Directive 2011/65/EU and its amendment directives 2015/863/EU		

Cleaning and Maintenance

Cleaning

1. Before cleaning, turn off the power and unplug the power adapter from the socket. Use a soft cloth with alcohol to clean the surface of the product.
2. Do not use corrosive cleaners such as benzene or diluents to clean the product, otherwise it will cause discoloration, surface aging, structural damage and other conditions.

Storage

1. If you do not use it for a long time, please cover it with a storage bag, and place the product in a dry and ventilated room to avoid sun exposure.
2. This product contains batteries, do not approach high temperature or put in the fire.

Troubleshooting

Problem	Possible causes	Take actions
Eye massager turn off automatically	<ol style="list-style-type: none">1. The massage time was finished.2. The Battery power is low.	<ol style="list-style-type: none">1.You can long press Power button 3s to turn on the eye massager for another massage.2.Please charge the eye massager.
The intensity is so strong	<ol style="list-style-type: none">1. Elastic band is too tight.2.May choose the hard intensity.	<ol style="list-style-type: none">1.Please loosen the elastic band to the comfortable position.2.Please choose the soft intensity on device or on APP.
The intensity is so weak	<ol style="list-style-type: none">1. Elastic band is too loosen.2.May choose the soft intensity.	<ol style="list-style-type: none">1.Please tighten the elastic band to the comfortable position.2.Please choose the hard intensity on device or on APP.
The device can not be charged	<ol style="list-style-type: none">1.Please check power supply at home is normal.2.The adapter may be overvoltage 5V, so it cannot charge the eye massager.3.USB cable or adaptor may be damaged.	<ol style="list-style-type: none">1.Please Please make sure that the power supply at home is normal .2.Please use the adaptor which support 5V/1A to charge the eye massager.3.Please use another adaptor or USB cable to charge it.

Guarantee

This product comes with 1-year warranty from the date of purchase. Warranty will be voided due to any of the following:
Damaged due to the improper usage or storage.
Damaged by attempting to repair or disassemble the device.
Period of warranty exceeded.

Product parameters

Product Name: Breo iSee M Eye Massager
Product Model:iSee M
Charging Input:5V=1A
Rated Power:5W
Automatic power off:10 minutes
Power Supply:3.7V=1150mAh rechargeable lithium battery
Charging Time: 2-3 hrs
Working time : 1.5hrs (+-0.5h)
Net Weight: 300g
Charging Port: micro USB
Dimension: L210*W75*H110mm

FCC Regulatory Compliance

FCC ID: PXU-ISEEM

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

FCC Warning Statement

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment. This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Disposal



Notice:

At the end of the product life cycle, do not throw this product into the normal household garbage, but bring it to a collection point for the recycling of electronic equipment and battery.

Waste Electrical and Electronic Equipment may have potentially harmful effects on the environment.

Improper disposal may lead to the accumulation of harmful toxins in air, water and soil, which is harmful to human health. you have obligation to dispose of device correctly. Consult your municipal authority for information about disposal.

Customer Service

Please feel free to contact us if you have any questions or concerns.

Breo Customer Service Team guarantees a quick response and hassle-free solutions to any issue you may have within business hours.

Email: support@breocare.com

Web: us.breo.com

Tel: 1-866-546-3587

Working time :

PST 6:00am--11:00pm (Monday to Friday)

PST 7:00am--3:00pm (Saturday to Sunday)

Made in China



Shenzhen Breo Technology Co.,Ltd
19/F, Hisense Southern Building, No.1777 Chuangye Road, Nanshan District,
Shenzhen, Guangdong, CN, 518056

Version:2.0 @202306