
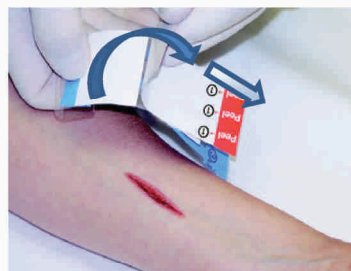
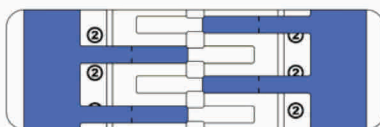
Instructions for Use

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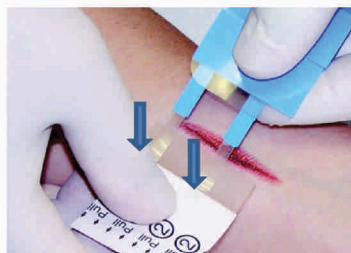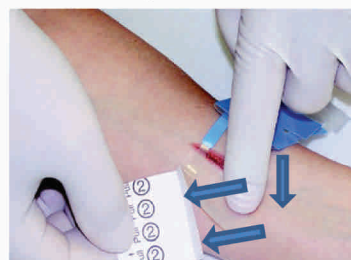
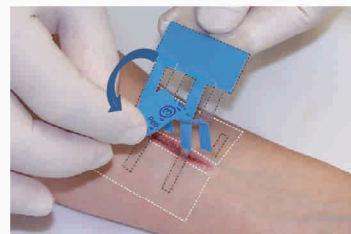
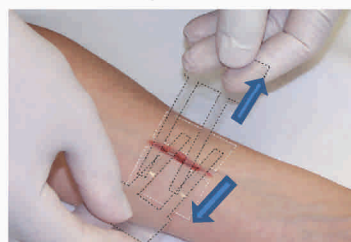
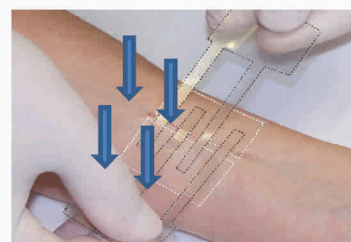
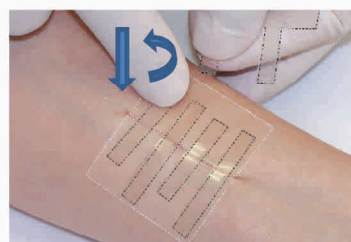
English
ClozeX® Skin Closures
Description:
ClozeX® closures are non-invasive and made of polymeric films coated with non-latex, pressure sensitive skin adhesive for single use.
Intended Use:
ClozeX® closures are indicated for use as skin closure devices to treat lacerations or surgical incisions. The closure devices can also be used as an adjunct to suture closures or as reinforcement for wounds after early suture or staple removal.
Contraindications:
ClozeX® closures are contraindicated where adhesion cannot be obtained. Potential causes of inadequate adhesion are presence of wound fluids, skin oils, moisture, or hair. Surgical skin closures are contraindicated in areas of high tension, and for use on infected wounds.
Warnings:
The development of postoperative edema may cause skin shearing, skin blistering or loss of tape adhesion to occur. Application of any surgical tape or adhesive skin closures may result in skin stripping upon removal. As with all adhesive products applied to the skin, a small percentage of individuals may experience hypopigmentation or hyperpigmentation following removal.
Precautions:
A. The skin should be clean, dry, and free of skin oils for good adhesion.
B. Do not apply skin closures under excessive tension. Skin shearing, skin blistering or loss of adhesion may result if excessive tension is applied.
Directions for use: (Instructions & videos available at: www.clozex.com)
A. Clean the skin at least 5-7 cm around the wound, using a sponge moistened with either isopropyl alcohol or normal saline solution taking care to remove wound fluids and excessive skin oils.
B. Peel open the pouch using sterile technique and remove the skin closure device.
C. ClozeX® closures have color-coded and numbered liners in familiar sequences to facilitate ease of use. The red, white and blue and 1-2-3 format simplifies the liner removal sequences.
Step 1. Remove the RED liner, also marked 1, from either RED edge to expose the adhesive on both clear pads of the device. Turn the closure device over with the adhesive pads facing the skin. Pull skin closure device apart as needed for application.
Step 2. Holding the WHITE liners, marked 2, apply clear pads to skin 1-2 mm from wound edge and gently press to secure each half one at a time. Apply pads to dry skin.
Step 3. Gently, remove WHITE liners marked 2 on both sides and gently press the pads to skin to secure them.
Step 4. Hold the clear pulling tabs and the BLUE liners marked 3, perpendicular to the wound being careful not to pull up on pulling tabs. Holding the clear pulling tabs up, slowly peel down blue liners to expose adhesive on the filament straps. Before lowering into place, Be Sure Tops of Clear Pads are Free of Wound Fluids by drying them with gauze or lap sponge.
Step 5. and 6. Draw skin closure device closed using pulling tabs in opposite directions and simultaneously align wound edges to achieve optimum approximation. Check wound edges to ensure planar approximation. Gently lower straps to adhere them to the tops of the closure device pads. MAKE SURE TO SMOOTH APPLIED PAD AND FILAMENTS FIRMLY, to maximize adhesion.
Step 7. and 8. GENTLY TWIST OFF each removable strap while placing finger at perforations to hold the strap to the pad.
D. Cover the skin closure device if desired to manage wound fluids and protect against friction or patient access. Be sure that no cover dressing adhesive, if applicable, is applied directly on the skin closure device.
Instructions for Care: (Instructions available at: www.clozex.com)
As necessary the wound area can be rinsed and dried carefully. Patient should not: swim, soak the skin closure device in a bath or unnecessarily rub the closure device as this may weaken the adhesives. Provide patient, caregiver or parent with the appropriate after care instructions. To remove closure device, begin at one end of incision and gently peel closure device off toward the other end of the incision. See the ClozeX® Skin Closure Patient Care Information for complete details.
Recommended Storage:
I. Store at room temperature.
Explanation of Symbols:
 This product and package do not contain natural rubber latex.
 Sterility guaranteed unless pouch is damaged or open.

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