# Breastmilk Storage Bags

## Directions for Use

### TO FILL:

- 1 Open bag by tearing away safety seal and pulling zipper apart.
- Use ballpoint pen or permanent marker to fill in the name, date, time and volume. Avoid using felt tip markers.


- 2 Pour milk into bag.
- Caution:** Do not fill past 6oz. Breastmilk will expand when frozen.

- 3 Remove air before sealing.

- 4 Seal the zippers completely.
- Caution:** Do not stand bag upright on its own until zipper is sealed.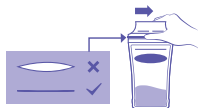

### TO PUMP INTO BAG (2 WAYS):

- 1 **PUMP INTO BAG USING LANSINOH® PUMP.**

Compatible with Lansinoh® Signature Pro®, Smartpump™, Smartpump 2.0, and Lansinoh® Manual Breast Pump.

figure 1B

figure 1C

figure 1F

figure 1G

- A)** Open bag by tearing away safety seal and pulling zipper apart.
- B)** Tilt the pump kit and slide it into the bag. (see figure 1B)
- C)** Make sure the bottom of the pump kit set is level with the white “pump into bag” line on the bag. (see figure 1C)
- D)** Tug gently on bottom of bag to ensure bag is secure.
- E)** Repeat as needed for a double pump, then start pumping.
- F)** To remove, securely hold the bag, gently tilt the pump kit and slide off bag. Do not pull the bag straight down. (see figure 1F)
- G)** Remove air before sealing bag. (see figure 1G)

## ② PUMP INTO BAG USING MOST STANDARD SIZE PUMPS WITH LANSINOH® PUMP ADAPTER.

Adapter sold separately on Lansinoh.com and Amazon.com. Adaptor compatible with: Evenflo Advanced Double Electric Breast Pump, Medela Pump in Style Advanced, Medela Freestyle Breast Pump, Medela Swing Breast Pump, Medela Harmony Breast Pump, Evenflo Deluxe Advanced Breast Pump, Ameda Purely Yours Ultra Breast Pump, Purely Yours Breast Pump, Ameda Elite Breast Pump, Ameda Platinum Breast Pump, Ameda Manual Breast Pump.

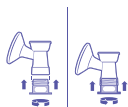
figure 2B

figure 2D

figure 2E

figure 2I

figure 2J

### A) Clean the adapter

Before first use

- Wash and dry hands with soap and water before handling the adapter.
- Carefully place adapter in boiling water to sanitize. Boil for 5 minutes.
- Place on a clean towel to dry. Dry completely before use.

After use

- Disassemble adapter from pump.
- Hand wash the adapter using warm soapy water. Top rack dishwasher safe.
- Place on a clean towel to dry. Dry completely before use.

**B)** Screw pump adapter into breast pump flange body, making sure it is secure. (see figure 2B)

**C)** Open bag by tearing away safety seal and pulling zipper apart.

**D)** Tilt the adapter and slide it into the bag. (see figure 2D)

**E)** Make sure the bottom of the adapter is level with the white “pump into bag” line on the bag. (see figure 2E)

**F)** Tug gently on bottom of bag to ensure bag is secure.

**G)** Repeat as needed for a double pump, then start pumping.

**H)** To remove, do not pull the bag straight down.

**I)** Securely hold the bag. Gently tilt adapter as you remove. (see figure 2I)

**J)** Remove air before sealing bag. (see figure 2J)

### LANSINOH® SIMPLE WISHES® HANDS-FREE PUMPING BRA

Use Lansinoh's pumping bra to free up your hands while pumping. This will make it easier to remove your milk storage bags when you're done!

## TO STORE:

### BREASTMILK STORAGE GUIDELINES

Source: Centers for Disease Control and Prevention. (2020)


TYPE OF BREASTMILK	STORAGE LOCATIONS AND TEMPERATURES		
	ROOM TEMPERATURE 77°F (25°C) or colder	REFRIGERATOR 40°F (4°C)	FREEZER 0°F (-18°C) or colder
Freshly expressed or pumped	Up to 4 hours	Up to 4 days	Within 6 months is best. Up to 12 months is acceptable
Thawed, previously frozen	1-2 hours	Up to 1 day (24 hours)	NEVER refreeze breastmilk after it has been thawed
Leftover from a feeding (baby did not finish the bottle)	Use within 2 hours after the baby is finished feeding		

These guidelines are for healthy full-term babies only and may vary for premature or sick babies. Check with your health care provider.

- 1 Remove air before sealing.

- 2 Lay bag flat to freeze. To prevent sticking, separate bags with paper towels until frozen.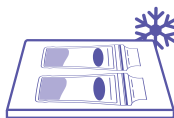

## TO THAW:

- 1 Place sealed bag in warm tap water or in a clean, empty container.

**Caution:** Do not put in boiling water or on stovetop.

**Caution:** Do not microwave.

- 2 It is normal for milk to separate. Massage bag to recombine.
- Transfer milk into desired feeding container.

- 3 Do not reuse bag. Dispose of used plastic bag safely.

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