

Moxxly Flow Spectra™ Adapter Kit

With a little assembly, this kit will make your Moxxly Flow compatible with Spectra™ S1 Plus, S2 Plus, and 9 Plus.

Spectra™ Adapter Kit:

1. Two Moxxly Spectra™ adapters
2. Two 3" silicone tubing

Not Included:

1. Spectra™ S1 Plus, S2 Plus, or 9 Plus breast pump motor
2. Spectra™ Backflow Protector

Assemble the Spectra™ Adapter Kit

- 1.** Gather your Moxxly Flow Kit and your Spectra™ pump and Backflow Protector with this adapter kit.
- 2.** Disconnect the Spectra™ tubing from the Backflow Protector. Keep the Backflow Protector together.
- 3.** Insert Moxxly's Spectra™ adapter into the open end of the Moxxly Flow tubing (found in the main Moxxly Flow Kit).
- 4.** Connect Moxxly's Spectra™ adapter to the wider end of the Spectra™ Backflow Protector by pushing it in all the way.
- 5.** Connect the 3" silicone tubing (found in this adapter kit) to the narrower end of the Spectra™ Backflow Protector by pushing it in all the way.
- 6.** Repeat steps 2 - 5 for the other tubing. Once both sides are assembled, you can leave them together.

Connect Flow to Your Spectra™ Pump

- 1.** Connect the 3" silicone tubing to your Spectra™ pump by pushing it onto the tube port all the way.
- 2.** Slide the Moxxly Flow tube connector into the tubing hole in the top of the neck of the Flow flange and push it into place.

Example Setup



Additional Tips

- Double check that your Spectra™ Backflow Protector has a good seal before starting your pump session.
- During your first few sessions with the Moxxly Flow and Spectra™ pump, pay attention to your suction levels and adjust it for comfort.
- Some condensation in the tubing is normal. If you notice condensation, run your pump motor for a few minutes after your pump session with just the tubing still attached.
- Refer to the Moxxly Flow User Manual for complete instructions.