



## Sample Urgent Care Room Layout

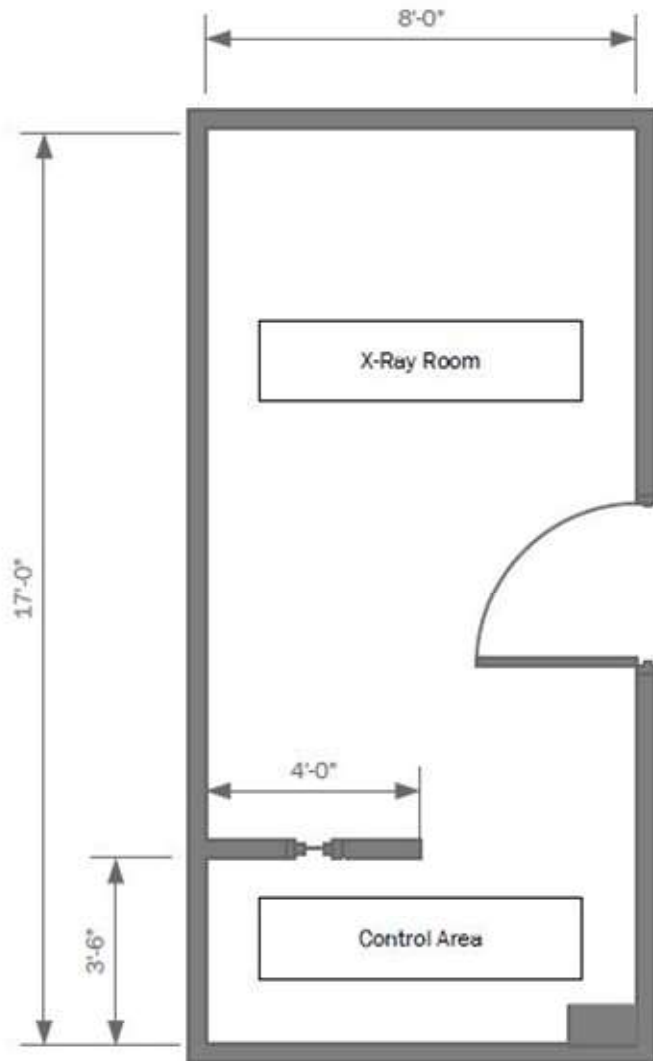

Maven Imaging provides room drawing as part of the purchase of any X-ray room installation. Complete room layout with all the electrical requirements and information to pass to a physicist for the shielding report.



## Sample Urgent Care Room Layout

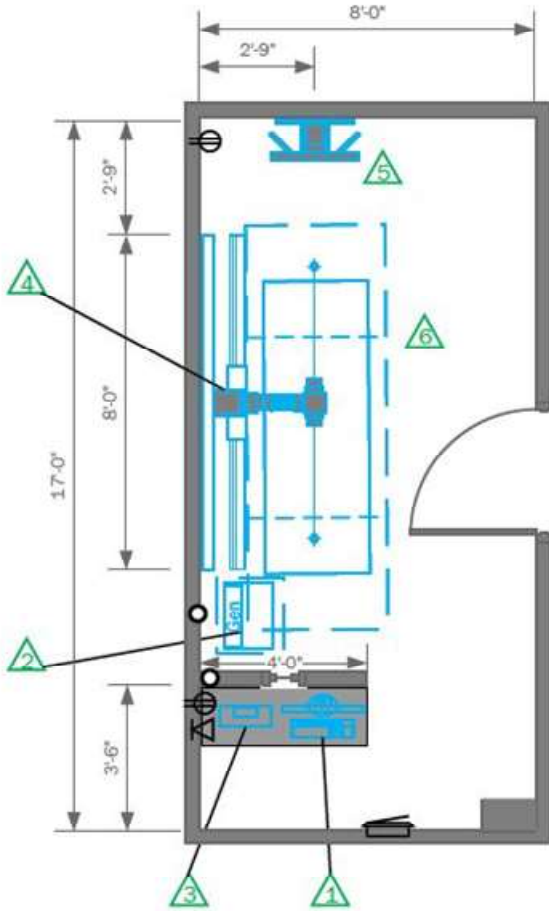

1	NEW DR computer workstation DR System. Installed on customer provided countertop/shelf/cabinet.
2	Summit Spectra 30KW HF X-ray generator. Line power needed for this generator is Single Phase (2 hots and 1 ground) 208-240 VAC, 125 AMP circuit. More info on Electrical page. <b>Customer/Electrical contractor must verify available power in building. Call Josh with any questions or concerns.</b>
3	Operator Console. Exposure button.
4	S105 Floor to Wall mounted Tubestand. Focal center =32.50". Minimum Mounting Height is for top rail is 91.25".
5	S109 wallstand with 17"x17" grid cabinet.
6	Four-Way Float-Top X-Ray Table



We understand the importance of the costs associated with the room build out and we work with you to utilize and maximize every inch of the space.