
Ergotopia® Akupressurmatte + Kissen

Ergotopia® Acupressure Mat + Cushion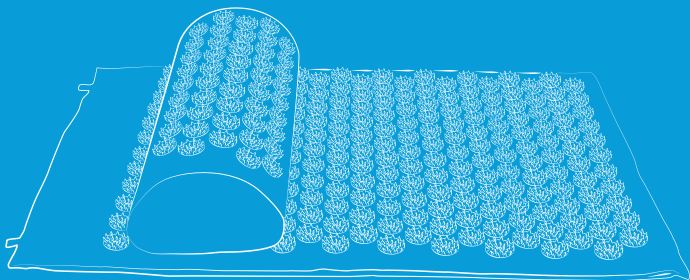

DE Gebrauchsanweisung
EN Instructions for use
FR Mode d'emploi
ES Instrucciones de uso
IT Istruzioni per l'uso

Deine Ergotopia®

Akupressurmatte

DE

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Matte: 68 cm x 42 cm, 500 g

Kissen: 35 cm x 16 cm x 10 cm, 200 g

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Recommended user weight: 40-120 kg

Intended purpose

- Stimulation of blood circulation in the application area
- Relief of muscle tensions
- Stress reduction through relaxation

Fields of application

- Relaxation
- Stress reduction
- Stimulation of blood circulation
- Beneficial in case of tenseness or sore muscles
- We recommend the use of the mat for the back area only
- If the usage of the pillow feels to be too intense, use a normal pillow instead.
- If the feeling on the mat is too intense in direct contact with the skin, use a thin fabric between your skin and the mat.

Indications

- Muscle tensions
- Sore muscles
- Stress, sleeping problems

Contraindications

- Blood coagulation disorders (bruises may occur)
- Open wounds in the area of application
- Heart problems (due to the increased blood circulation the application may only be carried out in consultation with the attending physician).
- Pregnancy (the mat should not be used or should be discussed with your doctor beforehand).
- Acne, eczema, psoriasis or any form of major skin damage in the area of application (use the mat – if at all – only with an overlay and reduce application time – ask your doctor!)
- Inflammations in the area of application (due to increased blood circulation and the warming effect, the inflammation can worsen sometimes).
- User weight of less than 40 kg
- User weight of more than 120 kg (it could be too much strain on the skin)
- If you suffer from fibromyalgia and increased sensitivity to pain, the application time should initially be limited. It is also advisable to place a thicker layer of fabric on the mat – consult your doctor!

Possible side effects

- The first few minutes after lying down can be a little unpleasant, but after that a warm and soothing feeling should develop.
- Slight feeling of dizziness or headache may occur occasionally shortly after usage.
- In case of whiplash injuries, dizziness and neck pain can increase. The ideal treatment duration and intensity must be individually optimised – ask your doctor!

Warnings/important information for usage

- Be careful when handling the mat (lying down, standing up, unpacking and packing) to protect your hands, eyes and other sensitive body parts from injuries caused by the tips.
- We advise against sleeping on the mat. Many users report that they fall asleep while using the Ergotopia® Acupressure Mat. As the application time should be approx. 10 to 40 minutes, it is recommended to set an alarm clock, if you are unsure whether you will fall asleep. Lying on the mat for several hours at a time may irritate the skin and/or lead to undesired side effects.
- Be sure to carefully read the contraindications and consult your doctor.
- We recommend the mat for use on the back.
- The cushion is initially felt by many to be too intense – if you feel the same way, use a thin layer of fabric or simply replace it with a normal pillow.
- Start with 10 to 15 minutes every 2 to 3 days and slowly increase to up to 40 minutes daily (if necessary in consultation with your doctor or therapist).
- The most intensive application is the direct contact with the skin. However, the intensity of the application can be individually adjusted to one's own sensation with the help of a fabric layer (e.g. t-shirt, thin towel).
- Create a calm and relaxed atmosphere before using the Ergotopia® Acupressure Mat.
- The usage of the mat by children is possible, but should be done under the supervision of an adult. As children's skin is more sensitive, it is advisable to use a fabric overlay or a t-shirt.

The material

- The filling material of the mat and the cushion is made of 100 % polyurethane.
- The acupressure tips are made of 100 % acrylonitrile-butadiene-styrene copolymer (ABS).
- The covers of the cushion and the mat are made of 100 % cotton.

Safety notice

The materials used are flammable and do not contain any flame-retardant chemicals. Do not expose the mat or cushion to open fire, sparks or other fire-inducing heat sources

Correct cleaning

Care instructions

The cotton cover is removable and can be washed by hand. Beware of the tips – turn the cover to the left and be careful. Do not expose the acupressure set to direct sunlight when drying. Not suitable for the dryer.

The carrier bag can be washed in the washing machine at a maximum temperature of 40 °C. The bag can also be washed in a washing machine. Please wash them with similar colours. Not suitable for the dryer.

Disinfection data

To disinfect your acupressure set, first remove the foam cores. Then apply / spray a disinfectant suitable for polyurethane foam onto the filling material. Air the foam cores for at least 12 hours before reinserting them into the cover.

Use a disinfectant detergent suitable for cotton and ABS to wash the covers. Recommended: Hand wash at max. 40 °C.

To also disinfect the bag, wash it in the washing machine with disinfectant detergent as indicated above.

Further helpful information on the Ergotopia® Acupressure Mat

Storage and transport conditions

The acupressure set should be stored and transported under constant conditions. The optimum temperature is 20 to 24 °C with an humidity of 40 to 60 %.

Scope of delivery

1 x the Ergotopia Acupressure Set is supplied, consisting of a mat and a cushion. The covers are made of cotton fabric with ABS tips, the filling of polyurethane foam. A carrying bag and this manual are also supplied.

Note on the scope of delivery:

If the acupressure set arrives differently than shown on the picture or if you have any defects, please contact us immediately at info@ergotopia.de or +49 (0) 6262-8542-977 and refrain from using it for the time being.

Application problems

In case of problems during application, please contact the responsible dealer or the responsible customer consultant of Ergotopia GmbH and do not use the acupressure set until clarification for the time being.

E-mail: info@ergotopia.de | Telephone: +49 (0) 6262-8542-977

Information to authorities in case of incidents

If serious incidents occur while using the Acupressure Set, these must be reported immediately to us and the responsible authorities:

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Medicines & Healthcare products Regulatory Agency (MHRA)
MHRA, 10 South Colonnade, London, E14 4PU, United Kingdom
info@mhra.gov.uk
<https://www.gov.uk/government/organisations/medicines-and-healthcare-products-regulatory-agency>

Information about repeat usage

The acupressure set is not intended for repeat usage.

Notes on safe disposal

Disposal with household waste.

Recommended duration of usage

3 years

We wish you a painless and productive time with your new relaxing companion!

If you are satisfied with the Ergotopia® Acupressure Mat and it helps you, we would be very happy about a positive product review! Please share your experiences with others who might benefit from the mat: Why did you choose the Ergotopia® acupressure mat? How do you feel during and after the application? To whom and for what would you recommend the mat?

We are very grateful for your opinion and your support for the Ergotopia® mission to create the ideal healthy and painless working and living environment for people around the world. :-)

You have questions, wishes or suggestions?
Feel free to contact us at any time!

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Explanation of the pictograms: Symbols used in DIN EN ISO 15223-1:2017

	MANUFACTURER Shows the manufacturer of the medical device according to the EU regulation 2017/745.
	DATE OF MANUFACTURE Displays the date on which the medical device was manufactured.
	BATCH NUMBER, CHARGE Displays the manufacturer's batch name so that the batch or lot can be identified (batch number-year-month).
	FOLLOW THE INSTRUCTIONS FOR USE Indicates the need for the user to consult the instructions for use.
	ATTENTION Indicates the need for the user to review the instructions for use regarding important, safety-related information, such as warnings and precautions, which cannot be placed on the product itself, for a multitude of reasons.

Washing labels for the cover

	Hand wash at maximum 40°C		Do not put into the dryer
	Do not iron		Do not bleach

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Ergotopia GmbH, Reichweinstr. 25, 74867 Neunkirchen