


Sheep A/V Complete

Ovine Artificial Vagina & Accessories

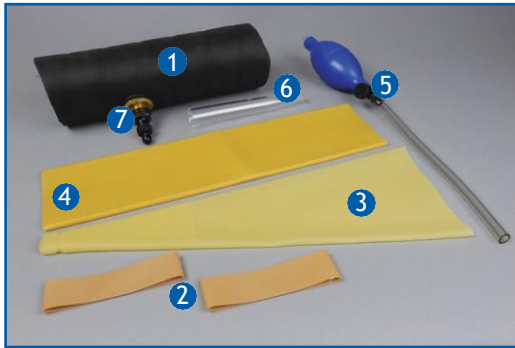

SHEEP A/V COMPLETE KIT INCLUDES:

1. Outer Casing
2. Rubber Bands (2)
3. Director Cone
4. Inner Liner
5. Pressure Bulb
6. Collection Vial
7. Water / Air Filling Valve

INSTRUCTIONS FOR USE:

STEP 1 - ASSEMBLY OF INNER LINER INTO OUTER CASING

1. Place the Inner Liner into the Outer Casing.
2. Fold the ends of the Inner Liner over the Outer Casing. Keep the Inner Liner as smooth as possible when folding over.
3. If using the Collection Vial, cut off the end of the Director Cone and place the vial into the end of the Cone. If not using the Collection Vial, the end of the Director Cone can be left uncut.

STEP 2 - ASSEMBLY OF DIRECTOR CONE ONTO OUTER CASING AND INNER LINER

1. Place Collection Cone directly onto the end of the Outer Tube.
2. Secure the Collection Cone in place with one of the Rubber Bands.
3. Place the other Rubber Band over the end of the Inner Liner to secure in place.

IF USING A SHORTER COLLECTION CONE:

1. Place Collection Cone, with or without Collection Vial, into the Outer Casing.
2. Fold the open end of the Collection Cone over the end of the Outer Casing.
3. Secure both ends of the Collection Cone in place with the Rubber Bands.

