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ZIRCON-ACRYLIC-COMPOSITE SYSTEM.

Indication:

The zirconium bonder serves as adhesive between plastics / composites and zirconium dioxide of all types.

Application:

Before the application the surface should be clean, dry and free from dust and fat. It is most easy to blow the surface off with oil-free compressed air. Alternatively the surface can also be cleaned with steam or acetone. When cleaning with steam and acetone the zirconium surface must be sufficiently dry.

Caution: the frame must be completely dry! The zirconium bonder must be applied thickly with a brush on the frame and aired for approx. 1 minute. It may be necessary to apply 2 coats. Subsequently, the pre-treated zirconia frame with all usual, light-hardening composites can be lined.

Storage:

The zirconium bonder should be stored under temperatures between 5–25 °C. Make sure that the bottle is closed immediately after use and protected against influence of light.

Durability:

See label for maximum durability. After expiry date has ended, the zirconium bonder should no longer be used.

Warranty:

Our application technology information, whether they are transferred verbally, in writing or practically, are based on our own experiences and can only be seen from here as recommendations. We develop our products continuously. For this reason we reserve the right to make possible changes.

Note:

Please forward all above information to the dentist in charge. Our existing safety data sheets are very helpful and important for the processing.

Side effects:

In case of appropriate application and processing of our product unwanted side effects occur very rarely. Immune reactions (e.g. allergy) or local discomfort cannot however be completely excluded. If unwanted side effects are communicated to you, we would be very grateful, if you would forward this to us.

Contraindications / interactions:

If the patient reacts with hypersensitivity to one of the components, this product may not be used or only under strict supervision of the treating physician/dentist. In these cases the composition of the medicine product supplied by us is available on request. Well-known cross reactions or interaction of the medical product with other materials already in the mouth must be considered by the dentist when using.

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anax dent GmbH Olgastr. 120a D-70180 Stuttgart